

Philips Avent Breastcare thermo pads

2-in-1

SCF258/02



Helps to stimulate and soothe

To stimulate breast milk flow before feed

The Philips Avent SCF258/02 helps to stimulate and soothe.

Soothe and stimulate

- Cold use to sooth sore or engorged breasts
- Warm use to stimulate milk flow before breast feed

Other benefits

- Developed with breastfeeding expert
- Includes soft sleeves



Specifications

Country of origin

• China

What is included

• 2-in-1 Thermopad: 2 pcs

Dimensions & weight

• Dimensions: 175x175x40 mm

Design

- Breast shaped design
- · Soft feel slave: Comfort on skin
- · Design by breastfeeding expert

Functions

- Cold use: Sooth sore breasts
- · Warm use: Stimulate milk flow

Highlights

Cold use



To sooth sore or engorged breasts. Keep in fridge or freezer. Always use with sleeves.

Developed with expert



Developed with midwife and baby adviser Vicky Scott who has been supporting mums with breastfeeding for 15 years.

Includes soft sleeves



For greater confort on skin

Warm use



To stimulate milk flow before breast feeding. Lightly dampen the sleeves, as moist heat gives an extra comfortable feel and helps stimulation. Heat the pad in hot water for 10 minutes.



Version: 4.2.1

© 2016 Koninklijke Philips N.V. All Rights reserved.

Specifications are subject to change without notice. Trademarks are the property of Koninklijke Philips N.V. or their respective owners.

