


Installation Guide

Installation Kit for Mounting Philips IntelliVue MP5/20/30/40/50/60/70, MX400/450/500/550/600/700/800 on
GE Healthcare Aisys Carestation

The purpose of this guide is to:


1. Describe mounting of Top Plate/Camlock Rail Assembly on top of machine.
2. Describe attachment of Table Mount to Camlock Mounting Plate.
3. Describe mounting of Camlock Mounting Plate in camlock rails.
4. Describe attachment of Display Mounting Bracket to display arm.
6. Describe mounting of Articulating Arm on left side of machine.

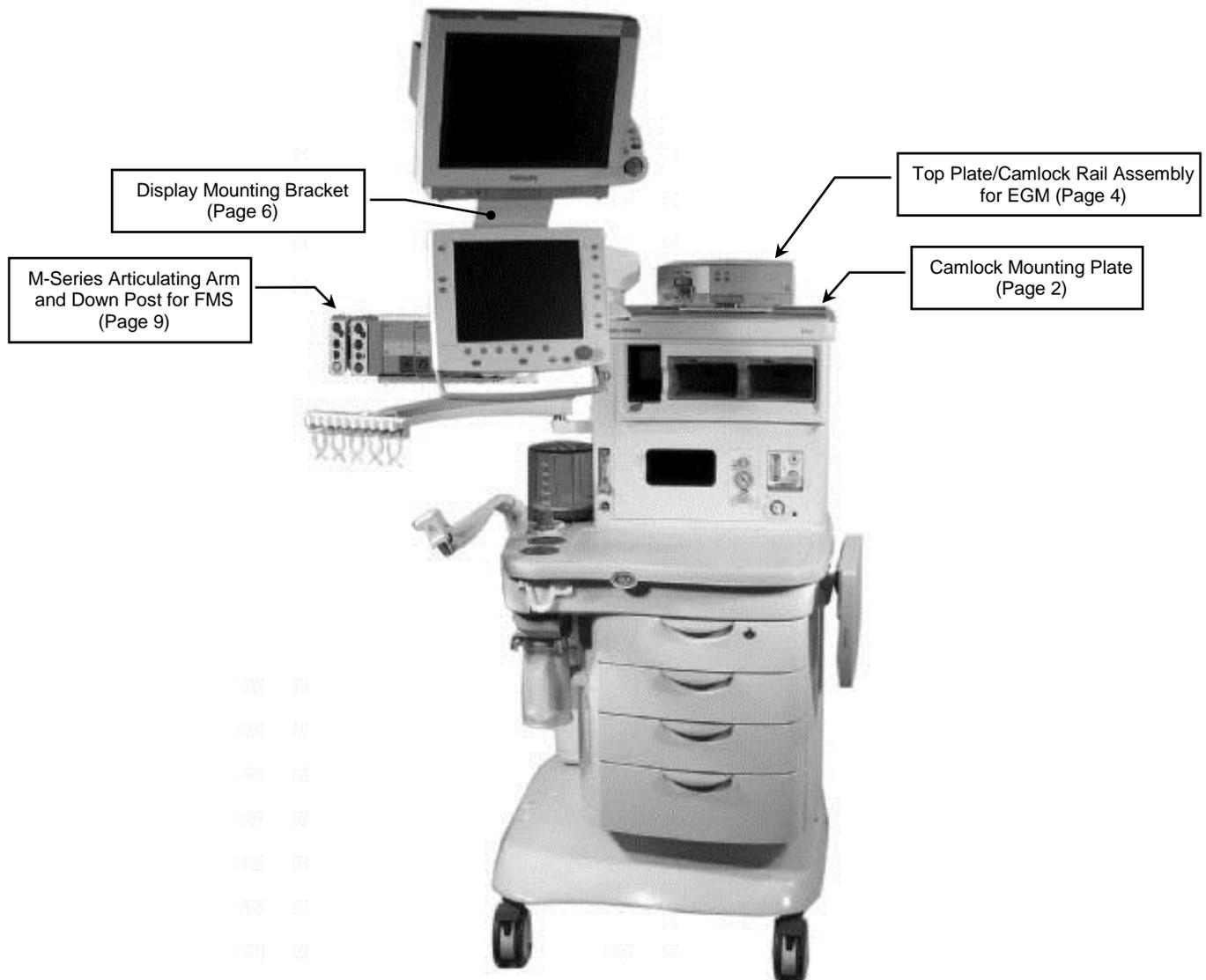

Parts Reference

The following parts and hardware are included with this installation kit (hardware not shown):


Item #	Description	Qty
1	Top Plate/Camlock Rail Assembly	1
2	M5 x 12mm Flat Head Machine Screw (FHMS)	4

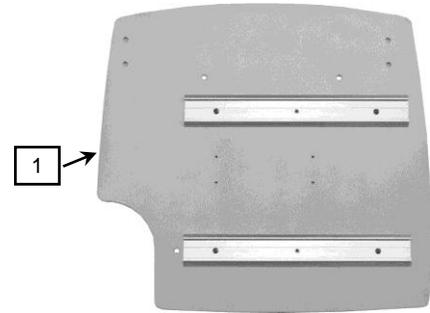


Tools required: Phillips screwdriver (not provided).

Attaching Top Plate to Top of Anesthesia Machine


1. Locate the four (4) threaded inserts shown in photo below. **Installation Notes:** It may be necessary to remove hole covers to expose the threaded inserts. The right front mounting hole will be covered by the front Camlock Rail (see step 3).

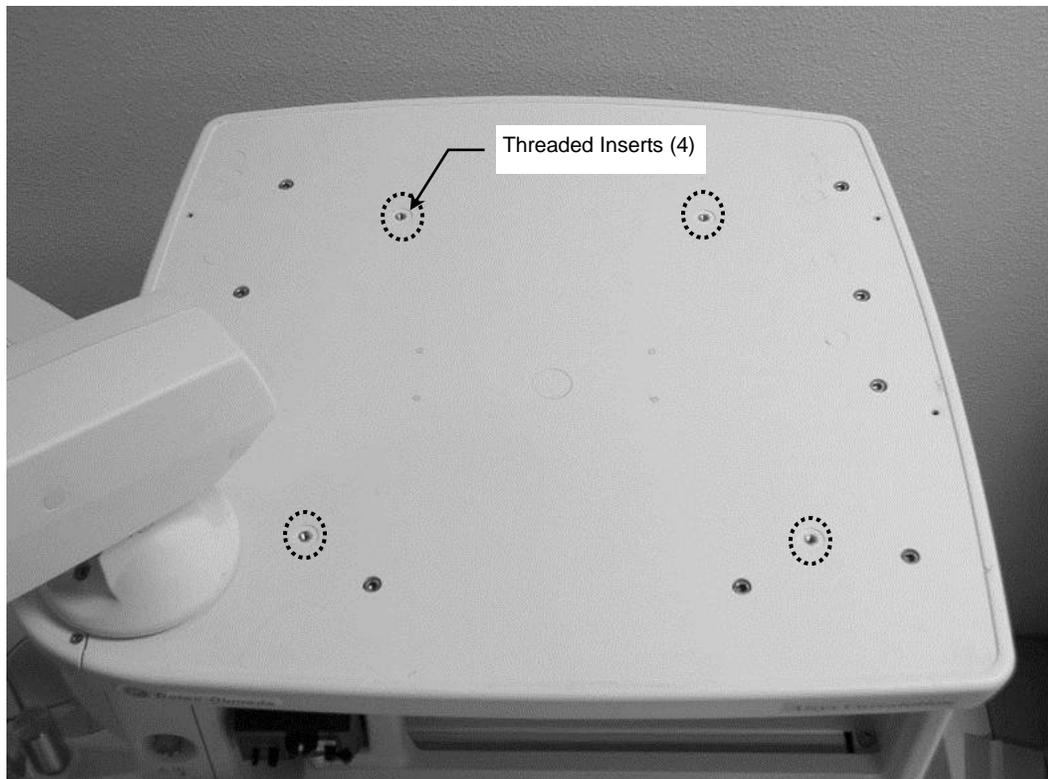

2. Align three (3) mounting holes in Top Plate with corresponding threaded inserts in top of machine. Fasten Top Plate to machine with three (3) M5 x 12mm FHMS. **Installation Note:** A fourth screw will be installed in step 3.

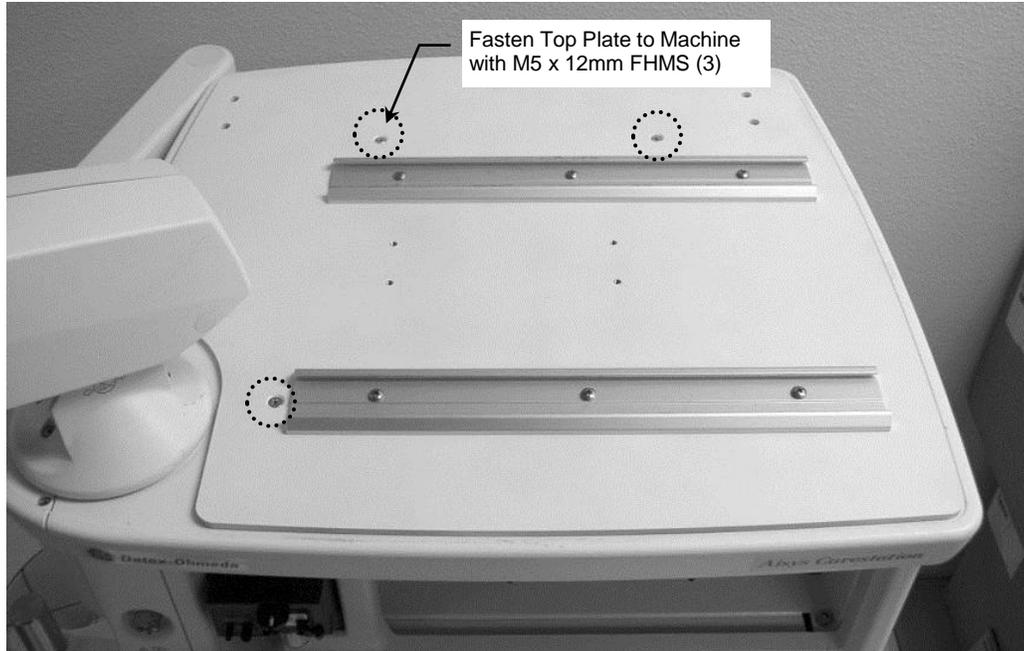


3. Unscrew and remove right and center screws from front Camlock Rail. Loosen left screw and rotate Rail enough to allow access to fourth mounting hole. Install M5 x 12mm FHMS in threaded insert. Return Rail to original mounting position, reinstall right and center screws, and tighten left screw in Camlock Rail.

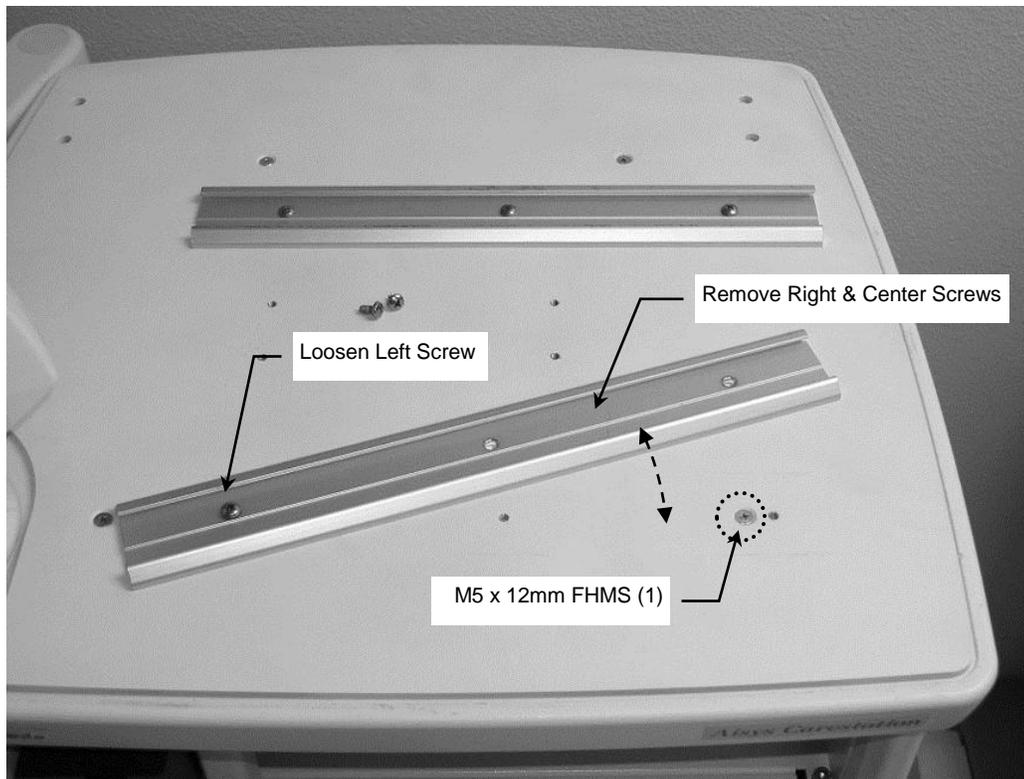

Attaching Table Mount to Camlock Mounting Plate

The following parts and hardware are included with this installation kit (hardware not shown):

Item #	Description	Qty
1	Camlock Mounting Plate	1
2	M6 x 10mm Flat Head Machine Screw (FHMS)	3
3	M4 x 20mm PHMS	2
4	#6-32 x 1/2" PHMS	1

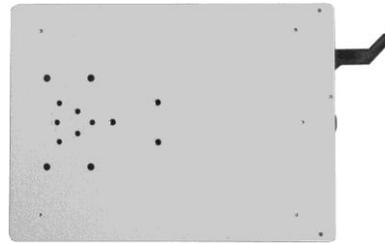

Camlock Mounting Plate

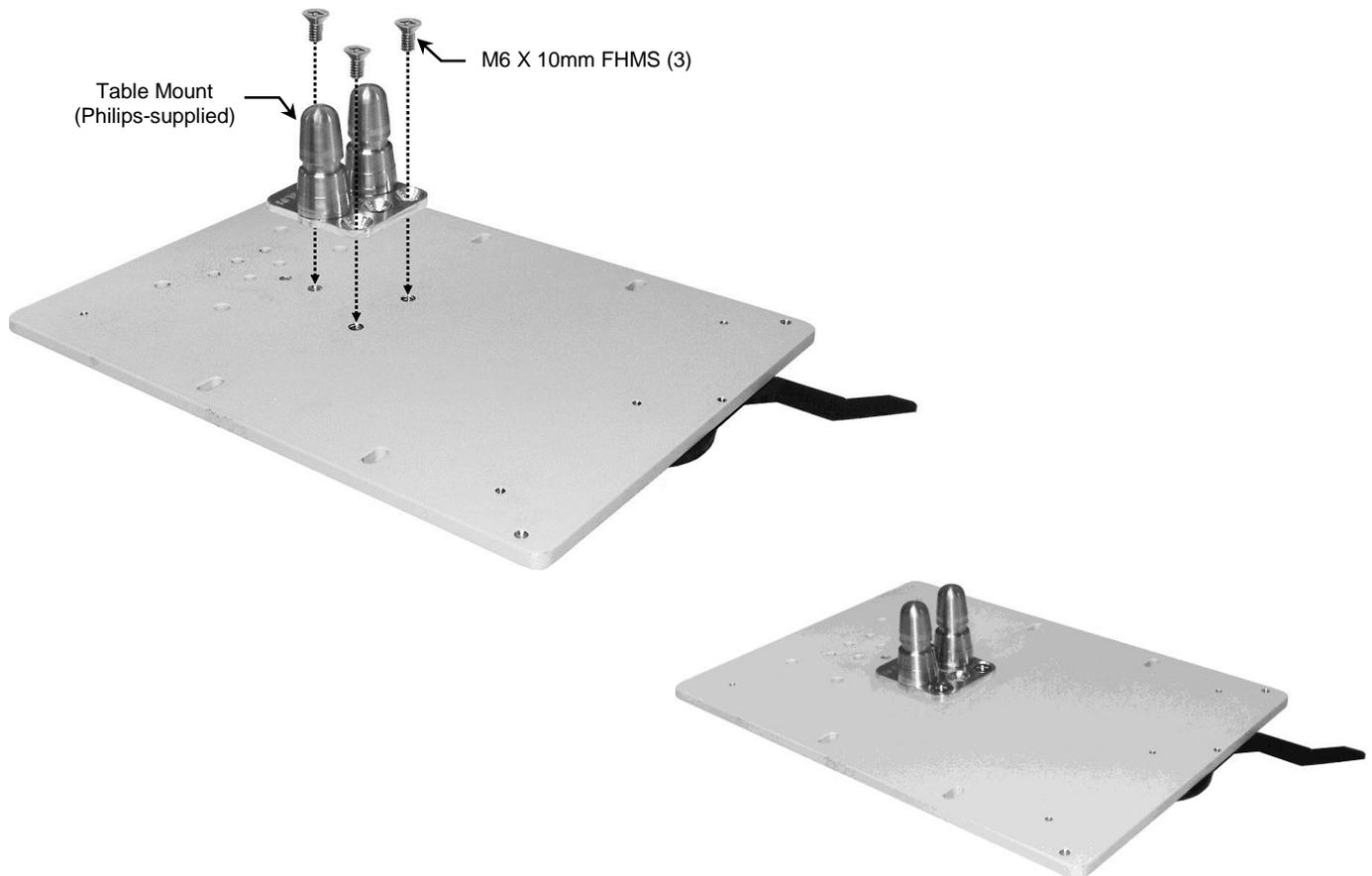


Table Mount
(Supplied by Philips)

Tools required: Phillips screwdriver (not provided).


Installation Note: Table Mount is supplied by Philips.

1. Fasten Table Mount to Camlock Mounting Plate with three (3) M6 x 10mm FHMS.




Mounting Camlock Mount in Camlock Rails

1. Insert four (4) feet (bottom of Camlock Mount) into Camlock Rails. **Installation Note:** Camlock Lever must be in the "unlocked" position when mounting feet in Rails.

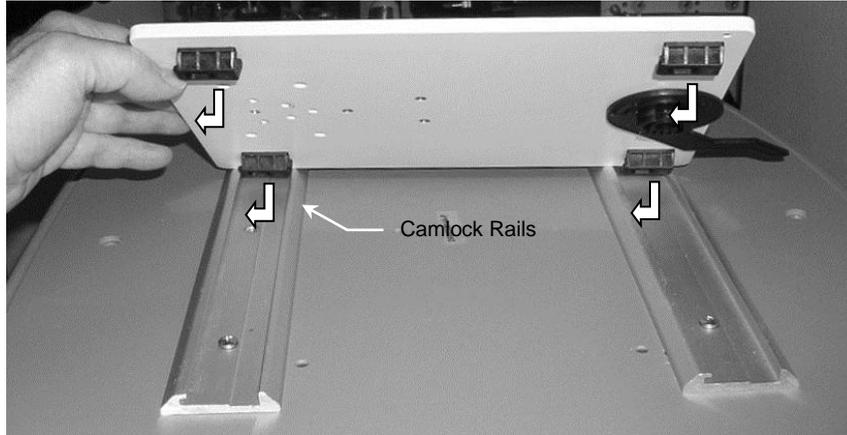

2. Move Camlock Lever from right to left to lock Mount in Rails. To prevent Camlock Lever from being unlocked, insert one (1) #6-32 x 3/8" PHMS in threaded hole to the right of Lever. Insert one (1) M4 x 20mm PHMS in each of the two (2) threaded holes at rear corners of the Mount as shown below. Stop tightening when screws contact bottom of Rail.

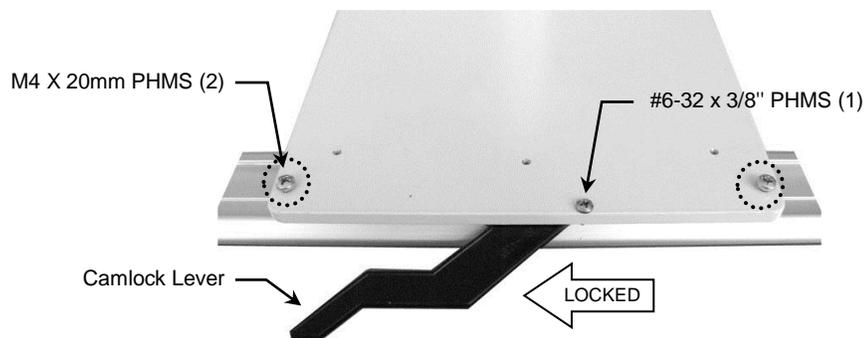

Mounting the Monitor


Mount the monitor in accordance with the Philips Table Mount instructions.



Attaching Mounting Bracket to Display Arm

The following parts and hardware are included with this installation kit (hardware not shown):


Item #	Description	Qty
1	Mounting Bracket, Ventilator Display	1
2	M6 x 16 mm Socket Head Cap Screw (SHCS)	4
3	M4 x 12 mm SHCS	4
4	M6 x 8 mm Flat Head Machine Screw (FHMS)	3
5	3 mm Hex Wrench	1
6	5 mm Hex Wrench	1

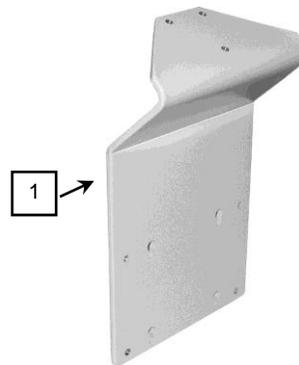

Tools Required

- 3 mm hex wrench (provided)
- 5 mm hex wrench (provided)
- Phillips screwdriver (not provided)


NOTICE! Before installing the Philips MP70 Display Mounting Kit on the Aisys anesthesia machine, confirm the latest revision, double-pin Wrist Casting is mounted on the Aisys Machine. This can be easily verified by gently lifting the end cap on the **left side arm** of the Wrist Casting and folding back the plastic coating under the cap about 1/2 inch as shown below. The head of the left side stop pin will be visible if the Wrist Casting is the latest revision, double-pin Wrist Casting Assembly. If the left side stop pin is not present, obtain the double-pin Wrist Casting from GEHC before installing this Kit.

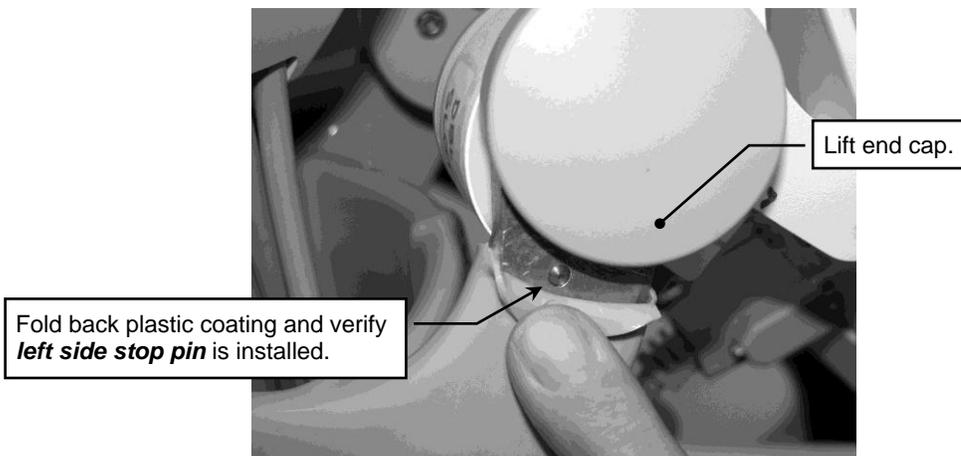

1. Remove ventilator display from display arm.




2. Using the 5 mm hex wrench provided, fasten the Mounting Bracket to the arm with four (4) M6 x 16 mm SHCS as shown.

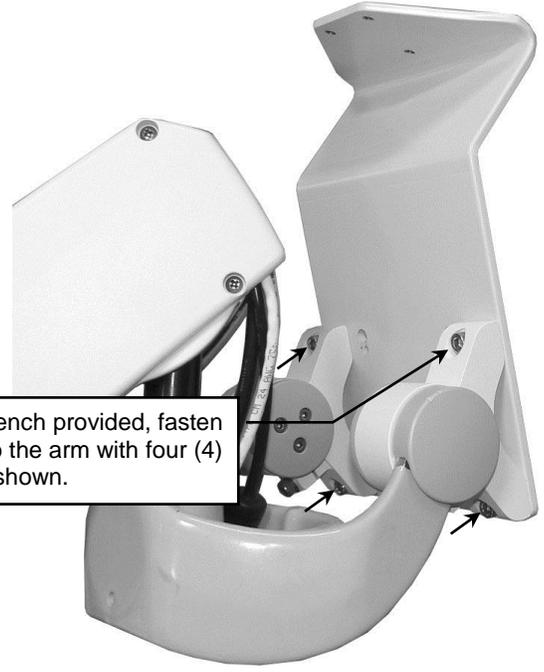


3. Thread four (4) M4 x 12mm SHCS into the mounting holes of the 75mm VESA mounting pattern on rear of ventilator display. Leave 8mm of thread exposed for inserting screw heads through keyholes in Mounting Bracket.



4. Mount ventilator display on Mounting Bracket by hanging the four (4) SHCS in the keyholes. Using the 3mm hex wrench provided, tighten four (4) SHCS.

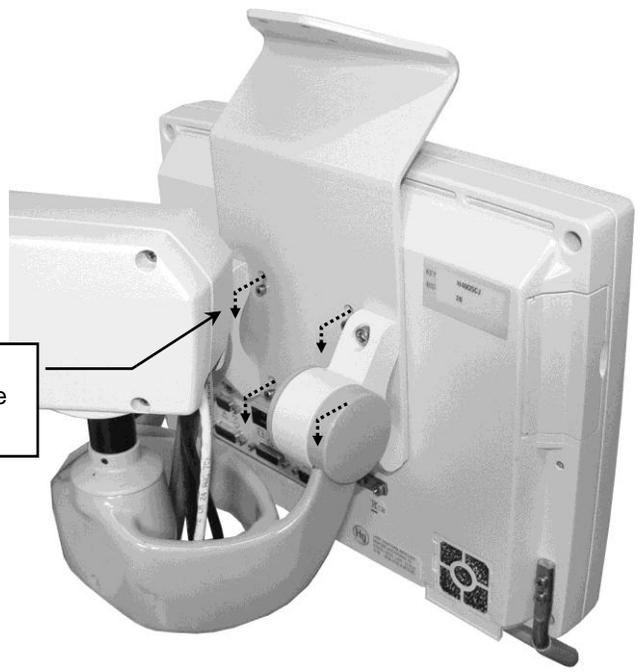

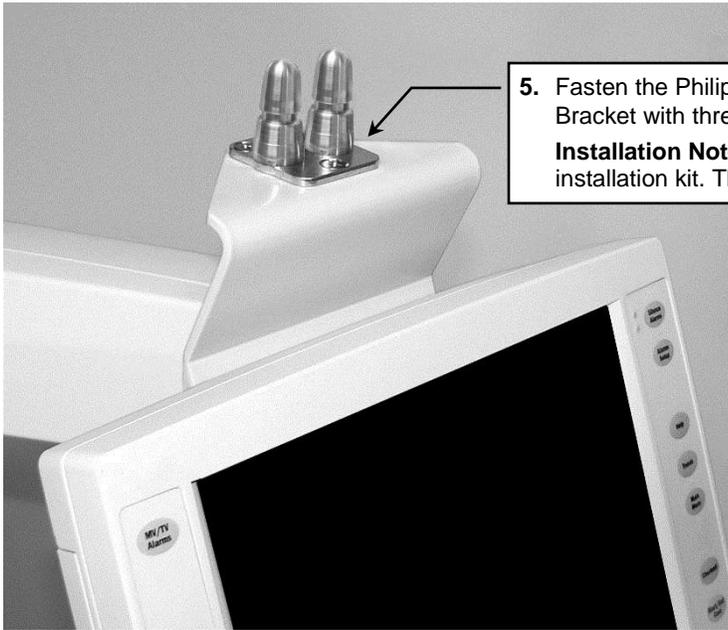


5. Fasten the Philips-supplied table mount to the top of the Mounting Bracket with three (3) M6 x 8 mm FHMS.

Installation Note: Three M6 x 8 mm FHMS are provided in this installation kit. The Table Mount is supplied by Philips.



CAUTION! Before mounting the Philips MP70 Display on the Mounting Bracket, ensure there is adequate friction being applied to both Wrist Casting Bearing Housings by torquing, uniformly, each clamping screw on the 2 Bearing Housings. The appropriate friction shall be able to prevent the Philips MP70 Display from rotating forward without assistance of the user.

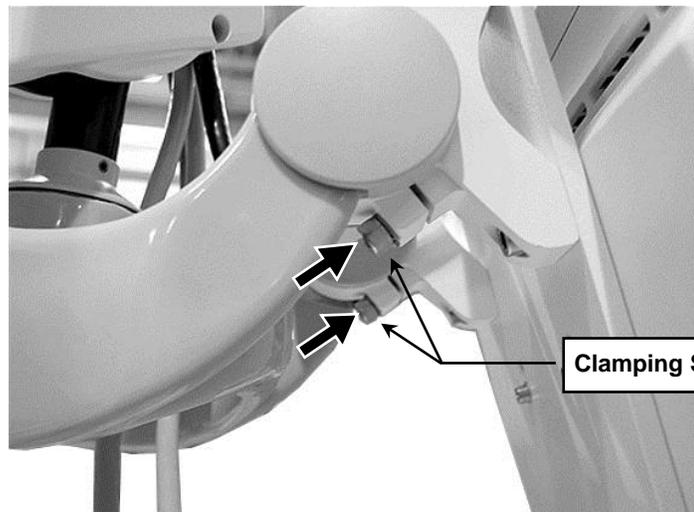

Mounting Monitor on Table Mount


Mount monitor in accordance with Philips Table Mount instructions



Attaching Down Post to Articulating Arm and Mounting Arm in Left Channel



The following parts and hardware are included for this installation procedure (hardware not shown):

Item #	Description	Qty
1	M-Series Articulating Arm, 12" x 12"	1
2	Down Post, 6"	1
3	#10-32 x 3/8" Socket Head Cap Screw (SHCS)	3
4	5/32" Hex Wrench	1
5	1/8" Hex Wrench	1

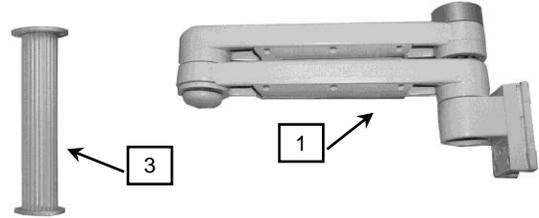


Tools required: 5/32" and 1/8" hex wrenches (provided).

- Using the 5/32" hex wrench provided, fasten Post to Swivel Cup with three (3) #10-32 x 3/8" SHCS as shown below.

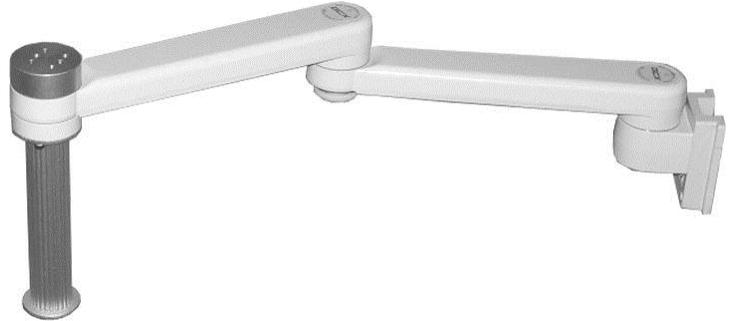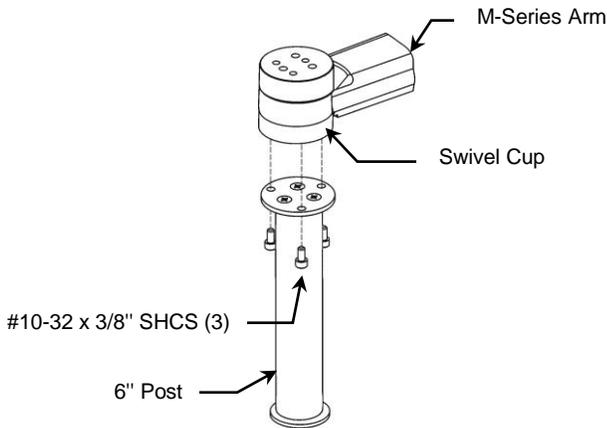


- Insert the Slide (rear of Arm) in bottom of left side channel and move to desired mounting position. Using the 1/8" hex wrench provided, tighten the two (2) socket head set screws at bottom of Slide.

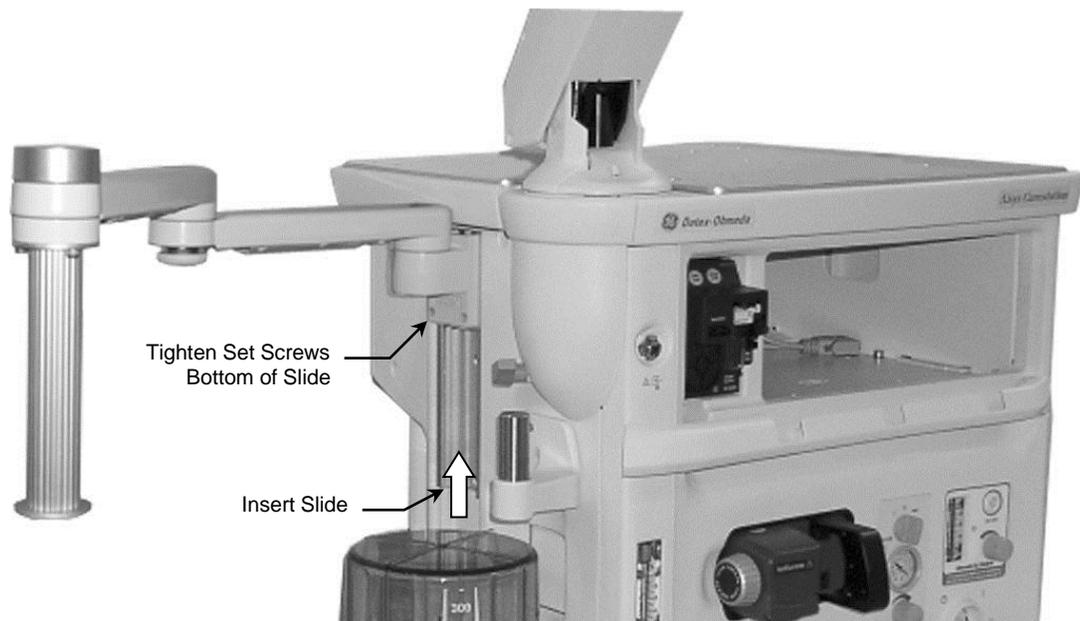

Mounting FMS on Down Post

Follow Philips instructions for mounting FMS on Down Post. A Philips-supplied clamp is required for this installation.

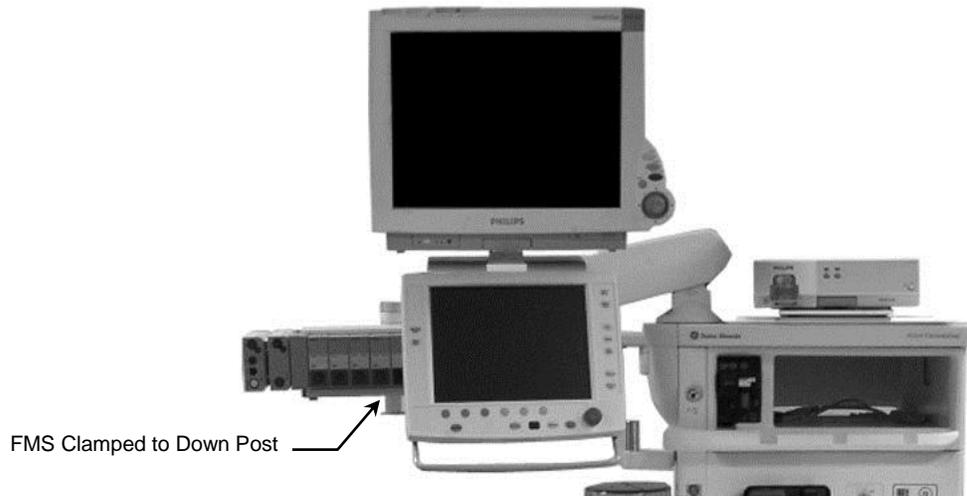

Routine Maintenance

Periodically check all mounting hardware. Tighten as necessary for optimal operation.

Cleaning the Mounting Assembly

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted instruments or accessory equipment, refer to the specific instructions delivered with those products.

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone and trichloroethylene.
3. Do not use steel wool or other abrasive material to clean the mounting assembly.
4. Damage caused by the use of unapproved substances or processes will not be covered by warranty. We recommended that you test any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the mounting assemblies. Wipe any cleaning agents off of the mounting assemblies immediately, using a water-dampened cloth. Dry parts thoroughly after cleaning.