Installation Note

Universal Docking Station

Assembly of the Universal Docking Station

Attach the universal mounting clamp to the Universal Docking Station and then affix the cable in the cable clamp according to the following instructions to ensure that the cable is attached permanently.



Printed in Germany 3/2011 Part number 4535 642 54381






1 Attach the universal mounting clamp (M8040A C05) to the Universal Docking Station (M8040A A03) with two M4x16 Torx screws in either vertical or horizontal position to suit your installation.








- 2 Remove the cable clamp from its position on the Universal Docking Station:
 - a. Applying gentle pressure to the cable clamp, and
 - b. using a small flat-bladed screwdriver, carefully free the retaining lugs (1) and push the cable clamp free of the Universal Docking Station.
- 3 Place the cable (1) on the cable clamp (2), as shown in the picture. Use the visual guide, "two black dots" (3) to ensure you have the cable correctly positioned.
- PHILIPS

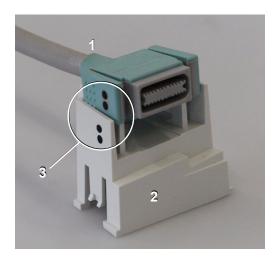

- 4 Squeeze the side clips (1) open slightly and ease the cable into the cable clamp so that the clips grip the pillars (2).
- 5 Press the cable down firmly to ensure that it is fully seated in the cable clamp.

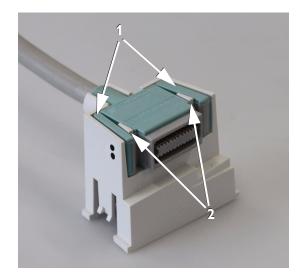

6 Return the cable clamp to its position on the Universal Docking Station.


7 Push the cable clamp into the housing until the retaining lugs lock in the Universal Docking Station.







Mounting the Battery Extension with MP2 or X2 to the Universal Docking Station

 Slide the rails (1) on the Battery Extension (865297) into the slots (2) on the Universal Docking Station until it locks in position.

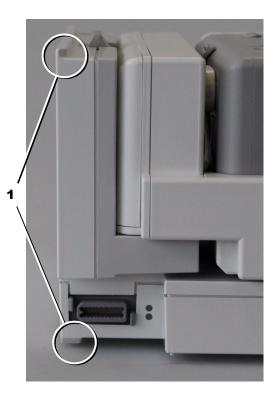

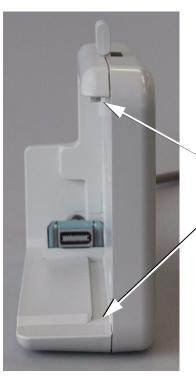

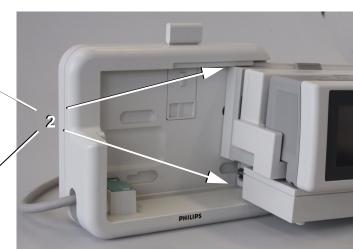

 To remove Battery Extension, MP2 or X2, plus MMS Extension combination from the Universal Docking Station, release the snap lock (1) and slide out the Battery Extension.

NOTE: The Battery Extension and equipment mounted on the Battery Extension must be considered as a single entity and must be removed from the Universal Docking Station as such and not individually.

- 3 If you wish to swap/remove equipment on the Battery Extension whilst it is mounted on the Universal Docking Station, for example, replace an MP2 with an X2, or an M3014A Capnography extension with anM3015A Microstream CO₂ extension:
 - a. Undock the Battery Extension from the Universal Docking Station
 - b. Remove/swap the selected equipment on the Battery Extension
 - c. Dock the Battery extension on the Universal Docking Station.



Mounting the MMS, MP2 or X2 to the Universal Docking Station

 Align the feet on the MMS, MP2 or X2 with the locating shoes (1) on the Universal Docking Station and slide the MMS, MP2 or X2 onto the Universal Docking Station until it locks in position.





2 You can also combine the MMS, MP2 or X2 with various MMS Extensions, for example the M3015A as shown in the illustration



3 To remove the MMS, MP2 or X2 (with or without MMS Extensions) from the Universal Docking Station, release the snap lock (1) and slide out the MMS, MP2 or X2.

