# TRAINING PADS CARTRIDGES

CARTOUCHES D'ÉLECTRODES DE FORMATION  
SCHULUNGSELEKTRODENPAKETE  
TRAININGSELEKTRODECASSETTE  
CARTUCHOS DE ELECTRODOS DE ENTRENAMIENTO  
CARTUCCE PER ELETTRODI DI ADDESTRAMENTO  
CARTUCHOS DE ELETRODOS DE TREINAMENTO  
ÖVNINGSELEKTRODPATRONER  
KASSETTER MED ØVELSESELEKTRODER  
KASSETTER MED TRÆNINGSELEKTRODER  
WKŁADY ELEKTROD SZKOLENIOWYCH  
ZÁSOBNÍKY NA VÝUKOVÉ ELEKTRODY  
КОНТЕЙНЕРЫ С УЧЕБНЫМИ ЭЛЕКТРОДАМИ  
ΘΗΚΕΣ ΕΚΠΑΙΔΕΥΤΙΚΩΝ PADS  
EĞİTİM PEDİ KARTUŞLARI

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**ENGLISH:** About this edition

The information in this guide applies to the models M5073A and M5074A training pads cartridges for use with HeartStart HSI defibrillators. These include the HeartStart OnSite Defibrillator M5066A, the HeartStart Defibrillator M5067A, and the HeartStart Home Defibrillator M5068A. This information is subject to change. Please contact Philips at [www.philips.com/productdocs](http://www.philips.com/productdocs) for information on revisions.

## Edition History

Edition 13

Publication date: June 2011

Publication #: 011870-0013

Printed in the U.S.A.

## Copyright

© 2011 Philips Electronics North America Corp.

No part of this publication may be reproduced, transmitted, transcribed, stored in a retrieval system or translated into any human or computer language in any form by any means without the consent of the copyright holder.

Unauthorized copying of this publication may not only infringe copyright but also reduce the ability of Philips Medical Systems to provide accurate and up-to-date information to users and operators alike.

## Device Manufacturer

Philips Medical Systems, Seattle WA 98121-1825, USA

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Philips Medical Systems, Seattle, WA 98121-1825, USA.

---

## M5073A AND M5074A TRAINING PADS CARTRIDGES

---

### DESCRIPTION AND USE

The HeartStart Adult and Infant/Child Training Pads cartridges are designed for use with these Philips automated external defibrillators: the HeartStart OnSite Defibrillator M5066A, the HeartStart Defibrillator M5067A, and the HeartStart Home Defibrillator M5068A (“HSI defibrillators”). The training pads cartridges are also for use with the Philips HeartStart Trainer M5085A and the Laerdal HeartStart Trainer M5086A. **These instructions are specific to use of the training pads cartridges with an HSI defibrillator.** Use of the training pads cartridge with a HeartStart Trainer is described in the *Instructions for Use* provided with the Trainer.

The training pads cartridges are reusable optional accessories for demonstration and occasional training applications. Replacement pads for the cartridges are also available. **These pads cannot be used to deliver defibrillation treatment.**

Each training pads cartridge has a removable clear protective lid with a handle, a resealable film cover, and a pair of reusable adhesive pads. It is packaged with:

- Illustrated directions for installing the cartridge, using the pads placement guide, and repackaging the cartridge after using it.
- A fold-out pads placement guide for adult or child, depending on the cartridge model. Metal areas on the placement guide allow the defibrillator to sense when the pads are properly applied.
- This *Instructions for Use* manual.

The illustrated directions are also printed on the inside front and back covers of this manual.

The Adult and Infant/Child Training Pads cartridges are yellow, to help prevent confusing them with the white SMART Pads cartridges used for patient treatment. In addition, the training pads cartridges have red labels that identify them as training cartridges.

Using a training pads cartridge installed in a HeartStart HSI defibrillator, the user can choose any of eight emergency scenarios. The scenarios are described in the table on page 3. The scenarios are designed to let the user become familiar with how to operate the HSI defibrillator.

To demonstrate or practice using the HSI defibrillator to treat an adult victim of sudden cardiac arrest, use the M5073A Adult Training Pads cartridge. Use the M5074A Infant/Child Training Pads cartridge to demonstrate or practice using the HSI defibrillator on an infant or a child less than eight years old or weighing less than 55 pounds (25 kg).

When the cartridge is installed, the HSI defibrillator recognizes whether it is an Adult or Infant/Child Training Pads cartridge. Depending on which cartridge is used, the defibrillator's voice instructions – including coaching in performing cardiopulmonary resuscitation (CPR) – will be appropriate for treating an adult or a child victim.

**NOTE:** Before using the HeartStart HSI Defibrillator, the user should read all instructions provided with the defibrillator, including information about when the defibrillator should and should not be used. Some models may also come with a training videotape. Be sure to watch the video before using the defibrillator. It is recommended that anyone who may use the HeartStart Defibrillator receive CPR/defibrillator training or other physician-authorized training.

**NOTE:** The HeartStart Training Pads cartridges cannot deliver energy and therefore cannot be used for patient treatment. If the HSI defibrillator is needed for patient treatment during a training session, replace the training pads cartridge with a HeartStart Adult (M5071A) or Infant/Child (M5072A) SMART Pads cartridge. Do not store the training pads cartridge with the defibrillator.

**NOTE:** It is recommended that the training pads be applied only to the pads placement guide or a properly equipped training manikin, not to people.

**NOTE:** Using the training pads requires power from the HSI defibrillator battery. The battery supports 10 hours of use in the training mode, after which the battery must be replaced.

## USING THE TRAINING PADS SCENARIOS

There are eight training scenarios available when using the training pads cartridges. They are described in the following table.

NO.	SCENARIO OVERVIEW	SCENARIO DETAILS
1	Shockable rhythm, one shock needed for conversion*	<ul style="list-style-type: none"> <li>defibrillator detects a shockable rhythm, tells user to deliver a shock</li> <li>defibrillator detects a non-shockable rhythm**</li> </ul>
2	Shockable rhythm, multiple shocks needed for conversion	<ul style="list-style-type: none"> <li>defibrillator detects a shockable rhythm, tells user to deliver a shock</li> <li>defibrillator still detects a shockable rhythm, tells user to deliver another shock (repeated until an entire shock series -- typically three shocks -- is delivered)</li> <li>defibrillator provides pause for CPR</li> <li>defibrillator detects a shockable rhythm, tells user to deliver a shock</li> <li>defibrillator detects a non-shockable rhythm</li> </ul>
3	Troubleshooting pads, one shock needed for conversion	<ul style="list-style-type: none"> <li>defibrillator detects poor pad contact, repeats pads placement instructions</li> <li>after one pad is removed and reapplied, defibrillator detects a shockable rhythm, tells user to deliver a shock</li> <li>defibrillator detects a non-shockable rhythm</li> </ul>
4	Shockable rhythm, conversion, return to shockable rhythm, conversion	<ul style="list-style-type: none"> <li>defibrillator detects a shockable rhythm, tells user to deliver a shock</li> <li>defibrillator detects a non-shockable rhythm</li> <li>defibrillator detects refribrillation (return to a shockable rhythm), tells user to deliver a shock</li> <li>defibrillator detects a non-shockable rhythm</li> </ul>
5	Non-shockable rhythm	<ul style="list-style-type: none"> <li>defibrillator detects a non-shockable rhythm throughout</li> </ul>
6	Shockable rhythm, two shocks needed for conversion	<ul style="list-style-type: none"> <li>defibrillator detects a shockable rhythm, tells user to deliver a shock</li> <li>defibrillator still detects a shockable rhythm, tells user to deliver another shock</li> <li>defibrillator detects a non-shockable rhythm</li> </ul>
7	Shockable rhythm, two shocks needed for conversion, return to shockable rhythm, conversion	<ul style="list-style-type: none"> <li>defibrillator detects a shockable rhythm, tells user to deliver a shock</li> <li>defibrillator still detects a shockable rhythm, tells user to deliver another shock</li> <li>defibrillator detects a non-shockable rhythm</li> <li>defibrillator detects refribrillation (return to a shockable rhythm), tells user to deliver a shock</li> <li>defibrillator detects a non-shockable rhythm</li> </ul>
8	Troubleshooting pads, two shocks needed for conversion	<ul style="list-style-type: none"> <li>defibrillator detects poor pad contact, repeats pads placement instructions</li> <li>after one pad is removed and reapplied, defibrillator detects a shockable rhythm, tells user to deliver a shock</li> <li>defibrillator still detects a shockable rhythm, tells user to deliver another shock</li> <li>defibrillator detects a non-shockable rhythm</li> </ul>

\* Conversion from ventricular fibrillation – the usual cause of sudden cardiac arrest – to a non-shockable rhythm.

\*\* When the defibrillator detects a non-shockable rhythm, it instructs the user to care for the patient and provide CPR if needed.

Not all scenarios are appropriate for all user training. Training scenario number 1 is the default scenario and is most commonly used when teaching how to use the defibrillator to treat a victim of sudden cardiac arrest.

If the training scenarios are used as part of a formal training course, the instructor may wish to:

- select only the scenarios appropriate for the training level goals,
- create case histories and patient details for the selected scenarios,
- highlight certain features of the defibrillator's performance,
- assure that users conform to local protocols,
- check pad placement, and
- evaluate user knowledge on topics such as CPR and assessing the patient.


### USING THE TRAINING PADS CARTRIDGE

To use the training pads cartridge, remove it from any packaging in which it is provided or stored, then follow the steps below.


**NOTE:** Do not remove the clear hard protective lid or resealable film cover from the training pads cartridge before installing it in the HSI defibrillator.

**NOTE:** To turn off the defibrillator at any time, press and hold the On/Off button until the defibrillator turns off.

1. Be sure that the HSI defibrillator has a battery and a white SMART Pads cartridge installed and that the green Ready light is blinking.
2. Remove the Adult or Infant/Child SMART Pads cartridge from the compartment on the front of the HSI defibrillator. To release the cartridge, slide the latch on the top of the unit over to the side (**A**). Then lift out the closed cartridge, as shown in the illustration (**B**).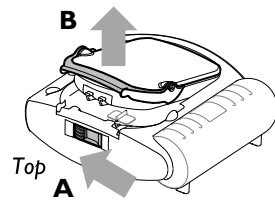

3. Insert the M5073A or M5074A Training Pads cartridge into the compartment on the front of the HSI Defibrillator (**A**). Press the cartridge down firmly until it clicks into place. Be sure the green handle is pressed all the way down (**B**).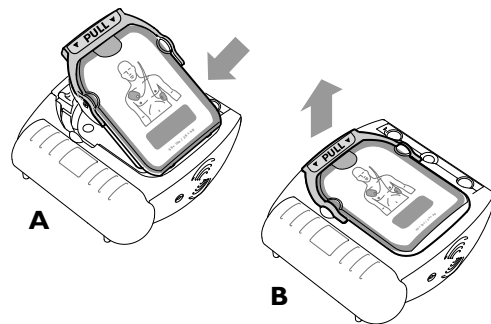

As soon as the training pads cartridge is installed, the defibrillator automatically turns on and verbally identifies the cartridge as either “Adult Training Pads” or “Infant/Child Training Pads,” then announces “Training.” The blue i-button on the defibrillator starts flashing. As a safety precaution, the defibrillator will instruct you to remove the training cartridge and insert a pads cartridge in case of an emergency. It will then invite you to press the flashing blue button to choose a training scenario.

**NOTE:** If you do not pull the handle or press the blue i-button within 30 seconds after installing the training pads cartridge, the defibrillator will go to standby training mode and the green Ready light will blink.

#### TO SELECT A TRAINING SCENARIO



The default training scenario is number 1. After installing the training pads cartridge, simply pull the cartridge handle or press the On/Off button to use this scenario. The defibrillator will not announce the scenario number.

To select a different scenario, press the flashing blue i-button to scroll through the eight scenarios in sequence. The defibrillator announces each scenario by number as you press the button. When you reach the scenario number you want, press and hold the On/Off button until the defibrillator turns off, or wait for 10 seconds and it will automatically go into standby training mode.

As long as the battery and training cartridge are not removed, you can delay the training session using the selected scenario for up to one hour. After one hour, the blue i-button will start blinking, and the defibrillator will start chirping to alert the user to either continue the training use or remove the training pads cartridge and install a HeartStart SMART Pads cartridge.

#### TO RUN A TRAINING SCENARIO

To run the selected training scenario, treat the training pads exactly as you would the SMART pads, using the Adult or Infant/Child Pads Placement Guide provided (shown at right) or a CPR training manikin with an External Manikin Adapter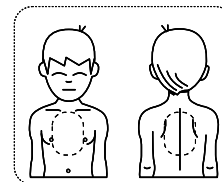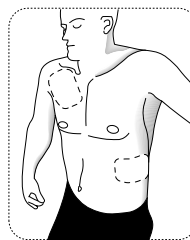


M5089A to simulate the patient. (Illustrated instructions for applying the manikin adapter to a training manikin are provided with the Adapter.) Most users will not need the adapter.

1. Pull the handle or press the On/Off button to turn on the defibrillator.


2. Remove the clear protective lid from the pads cartridge and set it aside.

Follow the HeartStart Defibrillator's voice instructions.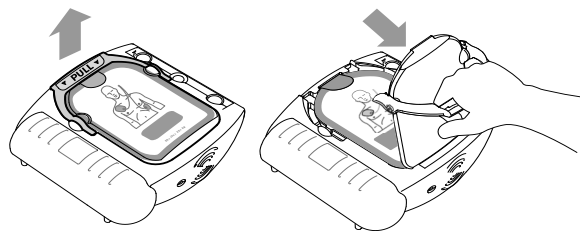


3. If using a clothed manikin, remove the clothing as instructed.

4. Pull the tab at the top of the pads cartridge to peel off the resealable film cover.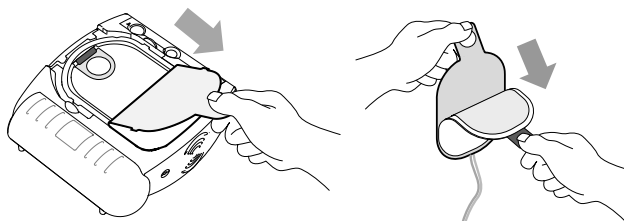

Inside are two adhesive pads on a yellow plastic liner. Remove the pads from the cartridge. Peel one pad off the liner.

5. Place the pad on the pads placement guide or the manikin, exactly as shown in the picture on the pad. (Note that pad placement for infants and young children differs from placement on adults.) Press the pad down firmly. Then peel and place the other pad. Be sure to remove the pads completely from the yellow liner before placing them. **PRECAUTION: Do not place the pads on a person.\***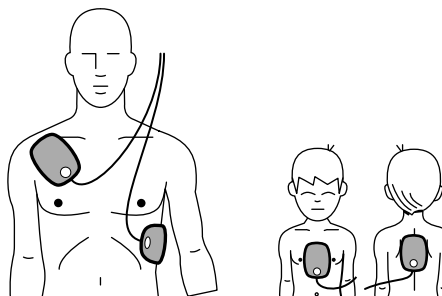

6. Continue to follow defibrillator instructions for remainder of scenario.

7. When the scenario has been completed, or to stop a scenario in process, press the On/Off button until the defibrillator turns off.

\* A Precaution is a condition, hazard, or unsafe practice that could result in minor personal injury.

8. To repeat the same scenario, repackage the pads as described below, then begin again by pulling the green handle or pressing the On/Off button. If you do this within one hour after the last training use, the defibrillator will automatically use the previously selected scenario. To select a different scenario, press the i-button. After replacing the battery, the defibrillator will default to scenario number 1.

## DEFIBRILLATOR PERFORMANCE DURING TRAINING

When a training scenario is running, the defibrillator behaves exactly as it would during actual emergency use, providing voice instructions, one or more simulated shocks, pauses for CPR, optional CPR guidance, etc.\*

For safety reasons, certain defibrillator features are not functional while the training pads cartridge is installed:

- The defibrillator cannot be used for actual patient care; it cannot analyze patient heart rhythm or deliver a shock.
- It will not perform the battery insertion and periodic self-tests.
- If it is not used (no buttons are pressed, or the cartridge handle is not pulled) for one hour, the defibrillator will start “chirping” to alert the user to either continue the training use or remove the training pads cartridge and install a HeartStart SMART Pads cartridge.
- If the i-button is pressed while the defibrillator is in training standby mode, the defibrillator will announce that it is training. If the i-button is pressed a second time, the defibrillator will tell you the number of the current training scenario. A third press of the i-button will allow you to select another scenario.

**NOTE:** If the battery power becomes low during training, the defibrillator will tell you to replace the battery immediately. It will begin chirping, and the blue i-button will start blinking. If you press the blue i-button or the green On/Off button, the defibrillator will repeat the instruction to replace the battery.

## AFTER USING THE TRAINING PADS CARTRIDGE

After you have finished using the training pads cartridge, press and hold the On/Off button to turn off the defibrillator. Remember that using the training


\* If the defibrillator's configuration has been changed from the default settings, the modifications are applied to its behavior during the training simulations.

pads cartridge requires battery power. The defibrillator battery is designed to support 10 hours of training use. If battery power is low after training, the defibrillator will chirp to alert you. Be sure to replace the battery with a new M5070A battery.



Prepare the training pads cartridge for its next use as follows:

1. Turn off the defibrillator. The following steps can be performed as illustrated or with the training pads cartridge in the defibrillator.


2. Reapply the pads to each side of the solid yellow liner, making sure that the red tabs are positioned at same end of the liner as the yellow tab **(A)**. Press the pads firmly together on the liner.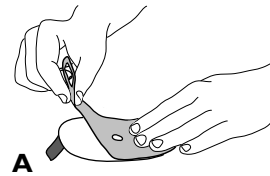


3. Untwist the wires. Then wind the wires around the defibrillator's cartridge recess, pressing them into the track **(B)**. Then place the pads over the wound wires in the cartridge. The chest pad should be on top, and the tab ends of the pads at the curved end of the cartridge **(C)**. Fold the tab down over the top pad **(D)**.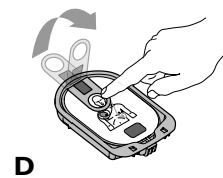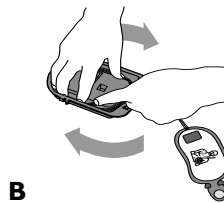

4. Press the resealable film cover over the cartridge, with the tab end at the top **(E)**. Be sure to fold the tab back down under the lid **(F)**.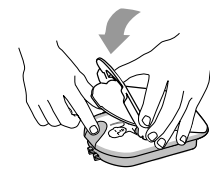

5. Fit the hooks on the bottom of the clear hard protective lid into the holes at the bottom edge of the cartridge, and press the lid down **(G)**. Be sure the cartridge handle snaps into place. The training cartridge is now ready for its next use.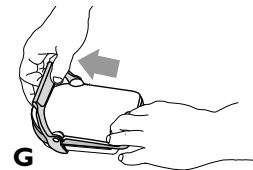

**NOTE:** If the cartridge is installed when you press down the protective lid, the defibrillator will turn on for scenario selection. Press and hold the On/Off button or wait 30 seconds and the defibrillator will automatically go to the training standby mode. If you turn the defibrillator off with the training pads cartridge installed without its protective lid, the defibrillator will chirp. Attach the lid and turn the defibrillator off to return it to standby training mode.

- If the training pads cartridge is installed at this point, slide the latch at the top of the defibrillator to release the cartridge. Remove it and store it in the resealable plastic bag. Then install a HeartStart Adult (M5071A) or Infant/Child (M5072A) SMART Pads cartridge so the defibrillator will be ready for use in the event of an emergency.


**NOTE:** To avoid confusion when using the defibrillator in an emergency, be sure to store training cartridges separately from the defibrillator.

#### MAXIMIZING SERVICE LIFE

The M5073A and M5074A Training Pads cartridges are designed for up to 100 uses. To help ensure continued performance, avoid placing the resealable film cover or adhesive side of the pads in heat, direct sunlight, or against surfaces that might transfer fiber, dirt, or lint. Periodically check the metal contacts on the back of the cartridge and in the defibrillator's cartridge recess and clean them with a damp soapy cloth if necessary. Do not use alcohol to clean the electrical contacts on the bottom of the cartridge or in the cartridge recess of the defibrillator.

When the original training pads have reached the end of their service life, you can order M5093A Replacement Adult Training Pads or M5094A Replacement Infant/Child Training Pads for the cartridge.

These include a set of pads and a replacement resealable film. Change the pads as shown in the figure.

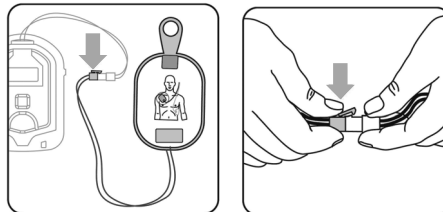NOTE: Please contact your local HeartStart distributor if you have any questions about using the training pads cartridge or need technical support. In the United States, if you have a HeartStart Home Defibrillator please call 1.866.DEFIBHOME (1.866.333.4246). If you have a HeartStart OnSite Defibrillator, please call 1.800.263.3342.

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# PHILIPS

Philips Medical Systems is part of Royal Philips Electronics

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Philips Medical Systems 隸屬 Royal Philips Electronics

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Philips Medical Systems

United States

Philips Medical Systems  
2301 Fifth Avenue, Suite 200  
Seattle, WA, USA 98121  
(800) 263-3342

Canada

Philips Healthcare, a Division of Philips Electronics Ltd.  
281 Hillmount Road  
Markham, Ontario  
L6C 2S3, Canada  
(800) 291-6743

Europe, Middle East, and Africa

Philips Medizin Systeme Boeblingen GmbH  
Cardiac and Monitoring Systems  
Hewlett-Packard Strasse 2  
71034 Boeblingen, Germany  
(+49) 7031-463-2254

Latin America

Philips Medical Systems Ltda.  
Rua Verbo Divino - 1400 - 7o Andar  
04719-002 -Sao Paulo, Brazil  
0800 7017789

Asia Pacific

Philips Electronics Hong Kong Ltd.  
6/F, Core Building I  
1 Science Park East Avenue  
Hong Kong Science Park  
Shatin, New Territories, Hong Kong  
(852) 2821 5888

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Niets uit deze uitgave mag in enige vorm of op enige manier worden verveelvoudigd, verzonden, getranscribeerd, opgeslagen in een geautomatiseerd gegevensbestand of vertaald in menselijke of computertaal zonder toestemming van de houder van het copyright.