

Installation Guide

Mounting Adapter for Philips Intellivue Docking Station

The purpose of this guide is to:


1. Describe attachment of Docking Station Mounting Adapter to Table Top Mounting Adapter.
2. Describe attachment of Docking Station to Mounting Adapter.

Parts Reference

The following parts and hardware are included for this installation procedure (see photos for parts; hardware not shown).

Item #	Description	Qty
1	Mounting Adapter, Docking Station	1
2	M6 x 10mm Flat Head Machine Screw (FHMS)	4
3	M4 x 8mm FHMS	4

Docking Station Mounting Adapter

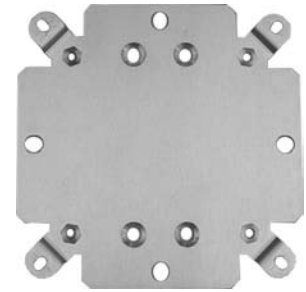

Tools Required: Phillips screwdriver.

Attaching Docking Station Mounting Adapter to Table Top Mounting Adapter

1. Fasten Mounting Adapter to Table Top Mounting Adapter with four (4) M6 x 8mm FHMS as shown below.

