
861394 Charger for the 989803150191 Battery  
Chargeur 861394 pour batterie 989803150191  
Ladegerät 861394 für den Akku 989803150191  
Batterijlader 861394 voor de 989803150191 batterij  
Cargador 861394 para la batería 989803150191  
Caricabatterie 861394 per la batteria 989803150191  
Carregador 861394 para bateria 989803150191  
Laddare 861394 til batteri 989803150191  
Laderen 861394 for batteriet 989803150191  
Akun 989803150191 laturi 861394  
989803150191 電池專用的 861394 充電器  
用于 989803150191 電池的 861394 充電器  
989803150191 バッテリ用充電器 (861394)  
989803150191 배터리용 861394 충전기

861394  
Edition 1

**PHILIPS**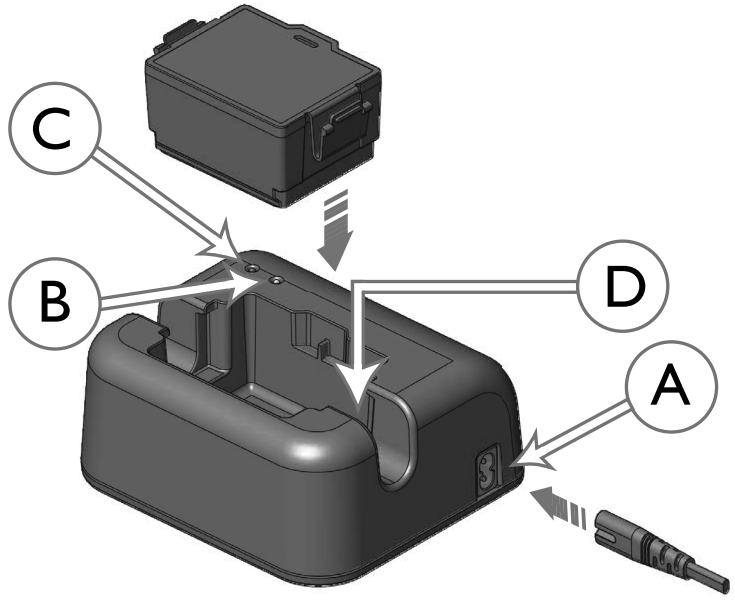

## 861394

Instructions for Use (ENGLISH) .....	1
Manuel d'utilisation (FRANÇAIS) .....	3
Gebrauchsanweisung (DEUTSCH) .....	5
Gebruiksaanwijzing (NEDERLANDS) .....	7
Instrucciones de uso (ESPAÑOL) .....	9
Istruzioni per l'uso (ITALIANO) .....	11
Instruções de uso (PORTUGUÊS) .....	13
Bruksanvisning (SVENSKA) .....	15
Bruksanvisning (NORSK) .....	17
Käyttöohjeet (SUOMI) .....	19
使用說明 (繁體中文) .....	21
使用说明 (简体中文) .....	23
取扱説明書 (日本語) .....	25
사용 지침 (한국어) .....	27

---

**ENGLISH: About This Edition**

The information in this guide applies to the 861394 charger for the 989803150191 training battery used with the 861389/861388 HeartStart FR3 defibrillator. This information is subject to change. Please contact Philips at [www.philips.com/productdocs](http://www.philips.com/productdocs) or your local distributor for information on revisions.

**Edition History****Edition I**

Publication date: January 2011  
Publication number: 453564069371

**Copyright**

© 2011 Koninklijke Philips Electronics N.V.  
No part of this publication may be reproduced, transmitted, transcribed, stored in a retrieval system or translated into any human or computer language in any form by any means without the consent of the copyright holder.

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Philips Medical Systems, Seattle, WA 98121-1825,  
USA

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Philips Medical Systems, Seattle, WA 98121-1825, USA

## 861394 Charger Instructions for Use

The 861394 charger is intended for indoor use *only*, with the rechargeable 989803150191 training battery for the HeartStart FR3 defibrillator. Refer to the FR3 *Instructions for Administrators* for battery and charger specifications.

See the inside front cover of this manual for a labeled illustration of the charger.

1. Place the charger on a level surface.
2. Ensure that the power cord supplied with the charger is correct for your AC power outlet. Connect the power cord to the connector port (A) on the left side of the charger and to an AC power outlet. Observe the LEDs to confirm operation:
  - AC-power indicator LED (B): on green continuously.
  - Charge status LED (C): flashes red and green alternately for one second, then turns off unless the battery is already inserted when the charger is powered on. (See step 4.)
3. Insert the rechargeable battery into the battery well (D) of the charger.
4. The charge process begins automatically. Observe the charge status LED (C).
  - Blinking green: The battery is charging.
  - Solid green: The battery is fully charged.
  - Red: Charging has stopped due to failure during the charge process. *Remove the battery from the charger and check the battery and charger contacts. If needed, disconnect the power cord from AC power, remove the battery and clean the contacts on the charger and the battery with isopropyl alcohol and a cotton swab. Reinstall the battery and reconnect the power cord to AC power. If the charge status LED is still red, check the battery and charger temperature.\* If temperatures are within range, disconnect the power cord and contact Philips customer service.*
  - Continuously alternating red and green: The battery cannot be charged and needs to be replaced.

**NOTE:** Typically, a full charge takes no more than 3 hours. The battery should be fully charged before being put back into use. When charging is complete, disconnect the battery from the charger and disconnect the charger from AC power.

\* For safety reasons, if the battery or charger temperature is hot to the touch — above 120° F (45° C) — it will not charge until cooled.

**NOTE:** The charger becomes warm during normal operation. This is not a malfunction.

**CAUTION:** Do not use with unapproved accessories or non-rechargeable batteries. • Do not allow metallic objects to contact the metal contacts in the charger's battery well. • Do not drop the charger or apply a mechanical shock to it. • Do not bend the power cord forcibly or place a heavy object on it. • Do not use the charger with a damaged cord or if the charger has been dropped or damaged. • Do not place the charger in locations that are extremely hot or cold, dusty or dirty, very humid, or vibrating. • Do not subject the charger to direct heat or direct sunlight. • Do not touch the battery contacts when the charger is plugged in. • Do not immerse the charger. • Do not clean the charger without disconnecting power cord. • Do not place the charger on or next to heat-sensitive materials. • Do not cover the charger. • Do not disassemble the charger. • Do not store the charger and batteries outside the temperature ranges specified in the *FR3 Instructions for Administrators*.

The charger requires no maintenance and contains no user-serviceable parts. If needed, the charger enclosure may be cleaned with a cloth dampened with water.

**FCC Statement (acc. to FCC 15.105):** This product has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

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Niets uit deze uitgave mag in enige vorm of op enige manier worden veeleelvoudigd, verzonden, getranscribeerd, opgeslagen in een geautomatiseerd gegevensbestand of vertaald in menselijke of computertaal zonder toestemming van de houder van het copyright.
Philips Healthcare, a Division of Philips Electronics Ltd.
Philips Medizin Systeme Boeblingen GmbH
Philips Electronics Hong Kong Ltd.