

PHILIPS


Mother & Child Care

ECG monitoring



Philips NeoLead electrodes

Application and removal instructions

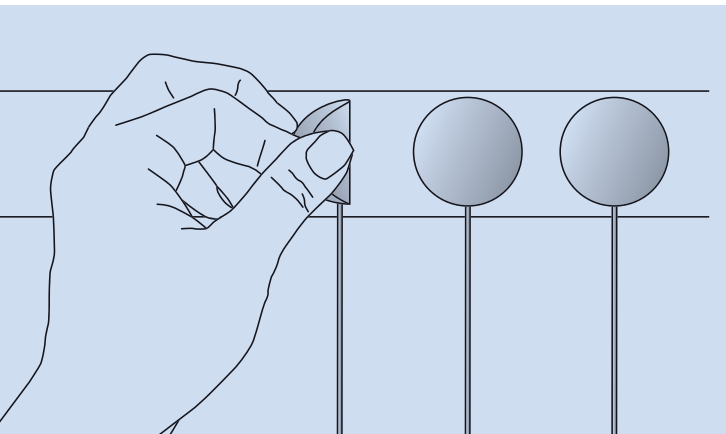

1. Skin must be intact, clean and dry. No skin preparation is recommended. Moisture, oils and lotions adversely affect the adhesion of hydrocolloid.
DO NOT USE ALCOHOL.

NOTE: While clear liner is in place, warm product in hands for 60 seconds to activate adhesive. Product may also be held near radiant warmer or heel warmer.

2. Remove NeoLead from liner.



3. Place NeoLead on skin where it is to be secured.

4. Hold in place for 60 seconds to allow proper adhesion. Heat will increase the bond between the product and patient.

5. Attach leadwire to ECG trunk cable.



6. To remove, saturate surface of electrode with water and gently peel away from skin as you continue to swab with water.

7. Change every 5–7 days or per hospital protocol.

NOTE: Discontinue immediately if skin irritation occurs.

Please see product instructions for use for a complete listing of contraindications, warnings and precautions.

© 2020 Koninklijke Philips N.V. All rights are reserved. Philips reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication. Trademarks are the property of Koninklijke Philips N.V. or their respective owners.



www.philips.com

Printed in the Netherlands.
4522 991 58601 * JUN 2020