

## Installation Guide

### Camlock Mounting Plate for Philips M1013A EGM, G1, or G5

The purpose of this guide is to:


1. Describe attachment of Philips Table Top Mount to Camlock Mounting Plate.
2. Describe mounting of Camlock Mounting Plate in Camlock Rails.



### Parts Reference

The following parts and hardware are included in this installation kit (hardware not shown).

Item #	Description	Qty
1	Camlock Mounting Plate	1
2	M6 x 10mm Flat Head Machine Screw (FHMS)	3
3	M4 x 20mm PHMS	2
4	#6-32 x 3/8" PHMS	1

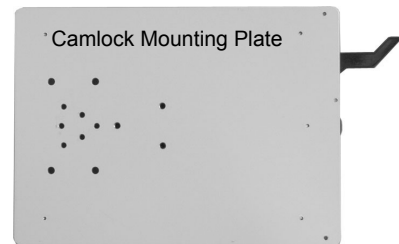


**Tools Required:** Phillips screwdriver.

### Attaching Table Top Mount to Camlock Mounting Plate



**Installation Note:** Table Top Mount is supplied by Philips.

1. Fasten Table Top Mount to Camlock Mounting Plate with three (3) M6 x 10mm FHMS.

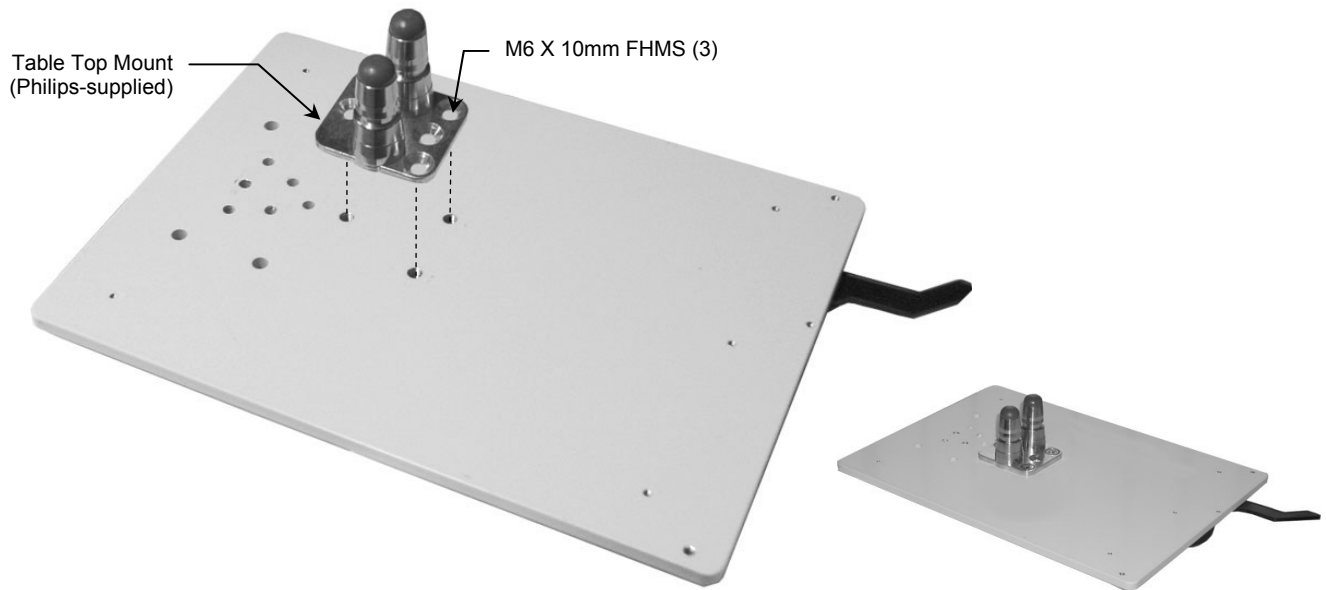


## Mounting Camlock Mount in Camlock Rails

1. Insert the four (4) feet (bottom of Camlock Mount) into the Camlock Rails (photo below left).  
**Installation Note:** Camlock Lever must be in the "unlocked" position when mounting feet in Rails. After inserting the feet, lock the Mount in place by moving the Camlock Lever from right to left.

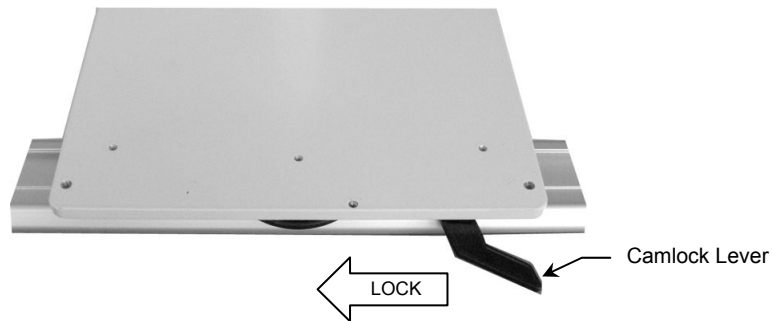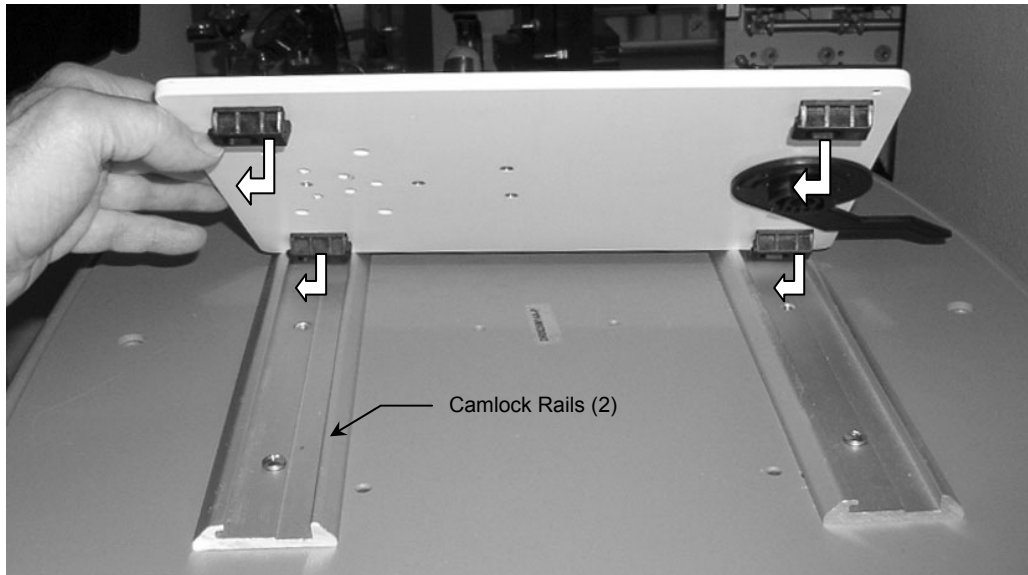

2. To prevent the Camlock Lever from being unlocked, insert one (1) #6-32 x 3/8" PHMS in the threaded hole to the right of the Lever (below right). To further secure the Camlock Mount in the Rails, insert one (1) M4 x 20mm PHMS in each of the two (2) threaded holes at the rear corners of the Mount (below right). Stop tightening these screws when they make contact with the bottom of the Rail.

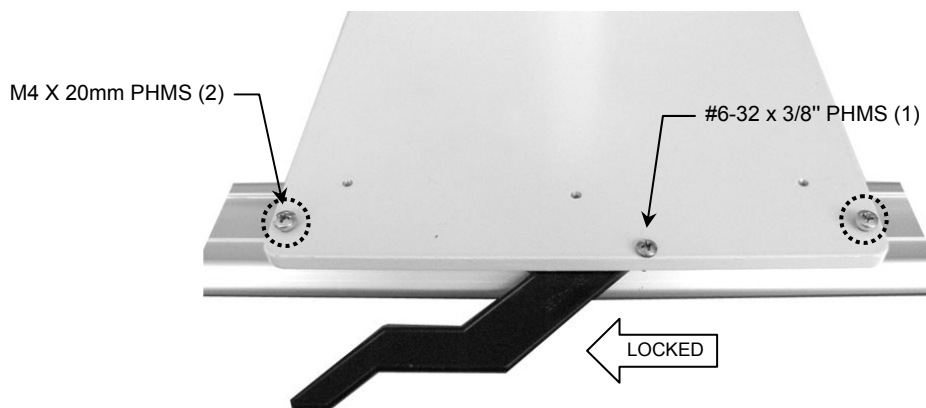

## Mounting the Monitor

Mount the monitor in accordance with the Philips Table Top Mount instructions.

