

### **Installation Guide**

# Philips MP2 / X2 Folding Bag Mount

The purpose of this guide is to:

- 1. Describe attachment of PRC Clamp and Wall Mount Slide
- 2. Describe attachment of MP2 Folding Bag to Mount

## **Parts Reference**

The following parts are included with this installation kit (hardware not shown).

Item #	Description	Qty
1	MP2 Folding Bag Mount	1
2	Post / Rail Clamp (PRC)	1
3	Channel Slide	1
4	#10-32 x 7/16" Socket Head Machine Screw (SHMS)	4
5	#10 x 5/8" Pan Head Sheet Metal Screw (PHSMS)	4
6	1/8" Hex Wrench	1
7	5/32" Hex Wrench	1


#### Tools Required:

Phillips screwdriver 1/8" Hex Wrench (provided) 5/32 Hex Wrench (provided)



## **Attaching PRC Clamp to Mount**

Using the 5/32 Hex Wrench, attach PRC Clamp to the rear of the Folding Shelf with (4) 10-32 x 7/16" SHCS provided. The PRC Clamp may be oriented for attachment to either a vertical or a horizontal rail.

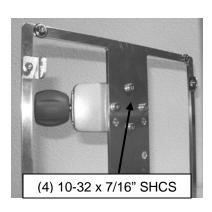





# Mounting Clamp on Post (.75" [19 mm] to 1.5" [38 mm] Dia.) or Rail (10 mm x 25 mm)

Open Clamp by turning knob counterclockwise (CCW), Position Clamp around post or over rail and turn knob clockwise (CW) to tighten.






Attaching Clamp to Pole


Attaching Clamp to Rail

## **Attaching GCX Channel Slide to Mount**


1. Attach the Channel Slide to the rear of the Folding Shelf using the (4) #10 x 5/8" PHSMS provided.

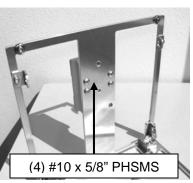

- 2. Slide the Folding Shelf into a GCX Wall Channel.

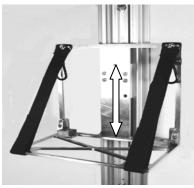


3. Secure its position by tightening the 1/4-20 Set Screw at the bottom of the Channel Slide with the 1/8 Hex Wrench.

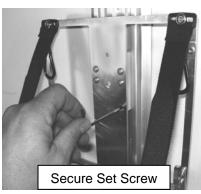

# **Mounting MP2 Transport Bag on Folding Shelf**

1. Fold shelf down to horizontal position.




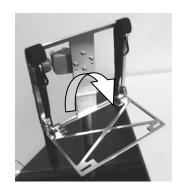

2. Place the MP2 Bag down onto the horizontal shelf.



3. Clip the (2) carabineers at the back of the Folding Shelf to the D-rings on the Transport Bag.

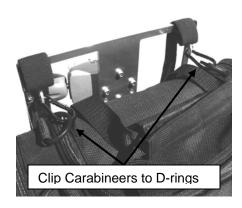

4. Slide the (2) Velcro straps through the slots at the front edge of the Lower Shelf. Fold the Fold strap back onto itself and press to secure Velcro.





#### **Routine Maintenance**

- 1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
- 2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone and Trichloroethylene.
- 3. Do not use steel wool or other abrasive material to clean the mounting assembly.
- 4. Damage caused by the use of unapproved substances or processes will not be covered by warranty. We recommend testing of any cleaning solution on a small area of the arm that is not visible to verify compatibility.
- 5. Do not submerge or allow liquids to enter the arm. Wipe any cleaning agents off of the arm immediately using a water dampened cloth. Dry the arm thoroughly after cleaning.

**CAUTION:** GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted instruments or accessory equipment, refer to the specific instructions delivered with those products

## **Cleaning the Mounting Assembly**

Periodically inspect all hardware associated with the mounting system. Tighten or adjust as necessary for optimal operation and safety.