
Installation Note

Installing the Fix Mount for IntelliVue Patient Monitors

This installation note describes the installation procedure for the Fix Mount for the IntelliVue MP20/30, MP40/50 and MP60/70 patient monitors.


Contents of the Fix Mount Kit

The fix mount kit contains the fix mount and two release knob covers. One for MP20-50 (white) and one for MP60/70 (grey).

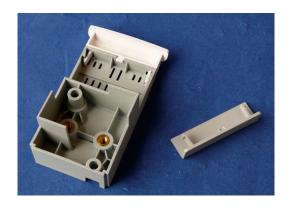

Installation Procedure for MP20/30

- **NOTE** MP20 monitors (M8001A) with a serial number < DE42601212 and MP30 monitors (M8002A) with a serial number < DE42602155 **do not** allow the use of a fix mount.
 - 1 Press the quick release knob down and release the lock with a small screwdriver. You can now pull out the knob.

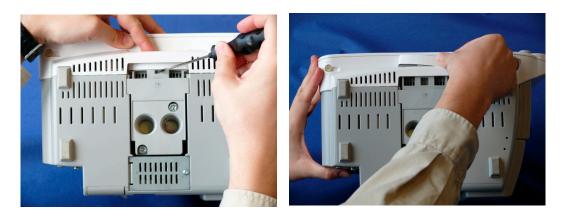


Printed in Germany 05/2007 Part number M8000-9382A 4512 610 23721





2 Remove the two screws securing the quick mount, press down the quick mount release mechansim and pull the mount out of the slot in the monitor.

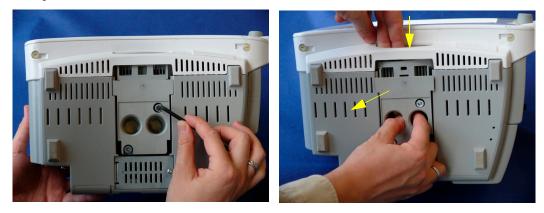

3 Insert the Fix Mount with the white release knob cover and secure it with two screws.



4 Perform the Visual and Power On Tests as described in the *Testing and Maintenance* section of your service guide.

Installation Procedure for MP40/50

1 Remove the two screws securing the quick mount.




2 Remove the battery from the battery compartment.



3 Remove the four screws at the back of the monitor, the lower ones first.



- NOTE Hold on to the display when loosening the screws so the display does not fall off.
 - 4 Pull off the front housing. Do not pull the display off completely, only tilt it to the front, because otherwise the flex panel adapter could be damaged.

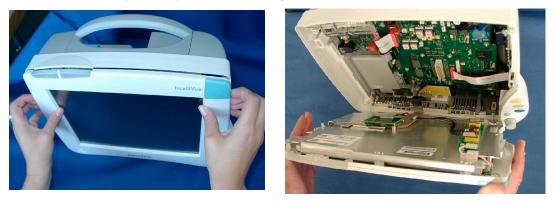

Reassembly Note: Make sure the metal sheet on the back of the display is inserted into the noses of the monitor rear housing when reassembling the two halves.



5 Depending on the version of your monitor, unscrew the screws holding the flex panel adapter cable and pull off the flex cable carefully towards you **or** unscrew the panel adapter board and disconnect the board..



Old LG-Philips/3M Version - Remove Flex Panel Adapter Cable




New Sharp/ELO Version - Remove Panel Adapter Board




Black Spacer is attached to main board with one screw

6 Remove the white plastic frame from the monitor rear chassis by releasing the five snaps in the rear chassis. You can use a flat-tipped screw driver to do this.

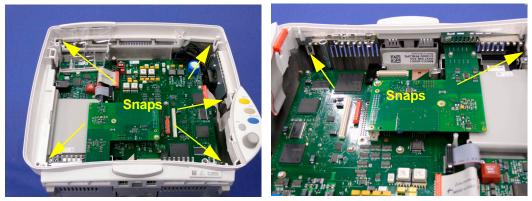

Bottom Edge of Housing


Reassembly Note: When reassembling the frame, make sure the navigation point assembly contacts are not bent and are positioned correctly inside the housing.

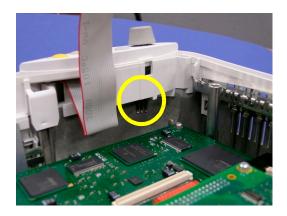

7 Pull out the quick mount.



8 Insert the fix mount with the withe release knob cover and secure it with two screws.

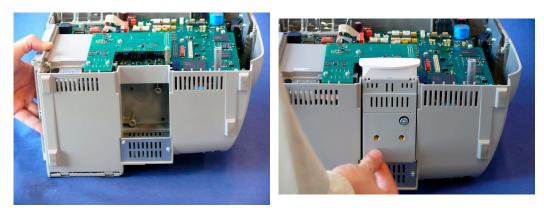

9 Reassemble the monitor by performing steps 2-6 in reverse order.




10 Perform the Visual, Power On and Safety Tests as described in the *Testing and Maintenance* section of your service guide.

Installation Procedure for MP60/70

1 Loosen the two screws securing the quick mount and pull out the quick mount



2 The fix mount is delivered with a white release knob cover. Remove this cover and replace it with the grey cover from your fix mount kit.

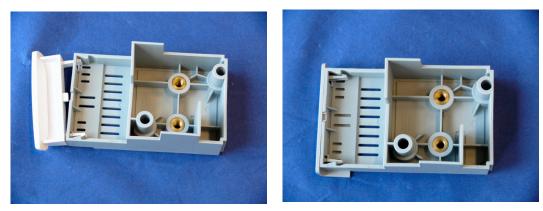

3 Insert the fix mount and secure it with two screws.



4 Perform the Visual and Power On Tests as described in the *Testing and Maintenance* section of your service guide.

####