Philips IntelliVue Family and M3/M4 Patient Monitors


Multi Measurement Module Mounting Installation Note


Attach the universal mounting clamp to the multi-measurement module (MMS) mount and then mount the MMS cable in the MMS clamp according to the following instructions to ensure that cable is attached permanently. Do not remove the MMS cable from the MMS clamp.




PHILIPS

1. Attach the universal mounting clamp to the mount with two M4x16 screws either in horizontal or vertical position to suit your installation.

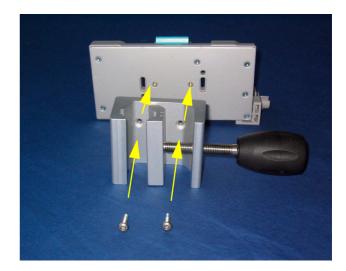


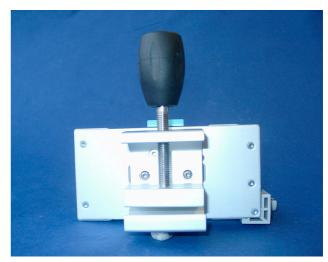

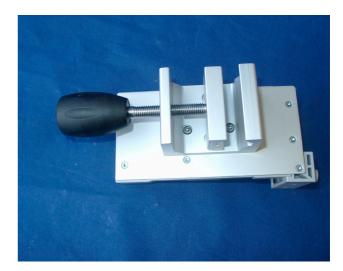


To install the multi measurement module:

2. Place the cable (1) on the clamp (2), as shown in the picture. Use the visual guide (3) and the positioning notch (4) to ensure you have the cable correctly positioned.

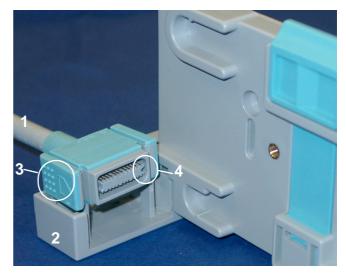

- 3. Squeeze the side clips (1) open slightly and ease the cable into the clamp so that the clips grip the gray pillars (2).
- 4. Press the cable down firmly to ensure that it is fully seated in the clamp.

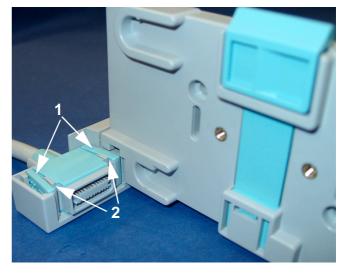

5. Align the feet on the MMS with the locating shoes (1) on the mounting plate and slide the MMS onto the mounting plate.



6. Ensure that the cable is securely fixed to the MMS. Once fixed, do not remove the cable from the MMS clamp.



Replacement Part Installation Instructions


M8000-60012 kit contains:

• 1 x MMS MSL (Multi Measurement Module Measurement Link) clamp.

Before mounting the multi measurement module onto the mounting plate, you must first screw the clamp to the mounting plate, using a Torx T10 screwdriver.

- 1 x Screw
- 1 x Installation Note

Tool required: Torx T10 screwdriver (not included).

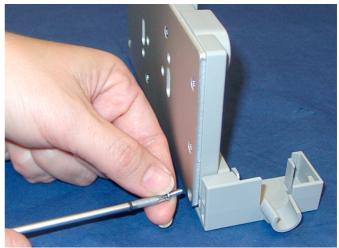

Philips Medical Systems is part of Royal Philips Electronics

INTERESTED?

Would you like to know more about our imaginative products? Please do not hesitate to contact us. We would be happy to provide specific information about our products and services, or put you on our mailing list for news about new product developments, upcoming events or for our clinical journal, MedicaMundi. We would be glad to hear from you.

On the web Contact us through our web site: www.medical.philips.com

Via e-mail Our e-mail for all remarks and requests is: medical@philips.com

By fax We can be reached at the following fax number: +31 40 27 64 887

By postal service Please write to us at the following address: Philips Medical Systems Global Information Center I.B.R.S. / C.C.R.I. Numéro 11088 5600 VC Eindhoven Pays-Bas / The Netherlands (no stamp required) *United States:* Philips Medical Systems Cardiac and Monitoring Systems 3000 Minuteman Road Andover, MA 01810 (800) 934-7372

Canada:

Philips Medical Systems Canada 281 Hillmount Road Markham, ON L6C 2S3 (800) 291-6743

Europe, Middle East and Africa: Philips Medizin Systeme Böblingen GmbH Cardiac and Monitoring Systems Hewlett-Packard Str. 2 71034 Böblingen Germany Fax: (+49) 7031 463 1552

Latin America Headquarters: Philips Medical Systems 1550 Sawgrass Corporate Parkway #300 Sunrise, Fl 33323 Tel: (954) 835-2600 Fax: (954) 835-2626

Asia Pacific Headquarters: Philips Medical Systems 30/F Hopewell Centre 17 Kennedy Road Wanchai Hong Kong Tel: (852) 2821 5888 Fax: (852) 2527 6727

© 2007 Koninklijke Philips Electronics NV All rights are reserved. Feb 2007 M8000-9490B 4512 610 20651

