Philips IntelliVue Family and M3/M4 Patient Monitors

Multi Measurement Server Mounting Installation Note


Mount the multi measurement server (MMS) cable in the clamp according to the following instructions to ensure that cable is attached permanently. Do not remove the MMS cable from the MMS clamp.






To install the multi measurement server:

1. Place the cable (1) on the clamp (2), as shown in the picture. Use the visual guide (3) and the positioning notch (4) to ensure you have the cable correctly positioned.

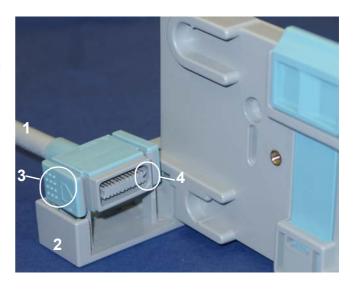



- 2. Squeeze the side clips (1) open slightly and ease the cable into the clamp so that the clips grip the gray pillars (2).
- 3. Press the cable down firmly to ensure that it is fully seated in the clamp.

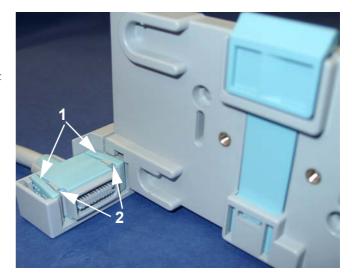

4. Align the feet on the MMS with the locating shoes (1) on the mounting plate and slide the MMS onto the mounting plate.



5. Ensure that the cable is securely fixed to the MMS. Once fixed, do not remove the cable from the MMS clamp.




Replacement Part Installation Instructions

M8000-60012 kit contains:

- $\bullet\,$ 1 x MMS MSL (Multi Measurement Server Measurement Server Link) clamp.
- 1 x Screw
- 1 x Installation Note

Tool required: Torx T10 screwdriver (not included).

Before mounting the multi measurement server onto the mounting plate, you must first screw the clamp to the mounting plate, using a Torx T10 screwdriver.

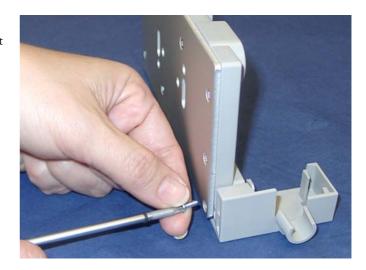

Philips Medical Systems is part of Royal Philips Electronics

INTERESTED?

Would you like to know more about our imaginative products? Please do not hesitate to contact us. We would be happy to provide specific information about our products and services, or put you on our mailing list for news about new product developments, upcoming events or for our clinical journal, MedicaMundi. We would be glad to hear from you.

On the web

Contact us through our web site:

www.medical.philips.com

Via e-mail

Our e-mail for all remarks and requests is:

medical@philips.com

By fax

We can be reached at the following fax number:

+31 40 27 64 887

By postal service

Please write to us at the following address:

Philips Medical Systems
Global Information Center
I.B.R.S. / C.C.R.I. Numéro 11088
5600 VC Eindhoven

Pays-Bas / The Netherlands

(no stamp required)

United States:

Philips Medical Systems Cardiac and Monitoring Systems 3000 Minuteman Road Andover, MA 01810 (800) 934-7372

Canada:

Philips Medical Systems Canada 281 Hillmount Road Markham, ON L6C 2S3 (800) 291-6743

Europe, Middle East and Africa:
Philips Medizin Systeme Böblingen GmbH
Cardiac and Monitoring Systems
Hewlett-Packard Str. 2
71034 Böblingen
Germany

Fax: (+49) 7031 463 1552

Latin America Headquarters:
Philips Medical Systems
1550 Sawgrass Corporate Parkway #300
Sunrise, Fl 33323

Tel: (954) 835-2600 Fax: (954) 835-2626

Asia Pacific Headquarters:
Philips Medical Systems
30/F Hopewell Centre
17 Kennedy Road
Wanchai
Hong Kong

Tel: (852) 2821 5888 Fax: (852) 2527 6727

© 2004 Koninklijke Philips Electronics NV All rights are reserved.

December, 2004 20041122 M8000-9490A 451261006621

