

#### Installation Guide

## Keyboard Mounting Plate and Keyboard Tray


The purpose of these instructions is to:


- 1. Describe attachment of Keyboard Plate and Tray on Swivel-Only Arms.
- 2. Describe attachment of Keyboard Plate and Tray on Adjustable L-Bracket.
- 3. Describe attachment of Keyboard Plate and Tray on Ergo Bracket.
- 4. Describe attachment of Keyboard Plate and Tray to Folding Keyboard Bracket.
- 5. Describe attachment of keyboard to Keyboard Plate and Tray.

## Mounting Keyboard Plate or Keyboard Tray on a Swivel-Only Arm

#### **Keyboard Plate**


Align 3- pattern on the Keyboard Plate with corresponding threaded holes in Swivel Fitting. Fasten Plate to Fitting with three (3)  $\#10-32 \times 7/16$ " FHMS.

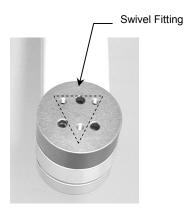


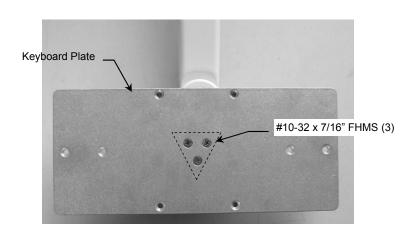

## **Keyboard Tray**

Align 3-hole pattern on the Keyboard Tray with corresponding threaded holes in Swivel Fitting. Fasten Tray to Fitting with three (3)  $\#10-32 \times 5/16$ " (100°) FHMS.

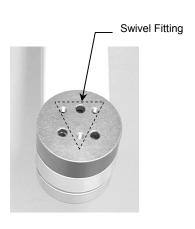


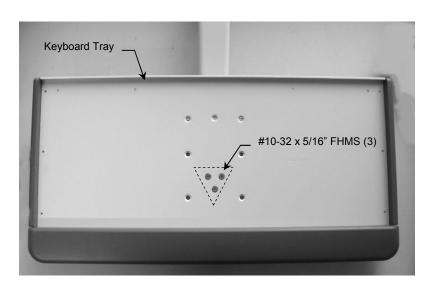

## Mounting Keyboard Plate or Keyboard Tray on Adjustable-Height L-Bracket


#### **Keyboard Plate**

Align 4-hole pattern on Keyboard Plate with corresponding mounting pattern on the L-Bracket. Fasten Plate to Bracket with four (4)  $\#10-32 \times 7/16$ " FHMS.

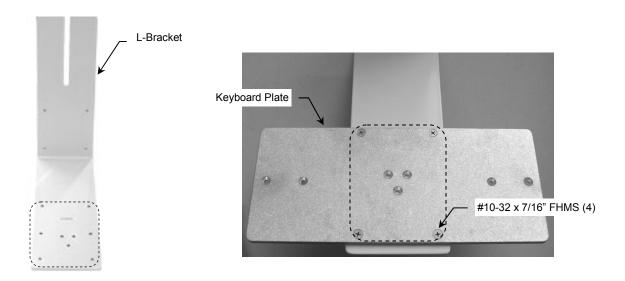

## **Keyboard Tray**


Align the 4-hole pattern on the Keyboard Tray with the corresponding hole pattern on the L-Bracket. Fasten Tray to Bracket with four (4) #10-32 x 5/16" FHMS. **Installation Note**: For greater depth separation between the keyboard and display screen, a second set of holes (solid outline below) is provided to mount the Keyboard Tray toward the end of the L-Bracket.

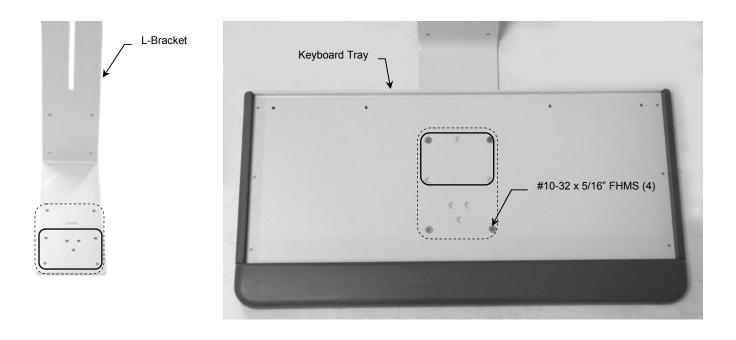

## Mounting Keyboard Plate or Keyboard Tray on Ergo Bracket


#### **Keyboard Plate**

Align 4-hole pattern on Keyboard Plate with corresponding mounting pattern on the Ergo Bracket. Fasten Plate to Bracket with four (4) #10-32 x 7/16" FHMS. **Installation Note:** For greater depth separation between keyboard and display screen, a second set of holes (solid outline below) is provided to mount the Keyboard Plate toward the front end of the Ergo Bracket.

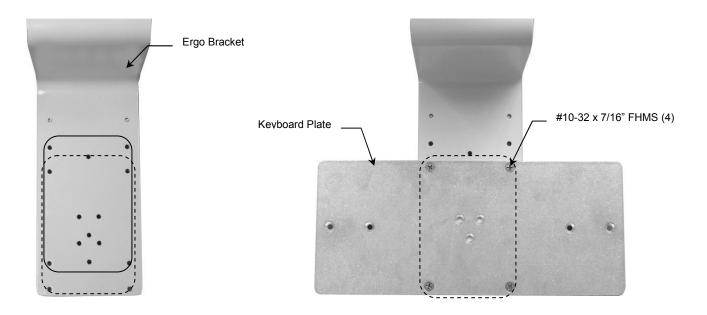


# **Keyboard Tray**

Align 4-hole pattern on Keyboard Tray with corresponding mounting pattern on the Ergo Bracket. Fasten Tray to Bracket with four (4) #10-32 x 5/16" (100°) FHMS. **Installation Note:** For greater depth separation between keyboard and display screen, a second set of holes (solid outline below) is provided to mount the Keyboard Tray toward the front end of the Ergo Bracket.

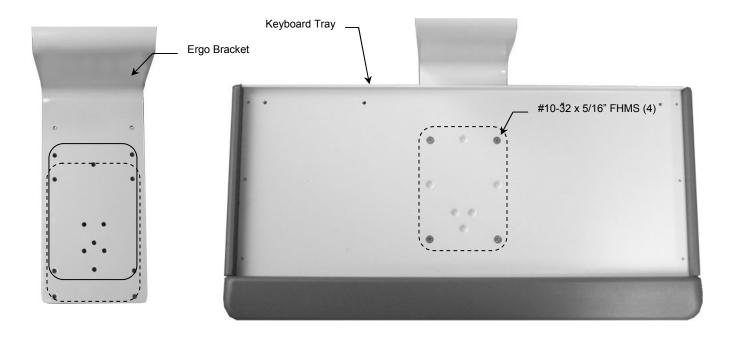

## Mounting Keyboard Tray or Plate on Folding Keyboard Bracket

Align 4-hole mounting pattern on Keyboard Tray or Plate with corresponding threaded holes in Bracket. Fasten Tray/Plate to Bracket with four (4) #10-32 flat head machine screws (FHMS). **Installation Note:** If Keyboard Tray is equipped with mouse trays, it may be necessary to slide both left and right mouse trays outward a few inches before attaching Tray to Bracket.

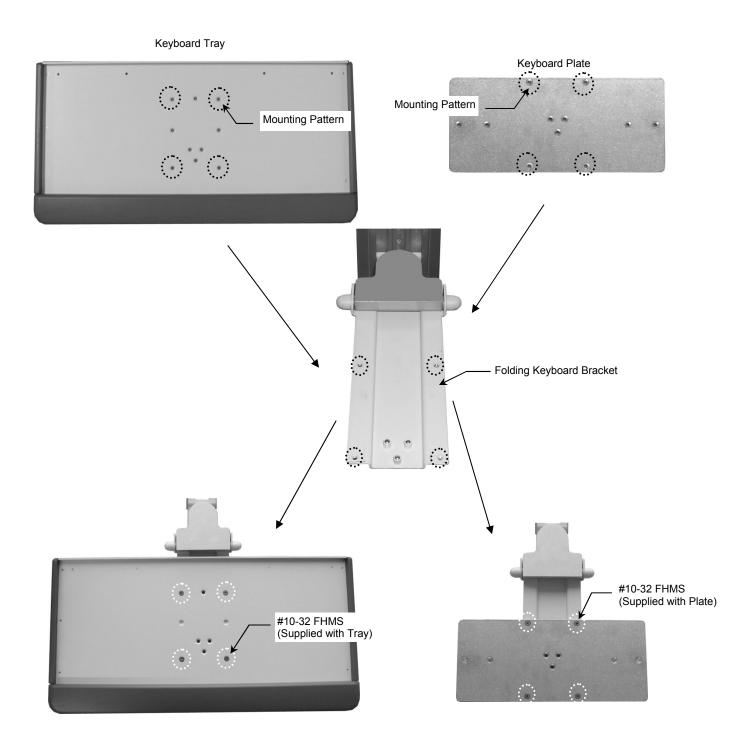

## Mounting the Keyboard with Hook-and-Loop Strips

Two (2) sets of Hook-and-Loop Strips are provided with the installation kit for securing keyboard to Keyboard Plate or Tray. **Installation Note:** Before attaching hook-and-loop strips, ensure hook-side is aligned opposite the loop-side. An example of a typical hook-and-loop strip attachment is shown below.

- 1. Attach hook-and-loop strips to the bottom of the keyboard (below left).
- 2. Attach corresponding strip to Keyboard Plate or Keyboard Tray (below right). Keyboard Plate shown in example below.
- 3. Press keyboard onto Plate or Tray (not shown).



