

Installation Guide

Wall Mount and 3.5" Utility Basket for Philips FM20/30 Monitor

The purpose of this guide is to describe attachment of Wall Mount to monitor, and mounting of monitor and Basket in Channel.


Parts Reference

The following parts and hardware are included with this installation kit (see photo for parts; hardware not shown):

Item #	Description	Qty
1	Wall Mount	1
2	M4 x 12mm Pan Head Machine Screw (PHMS)	4
3	Adjustable Stop	1
4	Utility Basket, 3.5"	1
5	1/8" Hex Key	1



Wall Mount

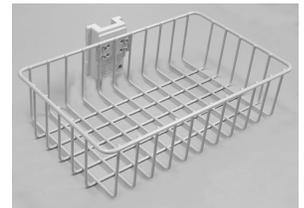


3.5" Utility Basket

Tools Required: Phillips screwdriver (not provided), 1/8" hex key (provided).

Attaching Wall Mount to Monitor

1. Align Wall Mount with four (4) threaded mounting holes on bottom of monitor (below left). **Installation Note:** Four (4) screw heads on rear of Wall Mount must be oriented as shown below right.
2. Fasten Wall Mount to bottom of monitor with four M4 x 10mm PHMS (right).

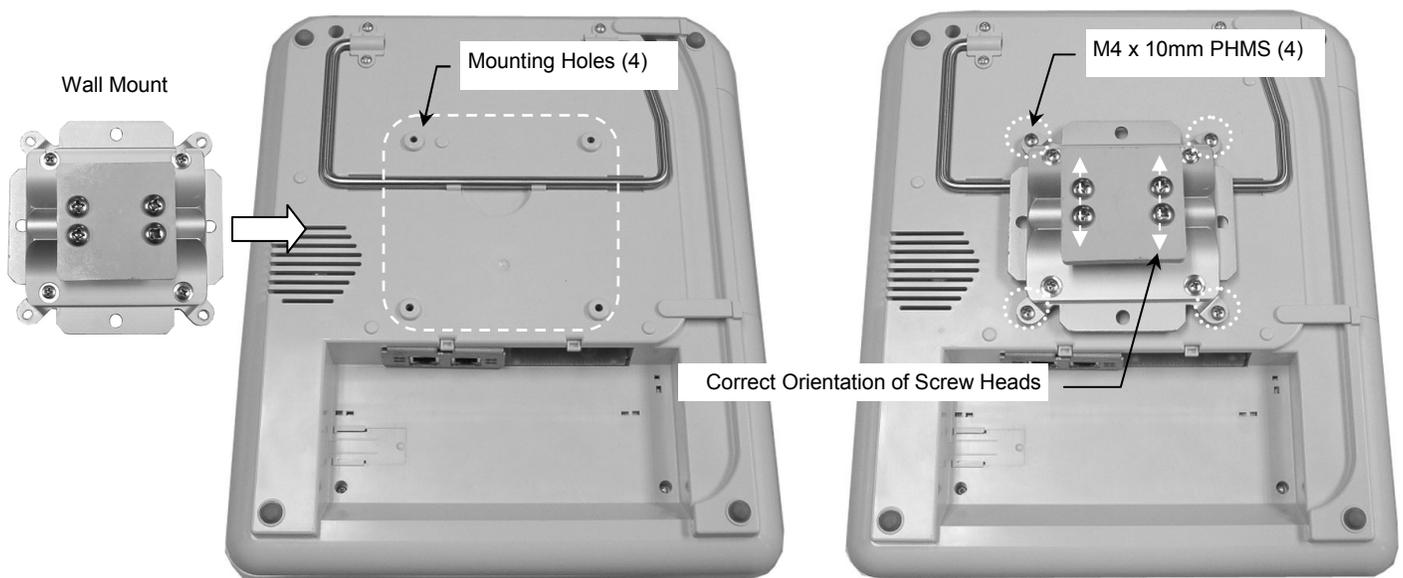

Mounting Utility Basket and FM20/30 Monitor

1. Insert Slide (rear of Basket) into top of Channel and slide Basket to desired mounting position. Using the 1/8" hex key provided, tighten the set screw in the bottom right corner of the Slide (below left).
2. Insert Adjustable Stop in Channel and slide to instrument-mounting position (below left). Tighten screw in center of Stop. **Installation Note:** There must be at least 7" separation between Adjustable Stop and top of Basket Slide (below left).
3. Insert Wall Mount Slide in Channel and carefully lower monitor to rest against Adjustable Stop (below right).

