

Installation Guide

Fixed (non-swiveling/non-tilting) Flush Mount for VESA-compatible Flat Panel Display

The purpose of this guide is to:


- 1. Describe attachment of Flush Mount to display.
- 2. Describe mounting of Flush Mount in Channel.




Parts Reference

The following parts and hardware are included in this installation kit (hardware not shown):

Item #	Description	Qty
1	Flush Mount	1
2	Mounting Kit for 75/100 VESA Compatible Flat Panel Displays	1
3	Adjustable Stop (not shown)	1

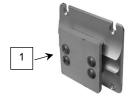


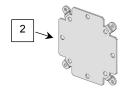

Tools Required: Phillips screwdriver (not provided).

Installation Note: This installation procedure requires hardware from the *Mounting Kit for 75/100 VESA Compatible Flat Panel Displays* (item #2 in Parts Reference). If the display has a 75 x 75mm mounting pattern (rear of display), follow the procedure below. If the display has a 100 x 100mm mounting pattern: 1) follow instructions (DU-FLP-0002-17) included with the *Mounting Kit for 75/100 VESA Compatible Flat Panel Displays* before attaching the Flush Mount.

1. Using M4 pan head machine screws (PHMS) from the Mounting Kit (Item #2), fasten Flush Mount to display (or Mounting Adapter) as shown below.

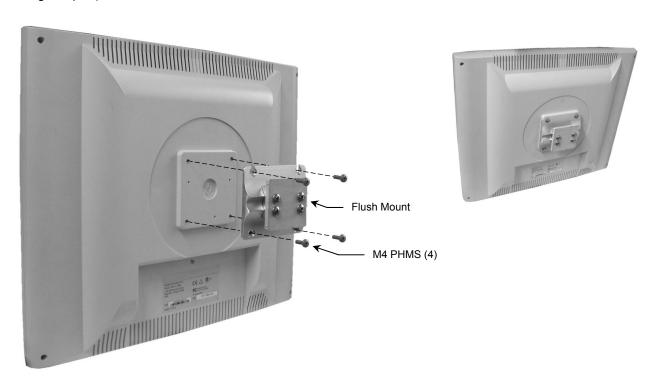

- 2. Insert Adjustable Stop in Channel, guide to desired display-mounting position, and tighten center screw (below left).
- 3. Insert the Channel Slide (rear of Flush Mount) into the channel and guide the display down the channel until the Slide contacts the Adjustable Stop.

