NM3

Instructions for Use
Mode d'emploi
Istruzioni per l'uso
Gebrauchsanleitung
Instrucciones de uso
Instruções para uso

CO₂/Flow Sensor

FOR SINGLE PATIENT USE ONLY

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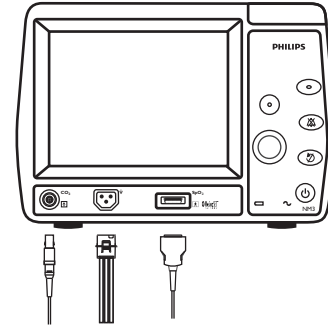
1

- Verify that the sensor, tubing and connector are intact and not damaged.
- Before connecting to the breathing circuit, attach the connector to the Flow Sensor receptacle on the monitor. The connector will "click" into place when properly seated.
- Connect CAPNOSTAT CO₂ and SpO₂ sensors to the monitor.

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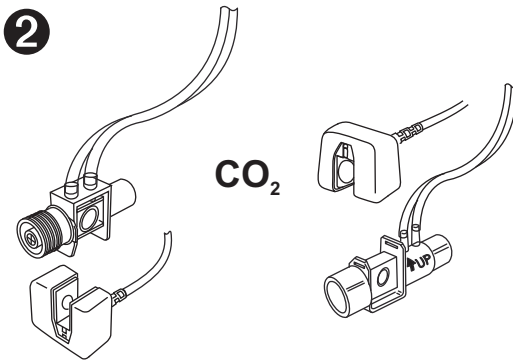
2

- Press the CAPNOSTAT CO₂ Sensor onto the adapter, it will "click" into place when properly seated.
- Apply SpO₂ sensor to patient.
- Perform a room air Adapter Zero only if prompted by the monitor (see User's Manual for further details).

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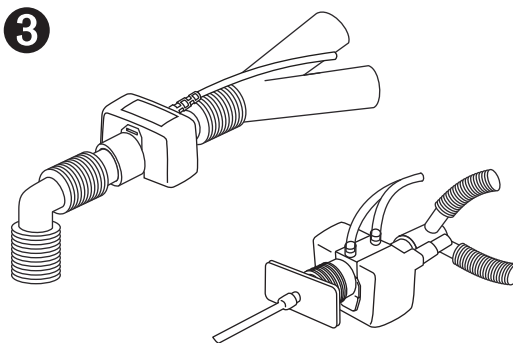
3

- Install the CO₂/Flow Sensor between the patient and the ventilator circuit wye. Be sure the CO₂/Flow Sensor tubing is positioned in an upward direction.
- Be sure the integrity of the patient ventilator circuit has not been compromised after insertion of the CO₂/Flow Sensor.

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RESPIRONICS®

WARNING: DO NOT APPLY TENSION TO THE FLOW SENSOR TUBING. DO NOT ALLOW THE SENSOR TO BE IN THE PATIENT CIRCUIT WHEN NOT CONNECTED TO THE MONITOR.

CAUTION: FOR SINGLE PATIENT USE ONLY. DO NOT STERILIZE.

CAUTION: USA law restricts this device to sale, distribution, or use by or on the order of a licensed medical practitioner.

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