Reordering Information	
Component	Reorder #
Respironics Pressure Valve	302418
Pressure Valve, Pack of 5	302423

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# Pressure Valve

## Instructions for Use

Reordering Information	
Component	Reorder #
Pressure Valve	302418
Pressure Valve, 5 PK	302423

**Warning:** The Pressure Valve may affect certain features of some Respironics equipment. See your system's operating manual for information.

**Caution:** Federal law restricts this device to sale by or on the order of a physician.

### Description

The Pressure Valve is designed for single patient use on all Respironics BiPAP and CPAP systems. It is a spring-loaded valve that helps prevent the backflow of oxygen or water from the patient circuit into the BiPAP or CPAP system when the unit is turned off. Any backflow passes out of the circuit through the vents. When the system is operating, the valve allows air to flow through the patient circuit. The vents are sealed to prevent flow out of the circuit.

The inlet side of the valve is a 22 mm connector that fits on the outlet port of the BiPAP or CPAP system. The outlet side of the valve fits a 22 mm connector on a bacteria filter or smooth inner lumen tubing. The minimum operating pressure is 2 cm H<sub>2</sub>O.

**Note:** The Pressure Valve is not an exhalation port and does not vent exhaust gas during operation.

### Instructions For Use

1. Place the inlet side of the Pressure Valve on the outlet port of the BiPAP or CPAP system.
2. If a bacteria filter is being used, place the bacteria filter inlet on the Pressure Valve outlet.
3. Attach the smooth inner lumen tubing to the outlet of the bacteria filter (or the outlet of the Pressure Valve if a bacteria filter is not used).
4. Continue the setup as instructed in your system's manual.

**Caution:** Do not block the vents on the valve. Blocking the vents will prevent the valve from expelling oxygen or water when the BiPAP or CPAP system is turned off.

### Cleaning

The Pressure Valve should be cleaned weekly.

**Caution:** Do not clean the pressure valve with alcohol, cleaning solutions containing alcohol, or any strong household cleaners. Do not sterilize the pressure valve. Do not use glutaraldehyde or quaternary ammonium compounds.

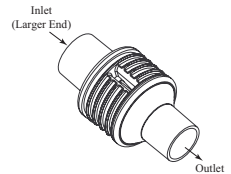
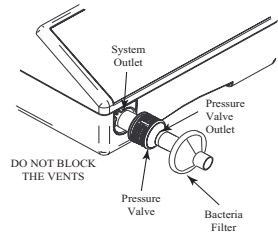
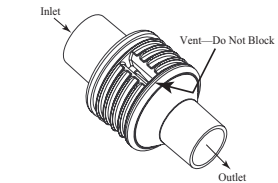
1. Remove the circuit tubing and bacteria filter from the outlet of the Pressure Valve. Remove the valve from the BiPAP or CPAP System.
2. Gently wash the Pressure Valve in a solution of warm water (less than 140 F) and mild detergent. Do not use any cleaners containing conditioners or moisturizers. Do not use bleach. Rinse thoroughly. Air dry completely before use.

### Verifying the Valve Function

After cleaning, depress and release the blue internal disk several times from the inlet side. If the valve is functioning properly, the disk will return to its normal position (towards the inlet) when released. If the valve fails to function properly, discontinue use and replace it with a new valve.

**Note:** Do not attempt to disassemble the valve for cleaning.

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