PHILIPS

MI708A

REF 989803104771

Ordering Information

Visit us on the web at:
http://shop.medical.philips.com

Asia Pacific Headquarters

Philips Medical Systems
30/F Hopewell Centre
17 Kennedy Road
Wanchai, Hong Kong
(852) 2821 5888

Canada

Philips Medical Systems Canada
281 Hillmount Road
Markham, ON
L6C 2S3
(800) 291-6743

Europe, Middle East and Africa:

Philips Medizin Systeme
Böblingen GmbH
Cardiac and Monitoring Systems
Hewlett-Packard Str. 2
71034 Böblingen, Germany
Fax: (+49) 7031 463 1552

Latin America Headquarters:

Philips Medical Systems
1550 Sawgrass Corporate
Parkway #300
Sunrise, FL 33323
(954) 835-2600

United States

Philips Medical Systems
3000 Minuteman Road
Andover, MA 01810, USA
(800) 934-7372

A-MI708A-2 Rev. E

ECG Recording Paper for use in Philips Medical Systems PageWriter Cardiographs with Thermal Printers
EKG-Schreiberpapier speziell für Philips Medical Systems PageWriter Kardiographen

Papier d'enregistrement ECG, destiné uniquement aux électrocardiographes PageWriter de Philips Medical Systems
Papel de registro de ECG para utilizarse sólo en los electrocardiógrafos PageWriter de Philips Medical Systems
Carta per la registrazione di ECG da utilizzare esclusivamente su elettrocardiografi PageWriter Philips Medical Systems

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Handling and Storage Precautions

Recordings on standard Chemical/ Thermal (CT) paper such as this paper require careful handling to avoid premature deterioration or fading of images. With proper storage and handling, the images on standard C/T paper have been observed to remain legible for approximately 5 years or longer.

Fading of images on standard C/T paper is accelerated when exposed to or in contact with: light, other papers, various chemicals, elevated temperature or humidity.

For more information on the proper storage and handling of C/T paper refer to Application Note number 5980-1343E

titled *Correct Storage of Chemical/ Thermal Recording Paper* included with this product's outer carton.

For long-term storage applications consider using PTP™ brand thermal paper M2485A.

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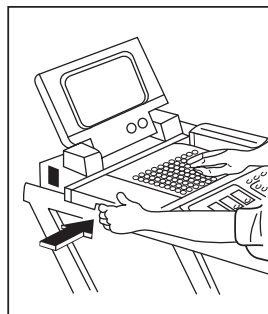
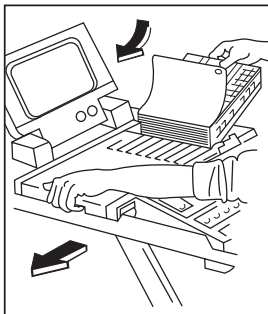
Loading the Paper

1. From the front of the cardiograph, release the latch on the left side and slide the paper door to your left.
2. Remove the outer packaging on the paper. Keep the cardboard backing on the bottom of the paper stack.
3. Place the paper stack in the compartment so that the top side of the paper feeds grid side up over the top panel. The paper sensing hole will be in the lower corner of the paper and the cardboard backing will be on the bottom.
4. Pull a half sheet out and over the paper door. Make sure that the paper is parallel with the sides of the paper door.
5. Slide the paper door back into place. Make sure that the door is latched.
6. Press Page Advance to advance the paper to the beginning of the next page.

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