Instructions for Use

Plateau Exhalation Valve

The Plateau Exhalation Valve is a multi-patient use exhalation device that provides a continuous leak path in the patient circuit when used with CPAP and bi-level systems.

⚠ Caution: U.S. federal law restricts this device to sale by or on the order of a physician.

⚠ Important Note: Using the Plateau Exhalation Valve may cause a Respiration Detachable Control Panel (DCP) to display Est V_t readings outside the stated DCP accuracy of $\pm 25\%$. When the Plateau Exhalation Valve is used in a circuit, the DCP should not be used as the only basis for establishing tidal volume.

Symbols

Warning

Note

Contains DINP
(diisobutyl phthalate)

⚠ Warnings:

- This warning applies to almost all CPAP devices. The Plateau Exhalation Valve should be used with the CPAP system recommended by your healthcare professional or respiratory therapist. As with any exhalation device, do not apply the mask to your face unless the CPAP device is turned on and operating properly. To allow a continuous flow of air out of the mask, the vent holes in the exhalation port must never be blocked. When the CPAP device is turned on and functioning properly, new air from the CPAP device flushes exhaled air out through the exhalation port's vent holes. However, when the CPAP device is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can, in some circumstances, lead to suffocation.
- As with most CPAP devices, at low CPAP pressures, the air flow through the exhalation port may not be enough to clear all the exhaled gas (CO_2) from the mask. You may rebreathe in some of the air that you have exhaled.
- Hand wash prior to first use. Inspect for damage or wear. Discard and replace any components as necessary.
- For children and those who are pregnant or breast feeding: This product may contain chemicals (phthalates) which may cause adverse health effects. Healthcare professionals should discuss the implications of using this product.

Connecting the Valve

Position the valve so that the exhaust vent is facing away from the body (Figure 1). Insert the outlet of the valve (the non-rotating end) into the user interface connector. Twist the valve slightly to ensure a good seal. Connect the tubing to the other end.

⚠ Note: The valve is equipped with two pressure pick-off ports. These may be used to monitor air pressure. Close the port caps when the ports are not in use.

Verifying the Valve Function

⚠ Warning: Prior to use, and especially after cleaning, verify the valve operation using the following procedure.

Make sure the diaphragm alignment tab is visible through the cap slot (Figure 2). Turn the unit ON. With the palm of your hand, completely block the outlet and verify that the diaphragm rises toward the cap. Verify that air is flowing from the exhaust vent on the valve. Remove your hand from the outlet and verify that the diaphragm falls away from the cap.

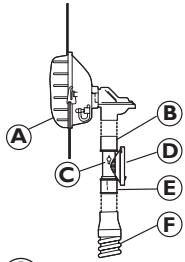
Figure 1

A = Mask or Other User Interface
B = User Interface Side
C = Port Cap
D = Exhaust Vent
E = Tubing Side
F = Circuit Tubing

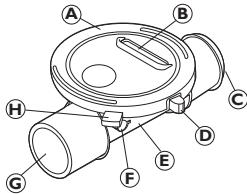
Figure 2

A = Valve Cap
B = Exhaust Port (Do not block)
C = Inlet
D = Diaphragm Alignment Tab
E = Valve Body
F = Port Cap
G = Outlet
H = Cap Removal Tab

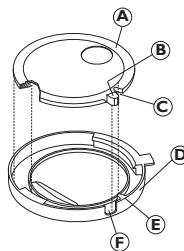
Figure 3

A = Diaphragm
B = Diaphragm Alignment Tab
C = Valve Cap Alignment Notch
D = Valve Cap
E = Valve Cap Alignment Tab
F = Diaphragm Alignment Notch

Note: If the valve is not operating correctly, make sure that the alignment tab is extending from the cap slot. If the tab is in place, disassemble the valve and inspect the parts for wear, cracks, and dirt. Verify that the diaphragm is seated correctly (Figure 3).

Cleaning Instructions

Disconnect the valve from the circuit. Position one hand so that four fingers are pressing down on the top of the valve cap. Cradle the outlet end of the valve between your thumb and forefinger. With the thumb of your other hand, press up on the cap removal tab. Be sure your thumb is as close to the inside of the tab as possible. Remove the diaphragm by pulling up on the alignment tab (Figure 3).

Note: It is normal for the valve cap to bend slightly when you are applying pressure to remove it.

1. Home Use: Gently wash the valve parts in a solution of warm water and a mild dishwashing detergent.

Institutional Use: Disinfect the valve parts with Cidex; refer to the manufacturer's instructions regarding the level of decontamination that can be achieved. Using a disinfectant may reduce the useful life of the valve.

Caution: Do not use bleach, alcohol, cleaning solutions containing bleach or alcohol, or cleaning solutions containing conditioners or moisturizers.

2. Rinse thoroughly and air dry completely before re-assembly.

3. Inspect the valve after each cleaning to ensure it maintains its integrity, and replace if damaged or worn. The valve should last approximately 10 cycles when cleaned according to the institutional use instructions. Verify the valve function.

Note: Replace the diaphragm after six months of use.

Reassembling the Valve

Inspect the diaphragm for tears and the valve body and cap for any damage. Discard and replace any damaged parts. Position the diaphragm so its rim faces toward the valve cap (Figure 3). Align the notch in the diaphragm with the tab on the valve cap, and align the tab on the diaphragm with the notch in the valve cap. Press down around the edge of the diaphragm so that the rim is secured in the valve cap. Align the tab on the valve cap with the notch on the valve body. Press the cap and valve body together.

Disposal

Dispose of in accordance with local regulations.

AUSTRALIAN SPONSOR DETAILS:

Philips Electronics Australia Ltd.
65 Epping Road, North Ryde, NSW 2113
Australia

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Respironics Inc.