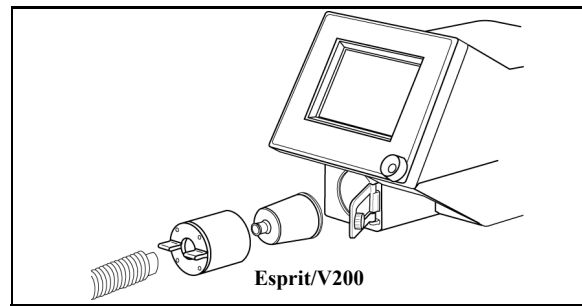
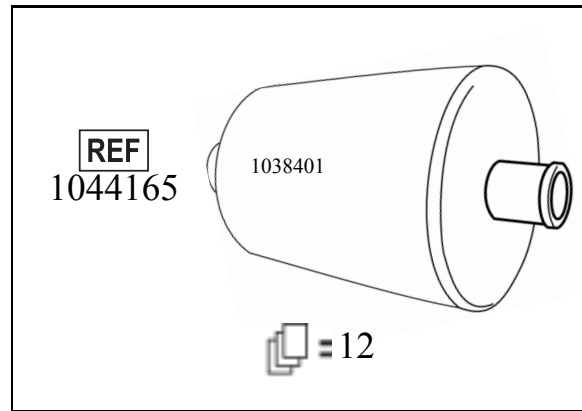
# PHILIPS

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## INSTRUCTIONS

### DESCRIPTION

The Single-Patient Use (SPU) Exhalation Bacteria Filter prevents bacterial and viral contaminants in the patient's exhaled gas from venting to room air. The SPU filter also protects the ventilator's exhalation and spirometry systems, which is critical when nebulized medications are administered.

**Warning** Discard SPU Filter after use, following your institution's infection control procedures.

**NOTE:** Approved for use only with: Philips Respironics Esprit, Philips Respironics V200, Cardinal Health (VIASYS) AVEA, Covidien (Puritan Bennett) 7200

### INSTALLATION: ESPRIT/V200

The SPU filter is ready to use and requires no disinfection. Handle the filter with care; abusive handling can damage filter media.

- Locate the heated expiratory filter and the receiving compartment on the lower center of the Esprit Ventilator front panel.
- Turn the knob counterclockwise to unlatch the retaining bracket.
- Open the retaining bracket by pulling it out and away from the ventilator.
- Use the tabs to gently pull the heater housing away from the ventilator.
- Ensure that the ventilator has been turned off for 15 minutes. If not, allow the heating housing to cool before touching it.
- Gently remove the heater housing. Tap the filter input port if the filter is not easily removed.
- Insert the new expiratory bacteria filter into the heater housing.
- Reinstall housing and filter into receiving compartment, then close the retaining bracket.
- Ensure the retaining bracket is securely closed, then turn the knob clockwise to secure the latch.
- Connect the exhalation limb and water trap of the patient circuit to the inlet port of the exhalation filter.

**Caution** Ensure all patient circuit connections are leaktight; perform an Extended Self-Test (EST) when the SPU filter is installed on a ventilator. For other ventilators, consult the respective operating instructions for appropriate circuit leak test procedures.

**Caution** To avoid damage, do not force the filter heater into the compartment.

**Caution** Ensure ventilator functions normally with filters in place. Monitor filter performance.

**Caution** If nebulized medication is used, monitor filters more frequently as nebulized medication build-up can occlude filter medium. Inspect and test expiratory filters at patient set-up and frequently during use.

**CONDITIONS FOR REMOVAL AND DISPOSAL OF FILTER**  
Remove and discard SPU filter for the following conditions:

- After each patient use
- After 15 days of continuous use on a single patient
- If resistance to flow exceeds 4 cmH<sub>2</sub>O @ 100 L/min
- If filter is visibly damaged or rattles

### SPECIFICATIONS

Viral/bacterial filtration efficiency:	Greater than 99.9999947%
Filter medium:	Minimum filter efficiency of 99.97% retention of particles 0.3 µm nominal at 100 L/min flow
Resistance to air flow:	Less than 2.5 cmH <sub>2</sub> O at 100 L/min when new
Size:	3½ inches diameter by 6 inches in length (8.9 cm by 15.3 cm)
Internal volume:	Approximately 350 mL
Leakage:	Less than 0.01 L/min at 3 psi (211 cmH <sub>2</sub> O) internal pressure
Connectors:	22 mm barbed fitting
Maximum length of use:	15 days

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