

Care Orchestrator

Adherence Profiler

At Philips Respironics, **your** efficiency is our priority.

Adherence Profiler is an automated process that identifies patients who are at risk of non-adherence. Our statistically validated algorithm determines who will be adherent at 90 days without intervention.

Adherence Profiler analyzes usage patterns and helps you focus on patients who need help meeting the CMS adherence requirements. Adherence Profiler helps DMEs to efficiently manage their businesses by helping clinicians prioritize their time allocated to manage patients' adherence. This helps you better deal with lower patient outcomes and increased workload.



Learn more

To learn more about Adherence Profiler[™], contact your Account Manager or a Philips Respironics representative at 1800 345 6443. Algorithm


The first 90 days of a patient's therapy is divided into six buckets. The algorithm runs every day and on every download to enable Care Orchestrator to create tasks for patients. Every time the algorithm runs, it assigns one of the probabilities shown at the right to every patient.

A specific type of task is created depending on the bucket in which the algorithm is running and probability assigned to the patient.

A graphical representation of this is shown below. Apart from these, there is a **No Usage Detected** bucket which creates a "no therapy" data task when therapy data is not received or when usage is zero, i.e., patient left the device on but usage is recorded as zero.

Note: The algorithm ignores any patients who meet adherence and any patients who are more than 90 days old from first therapy day.

The boxes underneath the buckets are the tasks created on Care Orchestrator

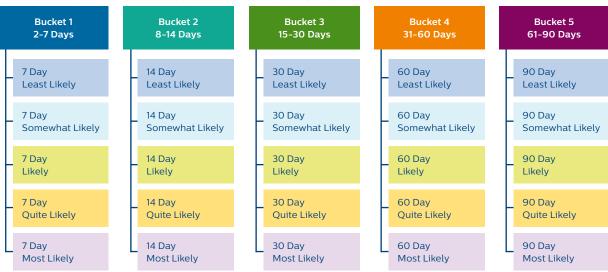


© 2019 Koninklijke Philips N.V. All rights reserved. Specifications are subject to change without notice.



Probabilities

- 5 Most likely to become adherent with an intervention
- 4 Quite likely
- 3 Likely
- 2 Somewhat likely
- 1 Least likely to become adherent with an intervention

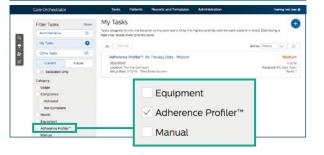

Caution: U.S. federal law restricts these devices to sale by or on the order of a physician.

RRDPGH ML 2/6/19 MCI 4108387 PN 1139794 1010 Murry Ridge Lane, Murrysville, PA 15668 800 345 6443 • 724 387 4000