

PHILIPS
RESPIRONICS

Care Orchestrator

Compliance reports



**Put power at
your fingertips**



© 2020 Koninklijke Philips N.V. All rights reserved.
Specifications are subject to change without notice.

Caution: U.S. federal law restricts these devices
to sale by or on the order of a physician.

RRDPGH ML 2/4/20 MCI 4109618 PN 1145719
1010 Murry Ridge Lane, Murrysville, PA 15668
800 345 6443 • 724 387 4000

www.philips.com/respironics

Conduct sleep and respiratory care in harmony

Care Orchestrator sleep and respiratory care management system
Remotely monitor and manage virtually all of your sleep apnea and respiratory patients with a single system. Care Orchestrator delivers actionable patient information directly to care teams – via smart phone, tablet or PC.



Care Orchestrator reporting options

To sign in to your Care Orchestrator (CO) account, navigate to www.careorchestrator.com and click the 'Sign In' button at the top-right corner. The sign in page will ask you for your **Username** and your **Password**.

Filter

- Contains options for running a new report
- Select the columns of information you prefer to have displayed by checking the boxes next to the information provided

Apply

Reset

Report view

- Choose how you would like to view this list



Identity

- First name
- Last name
- DOB
- Setup date (30, 60, 90, 120)
- Location (internal or external)
- External ID

Compliance

- Decide the compliance status you would like to view (compliant, noncompliant)
- Designate the compliance rules you would like to view per report

Usage

- Select the days used (7, 14, 30, 90, 180)
- Select the hours used (7, 14, 30, 90, 180)

Equipment

- Device
- Device name
- Data source
- Mask
- Connectivity serial number

Care team

- Assign / filter care team members

Optional custom fields

- Run reports based on customized fields created by the administrator and assigned to patient record


How to generate **compliance reports**

Compliance reports give physicians the ability to monitor compliance levels of individual patients, as well as their overall practice. The default report shows all compliance rules for patients in the last 120 days.

Access the compliance report by selecting Reports and Templates tab, Patient Reports, then choose the template under the Report Template.

Prioritize the results you want to highlight (last name, setup date, compliance level, etc.)
You can also sort by selecting the header of the column in the report.

Categorize your sort in ascending or descending order.

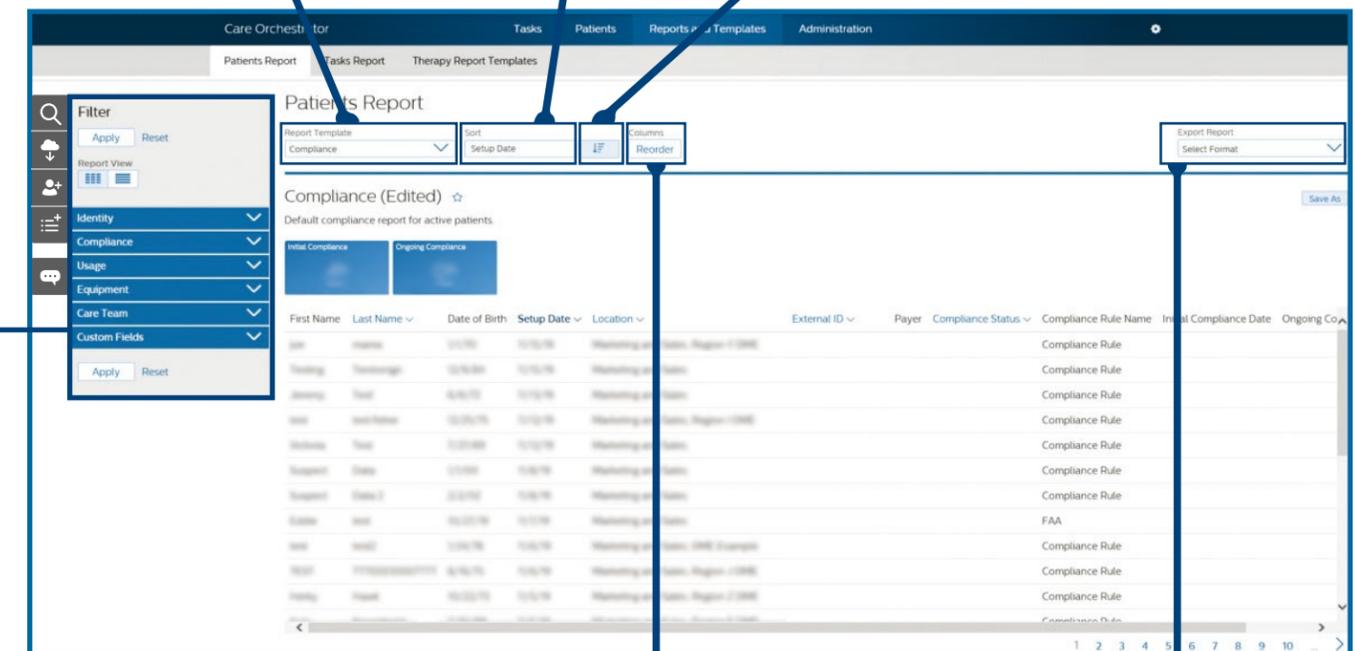

Sort the columns by Care Orchestrator's drag-and-drop feature to design the report in your preferred view.

Export your lists to XLS or CSV to further evaluate.

Note: When you attempt to use the **Data Card Utilities** in Care Orchestrator, you will be prompted to download or update **Data Card Utilities** if you do not have the latest version installed. Follow the on-screen instructions to download and install the latest version of **Data Card Utilities**.

For more information, contact product support at **1 844 780 0208** or software.support@philips.com