CAII TOLL FREE: 1.800.54.AVENT SANS FRAIS: 1.800.54.AVENT LÍNEA GRATUITA: 1.800.54.AVENT

LIMITED GUARANTEE

This product is a personal care item and therefore cannot be returned once opened. In the event of a manufacturing defect, we will gladly provide replacement parts or a possible exchange. If you experience any problems with this product, please DO NOT return it to the retailer. Call our Customer Service toll free at **1.800.54.AVENT**, where our representatives will be happy to assist you.

NOTE: For our guarantee to be valid the claim must not be a result of accident, misuse, improper care, neglect or normal wear and tear.

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Exclusively Imported and Distributed by AVENT America, Inc. 475 Supreme Drive, Bensenville, IL 60106, USA

www.aventamerica.com

AVENT®, Suffolk, CO10 7QS, England


ISIS [DUO[™]

Guide to Your Pump Mode d'emploi de votre tire-lait Guía para su Extractor

ISIS iQ DUO™ Pump Parts Guide

When assembled parts b – g make up an assembled pump body. When the Control Handle (a) is added to this assembled pump body, it becomes the First (Control) Pump. When the Double Pumping Connector (l) is added to this assembled pump body, it becomes the Second Pump.

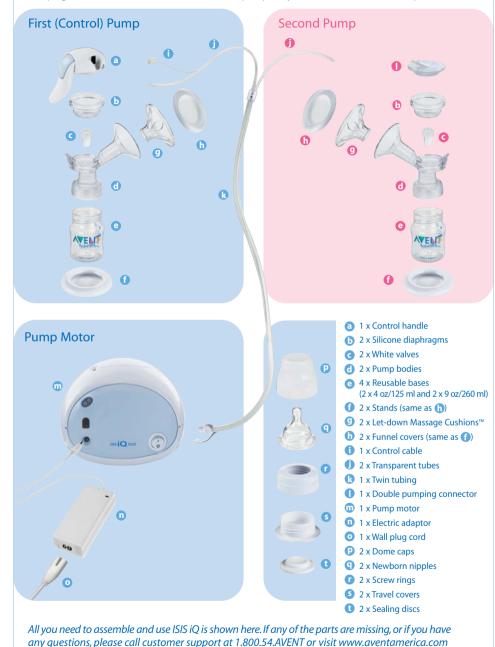ISIS iQ DUO™ Pump Parts Guide – Detailed

Warnings and Cautions

When using electrical products, especially when children are present, basic safety precautions should always be followed.

READ ALL INSTRUCTIONS BEFORE USING THIS PRODUCT.

WARNING – To reduce the risk of fire, electrocution, or other personal injury:

- ISIS iQ DUO™ is intended for repeated use by a single user only.
- Always unplug the breast pump motor from the electrical outlet immediately after use.
- Do not use while bathing.
- Do not place or store the product where it can be pulled or fall into water.
- Do not place or drop the pump motor, control handle, electric adaptor or wall plug cord into water or other liquid.
- Do not reach for a product that has fallen into water. Unplug from electrical outlet immediately.
- Close supervision is necessary when this product is used near children or persons with disabilities.
- Do not allow children or pets to play with the pump motor, tubing, power supply or accessories as these parts may pose a choking or strangulation danger.
- There are no serviceable parts inside the ISIS iQ DUO. Do not attempt to open, service or repair the ISIS iQ DUO yourself.
- Before each use inspect the product, including mains cord, for signs of damage. Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or has been submerged in water.
- Only use attachments recommended by AVENT.
- Do not use outdoors with the AC adaptor, where aerosol spray products are being used, or where oxygen is being administered.
- Equipment not suitable for use in the presence of flammable anesthetic mixture with air, or with oxygen or nitrous oxide.
- Only use the electric adaptor supplied with the product.
- Ensure the unit is not exposed to extreme heat or placed in direct sunlight.
- This product is a personal care item and should not be shared between mothers.
- Do not attempt to remove the pump body from your breast while under vacuum. Turn the unit off and break the seal between your breast and the pump funnel with your finger, then remove the pump from your breast.
- Never use while pregnant, as pumping can induce labor.
- Never use when sleepy or drowsy.

CAUTION – To reduce the risks of injury, device malfunction or failure:

- Do not leave product unattended when plugged into an electrical outlet.
- Never drop or insert any object into any opening or tubing.
- Do not shorten the tubing.
- Make sure the voltage of the electrical adaptor is compatible with the power source.
- First plug the adaptor into the breast pump motor. Then, attach the wall plug cord into the adaptor. Lastly, insert plug into the wall socket.
- Although compliant with applicable EMC directives, this equipment may still be susceptible to excessive emissions and/or may interfere with other more sensitive material.
- Wash and sanitize the appropriate pump components before each use.
- Do not continue pumping for more than five minutes at a time if no result is achieved.
- Do not use antibacterial cleaners or detergents when cleaning breast pump parts.
- Never put the control handle or breast pump motor in water or sterilizer, as you can cause permanent damage to the pump.
- If the pressure generated is uncomfortable or causing pain, turn the unit off, break the seal between the breast and the pump body with your finger and remove the pump from your breast.

Congratulations!

Soon you will discover what makes this ISIS iQ Breast Pump so uniquely yours. ISIS iQ is designed with an electronic memory that learns and continues your gentle pumping rhythm. It gives you complete control over your comfort, which means that pumping with ISIS iQ always feels natural and exactly right for you. *Intelligent and intuitive...* It's your iQ.

Let-down Massage Cushion[™] has petals that flex in and out to gently massage the area around your breast. This is intended to help stimulate a fast, natural milk flow.

One-Touch Memory Control is right at your fingertips. You can stop, start or adjust the speed and suction level without removing your hand from the pump.

Electronic Memory learns from you. Just begin a comfortable pumping rhythm, press a button and ISIS iQ automates it.

Pump, Store, Feed Convenience from the AVENT Natural Feeding System. AVENT products work together, so you never have to transfer milk between containers.

Ultra Comfortable, Quiet and Effective making it easier for you to continue giving your baby the benefits of breast milk.

Please take a few minutes to read through this entire instruction leaflet before using your ISIS iQ DUO™ for the first time.

PLEASE SAVE THESE INSTRUCTIONS.

What's Inside:

Electric Pumping

(Single and Double)	
ISIS iQ DUO Pump Parts Guide	1-2
Warnings and Cautions	3
Disassembling ISIS iQ DUO	5
Cleaning and Sanitizing ISIS iQ DUC	6
Preparing to Pump	7-8
Assembling Pump Bodies of ISIS iO DUO	9-10

Connecting the Electronics of ISIS iQ DUO 11-12
Pumping with ISIS iQ DUO 13-14

Manual Pumping with ISIS™

ISIS Manual Pump Parts Guide 15
Cleaning the ISIS Manual Pump 15
Assembling the ISIS Manual Pump 16
Using the ISIS Manual Pump 17

Keeping Parts Clean and Compact for Travel	18
Storing Your Expressed Breast Milk	19-20
Troubleshooting Guide	21
Technical Specifications	22

Disassembling ISIS iQ DUO™

Make sure that ISIS iQ pump motor is turned off and unplugaed before disassembling parts. See section titled, ISIS iQ DUO Pump Parts Guide on page 1 for a description of the First (Control) Pump and Second Pump.

First (Control) Pump

First (Control) Pump


On the first (control) pump, remove the control handle by pulling it away from the silicone diaphragm.

Second Pump

Second Pump

On the second pump, detach the transparent tube from the double pumping connector. Remove the double pumping connector by pulling it away from the silicone diaphragm.

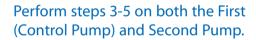Remove the silicone diaphragm by pulling it away from the pump body.

Remove the white valve from the pump body.

Remove the Let-down Massage Cushion™ by gently pulling it away from the funnel on the pump body.

Cleaning and Sanitizing ISIS iQ DUO™

Make sure that ISIS iQ is turned off and unplugged before cleaning.

Clean Before Every Use (including first use):

Silicone Diaphragm (b)

White Valve (c)

Pump Body (d)

Funnel Cover/White Stand (h/f)

Cleaning Instructions:

These pump parts should be sanitized after every use. Disassemble the pump before cleaning. Wash in warm soapy water and rinse, or wash on the top rack dishwasher. Sanitize in a steam sterilizer (electronic, electric or microwave) or by boiling in water for 10 minutes.

Caution: Do not use antibacterial cleaners or detergents when cleaning breast pump parts.

Clean with a Damp Cloth when Necessary:

Control Handle (a)

Tubing (i/j/k)

Note: The tubes are used for suction and do not carry milk.

Double Pumping Connector (I)

Pump Motor (m)

Caution: NEVER put the control handle or pump motor in water or a sterilizer, as you can cause permanent damage to the pump.

HELPFUL TIP!

The AVENT Electronic, Electric and Microwave Steam Sterilizers are designed to hold the washable components of the ISIS iQ DUO and the entire ISIS™ Manual Pump.

Preparing to Pump

When using the pump for the first time, your ISIS iQ will have retained factory settings stored in the memory as a result of quality control testing. If the blue One-Touch Memory Control is pressed prior to squeezing the control handle upon first use the pump will operate at these settings, which may or may not be comfortable for you. We recommend that prior to engaging the automatic mode, you establish a comfortable rhythm manually, then press the blue One-Touch Memory Control to continue your preferred rhythm.

If you find that you are not expressing a lot of milk the first few times you pump, don't be discouraged. Learning to express milk, like breastfeeding, takes practice. Here are some tips that might make pumping easier:

- 1. Choose a time when you're not rushed and won't be interrupted.
- 2. Choose a comfortable chair with a table nearby, so you can set the ISIS iQ motor within reach.
- 3. Look at a photograph of your baby to help encourage let-down.
- 4. Place a warm cloth on your breasts for a few minutes before you start pumping. This encourages milk flow and can soothe painful breasts.
- 5. Warmth and relaxation can encourage milk flow. Try expressing after a bath or shower.
- 6. Try expressing from one breast while your baby is feeding on the other, or continue expressing just after a feed.
- 7. Reposition the pump on your breasts from time to time. This can stimulate your milk ducts.

How quickly will the milk flow begin?

When pumping with the ISIS iQ DUO[™], your milk should begin to flow after a few compressions. If your milk does not begin to flow immediately, relax and continue pumping. *CAUTION: DO NOT CONTINUE PUMPING FOR MORE THAN 5 MINUTES AT A TIME IF NO RESULT IS ACHIEVED.* Stop, and try expressing milk at another time of day. If the process becomes painful, stop and consult your lactation consultant or breastfeeding advisor.

How much milk can you expect?

It has been shown that double pumping boosts prolactin levels and stimulates milk production so, if you can, use your ISIS iQ DUO regularly as a double pump. On average you will need to pump for 10 minutes to express 60-120 mls or 2-4 ounces of milk. However, this is just a guide and varies from woman to woman. If you find that you regularly express more than 4 ounces of milk per session, attach a 9 oz/260 ml AVENT reusable base to the pump body.

Preparing to Pump

You can use ISIS iO DUO™ as a:

Single Electric Pump

Manual Pump

(to use the ISIS™ Manual Pump, see page 15)

Assembling Pump Bodies of ISIS iQ DUO™

Before assembling, make sure that you have correctly washed and sanitized the pump. See section titled 'Cleaning and Sanitizing ISIS iQ DUO' on page 6.

Parts you will need for this step:

pump body

white valve

clear silicone diaphragm

Double pumping? Assemble two pumps as shown below. Single pumping? Assemble one pump as shown below.

Click the white valve into the pump body, so that it fits securely into the small round hole. Make sure the pointed side of the valve is facing down.

Fit the clear silicone diaphragm into the pump body. Ensure a perfect seal around the rim by pressing down with your fingers. (This is easier if assembled while the diaphragm is wet.)

Assembling Pump Bodies of ISIS iQ DUO™

Let-down Massage Cushion™


reusable base

white stand

Screw the pump body onto the reusable base until securely in place.

To ensure that the pump is completely stable, insert the pump base into the stand.

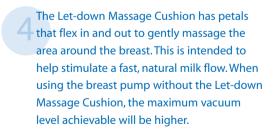Connecting the Electronics of ISIS iQ DUO™

Parts you will need for this step: Double Pumping? double pumping connector control (see pages 9-10) control cable transparent tubing electric adaptor wall plug cord

First (Control) Pump

First (Control) Pump

See section titled, ISIS iQ DUO Pump Parts Guide on page 1 for a description of First (Control) Pump and Second Pump.

- Push the control handle onto the assembled pump body so that it sits firmly on top of the silicone diaphragm. This assembly becomes the First (Control) Pump which contains the electronics of the pump.
- Click the control cable into the socket on the control handle.

Then securely attach one of the transparent tubes to the port on the control handle.

Connecting the Electronics of ISIS iQ DUO™


Double pumping? Complete the next step with your second assembled pump body.

See section titled, ISIS iQ DUO Pump Parts Guide on page 1 for a description of First (Control) Pump and Second Pump

Push the double pumping connector onto the pump body so that it sits firmly on top of the silicone diaphragm. Securely attach the other transparent tube to the double pumping connector. When ISIS iQ DUO is fully assembled, the Second Pump will mimic simultaneously the speed and suction of the First (Control) Pump.

Single pumping? Complete the next step with the remaining tube.

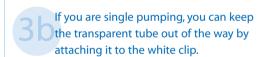Securely attach the twin tubing to the ports on the pump motor.

Connect the control cable into the adjacent socket.

Connect the electric adaptor to power socket on the pump motor.

Then connect the wall plug cord to the electric

Then connect the wall plug cord to the electric adaptor and insert the plug into the wall outlet.

Pumping with ISIS iQ DUO™

Before you begin pumping, make sure that you have correctly washed, sanitized and assembled the pump. Consult the 'Preparing to Pump' Section on page 7 for tips on stimulating let-down and milk flow.


Wash your hands thoroughly and make sure your breasts are clean.

Turn on Pump: When the flashing ORANGE light changes to a continuous GREEN light, the pump is ready to use.

If the light is flashing RED, refer to the 'Troubleshooting Guide' on page 21.

Form a Seal: Press the pump against your breast, so that your nipple fits into the funnel. A seal should form between the pump and your breast. (If double pumping, press a pump to each breast.)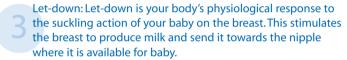

To initiate let-down, begin by squeezing the handle with shallow quick strokes. This is intended to imitate your baby's natural suckling pattern. At the same time you are doing this, you are enabling the petals on the Let-down Massage Cushion™ to flex in and out around the areola, creating a massaging effect. This is intended to help stimulate the natural let-down reflex. Let-down can occur immediately after a few strokes and should not take longer than 5 minutes (refer to page 7 on Preparing to Pump). You can choose to automate this let-down phase by pressing the blue One-Touch Memory Control or you can continue to pump manually.

Milk Expression: Milk Expression occurs after let-down is achieved and milk begins to flow. It is the phase that is induced by the baby's deeper more comfortable sucking rhythm.


If you did not automate the let-down rhythm, please skip to Step 5. If you have automated the let-down phase in Step 3, press the blue One-Touch Memory Control to end the let-down rhythm. You can now begin to establish a comfortable pumping rhythm and level of suction that can be maintained throughout your pumping session.

ISIS iQ Mimics Your Pumping: To create a natural breastfeeding rhythm, we suggest you hold down on the handle for 3 seconds, then release and repeat the pattern for the next compression.

Next, press the blue One-Touch Memory Control on the handle. The mode light on the pump motor will change from solid GREEN to flashing GREEN and ISIS iQ will mimic the rhythm of the last two compressions.

Note: You do not need to compress the handle fully, only until it feels comfortable. The further you compress the handle, the more suction the pump will provide. If the pressure is uncomfortable, stop the automated pumping by pressing the One-Touch Memory Control and breaking the seal between your breast and pump funnel with your finger.

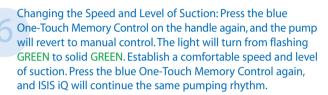Ending Pumping Session: Press the blue One-Touch Memory Control, break the seal between your breast and pump funnel with your finger and remove the pump from your breast. Turn the pump off and unplug at the power source. ISIS iQ will remember and save the speed and suction level of your last two compressions. Unscrew the base after milk is collected. If any milk remains in the pump body, pour it into the base. Do not overfill the base. Wash and re-sanitize all necessary parts before the next use.

14

Next time you turn your pump on: Resuming Saved Session? Turn on the pump as in Step 1. Complete Step 2. Press the blue One-Touch Memory Control. This engages the automatic pumping mode. The pump will continue with the rhythm of the last two saved compressions of the milk expression phase. Continue pumping as described in Step 4. Need to re-establish let-down? If you need to establish let-down, turn on the pump as in Step 1. Complete Step 2. Squeeze the handle with shallow quick strokes and proceed as described in Step 3.

CONSULT THE GUIDELINES FOR STORING EXPRESSED BREAST MILK ON PAGE 19 & 20.

HELPFUL TIP!

The light on top of the ISIS iQ DUO Motor indicates the pump mode:

Flashing ORANGE Light: Pump is in warm-up mode. Wait for the continuous GREEN light.

• Continuous GREEN Light: Manual pumping mode. You control the speed and pressure of the pump.

Flashing GREEN Light: Automatic pumping mode. ISIS iQ will mimic your last established pumping rhythm and level of suction.

Flashing RED Light: Consult the Troubleshooting Guide on page 21.

ISIS™ Manual Pump – Parts Guide

You may want to use the ISIS Manual Pump when you need a cord-free alternative to the ISIS $IQDUO^{m}$. While the ISIS Manual Pump uses many of the same parts as the ISIS IQDUO, please note the parts shaded in green are specific to the ISIS Manual Pump.

Parts specific to the ISIS Manual Pump are included in a separate bag:

- Clear pump cover
- Silicone diaphragm with a stem
- Forked handle
- White valve
- O Pump body
- Reusable base (4 oz/125 ml)
- **1** Stand (same as **b**)
- Let-down Massage Cushion[™]
- Funnel cover (same as •)

Cleaning the ISIS Manual Pump

Caution: Do not use antibacterial cleaners or detergents when cleaning breast pump parts.

Every time before use (including first use):

Clean all parts of the ISIS Manual Pump on the top rack of the dishwasher, or in warm soapy water and rinse. The pump should be sanitized after every use. All parts of the ISIS Manual Breast Pump are sterilizer safe. Sanitize in a steam sterilizer (electronic, electric or microwave), or by boiling in water for 10 minutes.

Assembling the ISIS™ Manual Pump

Wash and sanitize all parts before use. See section titled 'Cleaning the ISIS Manual Pump' on page 15.

Wash your hands thoroughly. Click the white valve into the pump body so that it fits securely into the small round hole. Make sure the pointed side of the valve is facing down.

Fit the silicone diaphragm with stem into the pump body. Ensure a perfect seal around the rim by pressing down with your fingers. (*This is easier if assembled while the diaphragm is wet.*)

Twist the reusable base onto the pump body until securely in place. To ensure the pump is completely stable, insert the pump base into the stand.

Fit the Let-down Massage Cushion™ onto the pump funnel. Secure the rim of the Let-down Massage Cushion around the funnel, so that there are no gaps. (This is easier if assembled while the Let-down Massage Cushion is wet.)

Place the forked end of the handle under the rounded tip of the diaphragm stem. Push down gently on the handle until it clicks into place.

Click the pump cover onto the pump body.

Using the ISIS™ Manual Pump

Before you begin pumping, make sure you have washed, sanitized and carefully assembled the ISIS Manual Pump. Consult the 'Preparing to Pump' on page 7 for tips about stimulating let-down and milk flow.

Wash your hands thoroughly and make sure your breasts are clean.

Form a Seal: Press the pump against your breast, so that your nipple fits into the funnel. A seal should form between the pump and your breast.

Initiate let-down: Squeeze the handle with shallow, quick strokes, until milk starts to flow. Do not use full compression on the handle.

Begin Pumping: When you see milk beginning to flow, you can establish a comfortable pumping rhythm and level of suction. Create a natural breastfeeding rhythm by holding down on the handle for 3 seconds, then releasing, and repeating that pattern for each compression.

Note: You do not need to compress the handle fully, only until it feels comfortable. The further you compress the handle, the more suction the pump will provide.

After milk is collected, unscrew the pump body from the base.

Caution: Do not continue pumping for more than five minutes at a time if no result is achieved. Try expressing at another time during the day.

Keeping Parts Clean and Compact for Travel

Pump

After sanitizing all parts, insert the white valve and diaphragm into the pump body. Clip the funnel cover over the Let-down Massage Cushion™ and screw the travel cover onto the bottom of the pump body. Then put the assembled items in the gray drawstring bag.

Travel cover (s)

Pump body (d)

Let-down Massage Cushion (g)

Funnel cover (h) (also used as a stand when pumping)

After sanitizing all parts, click the sealing disc into the screw ring. Screw the ring onto the reusable base.

Screw ring (r)
Sealing disc (t)
Reusable base (e)

AVENT Natural Shape Nipple

After sanitizing all parts, click the nipple into the screw ring. Screw the travel cover into the ring. Attach the dome cap over the top of the nipple.

Dome cap (p)

Screw ring (r)

Nipple (q)

Travel cover (s)

Storing Your Expressed Breast Milk

With AVENT, you never have to transfer milk between containers because you pump, store and feed with the same base.

By using the AVENT Natural Feeding System, you preserve valuable nutrients that can get lost if breast milk is transferred.

After expressing milk into the AVENT reusable base, insert a sealing disc into the screw ring and twist it onto the base. To feed from the same base, insert the AVENT Natural Shape Nipple into the screw ring.

AVENT° VIA": The Safest, Strongest Disposable Ever!

With your ISIS iQ DUO™ or ISIS™ Manual Pump, you can express milk directly into VIA Disposable Cups. VIA Cups are safer and sturdier than disposable liners, and offer better protection for pumping, storing and feeding breast milk.

Get VIA and Get Going!

- Twist-on lid seals tightly to VIA Cup for leak-proof storing or freezing
- Writable and stackable cups make it easy to organize
- Compatible system lets you pump, store and feed using the same cup
- Sturdy VIA Cups heat easily and quickly in a bottle warmer

With all the assurances you get from VIA, why trust any other disposable?

Storing Your Expressed Breast Milk

Guidelines for Storing Expressed Breast Milk

- Refrigerate breast milk immediately. Refrigerate for up to 48 hours in the back of the refrigerator, not in the door.
- Freeze for up to 3 months, or 6 months in a zero-degree freezer.
- Frozen breast milk can be defrosted overnight in the refrigerator, thawed in warm water or heated on the lowest setting of a bottle warmer or baby food warmer.
- NEVER refreeze breast milk.
- If you express small amounts of milk throughout the day, you can top up the milk container by keeping it in the refrigerator and adding to it. Make sure the milk is used within 48 hours of the first expression, or store it in the freezer.
- DO NOT add fresh breast milk to already frozen milk.
 Note: Only milk collected with a clean, sanitized pump should be stored to be fed to your baby. Sanitize
 Breast Milk Storage Containers before use or express milk directly into VIA™ Disposable Cups.

Never immerse a frozen bottle or liner in boiling water as this may cause it to crack/split.

Transporting Expressed Breast Milk

- Please refer to Tote Bag Instructions for complete directions on how to store and transport breast milk using the Thinsulate™ carriers.
- When removed from the freezer, four AVENT Gel Packs will keep one 4 oz or one 9 oz container of freshly expressed breast milk sufficiently chilled inside a Thinsulate carrier and insulated bag for up to 6 hours. Ensure that both the storage containers and carriers are placed upright in the bag to prevent leaking. Also make sure that the Thinsulate carriers are completely zipped.
- Be sure to keep the gel packs frozen until they are ready to use and refrigerate milk immediately after removing it from the insulated bags.
- In hot climates, avoid storing Thinsulate carriers or the travel bag containing expressed milk in a car, office, cabinet etc. Milk must be sufficiently chilled to avoid spoilage.

Please note: Milk storage times vary based on collection and storage conditions. This storage information is only a suggestion. Please contact your doctor or lactation consultant about milk storage guidelines.

Troubleshooting Guide

 Check that all the pump components have been assembled correctly and that the white valve is positioned correctly.
• Ensure that the silicone diaphragm and Let-down Massage Cushion™ are fitted securely to the pump body and a perfect seal is created, making sure to push the Let-down Massage Cushion far enough down the center of the funnel.
• Make certain that the tubing connections and control handle are firmly in place.
• Ensure the pump is correctly assembled and that suction is being created.
• Relax and try again, as expression may improve after practice.
• Refer to the 'Preparing to Pump' section on page 7.
• You may be pumping too hard. You do not need to use all the suction the pump can generate. Try compressing the handle halfway.
\circ Try holding down the pump handle for 2-3 seconds per compression.
\circ Consult your health professional or breastfeeding advisor.
∘ Try leaning slightly forward.
• Remove and re-fit the Let-down Massage Cushion to ensure it is firmly attached and pushed far enough down the center of the funnel.
Some women may prefer to use the pump without the Let-down Massage Cushion. The Let-down Massage Cushion has petals that flex in and out to gently massage the area around the breast. This is intended to help stimulate a fast, natural milk flow. When using the breast pump without the Let-down Massage Cushion, the maximum vacuum level achievable will be higher.
• Avoid contact with antibacterial detergents or cleaners as they can damage the plastic. Combinations of detergents, cleaning products, sanitizing solution, softened water and temperature fluctuations may, under certain circumstances, cause the plastic to crack. If this occurs, do not use. The pump bodies and breast milk containers are dishwasher safe but food colorings may discolor them. Follow guidelines in 'Cleaning and sanitizing your ISIS iQ DUO™ on page 6 of the instruction booklet.
• Contact AVENT for replacements. Call 1.800.54.AVENT or visit www.aventamerica.com
• Ensure the pump is correctly assembled and switched on with the power indicator illuminated GREEN.
• Make sure the cables between the electric wall socket and the pump are correctly connected.
\circ If the pump stops working and the power light is flashing RED then a fault has been detected.
 3 flashes, delay, 3 flashes, etc – the ISIS iQ has detected a vacuum fault. Switch off the motor and check tubing and connections for obstructions or damage.
• 2 flashes, delay, 2 flashes, etc – the ISIS iQ has detected that it is over the acceptable temperature. Switch off the motor. Ensure unit is not exposed to heat sources or sitting in direct sunlight. Allow unit to stabilize before trying again.
° Continuous flashing – the pump has detected that the control cable has been incorrectly fitted/not fitted or the cable is damaged.

Technical Specifications

Key to product base label

TYPE B APPLIED PART

GENERAL WARNING

FOR INDOOR USE ONLY

CLASS II EQUIPMENT

Technical Specifications:

- Mains Input Voltage 100 240 VAC
- Mains Input Current 1.5A
- Mains Input Frequency 50 60 Hz
- Supply voltage fluctuations not exceeding
 +/- 10% of the nominal voltage
- Ambient temperature range: +10° C to +40° C (+50° F to +104° F)
- Relative Humidity range: 30% to 75%
- Atmospheric Pressure: 700 to 1060hPa
- Equipment rated for continuous operation

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