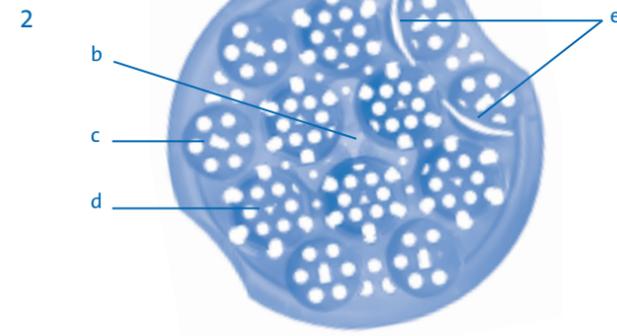
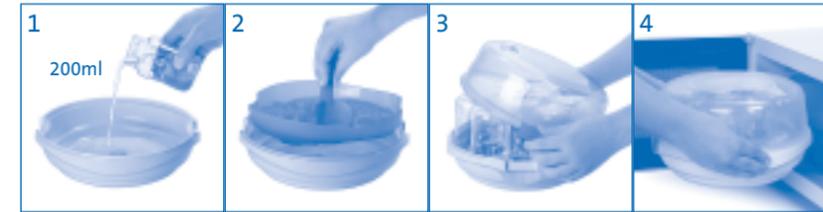
I

II


III

WATTAGE WATTAGE VOLTAJE	FULL POWER PUISSANCE MAXIMUM POTENCIA MÁXIMA	COOLING TIME TEMPS DE REFROIDISSEMENT TIEMPO DE ENFRIAMIENTO
1100 – 1850W	2 mins	2 mins
850 – 1000W	4 mins	2 mins
500 – 800W	6 mins	2 mins


IV

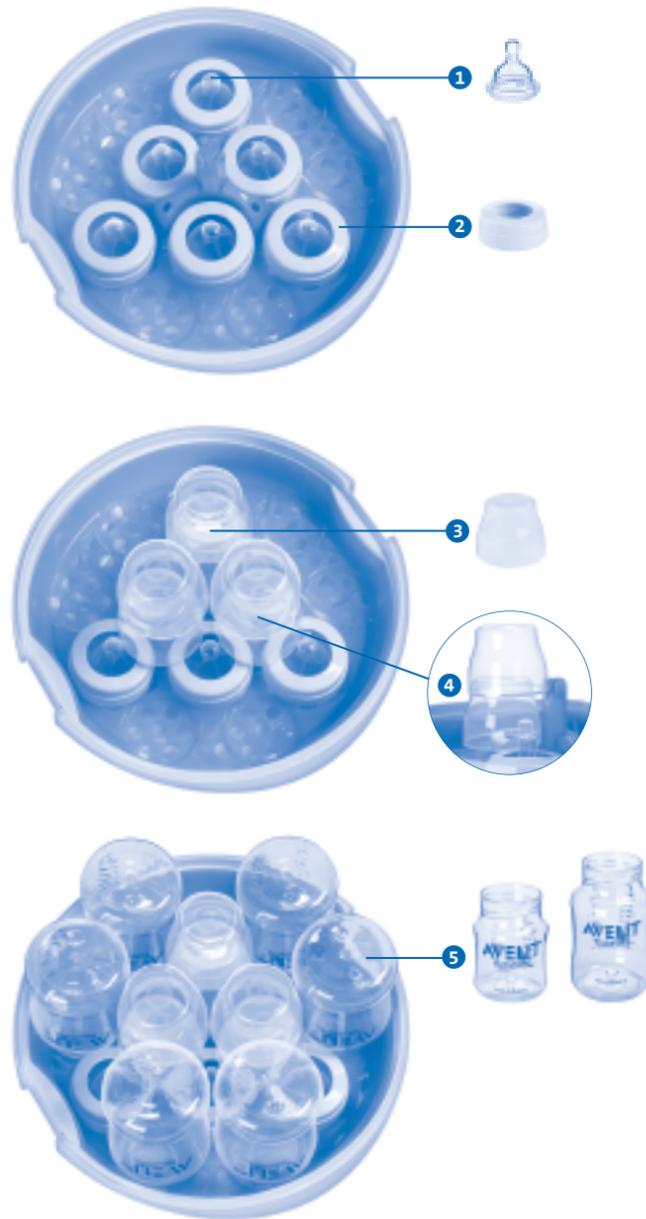
A


B

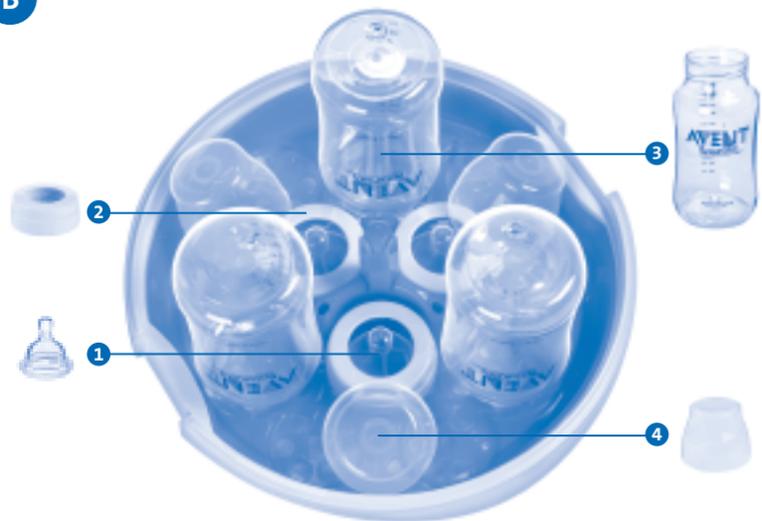
C

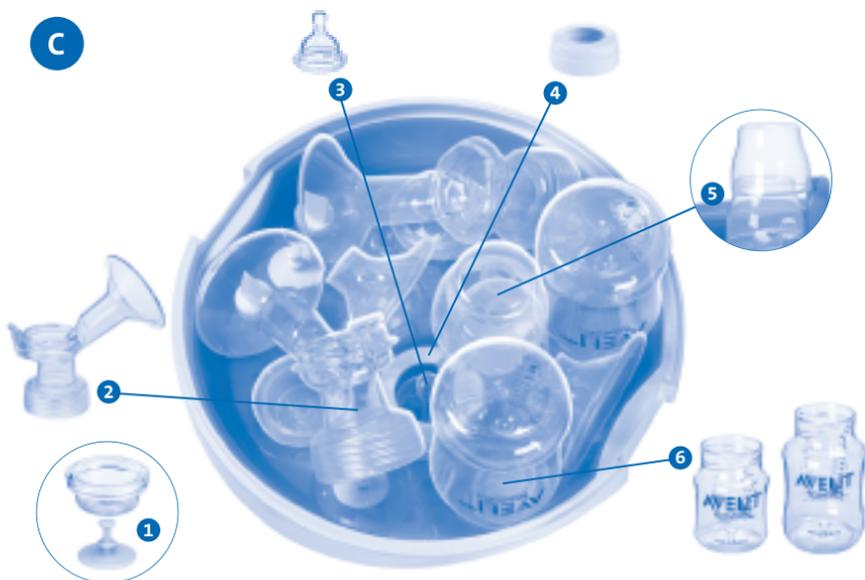D

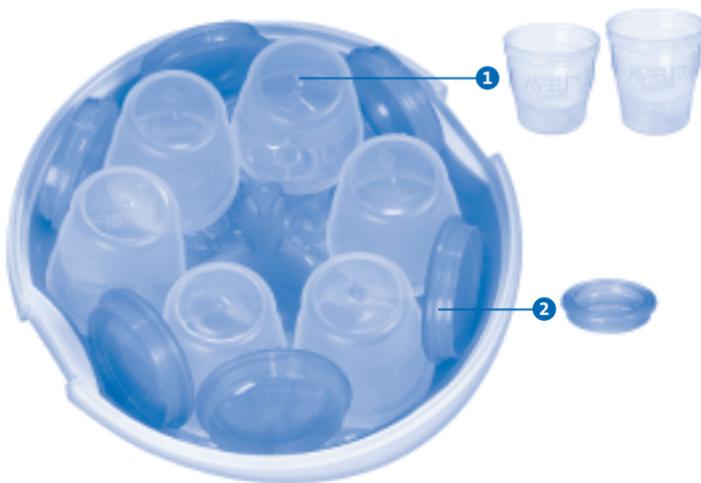E

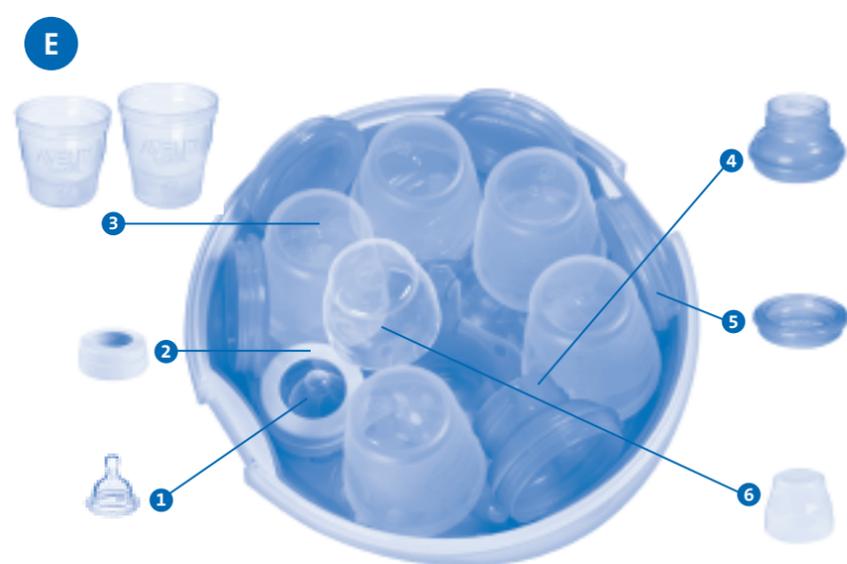US AVENT Express II Microwave Steam Sterilizer

Thank you for choosing AVENT

Now you can enjoy the convenience and ease of using the AVENT Express II Microwave Steam Sterilizer, which effectively eliminates household bacteria in just minutes! Its large capacity holds more than any other microwave sterilizer – up to six bottles – yet it is compact and lightweight enough for travel. Advanced features such as cool safety grips aid in safe handling, and make it easier than ever to sterilize AVENT Breast Pumps, Bottles, Cups and more.

Please take a few minutes to read through the ENTIRE INSTRUCTION LEAFLET including the warnings before using your AVENT Express II Microwave Steam Sterilizer for the first time. PLEASE RETAIN THESE INSTRUCTIONS. Also available at www.aventamerica.com

Assembly and Use

I Contents

- Lid
 - Clips
 - Dome cap stem
 - Bottle cavity
 - Nipple cavity
 - ISIS™ Breast Pump slots
- Base
 - Safety grips

II Before each use

- Wash all items in warm soapy water and then rinse in water.
- Wash your hands before handling your sterilized products.
- Remove rack from the sterilizer and pour 200ml of water into the base.

IMPORTANT: Failure to add enough water may cause damage to the AVENT Express II Microwave Steam Sterilizer and/or your microwave. Adding too much water may mean your items are not properly sterilized.

- Put the rack back into the base.
- Position the lid over the sterilizer base and lock the clips into place until you hear them "click".

4. Ensure that the microwave is clean and dry before use – Place the sterilizer into the center of the microwave.

III Select microwave wattage and timings – See Table III – do not exceed these times.

Loading the sterilizer

Before using the AVENT Express II Microwave Steam Sterilizer, always ensure that the items are positioned according to the below loading instructions.

A Loading six 4oz/125ml or 9oz/260ml AVENT Natural Feeding Bottles

- Position the nipples in the nipple cavities around the dome cap stem.
- Place the screw rings over the nipples.
- Place the first three dome caps in the lower slots of the dome cap stem.
- Place the remaining three dome caps in the upper slots of the dome cap stem.
- Position the bottles upside down around the previously loaded nipples and screw rings.

B Loading three 11oz/330ml AVENT Natural Feeding Bottles

- Position the nipples in the three nipple cavities surrounding the dome cap stem.
- Place the screw rings over the nipples.
- Position the bottles upside down in the three outer nipple cavities.
- Arrange the three dome caps around the edge of the rack in between the inverted bottles.

C Loading one or two AVENT ISIS Breast Pumps (non-electrical components only) and two 4oz/125ml OR two 9oz/260ml AVENT Natural Feeding Bottles

- Disassemble the ISIS Breast Pump ensuring the stem is removed from the diaphragm.
- Position the ISIS Breast Pump bodies into the two slots provided.
- Position the nipples in the nipple cavities around the dome cap stem.
- Place the screw rings over the nipples.
- Place one dome cap in the unobstructed lower slot of the dome cap stem, then the second in the upper slot on top of the inserted dome cap.
- Position the bottles upside down in the unobstructed bottle cavities.
- Place the remaining NON-ELECTRICAL pump components in any unobstructed space on the rack.

D Loading six 6oz/180ml OR 8oz/240ml VIA™ Cups and Lids

- Position the VIA Cups upside down around the dome cap stem.
- Position the VIA Lids vertically around the racks edge. Rims should face out for easier loading.

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ES

IV After each cycle

IMPORTANT: Allow the unit to cool for at least two minutes before removing the sterilizer from the microwave.

1. **Removing the sterilizer from the microwave:** After the unit has cooled, remove it from the microwave by holding the skirt and/or safety grips and then place on an even, heat-resistant surface.

NOTE: Items will remain sterile in the AVENT Express II Microwave Steam Sterilizer for up to 24 hours providing the lid is not removed.

2. **After removing the sterilizer from the microwave:** Remove the lid by unlocking the clips – When removing the lid, point it away from you to allow steam to escape.

Care and cleaning

- Always drain the excess water from the unit and rinse to prevent the build up of limescale.
- Always store when completely dry.
- The AVENT Express II Microwave Steam Sterilizer is dishwasher safe.

E Loading one VIA Disposable Nurser* and up to four 6oz/180ml OR 8oz/240ml VIA Cups and Lids

1. Position the nipple in an outer nipple cavity.
2. Place the screw ring over the nipple.
3. Position the VIA Cups upside down around the dome cap stem.
4. Place the VIA Adaptor on its side around the racks edge (as shown).
5. Position the VIA Lids vertically around the racks edge. Rims should face out for easier loading.
6. Place the dome cap in the unobstructed upper slot of the dome cap stem.

Loading other feeding and baby product items

When sterilizing other **NON-ELECTRICAL AVENT PRODUCTS** simply place the components in the rack allowing enough space so that steam can effectively circulate.

NOTE: Do not overload the sterilizer.

NOTE: Do not place dome caps in the sterilizer upside down.

NOTE: The AVENT Express II Microwave Steam Sterilizer is designed to specifically hold the wide shape of AVENT Natural Feeding Bottles.

* Consists of VIA Cup, Lid and Adaptor, an Extra Soft Slow Flow Nipple, Dome Cap and Screw Ring

WARNINGS –

FAILURE TO FOLLOW INSTRUCTIONS MAY CAUSE SCALDING OR DAMAGE TO THIS PRODUCT AND/OR YOUR MICROWAVE

- Always ensure you use 200ml of water before sterilizing. (The unit should be emptied of residual water before each subsequent use.)
- Always follow loading instructions and do not overload.
- Do not use bleach or chemical sterilizing solutions/tablets in sterilizer or on products to be sterilized.
- Do not sterilize metallic or electronic items in the sterilizer.
- Do not microwave bottle brush.
- Always use the appropriate heating time for the power of your microwave unit. (see Table III)
- In combination ovens, ensure grill is switched off and has cooled before use.
- Be sure that the lid is properly fastened to the base before placing in microwave.
- Always allow at least 2 minutes cooling time before removing from the microwave.
- Do not remove the lid from the sterilizer immediately after cycle. Allow to cool as the steam will be hot.
- When removing from microwave, be sure to hold the sterilizer level to avoid leaking or spilling of hot water.
- When taking the sterilizer out of the microwave, **BE CAREFUL** – it will still be hot.
- When removing the lid, point it away from you to allow steam to escape.
- Be careful when handling the sterilized products as they may be hot.
- Always keep out of reach of children.

Some products featured may not be available in all countries. Please contact AVENT to check availability.

AVENT® is here to help
1.800.54.AVENT
www.aventamerica.com

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