Register your product and get support at  
[www.philips.com/welcome](http://www.philips.com/welcome)

SCF755, SCF753, SCF751




PHILIPS  
**AVENT**

Philips Consumer Lifestyle BV  
Tussendiepen 4  
9206 AD Drachten  
The Netherlands

 100% recycled paper  
 100% papier recyclé

[www.philips.com](http://www.philips.com)

4213.354.0103.1

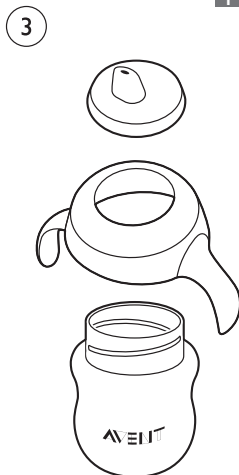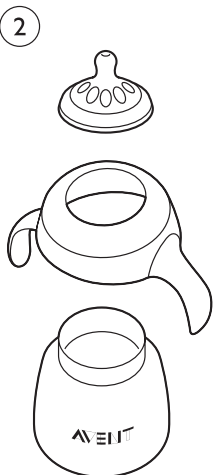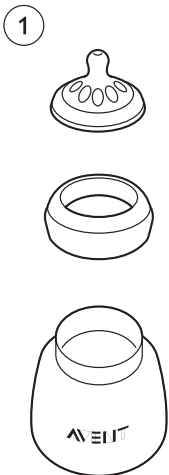
1

SCF751: 200ml/7oz

SCF753: 270ml/9oz, SCF755: 340ml/12oz

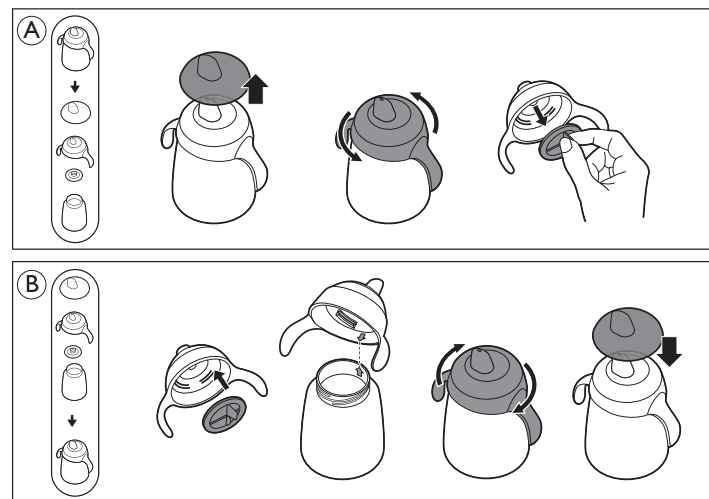**Important**

To fully benefit from the support that Philips offers, register your product at [www.philips.com/welcome](http://www.philips.com/welcome).

Read this user manual carefully before you use the spout cup and save it for future reference.

**For your child's safety and health:**

**WARNING!**

- Check the spout cup before each use. If any damage or crack is detected, stop using the spout cup immediately.
- Always use this product with adult supervision.
- Continuous and prolonged sipping of fluids will cause tooth decay.
- Always check food temperature before feeding.
- Prevent your child from running or walking while drinking.
- Keep all components not in use out of the reach of children.
- Do not heat up any liquid or food in the microwave.

**Caution**

- Before first use, disassemble all parts (fig. A) and clean and sterilize them thoroughly. Place the spout cup in boiling water for 5 minutes. This is to ensure hygiene.
- Do not sterilize any part of the spout cup in the microwave without using a microwave sterilizer.
- Make sure all parts are properly assembled (fig. B) before you give the spout cup to your child.
- This spout cup is intended for use with diluted drinks. Water and milk are the best drinks for children of any age.
- Thick, fizzy or pulpy drinks may block the valve and cause it to break or leak.

- This spout cup is not designed for preparing infant formula. Do not use the spout cup to mix and shake infant formula as this can clog the holes and cause the spout cup to leak.
- Do not overtighten the drinking top on the spout cup.
- SCF751 only: Do not use the spout as a soother.
- Do not use the spout cup with a glass bottle.
- Do not put the spout cup in the freezer compartment.
- To prevent scalding, let hot liquid cool down before you fill the spout cup.
- SCF751 only: For hygiene reasons, replace spouts after 3 months of use. Use only Philips Avent spouts.

**Cleaning and sterilization**

- After each use, disassemble all parts (fig. A) and clean them thoroughly in warm water and some washing-up liquid. You can also clean all parts on the top rack of the dishwasher.
- Do not use abrasive or anti-bacterial cleaning agents or chemical solvents.
- Do not place parts directly on surfaces that have been cleaned with anti-bacterial cleaners.
- Food coloring can discolor parts.
- After cleaning, sterilize the spout cup in boiling water. The spout cup is also suitable for all standard sterilization methods.

**Storage**

- For hygiene and maintenance, disassemble the spout cup (fig. A) and store it in a dry and covered container.
- Keep the spout cup away from heat sources and direct sunlight.

*Tip: The drinking top and bottle of the spout cup are compatible with Philips Avent Natural bottles.*

**Ordering accessories**

To buy accessories or spare parts, visit [www.shop.philips.com/service](http://www.shop.philips.com/service) or go to your Philips dealer. You can also contact the Philips Consumer Care Centre in your country.

**Guarantee and support**

If you need information or support, please visit [www.philips.com/support](http://www.philips.com/support).

**Compatibility chart (fig. 1)**

All parts are interchangeable, with the following exceptions:

- It is not recommended to use the spout cup of SCF753 and SCF755 on the 4oz bottle.

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