
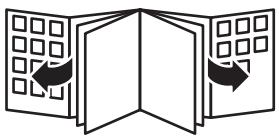
Register your product and get support at
www.philips.com/welcome

HP3641

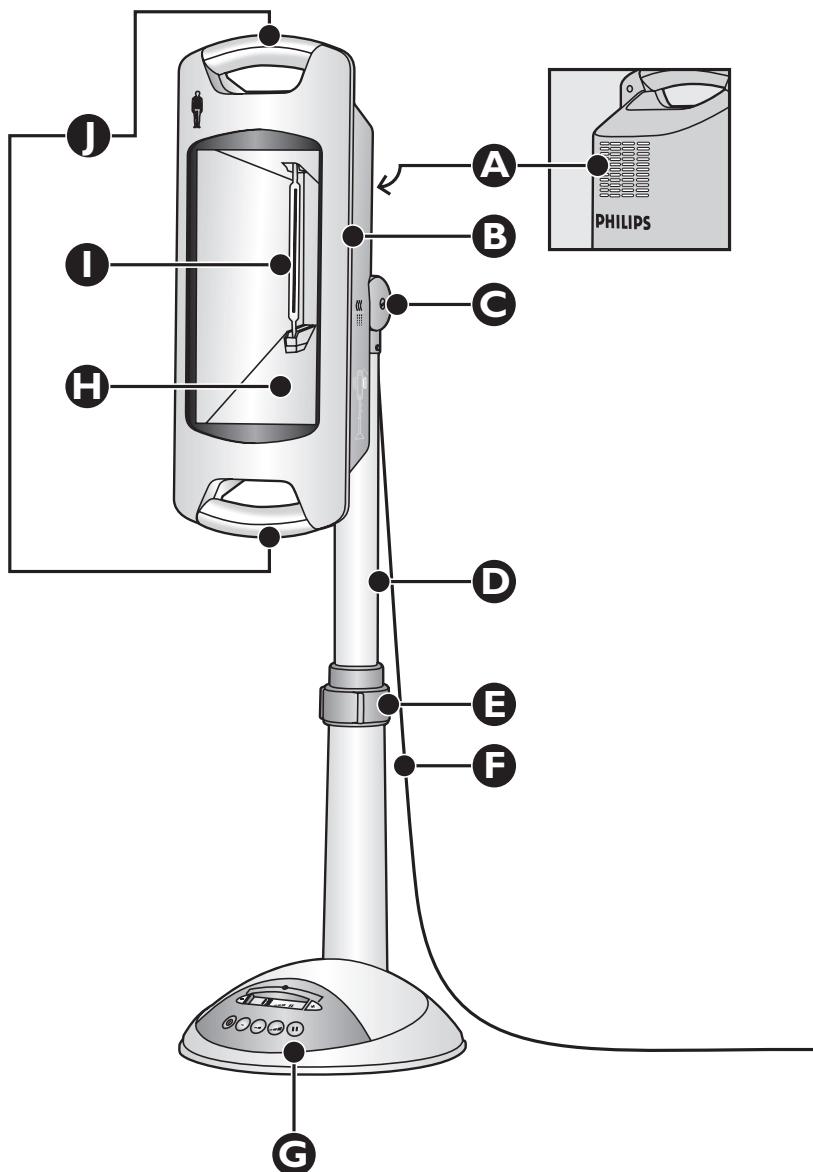PHILIPS

ENGLISH	6
DANSK	12
DEUTSCH	18
ΕΛΛΗΝΙΚΑ	25
ESPAÑOL	32
SUOMI	38
FRANÇAIS	44
ITALIANO	51
NEDERLANDS	57
NORSK	64
PORTUGUÊS	70
SVENSKA	77
TÜRKÇE	83
简体中文	89

Introduction

Congratulations on your purchase and welcome to Philips! To fully benefit from the support that Philips offers, register your product at www.philips.com/welcome.

With the InfraCare you can comfortably treat muscular problems and relieve muscular pain at home. It allows you to treat a large part of your body in one go. Just like the sun, the appliance emits infrared light. The special filter only passes the type of light required to achieve the intended therapeutic effect.

General description (Fig. 1)

- A** Fan
- B** Lamp housing
- C** Hinge for tilting lamp housing
- D** Pedestal pole
- E** Height adjustment ring
- F** Mains cord
- G** Control panel
- H** Filter
- I** Infrared halogen lamp
- J** Handgrips

Important

General

- The symbols on the InfraCare have the following meaning:
- 1 This symbol warns you against hot surfaces (Fig. 2).
- 2 This symbol indicates that you have to read the user manual carefully before you use the appliance. Save the user manual for future reference (Fig. 3).
- 3 This symbol indicates that this appliance is double insulated (Class II) (Fig. 4).
- This appliance meets the requirements of Medical Device Directive MDD93/42/EEC.

Danger

- Water and electricity are a dangerous combination! Do not use this appliance in wet surroundings (e.g. in the bathroom or near a shower or swimming pool).
- Do not let water run into the appliance or spill water onto the appliance.

Warning

- Check if the voltage indicated on the appliance corresponds to the local mains voltage before you connect the appliance.
- If the mains cord is damaged, you must have it replaced by Philips, a service centre authorised by Philips or similarly qualified persons in order to avoid a hazard.
- Do not leave the appliance unattended when it is switched on.
- This appliance is not intended for use by persons with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- This appliance is not intended for use by children. Children should be supervised to ensure that they do not play with the appliance.

Caution

- If the wall socket used to power the appliance has poor connections, the plug of the appliance becomes hot. Make sure you plug the appliance into a properly connected wall socket.
- Do not subject the appliance to heavy shocks.
- Always unplug the appliance after use and in case of a power failure.
- Place the appliance on a stable, level surface. Make sure there is at least 35cm free space around it to prevent overheating.

- Do not use the appliance at room temperatures lower than 10°C or higher than 35°C.
- Make sure the vents in the back of the appliance remain open during use.
- The appliance is equipped with an automatic protection against overheating. If the appliance is insufficiently cooled (e.g. because the cooling vents are covered), the appliance switches off automatically. Unplug the appliance and remove the cause of overheating. Once the appliance has cooled down, it can be switched on again. To do so, plug the appliance in again and press the on/off switch.
- To avoid getting your fingers stuck when you adjust the height, hold the lamp housing by one of the handgrips or by the hinge as long as the height adjustment rings are unlocked.
- Let the appliance cool down for approx. 15 minutes before you clean or store it.
- If you have just been swimming or have just taken a shower, make sure you dry your skin properly before you use this appliance.
- To prevent eye damage, do not look into the light of the lamp when it is on. Keep your eyes closed.
- The filter and the parts directly around it become very hot when the appliance is switched on. Do not touch them.
- Never use the appliance when the filter is damaged, broken or missing.
- Avoid too intense cooling of the body part immediately after treatment.
- Do not fall asleep during treatment.
- Painkillers reduce the sensitivity to heat. Do not use this appliance when you are on painkillers without having consulted your doctor first.
- If you suffer from a severe disease (e.g. diabetes, heart disease, acute inflammatory diseases, multiple sclerosis, thrombosis, blood coagulation disorders) consult your doctor before use.
- As a general rule, if you cannot stand a hot bath, do not use this appliance. If in doubt, consult your doctor.
- If you are oversensitive or insensitive to infrared light, be extra careful when you use this appliance. If in doubt, consult your doctor.
- Do not use the appliance if you have a swelling or an inflammation, as heat could aggravate the complaints. If in doubt, consult your doctor.
- If there is no improvement after 6-8 treatments, stop using the appliance and consult your doctor.
- To prevent overheating of the skin, avoid very short treatment distances.

Electromagnetic fields (EMF)

This Philips appliance complies with all standards regarding electromagnetic fields (EMF). If handled properly and according to the instructions in this user manual, the appliance is safe to use based on scientific evidence available today.

Using the appliance

The effect of the InfraCare

The InfraCare produces infrared light, which penetrates deeply into the skin and warms the tissues. Warmth modulates the neuronal signals to the brain and in this way reduces pain. Warming stimulates the blood circulation and dilates the blood vessels, which accelerates the transport of substances necessary to rebuild and nourish the body's tissues. Infrared light also speeds up the metabolic process and the removal of waste substances from the body. Because it makes tissues more pliable, warmth also reduces stiffness and makes joints more flexible. Because of these effects, treatment with the InfraCare can relieve pain due to muscular and joint problems.

Therapeutic applications

The InfraCare is a local health therapy appliance.
Possible applications:

- Treatment for occasional or chronic muscle pains, stiff muscles and stiff joints
- Treatment of lower-back ache (lumbago)
- Treatment of chilblains
- Preparation for another treatment such as a massage
- Treatment of sprains and bruises
- To relieve pain, e.g. due to rheumatic disorders

► For more information about the health benefits, see www.philips.com, search word InfraCare (Fig. 5).

Preparing for use

- 1** Fully unwind the mains cord.
- 2** Place the appliance on a stable, level surface and make sure there is at least 35 cm free space around it to prevent overheating (Fig. 6).

Thanks to the pedestal, you can use the appliance in many different positions. Below you find some examples.

- To treat your back from shoulders to hips, place the appliance beside a bed, with the lamp housing in highest position and tilted down, and lie down comfortably on your belly (Fig. 7).
- To treat your back, place the appliance beside a bed, with the lamp housing in lowest position, and lie down comfortably on one side (Fig. 8).
- To treat your upper back and shoulders, place the appliance behind a chair or other seat, with the lamp housing in horizontal position, and sit down comfortably with your back towards the appliance (Fig. 9).
- To treat your back, place the appliance beside a chair or stool, with the lamp housing in vertical position, and sit down comfortably with your back towards the appliance (Fig. 10).
- To treat the hip and upper leg area, place the appliance beside a chair; with the lamp housing in horizontal position, and sit down comfortably, with your leg towards the appliance (Fig. 11).

- 3** Adjust the lamp housing

- Turn the height adjustment ring anticlockwise, pull the lamp housing up or push it down to the desired height and then lock it into position by turning the ring clockwise (Fig. 12).
- To tilt the lamp housing upwards or downwards, grasp it by its handgrips and move it to the desired position (Fig. 13).
- You can also turn the lamp housing to horizontal or vertical position by grasping it by the handgrips and turning it to the required position (Fig. 14).
- Make sure the pedestal base is horizontal. If it is placed at an angle, the appliance will switch off automatically (Fig. 15).

- 4** Sit or lie down at the right distance from the appliance.

- At a distance of 40-50 cm to the body, the InfraCare treats an area of about 40 cm by 60 cm. At a larger distance, the area treated becomes larger and the infrared light and therefore the heat less intense. At a smaller distance, the area treated is smaller and the infrared light and therefore the heat more intense.
- Make sure the distance between your body and the appliance is large enough to prevent accidental contact with the hot filter.
- Choose a distance at which the heat is comfortable.

Note: The heat of the appliance does not reach its full intensity until several minutes after you switch on the appliance. This does not mean, however, that the appliance is not effective during these first minutes.

Duration of treatment

The duration of the treatment depends on the person to be treated and the type of use. We recommend a few short treatments a day, e.g. 2 treatments of 15 minutes, on a number of consecutive days to get a result.

Note: Practice has shown that a few short treatments a day produce a better result than one longer treatment.

If in doubt, always consult your doctor!

Using the InfraCare

- 1** Put the mains plug in the wall socket.
- The on/off button (A) and the timer buttons (D) blink for 1 minute, the maximum intensity button (B) lights up and 00 blinks on the display (Fig. 16).
- 2** Set the desired treatment time by pressing the + or - timer button. Keep the button pressed to adjust the time more quickly.
- 3** Adjust the light intensity of the lamp by pressing one of the intensity buttons. If you do not press any of these buttons, the lamp will operate at its default setting, which is maximum intensity.
- 4** Switch the appliance on by pressing the on/off button.
- 5** If you want to interrupt the treatment, press the pause button (C) (Fig. 16).
The lamp goes out, but the remaining treatment time is stored in the memory of the appliance.
- To resume the treatment, press the pause button again.
- 6** To change the treatment time and light intensity during the treatment:
- Press the + or - timer button to change the treatment time or one of the intensity buttons to change the light intensity.
- 7** The appliance automatically switches off after the set time has elapsed.

Note: During the last minute of the set time 01 blinks on the display.

Cleaning

Unplug the appliance and let it cool down for about 15 minutes before you clean it.

Never use scouring pads, abrasive cleaning agents or aggressive liquids such as petrol or acetone to clean the appliance.

- 1** Clean the lamp housing, the pedestal and the base with a moist cloth. Do not let any water run into the appliance.
- 2** You can clean the outside of the filter with a soft cloth sprinkled with some methylated spirits.

Storage

- 1** Unplug the appliance and let it cool down for about 15 minutes before you store it.
- 2** Turn the height adjustment ring anticlockwise, push the lamp housing down and then lock it into position by turning the ring clockwise (Fig. 12).
- Always lift or carry the appliance by its pedestal (Fig. 17).

Note: Store the InfraCare in a dry place.

Replacement

Filter

The filter protects against exposure to undesired light.

If the filter is damaged or broken, do not use the appliance anymore. Contact the Philips Consumer Care Centre for information.

Note: If the filter breaks or becomes damaged during the guarantee period, contact the Consumer Care Centre to have the appliance replaced.

Lamp

The effectiveness of the lamp does not diminish in the course of time.

Do not attempt to replace the lamp if the lamp stops working. Contact the Philips Consumer Care Centre for information.

Note: If the lamp stops working during the guarantee period, do not attempt to replace it yourself, but contact the Consumer Care Centre to have the appliance replaced.

Environment

- Do not throw away the appliance with the normal household waste at the end of its life, but hand it in at an official collection point for recycling. By doing this, you help to preserve the environment (Fig. 18).

Guarantee & service

If you need service or information or if you have a problem, please visit the Philips website at www.philips.com or contact the Philips Consumer Care Centre in your country (you find its phone number in the worldwide guarantee leaflet). If there is no Consumer Care Centre in your country, go to your local Philips dealer.

Troubleshooting

If the appliance does not work or does not work properly, first check the list below. If the problem is not mentioned in this list, the appliance probably has a defect. In that case we advise you to contact the Philips Consumer Care Centre.

Problem	Cause	Solution
The appliance does not go on.	The plug may not be inserted properly into the wall socket.	Insert the plug properly into the wall socket.
	There may be a power failure.	Check if the power supply works by connecting another appliance.
	The lamp may be defective.	Contact the Philips Consumer Care Centre.
	The mains cord of the appliance may be damaged.	Check the mains cord. If the mains cord is damaged, you must have it replaced by Philips or similarly qualified persons in order to avoid a hazard.

Specifications

Model	HP3641
Rated voltage (V) Hongkong/Singapore/Europe	220-230
Rated voltage (V) China/Korea	220
Rated voltage (V) Malaysia	240
Rated voltage (V) Taiwan	110

Model	HP3641
Rated frequency (Hz) Hongkong/Singapore/ Europe	50-60
Rated frequency (Hz) China/Malaysia	50
Rated frequency (Hz) Korea/Taiwan	60
Rated input (W)	650
Class	II
Classification	not classified
Ingress of water	IPX0
IR type	IR-A/B/C
Irradiance (W/m ²)	approx. 1000
Field dimensions (in cm ²)	60x40
Battery type	not applicable

Operation conditions

Temperature from +10°C to +35°C

Relative humidity from 30% to 90%

Storage conditions

Temperature from -20°C to +50°C

Relative humidity from 30% to 90% (no condensation)

Circuit diagrams, component part lists and other technical descriptions are available on request.
As user of this appliance, you should take measures to prevent the occurrence of electromagnetic interference or other types of interference between this appliance and other appliances.

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Philips Consumer Lifestyle B.V