Register your product and get support at
www.philips.com/welcome

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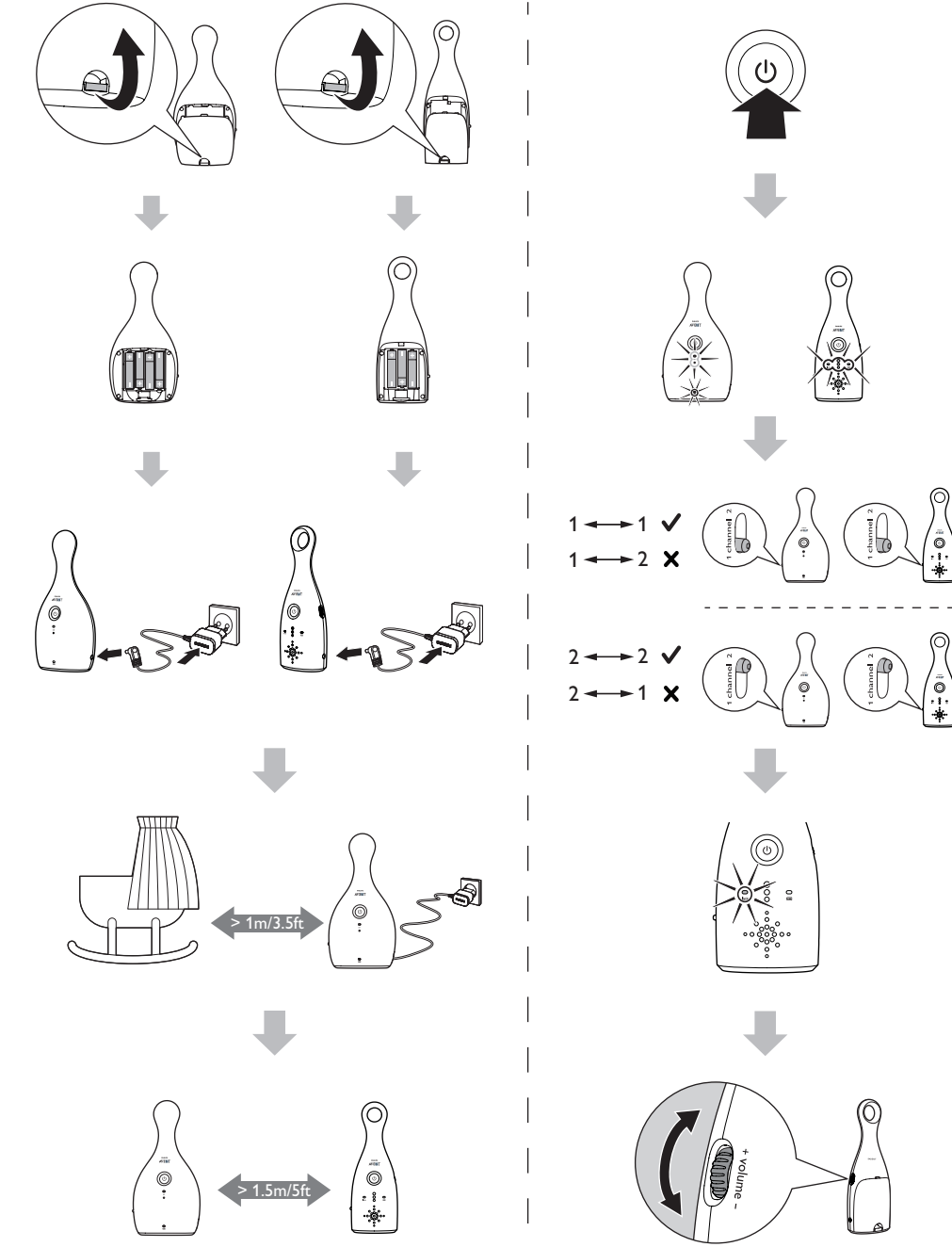
1 Introduction

Congratulations on your purchase and welcome to Philips AVENT! To fully benefit from the support that Philips AVENT offers, register your product at www.philips.com/AVENT. Philips AVENT is dedicated to producing caring and reliable products that give parents the reassurance they need when looking after the health and safety of their baby. This Philips AVENT baby monitor provides the essential reassurance of an ultra clear sound connection to your baby, so you will always know when your baby needs you. Adjustable channel settings, intuitive sound activated lights, sound and light indicators for link, range and low battery ensure you are always connected. SCD486 contains two battery-operated parent units so you can easily monitor your baby from multiple places in and around your home.

2 Important

Read this user manual carefully before you use the baby monitor and save the user manual for future reference.

WARNING: To prevent strangulation with the power cord, always keep the baby unit and the power cord out of baby's reach, at least 1 meter/3.5 feet away. Do not use extension cords.

Caution: Risk of explosion, electric shock, short circuit, or leakage

- The baby monitor shall not be exposed to dripping or splashing and no objects filled with liquids, such as vases, shall be placed on the baby monitor.
- Where the mains plug is used as the disconnect device, the disconnect device shall remain readily operable.
- Before you connect the baby monitor to the mains, make sure that the voltage indicated on the adapters of the baby monitor corresponds to the local mains voltage.
- Use the supplied adapters to connect the baby monitor to the mains.
- To prevent electric shock, do not open the housing of the baby unit or parent unit except for the battery compartments.
- Make sure that your hands are dry when you insert or replace batteries.
- To prevent battery explosion or leakage, which can damage the baby monitor and cause burns and skin or eye irritation:
 - insert batteries in the right direction (+/-),
 - remove batteries if you are not going to use the product for more than 30 days,
 - keep batteries away from excessive heat such as sunshine, fire or the like,
 - remove batteries as soon as they run out of power:
- To prevent batteries from heating up or releasing toxic materials, hydrogen, or oxygen, do not short circuit or mutilate batteries.

Adult supervision

- This baby monitor is intended as an aid. It is not a substitute for responsible and proper adult supervision and should not be used as such.
- Never place the baby unit inside the baby bed or playpen.
- To prevent overheating, do not cover the baby monitor with a towel, blanket, or any other items.
- When your baby monitor uses a power outlet, make sure that you can easily access the power outlet.
- For their safety, do not allow children to play with the baby monitor.
- To handle damaged or leaked batteries, wear protective gloves to protect your skin.
- Your baby monitor transmits over a frequency that is used by many household products and other baby monitors. Your conversation may be heard by another product that operates on the same frequency.

Cleaning and Maintenance

- Do not immerse the parent unit or baby unit in water, and do not clean them under the tap.
- Do not use cleaning spray or liquid cleaners.
- Unplug the parent unit and baby unit if they are connected to a power outlet.
- Clean the parent unit and baby unit with a damp cloth.
- Clean the adapters with a dry cloth.
- This baby monitor has no other user-serviceable parts.

Storage precautions

- Use and store the baby monitor between 10°C (50°F) and 35°C (95°F). Keep the baby monitor out of direct sunlight.
- If you store batteries in a fridge or freezer, protect them from condensation during storage and defrosting. Before you use the batteries, let them return to room temperature.

Replacement

- If you replace the adapters, use the type of adapters specified in this user manual.
- If you replace the batteries, use the type of batteries specified in this user manual. Replace all the batteries in the unit at the same time.

Electromagnetic fields (EMF)

This Philips AVENT baby monitor complies with all standards regarding electromagnetic fields (EMF). If handled properly and according to the instructions in this user manual, this baby monitor is safe to use based on scientific evidence available today.

Recycling

Your baby monitor is designed and manufactured with high quality materials and components, which can be recycled and reused. When you see the crossed-out wheel bin symbol attached to a baby monitor, it means the baby monitor is covered by the European Directive 2002/96/EC:

Never dispose of your baby monitor with other household waste. Please inform yourself about the local rules on the separate collection of electrical and electronic products. The correct disposal of your old baby monitor helps prevent potentially negative consequences on the environment and human health.

Batteries

Warning: Care utilization and disposal of batteries.

- Do not dispose batteries in household waste.
- The improper disposal of batteries can pose risks to the environment and human health.
- To contribute to environmental quality and your health, Philips will receive its batteries supplied with its products after end of life. Philips has collection places in Technical Assistance that helps in this process.
- To prevent the leakage of cells or batteries, which can result in damage to health, environmental or product, we recommend that:
 - In case of devices with removable batteries, remove it when not in use for a long period,
 - Do not mix batteries (old and new or carbon zinc and alkaline, etc.),
 - Do not expose your product with batteries inside the high temperatures.
- In case of devices with non-removable battery, promote the delivery to Philips technical assistance.
- Batteries used or scrap received by Philips will be forwarded to the environmentally friendly disposal.
- In case of doubt or query, please call 0800 701 0203 or 2121 0203 Grande Brazil Sao Paulo or access the site <http://www.philips.com.br>

3 Overview

Parent unit (Fig. 1)

- ① 
 - Press to turn the parent unit on or off
- ② **link**
 - In green: The parent unit is linked to the baby unit.
 - In red flashing: The parent unit is searching for the baby unit.
- ③ **Sound level lights**
 - Indicate the level of sound in your baby's room
- ④ **Speaker**
- ⑤ **Connector for the supplied AC/DC adapter**
- ⑥ 
 - In green: The parent unit is turned on.
 - In red flashing: The batteries are low in power.
- ⑦ **+ volume -**
 - Turn to change the volume up/down
 - Turn down fully to mute the volume
- ⑧ **Lid of battery compartment**
- ⑨ **Three 1.5V R6 AA batteries (not included)**
- ⑩ **1 channel 2**
 - Switch to Channel 1 or Channel 2
- ⑪ **Battery compartment**
- ⑫ **AC/DC adapter**
 - Input: Input: 100-240V AC
 - Output: 6.0V DC, 300 mA
 - Model number: SSW-2350BR

Baby unit (Fig. 2)

- ① 
 - Press to turn the baby unit on or off
- ② on
 - In green: The baby unit is turned on.
- ③ **Microphone**
- ④ 
 - In green: The baby unit is turned on.
 - In red flashing: The batteries are low in power.
- ⑤ **Connector for the supplied AC/DC adapter**
- ⑥ **Lid of battery compartment**
- ⑦ **Four 1.5V R6 AA batteries (not included)**
- ⑧ **1 channel 2**
 - Switch to Channel 1 or Channel 2
- ⑨ **Battery compartment**
- ⑩ **AC/DC adapter**
 - Input: Input: 100-240V AC
 - Output: 6.0V DC, 300 mA
 - Model number: SSW-2350BR

4 Get started

You can connect the parent unit or the baby unit to the mains or insert batteries to operate it. For added reassurance, insert batteries to guarantee a backup in case of power supply failure. If you use rechargeable batteries, make sure that you charge them separately.

To set up the baby unit:

- 1 Before you connect the baby unit to the mains socket, remove the lid of the battery compartment.
- 2 Insert four 1.5V R6 AA batteries with the correct polarity (+/-) as indicated.
- 3 Reattach the lid.
- 4 Connect a supplied adapter to the baby unit and to the mains socket.

To set up the parent unit:

- 1 Follow the same procedure as you do to set up the baby unit.

5 Use your baby monitor

- 1 Press  on the parent/baby unit.
 - ↳ The parent/baby unit turns on, and all indicators on the parent/baby unit light up briefly.
- 2 Keep the baby unit and the power cord at least 1 metre/3.5 feet away from your baby.
- 3 Use the parent unit in the operating range.
- 4 Set the parent unit and the baby unit to the same channel, either Channel 1 or Channel 2.
 - ↳ The **link** indicator flashes red and the parent unit starts to search for the baby unit.
 - ↳ When link is established, the **link** indicator lights up green.
 - ↳ If no link is established, the **link** indicator flashes red. The parent unit beeps from time to time.
- 5 To change the volume on your parent unit, turn **volume** up or down.
 - To mute the volume, turn **volume** down fully. When the volume is muted, the sound level lights operate as normal.

Tip

- Your baby monitor is specially designed to conceal the antenna in the top part of the parent unit. To avoid obstructions that may reduce the operating range, hold the parent unit at the base when carrying it.
- When the parent unit is out of range of the baby unit, the parent unit beeps from time to time. The **link** indicator on the parent unit flashes red.
- The battery indicator flashes red and a beep tone occurs 30 minutes before the batteries run out of power. When the batteries are completely empty, the parent unit automatically switches off and loses the link with the baby unit.

Operating range

- Outdoors: up to 150 metres/500 feet
- Indoors: up to 50 metres/150 feet
- This monitor is designed to operate at its best within a 150 m range, suitable for most household environments. The operating range and quality of the connection varies depending on the number of obstructions, like walls or buildings, in between the two units.

Link an additional parent unit

You can use multiple additional parent units with your baby monitor. Follow the simple two steps to link extra units.

Note

- Use the parent unit in the operating range of the baby monitor.

- 1 Press  on the additional parent unit.
 - ↳ The parent unit turns on, and all indicators on the parent unit light up briefly.
- 2 Set the parent unit and the baby unit to the same channel, Channel 1 or Channel 2.
 - ↳ When link is established, the **link** indicator lights up green.

6 Guarantee and service

If you need service or information or if you have a problem, please visit the Philips AVENT website at www.philips.com/AVENT welcome or contact the Philips Consumer Care Centre in your country. You can find the phone number of the Philips Consumer Care Centre in the worldwide guarantee leaflet. If there is no Philips Consumer Care Centre in your country, please visit the local Philips dealer.

7 Troubleshooting

If you don't find the solution you need in this section, you can get support at www.philips.com/AVENT or contact the Philips Consumer Care Center in your country.

Problems	Possible causes	Solutions
My parent/baby unit doesn't switch on.	The unit is not connected to the power supply.	Connect the unit to a power supply.
	The batteries are empty or inserted incorrectly.	Replace the batteries and make sure that they are correctly inserted.
I cannot establish a connection.	The parent and baby units are set to different channels.	Set the units to the same channel.
	The units are out of range.	Move the units closer (see 'Operating range' on page 9).
The parent unit makes a high pitched noise.	The units are too close.	Move the parent unit farther away from the baby unit.
		Lower the volume of the parent unit.
I get interference. I can only hear intermittently.	The parent unit picks up signals from other appliances or baby monitors.	Move the parent unit away from other electrical appliances, like microwaves or telephones.
		Change the units to the other channel.
	The units are out of range.	Move the units closer.
	The parent and baby units are set to different channels.	Set the units to the same channel.
	The batteries for the parent and/or baby are running low.	Connect the parent and/or baby units to a power supply. Replace the batteries.
I cannot hear anything through my parent unit.	The volume of parent unit is set too low or is muted.	Increase the volume of parent unit.
	The units are out of range.	Move the units closer.
The operating range of mine is less than 150 metres/500 feet.	Environmental obstructions affect the operating range.	Move the units closer.
The batteries in the parent unit run low quickly.	The volume is set too high and the unit consumes more power.	Decrease the volume.

Tip

- Product information is subject to change without prior notice.

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All rights reserved.