PHILIPS AVENT

GB: Call FREE on 0800 289 064
IRL: Call FREE on 1800 509 021
AU: 1 300 364 474
FR: Numéro Vert 0800 90 81 54
DE: 0800 180 81 74 (gebührenfrei)
AT: 0800 292 553 (gebührenfrei)
CH: 056 266 56 56
NL: 0900 1011015 (€ 0,25 per minuut)
BE: +32 (0)9 259 1050
LU: +32 (0)9 259 1050

www.philips.com/AVENT

† Randomised Trial Comparing the Efficacy of a Novel Manual Breast Pump With a Standard Electric Breast Pump in Mothers Who Delivered Preterm Infants.
M S Fewtrell, P Lucas et al. MRC Childhood Nutrition Research Centre, London.
Paediatrics June 2001.

AVENT, Suffolk, CO10 7QS, England

Philips of Holland. All logos brands and product names are trademarks or registered trademarks of their respective holders and are hereby recognised as such.

421335410150

ISIS Out & About Set

Coffret Vie Active ISIS

ISIS Aktiv-Set

ISIS Er-op-uit Set

**GB**

Pages 1-3

FR

Pages 4-6

DE

Seiten 7-9

NL

Blz. 10-12

GB GETTING STARTED

Please take time to read the Instruction Booklet to familiarise yourself with your pump. It is important that you know exactly how your Philips AVENT ISIS Manual Breast Pump works. You should use the pump at home until you feel completely at ease, before beginning to use it outside the home.

The ISIS Out & About Set has been specially designed for you to discreetly express and store your breast milk while away from your baby, then bring it home safely. You can maintain your milk supply by expressing at a time you would normally breastfeed and continue to breastfeed when you and your baby are together.

Before using your ISIS Out & About Set, please consult your health professional or breastfeeding advisor regarding your breastfeeding and breast milk expression schedule.

Contents:

1x ISIS MANUAL BREAST PUMP

Clinically proven as effective as hospital electric pumps[†]. The Let-down Massage Cushion is designed to naturally imitate your baby suckling at the breast and stimulate let down.

1x MICROFIBRE INSULATED BAG

Specially designed by Philips AVENT, this stylish and discreet bag will carry your ISIS Manual Breast Pump and accessories and will keep your breast milk cool for several hours with the help of the cool packs provided.

4x BREAST MILK CONTAINER (2x 125ml/4oz & 2x 260ml/9oz)

Freezer and dishwasher safe. Express directly into these storage containers, which can also be used to feed milk. Just replace the sealing discs with AVENT Airflex Teats.

2x FLEXIBLE COOL PACK


The cool packs must be placed in the freezer for at least 2 hours before each use. They are non-toxic and approved for use as described in this instruction leaflet. They remain flexible after freezing and can be refrozen again and again. To clean, wash with warm soapy water.

Caution: Do not swallow contents. Do not clean in a dishwasher. Suitable for use between -15°C and +70°C.

HOW TO PACK

Before you go out, make sure your breast pump body, breast milk containers and sealing discs are washed and sterilised. The pump body and breast milk storage components are steriliser and dishwasher safe*. Always ensure your hands are clean before packing your breast pump into the bag, using the funnel cover and sealing base to keep the pump sterile and compact for travel, as shown.

Do not attach containers to the pump. Fit the sealing discs into the screw rings and screw them onto the milk containers. Pack the milk containers in the insulated travel bag with the two frozen cool packs taken from the freezer.

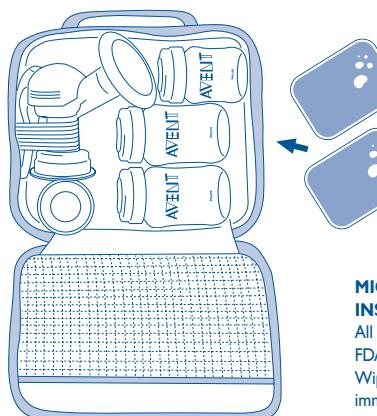COOL PACKS
conform to
EN 12546 pt3

**MICROFIBRE
INSULATED BAG**
All materials conform with
FDA & UK SI:3145 regulations.
Wipe clean only. Do not
immerse in water. Made in China.

ON ARRIVAL

As soon as you arrive at your destination, put the cool packs in the freezer until you are ready to go home. The cool packs must be in the freezer 2 hours before use.

When you are ready to express, you must wash your hands thoroughly and clean your breasts and nipples with a clean damp cloth.

Remove the funnel cover and sealing base and screw a breast milk container onto the pump as shown in the Instruction Booklet. When you have finished pumping, unscrew the container from the pump body and seal with the screw ring fitted with its sealing disc. Place the container of expressed breast milk into the refrigerator immediately. You can use the drawstring bag provided for discreet storage.

Wash the pump body, replace the funnel cover and sealing base and place in the insulated bag. If you wish to use the pump again, you must re-sterilise.

When you are ready to leave, take the container of breast milk from the refrigerator. Take the cool packs from the freezer and place them inside the bag on top of the container. This will keep the milk cool during your journey home. Ensure containers are transported upright within the bag to prevent leaking.

NB: When removed from the freezer, two cool packs will keep either 1x 125ml/4oz or 1x 260ml/9oz breast milk container sufficiently chilled for several hours.

In hot climates, avoid storing the bag containing expressed breast milk in a car, office, cabinet etc. Milk must be sufficiently chilled to avoid spoilage.

NB: Milk storage times vary based on collection and storage conditions. This storage information is only a suggestion. Please contact your health professional or breastfeeding advisor about milk storage guidelines.

WHEN YOU RETURN HOME

As soon as you reach home, put the cool packs into the freezer and the expressed breast milk into the refrigerator or freezer. Milk can be stored for up to 48 hours (from first expression) in the refrigerator and up to 3 months in the freezer. Always store breast milk in the back of the refrigerator, not in the door. You MUST sterilise your ISIS Manual Breast Pump and breast milk containers before each use*. Clean the insulated bag with a soft damp cloth after each use.

*See the Instruction Booklet for full details.

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GB Philips AVENT Bottle Warmers – the FAST, SAFE way to warm feeds.

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GB Philips AVENT Steam Sterilisers – Based on the hospital principle, the intensive heat of the steam eliminates harmful bacteria.

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42 code:	421335410150
Product:	Lft ISIS Manual O&A Instr DB
Languages:	GB/FR/DE/NL (4 langs)
To print colours:	FRONT: PMS647C
	BACK: PMS647C
Profile/dimensions:	W99mm x H153mm
Extent:	16pp booklet
same finishing as:	83 0081 0336#1
Barcodes:	—
PZN:	—
Point 6 ref:	9212-O&A_Lft_GBFRDENL.qxd
Date:	4th May 2007
Created by:	Point 6 Design. +44 (0)20 8962 5880

P6 PROOF DETAILS

Lft ISIS Manual O&A - REPRO