### Introduction

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Register your product and get support at  
[www.philips.com/welcome](http://www.philips.com/welcome)

SCF168

**PHILIPS**  
**AVENT**

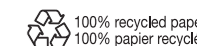
[www.philips.com/AVENT](http://www.philips.com/AVENT)

Philips Consumer Lifestyle BV  
Tussendiepen 4, 9206 AD Drachten, The Netherlands

**Trademarks owned by the Philips Group.**  
©2014 Koninklijke Philips N.V. All rights reserved

[www.philips.com/AVENT](http://www.philips.com/AVENT)

100% recycled paper  
100% papier recyclé

4213.354.3749.1

## ENGLISH

### Introduction

Congratulations on your purchase and welcome to Philips AVENT! To fully benefit from the support that Philips AVENT offers, register your product at [www.philips.com/welcome](http://www.philips.com/welcome).

- The Philips AVENT massage cushion is designed to feel soft and warm and imitates your baby's sipping action to provide a fast milk flow - comfortably and gently.
- Use this standard massage cushion if your Philips AVENT breast pump comes with the large cushion and this large cushion does not fit your breast properly (Fig. 1).
- Only use the massage cushion with Philips AVENT breast pump types SCF330, SCF332 and SCF334.

### Important

Read this user manual carefully before you use this massage cushion and save it for future reference.

### Warning

- This massage cushion is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the product by a person responsible for their safety.
- Children should be supervised to ensure that they do not play with the massage cushion.
- Inspect the massage cushion for signs of damage before each use. Do not use the massage cushion if it is damaged.
- Do not expose the massage cushion to extreme heat and do not place it in direct sunlight.

### Caution

- The massage cushion is only intended for repeated use by a single user.
- Never drop or insert any foreign object into the opening in the massage cushion.

- Never use any accessories or parts from other manufacturers or that Philips AVENT does not specifically recommend. If you use such accessories or parts, your guarantee becomes invalid.

- Wash, rinse and sterilize the massage cushion before each use.
- Do not use antibacterial or abrasive cleaning agents when you clean the massage cushion.

### Preparing for use

#### Cleaning and sterilizing

Clean and sterilize the massage cushion before first use as described in chapter 'Cleaning and sterilizing'.

#### Assembling the massage cushion in the Philips AVENT breast pump

*Note: Make sure that all parts of the breast pump have been cleaned, sterilized and assembled as described in the user manual of your Philips AVENT breast pump. Wash your hands thoroughly before you handle the cleaned parts.*

- 1 Place the massage cushion into the funnel-shaped section of the pump body (Fig. 2).
- 2 Push in the inner part of the massage cushion as far as possible and make sure that it is perfectly sealed all around the rim of the pump body (Fig. 3).
- 3 Press in between the petals to remove any trapped air.

#### Cleaning and sterilizing

Clean and sterilize the massage cushion before first use. Clean the massage cushion after each subsequent use and sterilize it before each subsequent use.

- 1 Clean the massage cushion in hot water with some washing-up liquid and then rinse it thoroughly. You can also clean the massage cushion in the dishwasher (on the top rack only).
- 2 Sterilize the massage cushion in a Philips AVENT steam sterilizer or by boiling it for 5 minutes.

### Storage

Keep the massage cushion out of direct sunlight as prolonged exposure may cause discoloration.

- 1 Store the massage cushion in a safe and dry place.

### Warranty and support

If you need information or support, please visit [www.philips.com/support](http://www.philips.com/support) or contact our Philips AVENT Consumer Care Centre in your country (you can find its number on the Philips website). If there is no Consumer Care Centre in your country, go to your local Philips AVENT dealer.

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