Breast Pump Instructions for use

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This product is a personal care item and therefore cannot be returned once opened. If the pump is defective, we will gladly provide replacement parts or a possible exchange. If you experience any problems with this product, please telephone our toll-free helpline at 1-800-54-AVENT. Our representatives will be happy to assist you. Please do not return this item to the retailer.

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Exclusively Imported and Distributed by AVENT America, Inc.
475 Supreme Drive, Bensenville, IL 60106, USA

www.aventamerica.com

AVENT®, Suffolk, CO10 7QS, England

83 0081 0361 #4

CLINICALLY PROVEN
AS EFFECTIVE AS ELECTRIC PUMPS
BUT WITH GREATER CONVENIENCE*

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







2.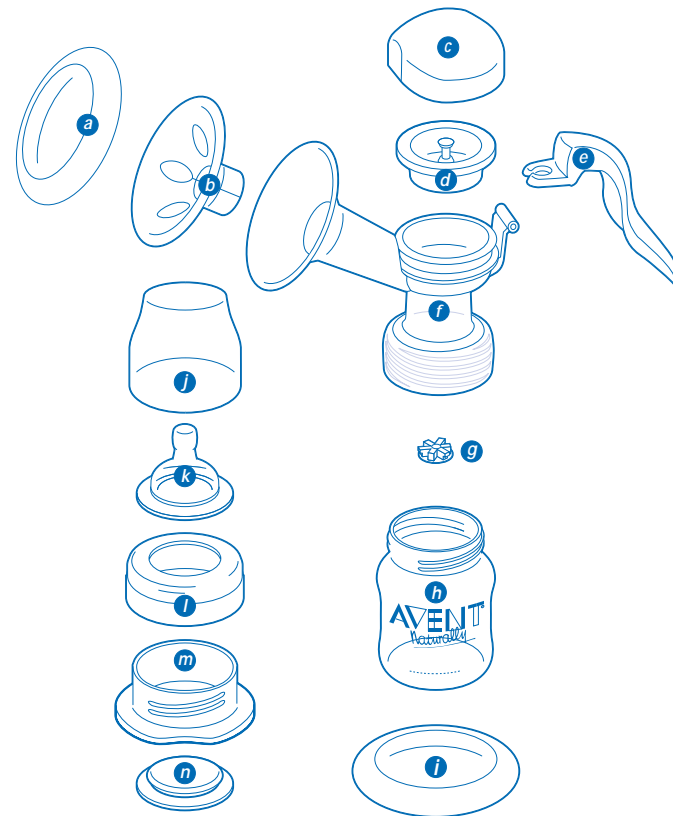

6. Assembly Instructions / Instruction d'assemblage / Instrucciones De Montaje

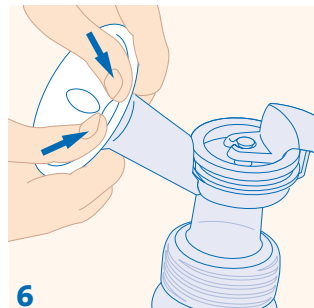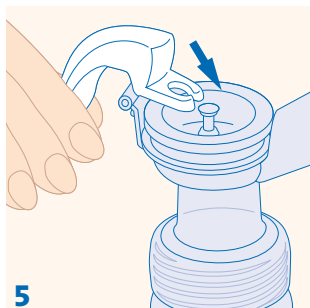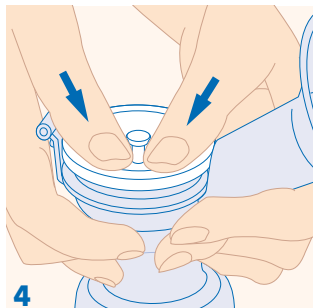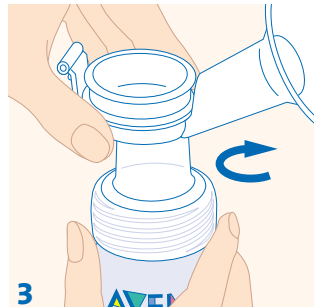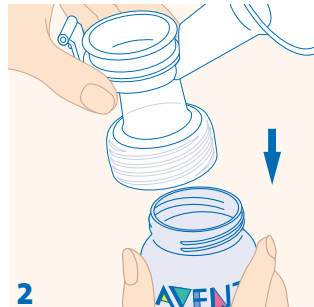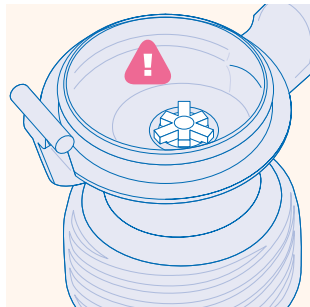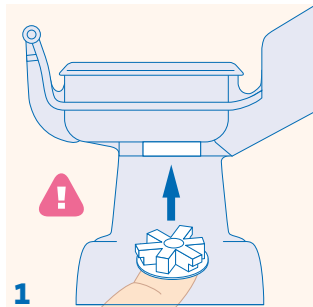IMPORTANT: DO NOT LOSE THE WHITE STAR VALVE. YOUR PUMP WILL NOT WORK WITHOUT IT OR IF IT IS FITTED INCORRECTLY. If lost, spare valves are available direct from AVENT.

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US

Instructions for use

Please take five minutes to read through this leaflet before you use your AVENT ISIS Breast Pump for the first time.

Contents

1. Introducing the AVENT ISIS Breast Pump
2. AVENT ISIS Breast Pump parts
3. To clean and sterilize your pump
4. To keep your pump sterile and compact for travel
5. Using the AVENT Disposable System with your ISIS Breast Pump
6. How to assemble your AVENT ISIS Breast Pump
7. When to use your AVENT ISIS Breast Pump
8. Hints to help you succeed
9. How to use your AVENT ISIS Breast Pump
10. How to store your expressed breast milk
11. Feeding your baby your expressed breast milk
12. Choosing the right nipple for your baby
13. Troubleshooting Guide
14. Other breastfeeding products from AVENT

Should you ever lose these instructions, they are available on our website: www.aventamerica.com

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1. Introducing the AVENT ISIS Breast Pump

AS EFFECTIVE AS ELECTRIC PUMPS BUT WITH GREATER CONVENIENCE*

The AVENT ISIS Breast Pump is a breakthrough in breast pump design. Its silicone diaphragm provides 100% reliable suction, while the let-down massage cushion naturally imitates your baby's suckling action to provide fast milk flow – silently, comfortably and gently. The pump is easy to assemble, discreet to use and all parts are dishwasher safe.

Doctors recommend that breast milk is the best nutrition for babies during the first year, combined with solid food after the first 4/6 months. Your milk is specially adapted to your baby's needs and contains antibodies which help protect your baby against infection and allergies.

Your AVENT ISIS Breast Pump will help you breastfeed longer. You can express and store your milk so that – whether you are out for a few hours, taking a well deserved rest, or back at work – your baby can still enjoy the benefits of your breast milk, even if you can't be there to give it yourself. As the pump is so silent and discreet you can take it with you anywhere, allowing you to express milk at your own convenience and maintain your milk supply.

Since the AVENT bottle and nipple is specially designed to mimic breastfeeding, your baby will switch easily between breast and bottle. This allows working mothers to continue providing breast milk to baby even when they're not around.

* In two recent clinical studies, mothers expressed the same amount of milk or more in ten minutes with the ISIS Breast Pump when compared with electric pumps, with greater comfort and convenience.

US

2. AVENT ISIS Breast Pump parts

PUMP

- a) Funnel cover
- b) Let-down massage cushion
- c) Pump cover
- d) Silicone diaphragm and stem*
- e) Handle
- f) Pump body and funnel
- g) White star valve*


BOTTLE

- h) Reusable bottle (125 ml/4 oz) or Disposable System
 - i) Stand
 - j) Dome cap
 - k) Newborn nipple (0m+)
 - l) Screw ring
 - m) Travel cover
 - n) Sealing disc
- *spares included

3. To clean and sterilize your pump

Before you use your breast pump for the first time and every time before use:

Separate all parts, then wash in the top shelf of the dishwasher or in warm soapy water and rinse. Do not use antibacterial cleaners/detergents.

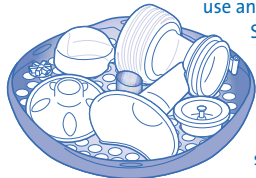Sterilize in an AVENT Steam

Sterilizer (electric or microwave), or by boiling for 10 minutes. If using an AVENT Steam Sterilizer, the pump will remain sterile in the Sterilizer for 3 hours provided the lid is not removed. Open lid to dry for storage/assembly. (Always use distilled water.)

4. To keep your pump sterile and compact for travel

After sterilizing all parts, you can secure the cover (a) over the funnel and let-down massage cushion and screw the travel cover (m) onto the pump body (f) in place of the bottle or bottle holder.

US

5. Using the AVENT Disposable System with your ISIS Breast Pump

Simply replace the AVENT Reusable Bottle with the AVENT Disposable Nurer fitted with a disposable liner.

6. How to assemble your AVENT ISIS Breast Pump

Please refer to diagram 6 at the beginning of this leaflet. Separate all parts and ensure pump has been cleaned and sterilized as in section 3:

- 1) Wash your hands thoroughly. Click the white star valve (g) into the pump body (f) from underneath making sure the flat side of the valve is downwards and the star side is facing up.
- 2) Place the pump body (f) onto the bottle (h).
- 3) Twist gently clockwise until it catches. **DO NOT OVERTIGHTEN!**
- 4) Place the silicone diaphragm with stem (d) into the pump body (f). Make sure it fits securely around the rim by pressing down with your fingers to ensure a perfect seal. (This is easier if assembled while wet.)
- 5) Place the forked end of the handle (e) under the diaphragm stem (d) and push down gently on the handle until it clicks into place.
- 6) Carefully insert the let-down massage cushion (b) into the pump funnel (f) making sure it is perfectly sealed all round the rim of the funnel. (This is easier if assembled while wet.)
- 7) Click the pump cover (c) onto the pump body (f). To make the pump completely stable insert the bottle into the stand (i).

US

7. When to use your AVENT ISIS Breast Pump

- 1) If possible, wait until your milk supply and breastfeeding schedule are well established, normally at least 2-4 weeks after birth, unless told otherwise by your doctor or breastfeeding adviser.
- 2) The exceptions to the above are:
 - a) If you are expressing milk for your baby to be given in hospital.
 - b) If your breasts are engorged (painful or swollen), you can express a small amount of milk before or between feeds to ease the pain and to help your baby latch on more easily.
 - c) If you have sore or cracked nipples, you may wish to express your milk until they are healed.
 - d) If you are separated from your baby and wish to continue to breastfeed when you are reunited, you should express your milk regularly to stimulate your milk supply.
- 3) You will need to find the right times of the day for you to express your milk, i.e.
 - a) Early in the morning when your breasts are full, just before or after your baby's first feed.
 - b) After nursing, if your baby has not emptied both breasts.
 - c) Between feeds, or on a break if you have returned to work.

8. Hints to help you succeed

Using a breast pump takes practice – you may need to make several attempts before you succeed, but because the AVENT ISIS Breast pump is so simple and natural to use, you will soon get accustomed to expressing your milk.

- 1) Familiarize yourself with the pump and pieces prior to use.
- 2) Choose a time when you're not rushed and won't be interrupted.
- 3) Having your baby or a photograph of your baby nearby can help encourage 'let-down'.

US

- 4) Placing a warm cloth on the breasts for a few minutes before you start pumping can encourage milk flow and soothe painful breasts.
- 5) Warmth and relaxation can encourage milk flow. Try expressing after a bath or shower.
- 6) Try expressing from one breast while your baby is feeding from the other, or continue expressing just after nursing.
- 7) Repositioning the pump on the breast from time to time can help stimulate your milk ducts.

Practice with your pump to find the best technique for you. However, if the process becomes very painful, stop and consult your breastfeeding adviser.

9. How to use your AVENT ISIS Breast Pump

- 1) Wash your hands thoroughly and make sure your breasts are clean. Gently squeeze a little milk from each nipple to ensure milk ducts are not blocked.
- 2) Make sure you have sterilized and carefully assembled your pump exactly as shown in sections 3 and 6.
- 3) Consult the 'Hints to help you succeed' section.
- 4) Relax in a comfortable chair, leaning slightly forward (use cushions to support your back). Make sure you have a glass of water nearby.
- 5) Press the funnel and let-down massage cushion of the pump firmly against your breast, making sure that no air can escape – otherwise there will be no suction.
- 6) As you gently push down on the pump handle you will feel the suction on your breast. You don't need to depress the handle fully to create a vacuum; only as much as is comfortable. Initially squeeze the handle with short quick compressions until let-down is achieved. Your milk will soon start flowing even though you may not use all the suction the pump can generate.
- 7) Begin by pumping 5-6 times rapidly to initiate let-down. Then, hold the handle down for 2-3 seconds, and allow it to return to its resting place. These 2-3 second cycles imitate your baby's natural suckling pattern and allow the milk to flow into the bottle between strokes.

US

- 8) Milk should begin to flow after the first few strokes. Don't worry if your milk does not flow immediately. Relax and continue pumping.

Do not continue pumping for more than 5 minutes at a time if NO result is achieved. Try expressing at another time during the day.


Please remember that, should the process become very painful or uncomfortable, you should stop using the pump and consult your doctor or breastfeeding adviser.

- 9) Some women may prefer to use the pump without the let-down massage cushion. Extensive trials show that, in most cases, the massaging effect of the petals flexing in and out around the areola stimulates the let-down reflex and allows you to express more milk more quickly.
- 10) On average you will need to pump for 10 minutes to express 60-120 ml or 2-4 ozs. However, this is just a guide and varies from woman to woman. If you find that you regularly express more than 4 ozs per session, please use a 9 oz/260 ml AVENT bottle.

Do not overfill the bottle and keep it upright, otherwise it may leak from below the pump body.

10. How to store your expressed breast milk

- Breast milk can be stored in the fridge or freezer in either sterile Breast Milk Storage Containers (4 oz/125 ml or 9 oz/260 ml bottles fitted with sealing discs) or in pre-sterilized disposable liners, sealed with a clip.
- Only milk collected with a sterile pump should be stored and fed to your baby.
- Expressed milk should be refrigerated immediately.
- Double bag if using disposable liners to prevent tearing.
- Expressed breast milk can be stored in the refrigerator for up to 72 hours (not in the door) or in the freezer for up to 3 months.
- If you are storing milk in the fridge to add to during the day, only add milk which has been expressed into a sterile container. Milk can only be kept

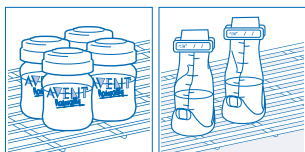US

like this for a maximum of 72 hours (from first expression) after which it must be used immediately or frozen for future use.

- NEVER refreeze breast milk.
- DO NOT add fresh breast milk to already frozen milk.

Storing your milk in AVENT Reusable Bottles

If your expressed breast milk will be fed to your baby within 72 hours, assemble the nipple, screw ring and dome cap and attach onto the bottle for storage in the fridge.

To store in the freezer (for up to 3 months), use an AVENT Sealing Disc in place of the nipple and dome cap. The storage container should be clearly labeled with the date and time of expression and older milk used up first.

Storing your milk in AVENT Disposable Liners

If using AVENT disposable liners for storage, secure liner with clip and place in disposable nurser base. Allow at least 25mm/1" of space between the milk and the clip as the milk will expand as it freezes. No more than 6 fl.oz. (170 ml) of milk should be stored in a disposable liner. Make sure it is well protected in the freezer to prevent crushing.

11. Feeding your baby expressed breast milk

If breast milk has been stored in an AVENT Breast Milk Storage Container, replace the sealing disc with a sterilized AVENT nipple.

If breast milk has been stored in a Disposable Liner, fit the liner into the AVENT Disposable Nurser. Assemble blue tinted nipple, screw ring and dome cap and screw onto the top of the Nurser, securing the liner.

Frozen breast milk should be defrosted overnight in the refrigerator. Once thawed use within 24 hours.

Breast milk can be heated in the same way as any other bottle feed. Stand the bottle of milk in a bowl of hot water, or alternatively, for a safe and fast way to warm feeds, use the AVENT Express Bottle Warmer. Always make sure you test the temperature of the milk before giving to baby.

Never heat breast milk or formula in a microwave as this may destroy valuable nutrients/ and antibodies. In addition, the uneven heating may cause hot spots in the liquid which could cause burns to the throat.

US

12. Choosing the right nipple for your baby

Nipples are numbered on the side to indicate flow rate.

Do not use clear AVENT nipples with your Disposable System as it will leak.

		Reusable (clear nipple)	Disposable (blue nipple)
	Newborn – one hole Recommended for newborn and breastfed babies.	✓	✓
	Slow Flow – two holes For a slightly faster flow. Recommended for 1 month+ and breastfed babies.	✓	✓
	Medium Flow – three holes Recommended for babies 3 months and older.	✓	✓
	Fast Flow – four holes Recommended for babies 6 months and older.	✓	✓
  	Variable – one slot With a slot instead of holes. Recommended for babies 3 months and older. The bulb is marked with three flow settings – I being the slowest and III being the fastest. To adjust the flow rate, turn the bottle while in baby's mouth and line up appropriate marker(s) to baby's nose. Start with slow flow (I) and regulate the flow according to baby's feeding needs.	✓	n/a

8

US

13. Troubleshooting Guide

Lack of suction	Check that all the pump components have been assembled correctly and that the white star valve is facing up with the flatside into the bottle. Ensure that the diaphragm and let-down massage cushion are fitted securely to the pump body and a perfect seal is created.
No milk being expressed	Ensure suction is created and the pump is correctly assembled. Relax and try again, expression may improve after practice. Please refer to the 'Hints to help you succeed' section.
Pain when expressing	You may be pumping too hard. You do not need to use all the suction the pump can generate. Try depressing the handle halfway and remember the 2-3 second pumping rhythm. Consult your health professional/breastfeeding adviser.
Milk is drawn up the petal massage cushion	Try removing and refitting the let-down massage cushion to ensure it is firmly attached and pushed far enough down the funnel. Try leaning slightly forward. Try releasing the handle more frequently to empty the breast milk into the storage container. Refer to "How to use your AVENT ISIS Breast Pump".
Cracking/discoloration of pump parts	Avoid contact with antibacterial detergents/cleaners as these can damage the plastic. Combinations of detergents, cleaning products, sterilizing solution, softened water and temperature fluctuations may, under certain circumstances, cause the plastic to crack. If this occurs, do not use. Contact AVENT Customer Services for spare parts. The pump is dishwasher safe but food colorings may discolor components.
Lost or broken parts	Spare parts are available through AVENT Customer Services, telephone 1-800-54-AVENT, visit our web-site at www.aventamerica.com

If you are still experiencing problems, please contact AVENT Customer Services at 1-800-54-AVENT, email us at www.aventamerica.com or contact your health care professional/breastfeeding adviser.

9

US

14. Other breastfeeding products from AVENT

ISIS On The Go Set – perfect for breastfeeding moms returning to work or traveling

ISIS Comfort Breast Shell Set with Ultrasoft Backing Cushion – to protect sore/cracked nipples, help ease engorgement and collect leaking breast milk

AVENT Breast Milk/Baby Food Storage Containers – for storing breast milk or baby food in the fridge or freezer

AVENT Nipple Protectors – Protects sore or cracked nipples during breastfeeding.

AVENT Breast Milk Storage Kit – pre-sterilized disposable liners for storing and freezing expressed breast milk

Ultra Comfort Disposable Breast Pads – unique ULTRA absorbent core guarantees total dryness and comfort

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