GB call free on **0800 331 6015** (GB Only)
call free on **1800 744 5477** (IRL Only)
call on **1 300 364 474** (AUS)

AR (+971) **800AVENT**

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www.philips.com/AVENT

Philips AVENT, a business of Philips Electronics UK Ltd.
Philips Centre, Guildford Business Park, Guildford, Surrey, GU2 8XH

Trademarks owned by the Philips Group.
© Koninklijke Philips Electronics N.V. All Rights Reserved.

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








2.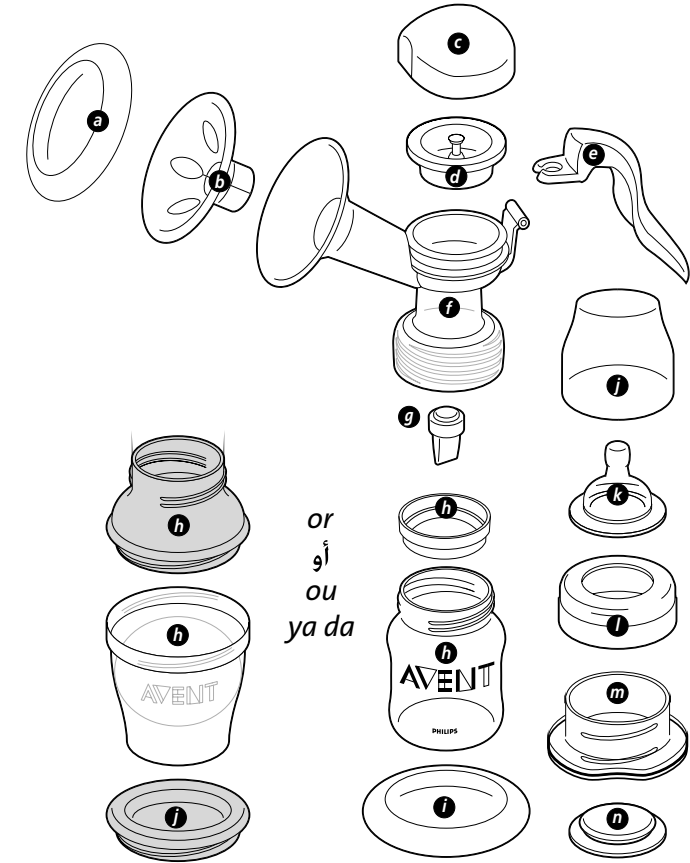

6. (1)–(7)

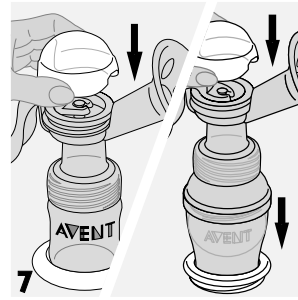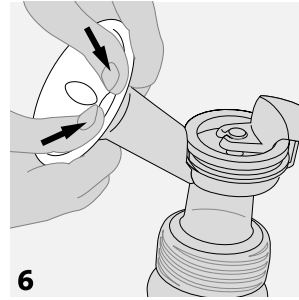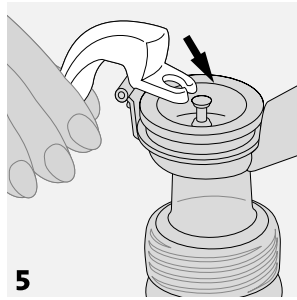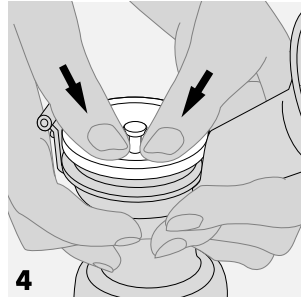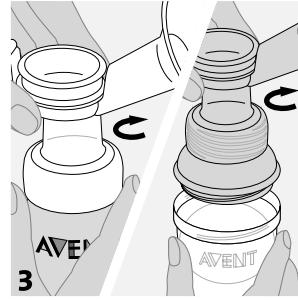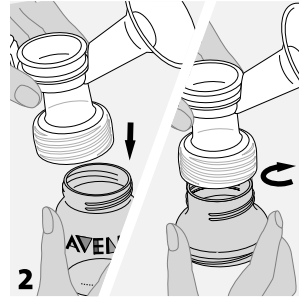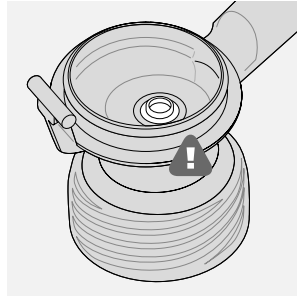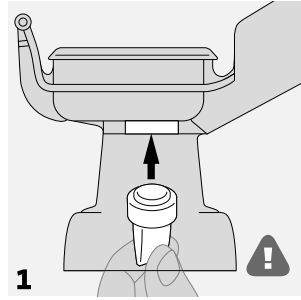IMPORTANT: DO NOT LOSE THE WHITE VALVE. YOUR PUMP WILL NOT WORK WITHOUT IT OR IF IT IS FITTED INCORRECTLY.
If lost, spare valves are available direct from Philips AVENT.

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Please take five minutes to read through the **WHOLE** of this leaflet **BEFORE** you use your Philips AVENT Manual Breast Pump for the first time.

Contents

1. Introducing the Manual Breast Pump
2. Detailed parts guide
3. Cleaning and sterilisation
4. Keeping your breast pump sterile for travel
5. Using the Manual Breast Pump with the AVENT VIA Storage System
6. How to assemble the Manual Breast Pump
7. When to express milk
8. Hints to help you succeed
9. How to use the Manual Breast Pump
10. Storing your breast milk
11. Feeding your baby your expressed breast milk
12. Choosing the right teat for your baby
13. Troubleshooting guide

Should you ever lose these instructions, they are available on our website: www.philips.com/AVENT

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