




Register your product and get support at
www.philips.com/welcome

SCH540

PHILIPS
AVENT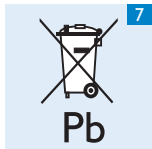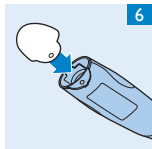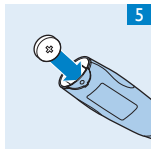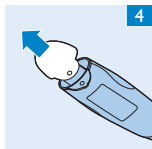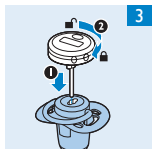

ENGLISH	6
DANSK	19
DEUTSCH	31
SUOMI	47
FRANÇAIS	60
NEDERLANDS	75
NORSK	89
SVENSKA	102
简体中文	114

Introduction

Congratulations on your purchase and welcome to Philips AVENT! To fully benefit from the support that Philips AVENT offers, register your product at www.philips.com/welcome.

The Philips AVENT Digital Thermometer Set accurately and conveniently measures your baby's body temperature orally, axially and rectally. The regular soother and the soother thermometer are suitable for use with children aged 0-12 months. The digital flexible-tip thermometer is suitable for children and adults of all ages.

The unique combination of a soother thermometer and a digital, flexible-tip thermometer offers you the choice of adapting the temperature measurement method to suit your baby.

The storage box contains three items:

- A soother thermometer to reliably measure your baby's temperature. It is very convenient to use and comfortable for your baby.
- A regular Freeflow soother to let your baby get used to the shape.
- A digital flexible-tip thermometer to measure your baby's body temperature axially or rectally if you want additional reassurance.

Make sure you follow the safety guidelines in chapter 'Important' when using these items so that you do not expose your child to any risk.

Use the soother thermometer to obtain a temperature indication. Use the flexible-tip thermometer to determine the actual body temperature. If in any doubt, consult your doctor.

Important

General

- The symbols on the thermometers have the following meaning:
 - 1 This symbol indicates that you have to read the user manual carefully before you use the appliance. Save the user manual for future reference (Fig. 1).
 - 2 This symbol is to identify a type BF applied part complying with IEC60601-1 (Fig. 2).

For your child's safety **WARNING!**

- Never attach ribbons or cords to a soother; your child may be strangled by them.
- Inspect carefully before each use, especially when the child has teeth. Pull the soother in all directions. Throw away at the first signs of damage or weakness.
- Do not store a soother in direct sunlight or near a source of heat, or store it in disinfectant ('sterilising solution') for longer than recommended, as this may weaken the teat.
- Keep the removable protection cap of the teat away from children to avoid suffocation.

- Before first use, place in boiling water for 5 minutes, allow to cool, and squeeze out any trapped water from the soother. This is to ensure hygiene.
- Clean before each use.
- Never dip the teat in sweet substances or medication. Your child may get tooth decay if you do.
- Replace the soother after 4 weeks of use, for safety and hygiene reasons.
- In the event that the soother becomes lodged in the mouth, DO NOT PANIC; it cannot be swallowed and is designed to cope with such an event. Remove from the mouth with care, as gently as possible.
- Always use this product under adult supervision. Keep in a dry covered container. Soothers and caps are suitable for all commonly used forms of sterilisation. Wash your hands thoroughly and ensure surfaces are clean before contact with sterilised components.
- Expose the handles of night-time soothers to light before use.
- The soother can be cleaned by washing with warm water.
- DO NOT use abrasive cleaning agents or anti-bacterial cleaners on the soothers. Excessive combinations of detergents may eventually cause plastic components to crack. Should this occur, replace your soother immediately.
- DO NOT sterilise the display case.
- For further advice on cleaning, see chapter 'Cleaning'.

Warning

- Do not immerse the measurement unit of the soother thermometer in water or any other liquid. Never sterilise or boil the measurement unit.
- Never leave your child when you are measuring its temperature with the soother thermometer or the flexible-tip thermometer.
- The thermometers are not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the thermometers by a person responsible for their safety.
- Children should be supervised to ensure that they do not play with the thermometers.
- Never use the soother thermometer measurement unit separately to avoid injury.
- Never use the soother thermometer as a regular soother.
- Battery compartment covers must be replaced properly if removed. If loose, they may choke your child.

Caution

- Do not drop the thermometers or expose them to shocks.
- Do not bend or disassemble the measurement unit of the soother thermometer or the tip of the flexible-tip thermometer

- Do not use the thermometer if the display shows the battery symbol. This means that the battery is low and the measurements will be inaccurate.
- Only use these thermometers to measure human body temperatures.
- When not in use, store the soother, the soother thermometer and the flexible-tip thermometer in the storage box supplied.
- Do not use this Philips AVENT soother thermometer in combination with parts of the old Philips soother thermometer (SCH530).

Electromagnetic fields (EMF)

These Philips AVENT thermometers comply with all standards regarding electromagnetic fields (EMF). If handled properly and according to the instructions in this user manual, the thermometers are safe to use based on scientific evidence available today.

Before first use

Clean the soother, soother thermometer and the flexible-tip thermometer before first use. See chapter 'Cleaning'.

Using the appliance

Ways to use the thermometers

The most convenient way to measure your baby's temperature is to use the soother thermometer according to this user manual. This method gives a reliable indication. If you need additional reassurance (for example if the first measured temperature is above 38°C), you can then use the digital flexible-tip thermometer to take an axial (underarm) or a rectal (anal) measurement.

Note: Oral measurements are usually 0.5 to 1°C lower than the actual body temperature.

Childrens' temperature

The temperature of a child is normally between 36.5 and 37.5°C. However, you have to keep in mind that a child's temperature varies depending on age, activity, time of day, etc. Also remember that some illnesses do not have high temperature as a symptom. Temperature measurement should not be a substitute for consulting your physician. When reporting your child's temperature to a professional, state the measurement method.

Soother thermometer

The most convenient way to measure your baby's temperature is to use the soother thermometer. This is also the most comfortable method for the baby.

Do not use the soother thermometer for 30 minutes after the baby has had hot or cold drinks, as this causes an inaccurate measurement.

Never use the soother thermometer as a regular soother.

- 1 Check if the measurement unit of the soother thermometer is firmly attached to the soother (i.e. turned completely to the closed lock position).

If the measurement unit is not firmly attached to the soother, the measurement may be inaccurate and the measurement unit may become a suffocation hazard.

- 2 Place the soother in the baby's mouth for approx. 3 minutes to let the thermometer warm up.

If you do not do this, the measured temperature may be inaccurate.

- 3 Press the on/off button  to switch on the thermometer.

Note: You can either press the on/off button while the soother is still in your baby's mouth, or you can remove the soother, press the on/off button and place the soother back into your baby's mouth.

- 4 Leave the soother in the baby's mouth until the thermometer beeps.

See section 'Audible signals' at the end of this chapter for the meaning of the beeps.

- ▶ The display now shows the measured temperature and the °C indication lights up continuously.

Note: If for some reason the temperature cannot be measured, the display shows 'Lo' or 'Hi'. This means that the temperature is outside the temperature range. Switch off the thermometer, check if it is correctly positioned and switch it on again to restart the measurement.

- 5 Remove the soother thermometer from the baby's mouth.

Do not leave the child alone with the soother thermometer.

- 6 Press the on/off button  to switch off the thermometer.

Note: Switch off thermometer after use to save battery power. If you do not switch off the thermometer, it switches off automatically: after 1 minute if the temperature measured is <math><32^{\circ}\text{C}</math>; or after 10 minutes if the temperature measured is >math>>32^{\circ}\text{C}</math>.

Digital flexible-tip thermometer

Do not insert the flexible-tip thermometer into the ear or nose.

The soother thermometer is just as accurate as the digital flexible-tip thermometer and is more convenient to use. However, because rectal measurement is inherently more accurate than oral measurement, the digital flexible-tip thermometer gives you an opportunity to reassure yourself.

For a rectal measurement of a baby's temperature, proceed as follows:

- 1** Remove the protection cap from the digital flexible-tip thermometer. Make sure that the metal probe of the flexible tip is still firmly attached to the thermometer.

Tip: If available, use special plastic hygienic covers to cover the tip of the thermometer (these are available from most chemists).

- 2** Lubricate the tip of the thermometer (or hygienic cover) with a water-soluble jelly.

Water-soluble jelly is available from most chemists.

Note: Do not use petroleum jelly.

- 3** Press the on/off button  to switch on the thermometer.

- 4** Lie the baby on its stomach or its side.

- 5** With one hand, separate the baby's buttocks. With the other hand, gently slide the tip of the thermometer no more than 12mm (1/2 inch) into the rectum. If you detect any resistance, stop immediately.

Hold the baby firmly so that it cannot turn while the thermometer is inserted.

Never push the flexible tip too far into the anus.

- 6** Keep the thermometer in place until it beeps.

See section 'Audible signals' at the end of this chapter for the meaning of the beeps.

Note: If for some reason the temperature cannot be measured, the display shows 'Lo' or 'Hi'. This means that the temperature is outside the temperature range. Switch off the thermometer, check if it is correctly positioned and switch it on again to restart the measurement.

- 7** Remove the thermometer and read the display.

- 8** Press the on/off button  to switch off the thermometer.

Note: Switch off thermometer after use to save battery power. If you do not switch off the thermometer, it switches off automatically: after 1 minute if the temperature measured is $<32^{\circ}\text{C}$; or after 10 minutes if the temperature measured is $>32^{\circ}\text{C}$.

Measuring the temperature of older children

For older children and adults, the digital flexible-tip thermometer can also be used to measure temperature orally (in the mouth) or axially (in the armpit).

Do not use the flexible-tip thermometer for oral temperature measurements on babies.

If the thermometer has been used rectally or in the armpit without hygienic covers, do not use it for oral measurement anymore for hygienic reasons, not even after cleaning.

Audible signals

Signal	Meaning
10 beeps in 10 seconds	Measured temperature is normal (below 37.8°C)
3 sets of short beeps in 10 seconds	Possible fever (measured temperature above 37.8°C)

Features

Last temperature recall

Both the soother thermometer and the digital flexible-tip thermometer have a memory function. When the soother thermometer and the digital flexible-tip thermometer are switched on, the display automatically shows the last measured temperature for approx. 2 seconds. The display then shows the normal body temperature of 37°C.

Battery-low indication

Both the soother thermometer and the digital, flexible-tip thermometer have a long-life battery which should last for about 2 to 3 years. If the display shows a battery symbol, the battery is low. Replace the battery of the flexible-tip thermometer (see chapter 'Replacing the battery').

Note: The battery of the soother thermometer cannot be replaced. If the battery is empty, discard the soother thermometer.

Automatic shut-off

Both the soother thermometer and the digital flexible-tip thermometer switch off automatically: - after 1 min if the temperature measured is <32°C; - or after 10 min if the temperature measured is >32°C.

Error function

Both the soother thermometer and the digital flexible-tip thermometer have an error function. In case of malfunction, the display shows an 'Err'. Proceed as follows if this happens:

- 1 Press the on/off button  to switch off the thermometer.
- 2 Press the on/off button  to switch on the thermometer again.

Cleaning

Do not use abrasive cleaning agents, anti-bacterial cleaning agents or chemical solvents. Excessive concentration of cleaning agents may eventually cause plastic components to crack. Do not use cracked parts.

Clean the regular soother, the soother thermometer and the flexible-tip thermometer after each use to ensure optimal hygiene.
Be careful not to damage the parts when you clean them.

Cleaning the soother thermometer

Before first use, wash and rinse the soother thoroughly.
Clean the soother before each use.

Before cleaning, always remove the measurement unit from the teat and shield.

- 1 Turn the measurement unit 90° anticlockwise with respect to the shield.
- 2 Pull the measurement unit straight out of the teat and shield.

Note: Be careful not to bend the tip of the measurement probe.

- 3 Clean the thermometer measurement unit with a soft damp cloth.

Do not immerse the measurement unit in water.

Do not sterilise or boil the measurement unit.

Do not clean the measurement unit in the dishwasher, in the microwave or in a microwave steriliser.

- 4 Sterilise or boil the soother and protection cap (not the measurement unit!).

The soother and protection cap are suitable for all forms of sterilisation. Sterilise using a Philips AVENT Steam Steriliser or boil in water for 5 minutes. If you use a sterilising solution, follow the instructions on the package. Do not leave the soother in the sterilising solution for longer than recommended, as this may weaken the teat.

Wash your hands thoroughly and ensure surfaces are clean before contact with sterilised components.

- 5 After sterilisation, let the soother cool down for 5 minutes and squeeze out any trapped water from the teat. This is to ensure hygiene.

To reassemble the soother thermometer:

- 1 Align the blocks of the measurement unit with the matching slots in the back of the shield.

2 Insert the measurement unit into the shield. Make sure that the tip of the probe goes into the pocket inside the teat. (Fig. 3)

3 Turn the measurement unit 90° clockwise to lock it into position.

Cleaning the flexible-tip thermometer

Do not sterilise or boil the thermometer.

Do not clean the flexible-tip thermometer in the dishwasher, in the microwave or in a microwave steriliser.

1 Clean the flexible-tip thermometer with a soft damp cloth.

Cleaning the regular soother

Before first use, wash and rinse the soother thoroughly.

Clean the soother before each use.

1 **Sterilise or boil the soother.**

The soother and protection cap are suitable for all forms of sterilisation.

Sterilise using a Philips AVENT Steam Steriliser or boil in water for 5 minutes.

If you use a sterilising solution, follow the instructions on the package. Do not leave the soother in the sterilising solution for longer than recommended, as this may weaken the teat.

Wash your hands thoroughly and ensure surfaces are clean before contact with sterilised components.

2 After sterilisation, let the soother cool down for 5 minutes and squeeze out any trapped water from the teat. This is to ensure hygiene.

Cleaning the storage box

Do not sterilise the storage box.

1 Wipe the storage box with a damp cloth.

Storage

You can use the storage box to store the soother, the soother thermometer and the flexible-tip thermometer or to take them with you when you travel.

1 Put the protection caps on the soother, the soother thermometer and the flexible-tip thermometer after use to keep them clean.

2 Put the soother, the soother thermometer and the flexible-tip thermometer in the storage box.

- Put the storage box in a safe, dry place away from direct sunlight and heat sources.

Replacing the battery

Swallowing a button battery is extremely dangerous.

Batteries may explode if exposed to high temperatures or fire.

Never attempt to recharge the battery.

If the display shows a battery symbol, the battery is low.

Replace the battery of the flexible-tip thermometer as described below.

Note: The battery of the soother thermometer cannot be replaced. If the battery is empty, discard the soother thermometer.

- Remove the battery compartment cover (Fig. 4).
- Use an insulated probe to remove the battery (e.g. a toothpick).
- Insert a new battery (LR41) with the + pole facing upwards (Fig. 5).
- Reattach the battery compartment cover (Fig. 6).
- Dispose of the empty battery in the proper way (see chapter 'Environment').

Environment

- Non-rechargeable batteries contain substances that may pollute the environment. Do not throw away non-rechargeable batteries with the normal household waste, but dispose of them at an official collection point for batteries. Always remove the non-rechargeable batteries before you discard and hand in the thermometers at an official collection point (Fig. 7).
- Do not throw away the thermometers with the normal household waste at the end of their life, but hand them in at an official collection point for recycling. By doing this, you help to preserve the environment (Fig. 8).

Guarantee and service

If you need service or information or if you have a problem, please visit the Philips AVENT website at www.philips.com/AVENT or contact the Philips Consumer Care Centre in your country (you find its phone number in the worldwide guarantee leaflet). If there is no Consumer Care Centre in your country, go to your local Philips dealer.

Technical specifications

	Digital flexible-tip thermometer	Soother thermometer	AVENT soother
Battery	1.55VDC (LR41)	Battery not replaceable	-
Battery life	> 100 operating hours	> 100 operating hours	-
Measurement range	32°C to 42.9°C	32°C to 42.9°C	-
Accuracy	± 0.1°C for 35°C to 42°C range	± 0.1°C for 35°C to 42°C range	-
Teat size	-	Suitable for newborn babies	Suitable for newborn babies
Compliance	Conforms to MDD93/42/EEC	Conforms to MDD93/42/EEC	Conforms to EN1400

Frequently asked questions

This chapter lists the questions most frequently asked about the thermometers. If you cannot find the answer to your question, contact the Customer Care Centre in your country.

Question	Answer
Do I get a different temperature reading with the soother thermometer than with the digital flexible-tip thermometer?	Since oral temperature is slightly lower than actual body temperature, the soother thermometer gives a reading that is 0.5 to 1°C lower than that of the flexible-tip thermometer.
What does the battery symbol on the display of the thermometers mean?	If the display shows a battery symbol, the battery is empty and needs to be replaced. Only the battery of the flexible-tip thermometer can be replaced. The battery of the soother thermometer cannot be replaced, and the soother thermometer must be discarded when the battery is empty.

Question	Answer
Can I replace the batteries of the thermometers myself?	You can replace the battery of the flexible-tip thermometer yourself. For instructions on how to replace this battery, see chapter 'Replacing the battery'. The battery of the soother thermometer cannot be replaced. When this battery is empty, the soother thermometer must be discarded.
What does the 'Err' indication on the display mean?	The 'Err' indication can appear on the display of both the flexible-tip thermometer and the soother thermometer and means that there is a malfunction. Press the on/off button to switch off the thermometer; then press the on/off button again to switch on the thermometer. Proceed to measure the temperature as usual.
What does 'Lo' or 'Hi' on the display mean?	The 'Lo' or 'Hi' indications on the display mean that the temperature cannot be measured because the temperature is outside the temperature range. Press the on/off button to switch off the thermometer. Check if the thermometer is correctly positioned. Press the on/off button again to switch on the thermometer and restart the measurement.
The thermometer beeps when it has finished measuring. What do these beeps mean?	When you measure your baby's temperature, there are 2 different types of beeps that sound once the temperature has been measured. These beeps are the same for the flexible-tip thermometer and for the soother thermometer. When the thermometer gives 10 beeps in 10 seconds, the measured temperature is normal, i.e. below 37.8°C. When it gives 3 sets of short beeps in 10 seconds, this indicates possible fever; i.e. the measured temperature is above 37.8°C.
Can I use honey or other sweet substances on the soother?	Never dip the teat in sweet substances or medication. Your child may get tooth decay if you do.

Question	Answer
What can I do when some parts of the thermometers are broken (e.g. the flexible tip)?	Do not use the thermometers if any part of them is cracked. If the thermometers are damaged or broken, they have to be discarded.
The thermometers do not work. What is wrong?	The batteries of the thermometers may be empty. For instructions on how to replace the battery of the flexible-tip thermometer, see chapter 'Replacing the battery'. The battery of the soother thermometer cannot be replaced. When this battery is empty, the soother thermometer must be discarded.
	Check if the measurement unit of the soother thermometer is firmly attached to the soother and completely locked into position, i.e. turned all the way to the 'closed lock' symbol.
Can I sterilise the thermometers?	You cannot sterilise the flexible-tip thermometer. You can only boil or sterilise the soother and the protection cap of the soother thermometer. Make sure you remove the measurement unit before boiling or sterilising. For instructions on how to clean the thermometers, see chapter 'Cleaning'.
The display does not show anything. What is the cause?	The batteries of the thermometers may be empty. For instructions on how to replace the battery of the flexible-tip thermometer, see chapter 'Replacing the battery'. The battery of the soother thermometer cannot be replaced. When this battery is empty, the soother thermometer must be discarded.
	Perhaps the battery of the flexible-tip thermometer makes poor contact. Clean the battery and the contact surface with a cotton swab. Please note that the measurement unit of the soother thermometer cannot be opened.

Question	Answer
	Perhaps you inserted the battery of the flexible-tip thermometer in the wrong way. Make sure the + pole is facing upwards. For instructions on how to place the battery, see chapter 'Replacing the battery'.
The measured temperature of the thermometers is not accurate.	The measured temperature may differ slightly from other thermometers you have at home. Temperature readings may deviate about 1°C.
Why is the temperature reading on the soother thermometer not accurate?	The temperature reading may be inaccurate because you remove the soother thermometer from the baby's mouth too soon. Place the soother back into the baby's mouth and measure again.
	The temperature reading may also be inaccurate if the baby has drunk a hot or cold liquid just before you measure the temperature. Wait 30 minutes before you place the soother in the baby's mouth.
	Check if the measurement unit of the soother thermometer is firmly attached to the soother and completely locked into position, i.e. turned all the way to the 'closed lock' symbol.

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Philips Consumer Lifestyle B.V