Awas

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Register your product and get support at

www.philips.com/welcome

SCF167

PHILIPS
AVENT

100% recycled paper
100% papier recyclé
100% 循环再生纸

4213.354.4149.2

www.philips.com/AVENT

PHILIPS

- 1

- 2

PHILIPS

PHILIPS

PHILIPS

ENGLISH

Introduction
Congratulations on your purchase and welcome to Philips AVENT! To fully benefit from the support that Philips AVENT offers, register your product at www.philips.com/welcome.

- The Philips AVENT massage cushion for larger nipples is designed to feel soft and warm and imitates your baby’s sucking action to provide a fast milk flow - comfortably and gently.
- Use the massage cushion for larger nipples if the standard massage cushion provided with the breast pump does not fit properly (Fig. 1).
- Only use the massage cushion for larger nipples with Philips AVENT breast pump types SCF330, SCF332 and SCF334.

Important
Read this user manual carefully before you use the massage cushion for larger nipples and save it for future reference.

- Warning**
- This massage cushion is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the product by a person responsible for their safety.
- Children should be supervised to ensure that they do not play with the massage cushion.
 - Inspect the massage cushion for signs of damage before each use. Do not use the massage cushion if it is damaged.
 - Do not expose the massage cushion to extreme heat and do not place it in direct sunlight.

- Caution**
- The massage cushion is only intended for repeated use by a single user.
 - Never drop or insert any foreign object into the opening in the massage cushion.
 - Never use any accessories or parts from other manufacturers or that Philips AVENT does not specifically recommend. If you use such accessories or parts, your guarantee becomes invalid.
 - Wash, rinse and sterilise the massage cushion before each use.
 - Do not use antibacterial or abrasive cleaning agents when you clean the massage cushion.

Preparing for use

Cleaning and sterilising
Clean and sterilise the massage cushion before first use as described in chapter "Cleaning and sterilising". Clean the massage cushion after each subsequent use and sterilise it before each subsequent use.

Assembling the massage cushion in the Philips AVENT breast pump

Note: Make sure that all parts of the breast pump have been cleaned, sterilised and assembled as described in the user manual of your Philips AVENT breast pump. Wash your hands thoroughly before you handle the cleaned parts.

- Place the massage cushion for larger nipples into the funnel-shaped section of the pump body. Push in the inner part of the massage cushion as far as possible and make sure that it is perfectly sealed all around the rim of the pump body. Press in between the petals to remove any trapped air (Fig. 2).

Cleaning and sterilising
Clean and sterilise the massage cushion before first use. Clean the massage cushion after each subsequent use and sterilise it before each subsequent use.

- Clean the massage cushion in hot water with some washing-up liquid and then rinse it thoroughly.You can also clean the massage cushion in the dishwasher (on the top rack only).

- Sterilise the massage cushion in a Philips AVENT steam steriliser or by boiling it for 5 minutes.

Storage
Keep the massage cushion out of direct sunlight as prolonged exposure may cause discolouration.

- Store the massage cushion in a safe and dry place.

Guarantee and service
If you need service or information or if you have a problem, please visit the Philips AVENT website at **www.philips.com/support** or contact the Philips AVENT Consumer Care Centre in your country. You find its phone number in the worldwide guarantee leaflet. If there is no Consumer Care Centre in your country, go to your local Philips AVENT dealer.

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