THIS PAGE DOES NOT PRINT!

42 codes:	421335411030
Product:	LFT ISIS IQ INSTRUCT UNO DB
Languages:	USESFR (3 langs)
To print colours:	(cover): cyan magenta yellow black PMS647C
	(text): cyan magenta yellow black PMS647C
Dimensions:	W105mm x H148.5mm
Extent:	76pp
Replaces existing:	421335410040
Matches size/folding spec:	-
Point 6 ref:	9977-iQUNO_USESFR_1030.qxd
Date:	30 June 2008
Created by:	Point 6 Design. +44 (0)20 8962 5880

PROOF DETAILS

USESFR

Any problems please phone Andy on 020 8962 5895

BPA-Free	Sin BPA	Sans BPA
BPA is Bisphenol-A. This product uses Polyether Sulfone (PES), which is BPA-free and has a natural honey-colored tint.	BPA significa Bisfenol-A. Este producto está fabricado con polietersulfona (PES), material de un suave color miel que no contiene BPA.	BPA = bisphénol A. Ce produit utilise le polyéthersulfone (PES), qui ne contient pas de BPA et a une teinte naturelle de couleur miel.

PHILIPS
AVENT

BPA-Free

Sin BPA

Sans BPA

EN Call TOLL-FREE: **1.800.54.AVENT**

ES Llame gratuitamente: **1.800.54.AVENT**

FR Appelez SANS FRAIS : **1.800.54.AVENT**

www.philips.com/AVENT

USA: Philips Consumer Lifestyle, a division of Philips
Electronics North America Corporation. 1600 Summer Street,
PO Box 120015, Stamford, CT 06905-5125

Canada: Philips Electronics Ltd, 281 Hillmount Road,
Markham, ON L6C2S3

Trademarks owned by the Philips Group.
© 2008 Koninklijke Philips Electronics N.V. All Rights Reserved.

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EN Single Electronic Breast Pump

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FR Tire-lait Electronique Simple

Single Electronic Breast Pump Parts Guide

All you need to assemble and use your Philips AVENT Single Electronic Breast Pump is shown here. If any of the parts are missing, or if you have any questions, please call customer support at 1.800.54.AVENT or visit www.philips.com/AVENT.

Pump Parts

Bottle Parts

- a 1x Control handle
- b 1x Silicone diaphragm with stem
- c 1x White valve
- d 1x Pump body
- e 2x Milk containers (1x 4oz/125ml and 1x 9oz/260ml)
- f 1x Stand (same as h)
- g 1x Let-down Massage Cushion
- h 1x Funnel cover (same as i)
- i 1x Battery pack
- j 1x Wall plug cord
- k 1x Dome cap
- l 1x Extra Soft, Newborn Flow Nipple (0m+)
- m 2x Screw ring
- n 1x Sealing base
- o 2x Sealing disc

1

Need Help? Call us at 1.800.54.AVENT or visit www.philips.com/AVENT

Single Electronic Breast Pump Parts Guide – Detailed

Control Handle

– adjusts the pumping suction, speed and pattern so that your Single Electronic Breast Pump can learn and then repeat your rhythm after only 2 compressions.

Power socket →

← Light

One-touch
Memory Control
– turns pump on and off and automates your pumping rhythm.

Control handle →

NEVER SANITIZE OR IMMERSE IN WATER

Battery Pack

Power cable →

Only the following batteries are compatible with the Single Electronic Breast Pump Battery Pack: Duracell (Standard Coppertop), Duracell Ultra Digital, Energizer, Energizer Max and Ansmann (NiMH). For Ansmann (NiMH) use/charge in accordance with manufacturer's guidelines. Dispose of batteries as per manufacturer's instructions.

NEVER SANITIZE OR IMMERSE IN WATER

Wall Plug Cord

2

IMPORTANT SAFEGUARDS

When using electrical products, especially when children are present, basic safety precautions should always be followed, including the following.

READ ALL INSTRUCTIONS BEFORE USING YOUR PHILIPS AVENT SINGLE ELECTRONIC BREAST PUMP.

WARNING – To reduce the risk of fire, electrocution, or other personal injury:

- The breast pump is intended for repeated use by a single user only.
- Always unplug from the electrical wall outlet immediately after use.
- Do not use while bathing.
- Do not place or store the product where it can fall or be pulled into a tub or sink.
- Do not place or drop the control handle, battery pack or wall plug cord into water or other liquids.
- Do not reach for a product that has fallen into water. Unplug from the wall outlet immediately.
- Close supervision is necessary when this product is used near children or persons with disabilities.
- Do not allow children or pets to play with the control handle, battery pack, wall plug cord or accessories as these parts may pose a choking or strangulation danger.
- Before each use inspect the product, including wall plug cord, for signs of damage. Never operate this product if it has a damaged cord or plug, if it is not working properly or if it has been dropped, damaged or submerged in water.
- Only use attachments recommended by Philips AVENT.
- Do not use outdoors or where aerosol spray products are being used, or where oxygen is being administered.
- Equipment not suitable for use in the presence of flammable anaesthetic mixture with air, or with oxygen or nitrous oxide.
- Only use the wall plug cord and battery pack supplied with the product.
- There are no user serviceable parts inside the breast pump.

3

Do not attempt to open, service or repair the breast pump yourself.

- Keep cords away from heated surfaces.
- This product is a personal care item and should not be shared between mothers.
- Do not attempt to remove the pump body from your breast while under vacuum. Turn the unit off and break the seal between your breast and the pump funnel with your finger, then remove the pump from your breast.
- Do not operate control handle unless connected to a pump body.
- Never use while pregnant, as pumping can induce labor.
- Never use when sleepy or drowsy.

CAUTION – To reduce the risks of injury, device malfunction or failure:

- The product should not be left unattended when plugged into an electrical outlet.
- Never drop or insert any foreign object into any opening.
- Make sure the voltage of the wall plug cord is compatible with the power source.
- Plug the end of the wall plug cord into the control handle first, then securely attach the wall plug into the outlet.
- Although compliant with applicable EMC directives, this equipment may still be susceptible to excessive emissions and/or may interfere with more sensitive material.
- Wash and sanitize the appropriate pump parts before each use.
- Never put the control handle, battery pack or cord in water or a sterilizer as you can cause permanent damage to the pump.
- Do not continue pumping for more than five minutes at a time if no result is achieved.
- Do not use antibacterial or abrasive cleaners/detergents when cleaning breast pump parts.
- If the pressure generated is uncomfortable or causing pain, turn the unit off, break the seal between the breast and the pump body with your finger and remove the pump from your breast.

SAVE THESE INSTRUCTIONS

Need Help? Call us at 1.800.54.AVENT or visit www.philips.com/AVENT

4

Congratulations!

Soon you will discover what makes this Philips AVENT Single Electronic Breast Pump so uniquely yours. The breast pump is designed with an electronic memory that learns and continues your gentle pumping rhythm. It gives you complete control over your comfort, which means that pumping with your Single Electronic Breast Pump always feels natural and exactly right for you.

Let-down Massage Cushion has petals that flex in and out, gently massaging the area around the areola. This is intended to simulate a baby's feeding action and help stimulate a fast, natural milk flow.

One-touch Memory Control gives you instant control over comfort, right at your fingertips. By changing the speed and intensity with which you pump, you can imitate your baby's unique feeding pattern – fast and shallow at the beginning, followed by a slower and stronger rhythm as your milk flows.

Electronic Memory learns from you. Just begin a comfortable pumping rhythm, press a button and the Single Electronic Breast Pump automates it.

Pump, Store, Feed Convenience from the Philips AVENT Natural Feeding System. Products work together, so you never have to transfer milk between containers.

Ultra Comfortable, Quiet and Effective making it easier for you to continue giving your baby the benefits of breast milk.

Please take a few minutes to read through this entire instruction leaflet before using your breast pump for the first time.

PLEASE SAVE THESE INSTRUCTIONS.

Unless advised otherwise by your health professional it is best to wait until your breastfeeding routine is established before expressing – usually around 4 weeks.

What's Inside:

Pumping with your breast pump (Battery or Electric)

Single Electronic Breast Pump Parts Guide	1-2
Important Safeguards	3-4
Disassembling Your Electronic Pump	6
Cleaning and Sanitizing Your Pump	7
Preparing to Pump	8
Assembling the Pump Body	9-10
Connecting the Electronics	11-12
Pumping with Your Electronic Breast Pump	13-15

Manual Pumping

Manual Breast Pump Parts Guide	16
Cleaning the Manual Breast Pump	16
Assembling the Manual Breast Pump	17-18
Using the Manual Breast Pump	19

Keeping Parts Clean and Compact for Travel	20
Storing Your Expressed Breast Milk	21
Technical Specifications	22
Limited Warranty	22
Troubleshooting Guide	23-24

Disassembling Your Electronic Pump

Before you use your breast pump for the first time and every time before use, you must disassemble, wash and sanitize as per 'Cleaning and Sanitizing Your Pump' on page 7.

Disassembly

- 1 If you have already used your breast pump, turn the pump off by pressing and holding down the One-touch Memory Control until the light goes off. Unplug from the electrical wall outlet and disconnect the wall plug or battery pack cord from the control handle.
- 2 **You can be quite firm when removing the control handle.** First pull directly upwards away from the diaphragm. Now unhook it from the stem by moving it towards the funnel.
- 3 Remove the funnel cover and gently pull the Let-down Massage Cushion away from the funnel.
- 4 Remove the silicone diaphragm and stem, unscrew the milk container and remove the white valve.

Cleaning and Sanitizing Your Pump

Clean with a Damp Cloth when Necessary:

These components **DO NOT** come into contact with your breast milk.

CAUTION: NEVER PUT THE CONTROL HANDLE, BATTERY PACK OR WALL PLUG CORD IN WATER OR A STERILIZER, AS YOU CAN CAUSE PERMANENT DAMAGE TO THE PUMP.

Disassemble and Clean Before Every Use (including first use):

Cleaning Instructions:

Wash in warm soapy water and rinse, or wash on the top rack of the dishwasher. Sanitize in a steam sterilizer (electronic or microwave) or by boiling in water for 5 minutes before every use.

CAUTION: DO NOT USE ANTIBACTERIAL OR ABRASIVE CLEANERS/DETERGENTS.

TO CLEAN THE VALVE, RUB GENTLY BETWEEN YOUR FINGERS IN WARM SOAPY WATER – DO NOT INSERT OBJECTS AS THIS MAY CAUSE DAMAGE.

HELPFUL TIP!

The Philips AVENT Electronic and Microwave Steam Sterilizers are designed to hold the washable components of the Electronic Breast Pump and the entire Manual Breast Pump.


Preparing to Pump

When using your Single Electronic Breast Pump for the first time, it will have retained factory settings stored in the memory as a result of quality control testing. If the One-Touch Memory Control is pressed prior to squeezing the control handle upon first use the pump will operate at these settings, which may or may not be comfortable for you.

We recommend that prior to engaging the automatic mode, you establish a comfortable rhythm manually, then press the One-Touch Memory Control to continue your preferred rhythm.

Using a breast pump takes practice. You may need to make several attempts before you succeed, but because your Single Electronic Breast Pump is so simple and comfortable to use, you will soon get used to expressing milk.

1. Choose a quiet time when you're not rushed and unlikely to be disturbed.
2. Having your baby or a photograph of your baby nearby can help encourage your milk to flow.
3. Warmth and relaxation can also help, so you might like to try expressing after a warm bath or shower.
4. Placing a warm cloth on your breasts for a few minutes before you start pumping can encourage your milk flow and soothe painful breasts.
5. Try expressing from one breast while your baby is feeding from the other, or continue expressing just after a feed.
6. Repositioning the pump on your breast from time to time can help stimulate your milk ducts.
7. Some women may prefer to use the pump without the Let-down Massage Cushion. The Let-down Massage Cushion has petals that flex in and out to gently massage the area around the areola. This is intended to help stimulate a fast, natural milk flow. When using the breast pump without the Let-down Massage Cushion, the maximum vacuum level achievable will be higher.
8. To relieve engorgement, we recommend gentle, slow single pumping to relieve the pressure.

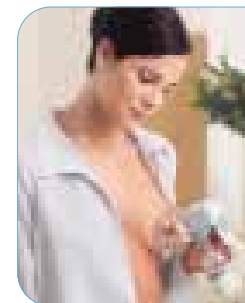CAUTION: IF AFTER 5 MINUTES NO RESULT IS ACHIEVED, TRY AGAIN AT ANOTHER TIME OF DAY. IF YOU EXPERIENCE ANY PAIN IN THE BREAST AREA, STOP AND CONSULT YOUR HEALTH PROFESSIONAL OR BREASTFEEDING ADVISOR.

How much milk can you expect?

On average you will need to pump for 10 minutes to express 2-4ozs or 60-125mls of milk. However, this is just a guide and varies from woman to woman. If you find that you regularly express more than 4ozs/125mls of milk per session, attach a 9oz/260ml milk container to the pump body.

Assembling the Pump Body

Before assembling, make sure that your hands are clean and you have correctly washed and sanitized the pump (see 'Cleaning and Sanitizing Your Pump' on page 7). Also check that none of the pump components are damaged.

Parts you will need for steps 1-4:

Pump body

White valve

Silicone diaphragm with stem


Let-down Massage Cushion

Milk container

White stand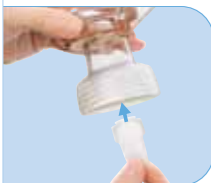

- 1 Insert the white valve into the pump body, so that it fits securely into the small round hole. Make sure the pointed side of the valve is facing down.

Assembling the Pump Body

- 2 Fit the clear silicone diaphragm with stem into the pump body. Ensure a perfect seal around the rim by pressing down with your fingers. (This is easier if assembled while the diaphragm is wet.)

- 3 Screw the pump body onto the milk container until securely in place.

To ensure that the pump is completely stable, you can use the stand with the 4oz/125ml milk container.

- 4 Insert the Let-down Massage Cushion and secure the rim around the funnel making sure there are no gaps. Run your thumb on the flat surface between each petal to ensure there are no air pockets. (This is easier if assembled while the Let-down Massage Cushion is wet.)

HELPFUL TIP!

Some women may prefer to use the pump without the Let-down Massage Cushion. The Let-down Massage Cushion has petals that flex in and out to gently massage the area around the areola. This is intended to help stimulate a fast, natural milk flow. When using the breast pump without the Let-down Massage Cushion, the maximum vacuum level achievable will be higher.

Connecting the Electronics

Parts you will need for steps 5-8:

Assembled
Pump Body
(see pages 9-10)

Control Handle

Wall Plug Cord

Battery Pack

- 5 Firmly push the control handle onto the pump body so that it sits securely on top of the silicone diaphragm.

Connecting the Electronics

- 6 For electric use connect the wall plug cord to the control handle and plug into an electrical wall outlet.

- 7 For battery use, open the top of the battery pack and insert 4 x AA batteries. (For compatible batteries, see page 2.)

Connect the power cord to the control handle.

HELPFUL TIP!

Batteries will last for approximately 6 hours. The light on the control handle will display a continuous **RED** pulsing flash to warn you when it is time to replace them. There will be enough power for approximately 1 more pumping session, with no drop-off in performance, before the pump switches itself off.

Pumping with Your Electronic Breast Pump

Before you begin pumping, make sure that you have correctly washed, sanitized and assembled the pump. Consult the 'Preparing to Pump' section on page 8 for tips on stimulating let-down and milk flow. Wash your hands thoroughly and make sure your breasts are clean. Relax in a comfortable chair and lean slightly forward so that your milk flows easily down the pump funnel into the milk container (you can use cushions to support your back).

- 1 Turn on Pump: Connect the power cord to the control handle and plug into an electrical wall outlet if necessary. Press and hold the One-Touch Memory Control until the light flashes **ORANGE** and you hear the pump turn on (this can take up to 5 seconds). Once the light turns a solid **GREEN**, the pump is ready to use.
If the light is flashing **RED**, refer to the 'Troubleshooting Guide' on page 23.

- 2 Create a Reliable Vacuum: Press the pump against your breast to form a complete seal, ensuring that your nipple fits into the funnel. Check you have a reliable vacuum by compressing and releasing the control handle so you can feel suction.

HELPFUL TIP!

Control Handle Light

-  Flashing **ORANGE** – Control Handle Setting Up
-  Solid **GREEN** – Motor Ready/Pump in Manual Control
-  Flashing **GREEN** – Pump in Automatic Mode
-  Flashing **RED** – Warning
(see 'Troubleshooting Guide' on page 24)

Pumping with Your Electronic Breast Pump

- 3 Initiate Let-down: To encourage let-down, lightly press the handle in a way that feels comfortable and similar to your baby's initial rapid suckling action. The petals on the Let-down Massage Cushion will flex in and out around the areola, creating a massaging effect, similar to baby's natural mouth and jaw movements during breastfeeding. You can choose to automate this let-down phase by pressing the One-touch Memory Control after two compressions (flashing **GREEN** light). Once your milk begins to flow, press the One-touch Memory Control to return to manual control (solid **GREEN** light).

CAUTION: LET-DOWN CAN OCCUR IMMEDIATELY, EVEN AFTER JUST A FEW STROKES AND SHOULD NOT TAKE LONGER THAN 5 MINUTES (REFER TO PREPARING TO PUMP ON PAGE 8).

- 4 Milk Expression: To create a natural and effective breast pumping rhythm that can be maintained for the rest of your pumping session, compress the handle slowly and more fully (to increase the intensity of the suction), hold for 2-3 seconds and then slowly release. Once you are happy with the level of suction and rhythm, after two compressions, press the One-touch Memory Control and your Electronic Breast Pump will continue your personal pumping rhythm (flashing **GREEN** light).

Note: You do not need to compress the handle fully, only until it feels comfortable. The further you compress the handle, the more suction the pump will provide. If the pressure is uncomfortable, stop the automated pumping by pressing the One-touch Memory Control and break the seal between your breast and pump funnel with your finger.

HELPFUL TIP!

Let-down is your body's physiological response to the suckling action of your baby on the breast. This stimulates the breast to produce milk and send it towards the nipple where it is available for baby.

Milk Expression occurs after let-down is achieved and milk begins to flow. It is the phase that is induced by the baby's deeper more comfortable suckling rhythm.

Pumping with Your Electronic Breast Pump

5

Changing or Ending your Pumping Session:

To change your pumping rhythm at any time, press the One-touch Memory Control to revert to manual control (solid GREEN light). You can then set a new pattern by repeating Step 4.

To end your pumping session, press the One-touch Memory Control to revert to manual control (solid GREEN light). Break the seal between your breast and the pump funnel with your finger and remove the pump from your breast. Turn the pump off by pressing and holding the control button until the light goes off. Remove the plug from the electrical outlet if necessary, then remove the power cord from the control handle.

6

Storing your milk:

Make sure that you have not over-filled the milk container and carefully unscrew it from the pump body. Add a sanitized screw ring and sealing disc to safely store your breast milk in the refrigerator or freezer.

7

Next time you turn your pump on:

Resuming saved session? Even after your Electronic Breast Pump is turned off, it will remember the last programmed rhythm. If you have had a short break and your milk is still flowing, ensure you have a good seal against your breast, press the One-touch Memory Control and your pump will continue as before.

Need to re-establish let-down? If you need to re-establish let-down, refer to the beginning of 'Pumping with Your Electronic Breast Pump' on page 13.

CONSULT THE GUIDELINES FOR STORING EXPRESSED BREAST MILK ON PAGE 21

Manual Breast Pump – Parts Guide

While the Manual Breast Pump uses many of the same parts as the Electronic Breast Pump, please note the parts shaded in green are specific to the Manual Breast Pump.

Parts specific to the Manual Breast Pump are included in a separate bag:

- p Clear pump cover
- q Forked handle

Manual Breast Pump

Cleaning the Manual Breast Pump

CAUTION: DO NOT USE ANTIBACTERIAL OR ABRASIVE CLEANERS/DETERGENTS. TO CLEAN THE VALVE, RUB GENTLY BETWEEN YOUR FINGERS IN WARM SOAPY WATER – DO NOT INSERT OBJECTS AS THIS MAY CAUSE DAMAGE.

Every time before use (including first use):

Disassemble and clean all parts of the Manual Breast Pump on the top rack of the dishwasher, or in warm soapy water and rinse. All parts of the Manual Pump are sterilizer safe. Sanitize your Manual Pump in a steam sterilizer (electronic or microwave), or by boiling in water for 5 minutes.

Assembling the Manual Breast Pump

Before assembling, make sure that your hands are clean and you have correctly washed and sanitized the pump (see 'Cleaning the Manual Breast Pump' on page 16). Also check that none of the pump components are damaged.

- 1 Insert the white valve into the pump body so that it fits securely into the small round hole. Make sure the pointed side of the valve is facing down.

- 2 Fit the silicone diaphragm with stem into the pump body. Ensure a perfect seal around the rim by pressing down with your fingers. (This is easier if assembled while the diaphragm is wet.)

- 3 Screw the pump body onto the milk container until securely in place.

To ensure that the pump is completely stable, you can use the stand with the 4oz/125ml milk container.

white stand

Assembling the Manual Breast Pump

- 4 Insert the Let-down Massage Cushion and secure the rim around the funnel making sure there are no gaps. Run your thumb on the flat surface between each petal to ensure there are no air pockets. (This is easier if assembled while the Let-down Massage Cushion is wet.)

- 5 Place the forked end of the handle under the rounded tip of the diaphragm stem. Push down gently on the handle until it clicks into place.


- 6 Click the pump cover onto the pump body.

Using the Manual Breast Pump

Before you begin pumping, make sure that you have correctly washed, sanitized and assembled the Manual Breast Pump. Consult the 'Preparing to Pump' section on page 8 for tips on stimulating let-down and milk flow. Wash your hands thoroughly and make sure your breasts are clean. Relax in a comfortable chair and lean slightly forward so that your milk flows easily down the pump funnel into the milk container (you can use cushions to support your back).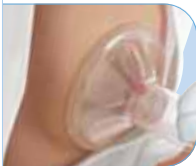

- 1 **Create a Reliable Vacuum:** Press the pump against your breast to form a complete seal, ensuring that your nipple fits into the funnel. Check you have a reliable vacuum by compressing and releasing the control handle so you can feel suction.

- 2 **Initiate Let-Down:** To encourage let-down, lightly press the handle in a way that feels comfortable and similar to your baby's initial rapid suckling action.

CAUTION: LET-DOWN CAN OCCUR IMMEDIATELY, EVEN AFTER JUST A FEW STROKES AND SHOULD NOT TAKE LONGER THAN 5 MINUTES (REFER TO PREPARING TO PUMP ON PAGE 8).

- 3 **Milk Expression:** To create a natural and effective breast pumping rhythm that can be maintained for the rest of your pumping session, compress the handle slowly and more fully (to increase the intensity of the suction), hold for 2-3 seconds and then slowly release.

Note: You do not need to compress the handle fully, only until it feels comfortable. The further you compress the handle, the more suction the pump will provide.

- 4 **Storing your milk:** Make sure that you have not over-filled the milk container and carefully unscrew it from the pump body. Add a sanitized screw ring and sealing disc to safely store your breast milk in the refrigerator or freezer.

CONSULT THE GUIDELINES FOR STORING EXPRESSED BREAST MILK ON PAGE 21

Keeping Parts Clean & Compact for Travel


Pump

After sanitizing all parts insert the white valve and fit the diaphragm with stem onto the pump body. Clip the funnel cover over the Let-down Massage Cushion and screw the sealing base onto the bottom of the pump body.


Breast Milk Storage Container

After sanitizing all parts, click the sealing disc into the screw ring and twist onto the container.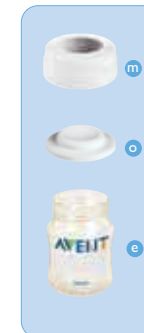

AVENT Naturally Shaped Nipple

After sanitizing all parts, insert the nipple into the screw ring and attach the dome cap over the top of the nipple. Screw on the sealing base.

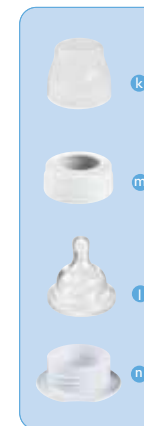Storing Your Expressed Breast Milk

Using Philips AVENT products interchangeably, you can pump, store and feed with the same container, preserving valuable nutrients that can get lost if breast milk is transferred.

After expressing milk into an AVENT container, click a sealing disc into the screw ring and twist onto the container. To feed, just replace the sealing disc with an AVENT Naturally Shaped Nipple.

Guidelines for Storing Expressed Breast Milk

Breast milk can be stored in the fridge or freezer in either sterile AVENT Breast Milk Containers (4oz/125ml or 9oz/260ml fitted with sealing discs) or in pre-sterilized VIA Cups.

Do not overfill the milk container and be sure to keep it upright.

Expressed milk should be refrigerated immediately.

Only milk collected with a sterile pump should be stored to be fed to your baby.

Expressed breast milk can be stored in the refrigerator for up to 24 hours (not in the door), the freezer for up to 3 months or 6 months in a zero-degree freezer.

If you are storing freshly expressed milk in the fridge to add to during the day, only add milk which has been expressed into a sterile container. Milk can only be kept like this for a maximum of 24 hours from first expression, after which it must be used immediately or frozen for future use.

Frozen breast milk can be defrosted overnight in the refrigerator, heated gradually in warm water or heated on the lowest setting of the Philips AVENT Bottle and Baby Food Warmer.

Note:

- NEVER refreeze breast milk.
- DO NOT add fresh breast milk to already frozen milk.
- ALWAYS discard any milk that is left over at the end of a feed.

CAUTION: NEVER IMMERSE A FROZEN BOTTLE OR CUP IN BOILING WATER AS THIS MAY CAUSE IT TO CRACK/SPLIT.

Technical Specifications

Key to Control Handle Symbols

TYPE B APPLIED PART

GENERAL WARNING

FOLLOW OPERATING INSTRUCTIONS

ETL LISTED MARK – CANADA/UNITED STATES

FOR INDOOR USE ONLY

CLASS II EQUIPMENT

Technical Specifications

- Mains Input Voltage 100 – 240 VAC
- Mains Input Frequency 50 – 60 Hz
- Supply voltage fluctuations not exceeding +/- 10% of the nominal voltage
- Ambient Temperature range: +10°C to +40°C
- Atmospheric Pressure: 700 to 1060hPa
- Equipment rated for continuous operation
- Maximum speed at maximum vacuum is 60 cycles per minute

FULL ONE-YEAR WARRANTY

Philips Electronics North America Corporation (USA) and Philips Electronics Ltd (CANADA) warrant each new Philips AVENT product, model SCF302/01, against defects in materials or workmanship for a period of one year from the date of purchase, and agrees to repair or replace any defective product without charge. **IMPORTANT:** This warranty does not cover damage resulting from accident, misuse or abuse, lack of reasonable care, or the affixing of any attachment not provided with the product. **NO RESPONSIBILITY IS ASSUMED FOR ANY SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES.**

In order to obtain warranty service, simply call toll-free 1.800.54.AVENT.

Troubleshooting Guide

<i>Lack of, or no suction</i>	◦ Check for any damaged pump parts and replace if necessary. ◦ Check that all the pump components have been assembled correctly and that the white valve is positioned correctly. ◦ Ensure that the silicone diaphragm and Let-down Massage Cushion are fitted securely to the pump body and that a perfect seal is created around your breast. ◦ Make sure the Let-down Massage Cushion is pushed far enough down the center of the funnel. Run your thumb on the flat surface between each petal to ensure there are no air pockets. ◦ Make certain that the control handle is firmly in place.
<i>No milk being expressed</i>	◦ Ensure the pump is correctly assembled and that suction is being created. ◦ Relax and try again, as expression may improve after practice. ◦ Refer to the 'Preparing to Pump' section on page 8.
<i>Pain in the breast area when expressing</i>	◦ You may be squeezing too hard. You do not need to use all the suction the pump can generate. Try compressing the handle halfway. ◦ Try squeezing the handle more gently and remember the 2-3 second pumping rhythm (refer to 'Pumping with Your Electronic Breast Pump' on page 13). ◦ Consult your health professional/breastfeeding advisor.
<i>Milk is drawn up the Let-down Massage Cushion</i>	◦ Try leaning slightly forward so your milk flows easily down the pump funnel and into the milk container (you can use cushions to support your back). ◦ Remove and re-fit the Let-down Massage Cushion to ensure it is firmly attached and pushed far enough down the center of the funnel (This is easier if assembled while wet). ◦ Some women may prefer to use the pump without the Let-down Massage Cushion. The Let-down Massage Cushion has petals that flex in and out to gently massage the area around the areola. This is intended to help stimulate a fast, natural milk flow. When using the breast pump without the Let-down Massage Cushion, the maximum vacuum level achievable will be higher.

23

Troubleshooting Guide

<i>Cracking or discoloration of pump parts</i>	◦ Avoid contact with antibacterial or abrasive detergents/cleaners as they can damage the plastic. Combinations of detergents, cleaning products, sterilizing solution, softened water and temperature fluctuations may, under certain circumstances, cause the plastic to crack. If this occurs, do not use. The pump body and breast milk containers are dishwasher safe but food colorings may discolor them. Follow guidelines in 'Cleaning and Sanitizing Your Pump' on page 7.
<i>Lost or damaged parts</i>	◦ Use the spare parts included with your Single Electronic Breast Pump or contact Philips AVENT for replacements. Call 1.800.54.AVENT or visit www.philips.com/AVENT
<i>Pump not responding</i>	◦ Ensure the pump is correctly assembled and switched on with the power indicator illuminated GREEN. ◦ Make sure the cord between the electrical wall outlet or battery pack and the pump is correctly connected. ◦ If using the battery pack, use recommended batteries and check they are inserted correctly.
<i>Pump light flashing RED</i>	◦ Pump light flashing RED with delays (3 flashes, delay, 3 flashes etc.): If the pump works properly in Manual Control Mode but does not replicate in Automatic Mode, the handle rhythm may have been too fast for optimal milk expression. The machine is set not to follow. Push the control button to return to Manual Control Mode and use a slower handle rhythm. If this does not help and the power light still flashes RED, the pump has detected a vacuum fault. Disconnect the power and check the control handle and diaphragm for obstruction or damage. ◦ Pump light flashing RED continuously: Low battery warning. Replace the batteries.
<i>Difficulties in separating the control handle from the pump body</i>	◦ The pump must be switched off via the control button in order to separate these parts. If the batteries go flat mid-pumping, you may experience difficulty. Replace the batteries or connect the pump to an electrical outlet. Turn the pump on by holding down the One-touch Memory Control until the light flashes ORANGE and you hear the pump turn on (this can take up to 5 seconds). When the light turns to a solid GREEN, press and hold the One-touch Memory Control again, to turn the pump off. The light will go out and you can now disconnect the pump from the power supply and separate the parts (refer to 'Disassembling Your Electronic Pump' on page 6).

Need Help? Call us at 1.800.54.AVENT or visit www.philips.com/AVENT

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