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GB Instruction Leaflet
SC 说明手册
TC 说明手册

GB Introduction

Ever since 1984, Philips AVENT has taken inspiration from nature in designing and manufacturing high quality parenting and baby products through extensive research and scientific testing, earning the acclaim of parenting experts and mothers worldwide. Now, we have developed a standard range of products specially based on the results of extensive surveys and interviews. Products in this line including the standard neck; breast pump, PP bottle, straight and curved glass bottle, offer mothers a complete pumping, storing and feeding solution.

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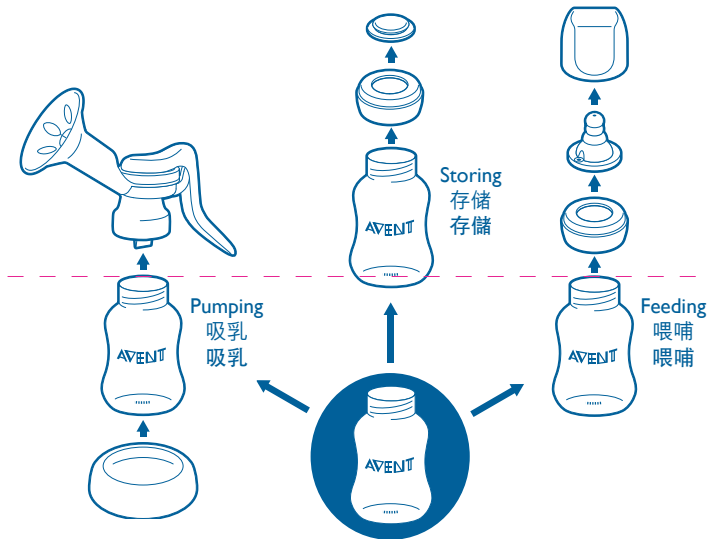
GB Pumping, storing and feeding in one easy step

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GB 1 Important safety notice

- To ensure your child's health and safety, use the product with adult supervision. After opening, immediately dispose of the product packaging or store it in a place out of babies' reach.
- Check the bottle before each use to ensure all components are attached correctly. If you discover any damage, cracks or broken pieces on the bottle, stop using it immediately. Glass bottles are fragile and should be used with care – do not let the baby use the bottle unaided.
- When the glass bottle is wet, handle with extra care to avoid slippage.
- When the surrounding temperature or the temperature of the glass bottle is low, do not immerse the bottle directly into boiling water to prevent cracks to the bottle.
- To avoid cracks, never use iced water to cool the glass bottle. Instead, use running water to cool the bottle, or immerse the bottle in a basin containing an appropriate amount of cold water.
- The glass bottle will remain hot following sterilization. To prevent burns, use the bottle tong to take the bottle or wait until cooling down.

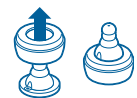
- Do not use metal spoons to stir glass bottles.
- Do not place the glass bottle in the freezer.
- Do not use the teat as a pacifier.

2 Before first use

- Disassemble and then wash and sterilize the product thoroughly.
- Sterilize each component in boiling water for 5 minutes or sterilize with Philips AVENT Sterilizer. For more sterilizing instruction, refer to the instruction manual for the device.

3 Assembly

- Wash your hands thoroughly before touching sterilized component.
- Be sure to use a feeding teat with the right flow when feeding your baby.
- Ensure the anti-colic valve (the valve on the side of the teat) is open. If the valve is blocked, pull the teat to the side to open it.
- Assemble the bottle when the teat has been moistened. Before each use, pull the teat in

every direction to test for damage. If you find any damage or flaws to the teat, please discard it immediately.

- To ensure the bottle works well and does not leak, always ensure you:
 - Clear all debris or residue around the edge of the bottle before assembly.
 - Do not over-tighten the screw ring when assembling onto the bottle base.

4 Feeding suggestions

- The bottle should only be used for the feeding of water and milk. For fruit juice or soup, use the Philips AVENT standard cross cut teat, sold separately.
- To avoid scalding, do not pour high-temperature liquids into the bottle.
- Do not use the microwave to heat contents in the bottle, as this may lead to uneven heating and cause injury to your baby.
- When cooling the bottle, do not use running water to rinse the teat or bottle.
- Hold on to the bottle rather than the bottle top.
- Do not make up formula milk next to your baby to avoid scalding.
- To prevent the milk from leaking or spurting, make sure the cap is on before mixing formula.
- When reading the scale on the streamlined

glass bottle model, use the longer scale marks next to the scale numbers as your guide.

- After mixing formula, place the bottle horizontally (placing the bottle on an uneven surface may result in leakage). Feed your baby once the milk begins to drip.
- Before feeding, check the milk temperature. You can do so by applying a small amount of the milk on the inside of your wrist and feed once the temperature is deemed appropriate.
- Formula milk should be fed within two hours of mixing. Discard any leftovers after two hours.
- Keep teething babies from chewing on the teat for extended periods to avoid damage to the teat.
- Long-term, prolonged sucking of liquids may lead to dental caries.
- Place sterilized components on a clean surface.
- Expressed breast milk may be stored in sterilized AVENT PP or glass bottles and kept in the refrigerator (but not the fridge door) for a maximum of 48 hours. If stored in the PP bottle, expressed breast milk may be kept in the freezer for as long as three months, but never store a glass bottle in the freezer.
- For hygiene reasons, we recommend you replace teat every three months. Use only compatible Philips AVENT standard teat with your Philips AVENT standard feeding bottle.

5 Cleaning and sterilizing

- Do not use corrosive antibacterial cleaners, chemical solutions or highly-concentrated detergents with your product.
- Do not place any components on surfaces that have been cleaned with antibacterial agents.
- Food colorings may discolor components.
- After each use, disassemble the product and clean it thoroughly.
- If necessary, clean with the Philips AVENT bottle and teat brush, sold separately.
- Sterilize each component in boiling water for five minutes or sterilize in Philips AVENT Sterilizer. Consult the sterilizer user manual for details.
- If sterilizing solution is used, do not allow the bottle teat to sit in the solution longer than the recommended time.
- Do not place the bottle in heated drier or sterilizing cabinets.

6 Storage

- Store small components out of the reach of children.
- Store the bottle and its components away from heat sources and away from direct sunlight.
- For hygiene reasons, remove all components and store in a dry container with a lid.

7 Accessories

- Spare bottle cap sets and teats are sold separately.
- Note: All Philips AVENT standard products are interchangeable.

* Based on December 2011 TNS online satisfaction survey conducted among more than 36,000 female users of childcare brands and products in the USA, the UK, France, Germany, Italy, the Netherlands, Poland, Spain, Russia and China.