PHILIPS AVENT

AVENT Ltd, Suffolk, CO10 7QS, England
(AVENT Ltd is a member of the Philips Group)

www.philips.com/AVENT


Philips of Holland. All logos brands and product names are trademarks or registered trademarks of their respective holders and are hereby recognised as such.

42133 5435 140

GB
FR
DE
NL
NO
SE
DK
ES
IT
PT
RU
PL
TR
AR
GR
SC

1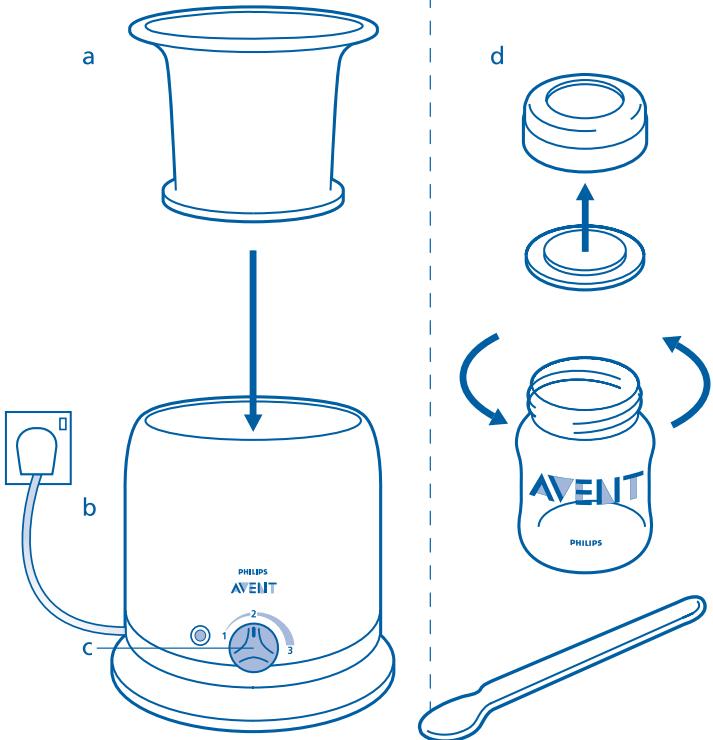

PHILIPS
AVENT

2

		TEMPERATURE OF FEED BEFORE WARMING*
125ml 4oz	5°C (40°F)	20°C (70°F)
260ml 9oz	7 mins	5 mins
330ml 11oz	8.5 mins	6 mins
130g	5 mins	4.5 mins
190g	7 mins	5.5 mins

*FR: TEMPÉRATURE DU LAIT AVANT QU'ON LE RÉCHAUFFE. DE: TEMPERATUR DER NAHRUNG VOR DEM AUFWÄRMEN.
 NL: TEMPERATUUR VAN DE VOEDING VOOR VERWARMEN. NO: TEMPERATUR PÅ MATEN FÖR OPPVARMING.
 SE: MATEMPERATUR INNAN UPPVÄRMNING. DK: STARTTEMPERATUR INDEN OPVARMING. ES: TEMPERATURA DE LA COMIDA ANTES DE CALENTAR. PT: A TEMPERATURA DA COMIDA ANTES DE AQUECER. IT: TEMPERATURA DEL CONTENUTO PRIMA DEL RISCALDAMENTO. PL: TEMPERATURA POKARMU PRZED PODGRZANIEM. RU: ИСХОДНЯЯ ТЕМПЕРАТУРА ПИТАНИЯ. TR: BESİNİN ISITILMASA ÖNCESİ İSİSLİ. AR: TEMPERATURE OF FEED BEFORE WARMING. GR: ΘΕΡΜΟΚΡΑΣΙΑ ΤΗΣ ΤΡΟΦΗΣ ΠΡΙΝ ΤΟ ΖΕΣΤΑΜΑ. SC: 加热前的食物温度

*TR: Dakika AR: mins GR: mins SC: 分钟

Please take 5 minutes to read through the whole of this leaflet before you use your Express Bottle and Baby Food Warmer for the first time and retain for future reference.

The Express Bottle and Baby Food Warmer allows you to warm all AVENT Airflex Feeding Bottles as well as baby food jars and tins safely and quickly. The Express Bottle and Baby Food Warmer isn't meant for keeping liquid or food warm as vitamins can be destroyed.

The speed at which the warmer heats the milk or food depends on the size of the bottle or container and the starting temperature of the milk or food (e.g. room temperature 20°C or out of fridge 5°C). Diagram 2 gives an indication of approximate warming time. Always test the temperature before feeding.

The following guidelines are the approximate temperature the water* surrounding the container can reach. This is not a heating guide for liquid or food!

Setting	Approximate water* temperature surrounding the container
1	35–45 °C
2	45–75 °C
3	75–85 °C

Contents

- a) Basket
- b) Warmer body
- c) Dial
- d) Baby food container and weaning spoon

Instructions for Use

For Airflex Feeding Bottles

- 1) Ensure basket is in place inside the warmer.
- 2) Place the assembled, filled bottle in the warmer.
- 3) For 125ml/4oz Airflex Bottles, fill the warmer with water to just above the level of milk in the bottle.
- 4) For larger Airflex Bottles (260ml/9oz), fill the warmer to 15mm (1/2 inch) below the rim of the warmer.
- 5) Never let the water level rise above the neck of the bottle or spill over the edge of the warmer.
- 6) Plug in the unit and switch on at the mains. Turn the dial to setting 3.
- 7) The light will illuminate to indicate the warmer is working.

- 8) During the warming process, the light will come on and off to indicate that the water surrounding the bottle is being heated to the maximum temperature of the setting.
- 9) Always test the feed and when it has reached the required temperature, remove immediately.
- 10) Shake the bottle gently and check the temperature of the milk by sprinkling a few drops on to your wrist before you feed your baby.
- 11) Take care not to overheat the milk.

NB: please refer to the timings in Diagram 2.

NB: The basket can be removed when heating the AVENT Magic Cup and other makes of bottles.

For baby food jars and tins

- 1) Ensure basket is in place inside the warmer.
- 2) Remove the lid and place jar or tin in the warmer.
- 3) Fill the warmer with water to 15 mm (1/2 inch) below the top of the jar/tin.
- 4) Plug in the unit and switch on at the mains. Turn the dial to setting 3.
- 5) Continue as 7-9 above.
- 6) The warmer will work in the same way as for feeding bottles. Take care when removing the jar or tin as they, and the water surrounding them, will be hot.
- 7) For small jars/tins you may find it beneficial to place a screw ring from an AVENT Airflex Bottle below the jar/tin before adding the water in order to remove the container easily.
- 8) Stir the food and test it on a spoon yourself to ensure it is not too hot for your baby.
- 9) Take care not to overheat the feed.

NB: please refer to the timings in Diagram 2.

Care of your warmer

Unplug and allow to cool before cleaning. Remove basket and wipe inside and outside with a damp cloth only, this will keep the warmer looking new. **DO NOT USE AN ABRASIVE CLEANER.** It may be advantageous to use pre-boiled or filtered water in your warmer in order to reduce limescale build up.

De-scaling

You should de-scale your Express Bottle and Baby Food Warmer every four weeks to ensure it continues to work 100 % effectively.

EITHER:

Mix one sachet of citric acid (10 g) with 200ml/7oz of water. Ensure basket is in place and pour solution into warmer. Heat for 10 minutes on setting 3. After 10 minutes, switch off by unplugging from the wall and leave to stand for 30 minutes. Pour the solution away and rinse the warmer with tap water. Citric acid is available from most chemists and through Philips AVENT Customer Service.

OR

Pour 50ml/2oz of vinegar (white) mixed with 100ml/4oz of cold water into the warmer (with basket in place). Allow to stand in the unit until any limescale has dissolved. Empty the unit and rinse thoroughly.

Warning!

The Express Bottle and Baby Food Warmer is fitted with a non-rewireable moulded plug. The mains supply cord of the appliance CANNOT be replaced. If the cord is damaged, the appliance should be discarded.

The lead supplied must not be used on any other appliance and no other lead should be used with the Express Bottle and Baby Food Warmer.

This product is manufactured to comply with the Radio Interference Requirements and the EEC Directive 89/336/EEC.

- When in use the warmer contains hot water.
- Always place container into warmer before adding water.
- Ensure water has been added before switching the unit on.
- If the feed is left in the warmer, the temperature will continue to increase as the thermostat cuts in/out whilst connected to the electric mains.
- Always test the feed and when it has reached the required temperature, remove immediately.
- Always unplug from the socket after each use to switch the bottle warmer off.
- **NEVER IMMERSE IN WATER.**
- This appliance is intended for use by responsible persons, keep out of the reach of small children and the infant.
- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience or knowledge, unless they have been given

supervision or instruction concerning use of the appliance by a person responsible for their safety.

Spoon

- For use 6+ months.
- Small-bowled spoons specially designed for small mouths and tender gums.
- Long handle – easy to hold, use and clean.
- Shaped for easier feeding without spills.

For your child's safety and health Warning!

Before first use, clean the product. Dishwasher safe. Sterilise using a Philips AVENT Steam Steriliser or boil for 5 minutes. Before each use, inspect the product. Throw away at the first sign of damage or weakness. Do not leave in sunlight when not in use or for longer than recommended in heat or sterilising solution, as this may weaken the product. Always use this product with adult supervision. Always check food temperature before feeding.

Guarantee

Philips AVENT guarantees that should the Express Bottle and Baby Food Warmer become faulty during twelve months from the date of purchase, it will be replaced free of charge during this time. Please return it to the retailer where it was purchased. Please keep your receipt as a proof of purchase, together with a note of when and where purchased. For the guarantee to be valid, please note:

- The Express Bottle and Baby Food Warmer must be used according to the instructions in this leaflet and connected to a suitable electricity supply.
- The claim must not be a result of accident, misuse, dirt, neglect or fair wear and tear.

This does not affect your statutory rights.

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