**Important****For your child's safety and health****WARNING!**

- Always use this product with adult supervision.
- Never use feeding teats as a soother.
- Continuous and prolonged sucking of fluids will cause tooth decay.
- Always check food temperature before feeding.
- Heating in a microwave oven may produce localised high temperatures. Take extra care when you heat up drinks in a microwave.
- Keep all parts not in use out of the reach of children.
- Do not allow child to play with small parts or walk/run while using bottles or cups.
- Do not place in a heated oven.
- Drinks other than milk and water, such as fruit juices and flavoured sugary drinks are not recommended. If they are used, they should be well diluted and only used for limited periods, not constantly sipped.
- Do not use abrasive cleaning agents or antibacterial cleaners. Do not place components directly on surfaces that have been cleaned with anti-bacterial cleaners.
- Expressed breast milk can be stored in sterilized Philips Avent bottles/containers in the refrigerator for up to 48 hours (not in the door) or in the freezer for up to 3 months. Never refreeze breast milk or add fresh breast milk to already frozen milk.
- Always discard any breast milk that is left over at the end of a feed.
- Plastic material properties may be affected by sterilizing and high temperatures. This can affect the fit of the cap.
- Throw away any part at the first signs of damage or weakness.
- Do not alter the product or parts of it in any way. This may result in unsafe functioning of the product.

**Teat**

- Inspect before each use and pull the feeding teat in all directions. Throw away at the first signs of damage or weakness.
- For hygiene reasons, we recommend replacing teats after 3 months.
- Keep teats in a dry, covered container.
- Do not leave a feeding teat in direct sunlight or heat, or leave in disinfectant.

**Before first use**

- Before first use, disassemble all parts, clean and sterilise the product.
  - Clean to ensure hygiene by washing and rinsing all parts thoroughly.
  - Sterilise by using a Philips Avent sterilizer or boil all parts for 5 minutes. This is to ensure hygiene.
  - During sterilising with boiling water, prevent the bottle or other parts from touching the side of the pan. This can cause irreversible product deformation or damage that Philips cannot be held liable for.
  - When using microwave steriliser: set to max 1000 Watt and follow user instructions of the sterilizer precisely.
  - Inspect the feeding teat before use and pull the feeding teat in all directions. Do not use damaged products.
- Note:** Every baby is unique and your baby's individual needs may vary from the description of flow rates.

**Assembly**

When you assemble the bottle, make sure you place the cap vertically onto the bottle so that the teat sits upright (see image for more details). The teat is easier to assemble if you wriggle it upwards instead of pulling it up in a straight line.

**Maintenance**

- Before each use, inspect all items.
- Pull the feeding teat in all directions.
- Throw away any part at the first signs of damage or weakness.
- For hygiene reasons, we recommend replacing teats after 3 months.
- Keep teats in a dry, covered container.
- When not in use, do not leave in sunlight or heat, or leave in disinfectant (sterilising solution) for longer than recommended, as this may weaken the teat. Do not place in a heated oven.

**Choosing the right teat for your baby**

The Philips Avent teats are available with different flow rates to help your baby with drinking. Over time you can change the teat accommodating the individual needs of your baby. Philips Avent teats are clearly numbered on the side to indicate flow rate (see image). Make sure you use a teat with the correct flow rate when you feed your baby. Use a lower flow rate if your baby chokes, is leaking milk or has trouble adjusting to the drinking speed. Use a higher flow rate if your baby falls asleep during feeding, gets frustrated or when feeding takes a very long time.

Teats are made of silicone and available for 0m+, 1m+, 3m+, 6m+, variable flow (**I/II/III**) and thick feed (**Y**).  
**(1)** The teat for 0m+ is ideal for newborn and breastfed babies of all ages. The teat has only one opening for fluid and the number 1 is shown on the teat.  
**(2)** The teat for 1m+ is ideal for breastfed babies of all ages. The teat has two openings for fluid and the number 2 is shown on the teat.  
**(3)** The teat for 3m+ is for bottle fed babies at 3 months of age and up. The teat has three openings for fluid and the number 3 is shown on the teat.

**(4)** The teat for 6m+ is ideal for bottle fed babies at 4 months of age and up. The teat has four openings for fluid and the number 4 is shown on the teat.  
**(I/II/III)** The variable flow teat with an adjustable flow rate to babies convenience is ideal for bottle fed babies at 3 months of age and up. The teat has one slot cut on the top for fluid and the markings I, II, III on the edges of the teat.

**(Y)** The thick feed teat designed for thicker feeds is ideal for bottle fed babies at 6 months of age and up. The teat has one Y cut for fluid and the symbol Y on the teat.

**Note:** Every baby is unique and your baby's individual needs may vary from the description of flow rates.

**Ordering accessories**

To buy accessories or spare parts, visit [www.shop.philips.com/service](http://www.shop.philips.com/service) or go to your Philips dealer. You can also contact the Philips Consumer Care Center in your country (see the international warranty leaflet for contact details).

**Compatibility**

- Spare teats are available separately. Make sure you use a teat with the correct flow rate when you feed your baby. To buy accessories or spare parts, visit [www.shop.philips.com/service](http://www.shop.philips.com/service) or go to your Philips dealer. You can also contact the Philips Consumer Care Centre in your country.
- The high quality Avent bottle is compatible with most Philips Avent breast pumps, spouts, sealing discs and cup tops.
- For more information about the materials used for Philips bottles: [www.philips.com/avent/materials](http://www.philips.com/avent/materials)

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**(3)** The teat for 3m+ is for bottle fed babies at 3 months of age and up. The teat has three openings for fluid and the number 3 is shown on the teat.

**(4)** The teat for 6m+ is ideal for bottle fed babies at 4 months of age and up. The teat has four openings for fluid and the number 4 is shown on the teat.

**(I/II/III)** The variable flow teat with an adjustable flow rate to babies convenience is ideal for bottle fed babies at 3 months of age and up. The teat has one slot cut on the top for fluid and the markings I, II, III on the edges of the teat.

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