Always here to help you

Register your product and get support at
www.philips.com/welcome


HP3616

Question?
Contact
Philips

PHILIPS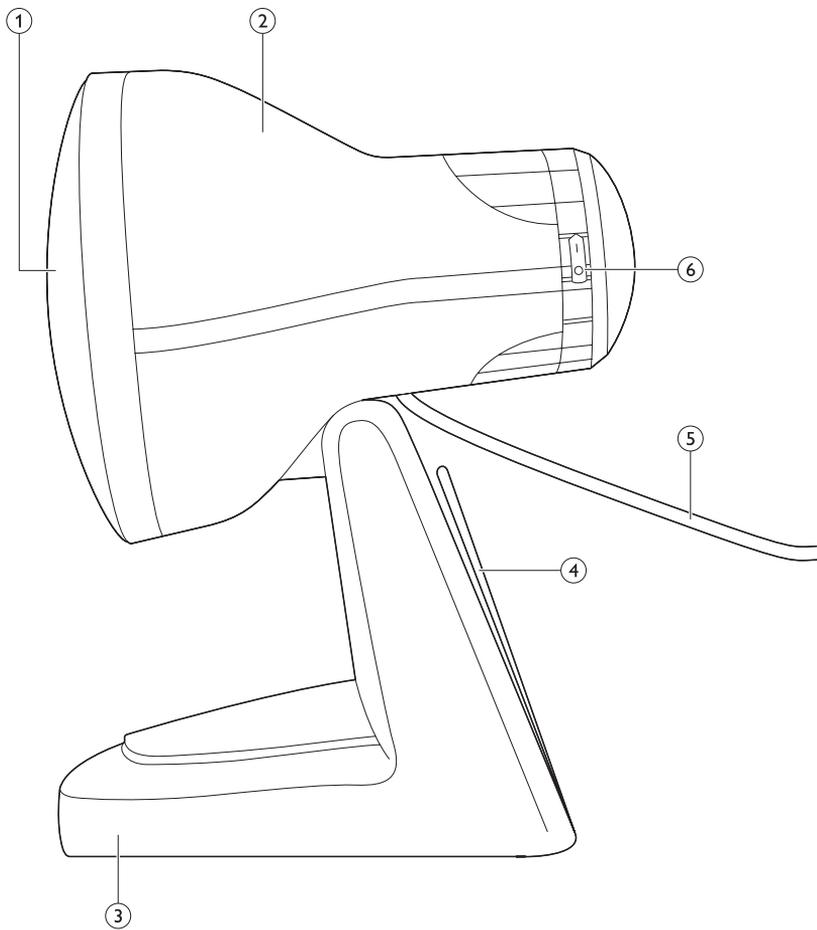

ENGLISH 6

FRANÇAIS 12

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BAHASA MELAYU 24

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Table of contents

Introduction	6
General description	6
Important	6
Preparing for use	8
Using the appliance	8
Cleaning	9
Storage	9
Replacement	9
Recycling	10
Guarantee and support	10
Troubleshooting	10
Specifications	10

Introduction

Congratulations on your purchase and welcome to Philips! To fully benefit from the support that Philips offers, register your product at www.philips.com/welcome.

With the InfraPhil, you can comfortably treat muscle and joint problems and relieve muscle and joint pain at home. Just like the sun, the appliance emits infrared light. The special filter only passes the type of light required to achieve the intended therapeutic effect. The InfraPhil can also be used for relaxation or for preparation for a massage.

General description (Fig. 1)

- 1 Lamp
- 2 Lamp housing
- 3 Base
- 4 Cord storage facility
- 5 Mains cord
- 6 On/off indication

Important

General

- The symbols on the InfraPhil have the following meaning:
 - 1 This symbol warns you against hot surfaces (Fig. 2).
 - 2 This symbol indicates that you have to read the user manual carefully before you use the appliance. Save the user manual for future reference (Fig. 3).
 - 3 This symbol indicates that this appliance is double insulated (Class II) (Fig. 4).
- This appliance meets the requirements of Medical Device Directive MDD93/42/EEC.

Danger

- Water and electricity are a dangerous combination! Do not use this appliance in wet surroundings (e.g. in the bathroom or near a shower or swimming pool).
- Do not let water run into the appliance or spill water onto the appliance.

Warning

- Check if the voltage indicated on the appliance corresponds to the local mains voltage before you connect the appliance.
- If the mains cord is damaged, you must have it replaced by Philips, a service centre authorised by Philips or similarly qualified persons in order to avoid a hazard.
- Do not leave the appliance unattended when it is switched on.
- This appliance is not intended for use by persons with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- This appliance is not intended for use by children. Children should be supervised to ensure that they do not play with the appliance.
- This appliance is intended for the treatment of muscle and joint pain and muscle and joint stiffness, not for the treatment of abdominal, facial and other sorts of pain.
- To prevent overheating of the skin, avoid very short treatment distances. The warmth should be comfortable throughout the treatment.
- To prevent eye damage, do not look into the light of the lamp when it is on.
- Do not fall asleep during treatment.
- Prevent the treatment area from cooling down too fast immediately after treatment.
- If there is no improvement after 6-8 treatments, stop using the appliance and consult your doctor.
- **Contraindications for InfraPhil treatment:**
- Do not use the appliance without consulting your doctor first if you suffer from severe diseases such as heart disease, acute inflammatory diseases, thrombosis, blood coagulation disorders, adrenal suppression, systemic lupus erythematosus or malignant diseases.
- Do not use the appliance if you are oversensitive to infrared light or if you have oedema. If in doubt, consult your doctor.
- Do not use the appliance if you are insensitive to infrared radiation and heat due to a disease (e.g. advanced diabetes) or the use of painkillers. If in doubt, consult your doctor.
- Do not use the appliance if you suffer from conditions associated with nerve demyelination such as carpal tunnel syndrome, multiple sclerosis and ulnar neuropathy. If in doubt, consult your doctor.
- Do not use the appliance if you have an implanted device, particularly not if it is located in the vicinity of the treatment area. If in doubt, consult your doctor.
- If you are pregnant, do not use the appliance to treat areas close to your abdomen or your lower back area, to avoid overheating of these areas. If in doubt, consult your doctor.
- Do not use the appliance if you have a swelling or an inflammation, as heat could aggravate the complaints. If in doubt, consult your doctor.
- Do not use the appliance on damaged or irritated skin or on wounds. If in doubt, consult your doctor.
- Do not use the appliance to treat areas on which you have recently applied topical counterirritants (creams or ointments containing e.g. menthol), as this can cause a mild inflammatory reaction.

Caution

- If the wall socket used to power the appliance has poor connections, the plug of the appliance becomes hot. Make sure you plug the appliance into a properly connected wall socket.
- Do not subject the appliance to heavy shocks.
- Always unplug the appliance after use and in case of a power failure.
- Place the appliance on a stable, level surface. Make sure there is at least 15cm free space around it to prevent overheating.
- Do not suspend the appliance from the ceiling (Fig. 5).
- Do not hang the appliance on the wall (Fig. 6).
- Do not use the appliance at room temperatures lower than 10°C or higher than 35°C.
- Do not cover the appliance with, for example, a cloth or a piece of clothing when it is switched on.

8 ENGLISH

- The appliance is equipped with an automatic protection against overheating. If the appliance is insufficiently cooled (e.g. because the cooling vents are covered), the appliance switches off automatically. Unplug the appliance and remove the cause of overheating. Once the appliance has cooled down, it can be switched on again. To do so, plug the appliance in again and press the on/off switch.
- Let the appliance cool down for approx. 15 minutes before you clean or store it.
- If you have just been swimming or have just taken a shower, make sure you dry your skin properly before you use this appliance.
- Make sure the beam of infrared light stays at a safe distance of at least 45cm from flammable objects and walls.

Electromagnetic fields (EMF)

This Philips appliance complies with all applicable standards and regulations regarding exposure to electromagnetic fields.

Preparing for use

Do not suspend the appliance from a ceiling or hang it on a wall.

- 1** Fully unwind the mains cord.
- 2** Place the appliance on a stable, level surface and make sure there is at least 15cm free space around it to prevent overheating.

Using the appliance

Effects of the InfraPhil

The InfraPhil produces infrared light, which penetrates deeply into the skin and warms the tissues. The warmth modulates the neuronal signals to the brain and thus reduces pain. Warmth also stimulates the blood circulation and dilates the blood vessels, which accelerates the transport of substances necessary to rebuild and nourish the body's tissues. Furthermore, infrared light speeds up the metabolic process and the removal of waste substances from the body. Because it makes tissue more pliable, warmth also reduces stiffness and makes joints more flexible. Because of these effects, treatment with the InfraPhil can temporarily relieve pain due to muscle and joint problems.

Note: Please read the 'Warning' section in chapter 'Important' and follow the instructions given there before you use the appliance.

Applications

Therapeutic applications

The InfraPhil is a local warmth therapy appliance. Possible therapeutic applications:

- Treatment of occasional or chronic muscle pains, stiff muscles and stiff joints.
- Treatment of lower-back ache.

Other applications

You can also use the InfraPhil for the following purposes:

- Muscle relaxation
- Preparation for a massage

Using the InfraPhil

1 Determine the distance between the body and the appliance:

- When you treat muscle pains, the distance between the lamp and the area to be treated must be approx. 30cm. Make sure the heat feels comfortable on your skin. Increase the distance if the heat becomes too intense.

2 Position the appliance properly.

- To treat a backache, place the appliance on a table at a distance of approx. 30cm (Fig. 7).
- To treat a sore knee, place the appliance on a low table. Sit down on a chair next to the table at a distance of approx. 30cm (Fig. 8).

3 To switch on the appliance, turn the rear cap of the lamp housing to 'I' (Fig. 9).

4 Aim the beam of infrared light at the part of the body to be treated (Fig. 10).

5 Switch off the appliance after use.

Do not look into the light of the lamp when it is on. Keep your eyes closed.

Duration of treatment

The duration of the treatment depends on the type of treatment and the person to be treated. Treatment of a single muscle or joint should last approximately 15 minutes. If necessary, the treatment can be repeated several times a day. A longer treatment time than 15 minutes does not lead to better results. You can repeat the course of several treatments a day on several consecutive days to get a result. If there is no improvement after 6-8 treatments, stop using the appliance and consult your doctor. For chronic problems, the appliance can be used for longer periods of time, as long as you follow the instructions given in the 'Warning' section in chapter 'Important'.

Cleaning

Unplug the appliance and let the lamp cool down for approx. 15 minutes before you clean the appliance.

Do not let water run into the appliance or spill water onto the appliance.

- 1 Clean the appliance with a cloth moistened with some distilled water or alcohol.

Storage

- 1 Let the appliance cool down for approx. 15 minutes before you store it.
- 2 Wind the mains cord around the cord storage facility (Fig. 11).

Replacement

Unplug the appliance and let the lamp cool down for approx. 15 minutes before you replace the lamp.

- The lamp must be replaced with an identical infrared lamp (Philips PAR38E/150W).
- You can obtain or order a new lamp from your Philips dealer.
- The lamp of this appliance does not contain substances that are harmful to the environment. You can throw away old lamps with the normal household waste.

Recycling

Do not throw away the product with the normal household waste at the end of its life, but hand it in at an official collection point for recycling. By doing this, you help to preserve the environment.

Guarantee and support

If you need information or support, please visit www.philips.com/support or read the separate worldwide guarantee leaflet.

Guarantee restrictions

The terms of the international guarantee do not cover the infrared lamp.

Troubleshooting

This chapter summarises the most common problems you could encounter with the appliance. If you are unable to solve the problem with the information below, visit www.philips.com/support for a list of frequently asked questions or contact the Consumer Care Centre in your country.

Problem	Possible cause	Solution
The appliance doesn't work.	The plug is not inserted into the wall socket properly.	Put the plug in the wall socket properly.
	There is a power failure.	Connect another appliance to check if the power supply works.
	The lamp is defective.	Replace the lamp (see chapter 'Replacement').
	The mains cord is damaged.	If the mains cord is damaged, you must have it replaced by Philips, a service centre authorised by Philips or similarly qualified persons in order to avoid a hazard.

Specifications

Model	HP3616
Rated voltage (V) Singapore/Europe/Bahrain/United Arab Emirates/Jordan	230
Rated voltage (V) China/Hong Kong/Korea/Kingdom of Saudi Arabia (KSA)/Lebanon	220
Rated voltage (V) Malaysia/Qatar	240
Rated voltage (V) Taiwan	110
Rated frequency (Hz) China/Malaysia/United Arab Emirates/Bahrain/Qatar/Jordan/Lebanon/Hong Kong/Singapore/Europe	50
Rated frequency (Hz) Korea/Taiwan/Kingdom of Saudi Arabia (KSA)	60

Model	HP3616
Rated input power (W)	150
Class	II (double insulated)
Ingress of water	IPX0
IR type	IR-A/B/C
Irradiance (W/m ²)	approx. 250
Field dimensions (in cm ²)	30x20
Battery type	not applicable

Operation conditions

Temperature	from +10°C to +35°C
Relative humidity	from 30% to 90%

Storage conditions

Temperature	from -20°C to +50°C
Relative humidity	from 30% to 90% (no condensation)

Circuit diagrams, component part lists and other technical descriptions are available on request. As user of this appliance, you should take measures to prevent the occurrence of electromagnetic interference or other types of interference between this appliance and other appliances.

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