
**PHILIPS
AVENT**

www.philips.com/avent

Philips Consumer Lifestyle BV
Tussendiepen 4, 9206 AD Drachten, the Netherlands

Trademarks owned by the Philips Group.
©2018 Koninklijke Philips N.V. All rights reserved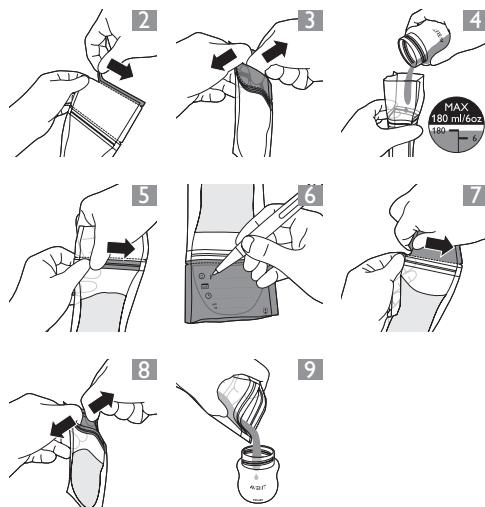

ENGLISH

Introduction

The Philips Avent milk storage bags offer a safe and convenient way to collect your precious breast milk. They are made of dual-layer, reinforced plastic and have a strong seal, to ensure your milk remains protected while being stored. A wide and sturdy opening ensures safe and easy filling and pouring. For ultimate hygiene, every bag comes presterilised with a tamper-evident seal.

Important

Read this user manual carefully before you use the milk storage bags and save it for future reference.

Warning

- Keep the milk storage bags out of the reach of children.
- Always check the temperature of the milk before you feed your baby.
- To prevent the milk from overheating, do not defrost the milk storage bag with breast milk in the microwave, in freshly boiled water or in a bottle warmer.

Caution

- The milk storage bags are intended for one-time use only.
- Thoroughly wash your hands before you fill and empty the milk storage bags.
- The actual amount of milk in the bag may deviate slightly from the level indications on the bag.
- Never refreeze breast milk or add fresh breast milk to already frozen breast milk.
- You can store the milk storage bag in the refrigerator for up to 48 hours (not in the door) or in the freezer for up to 3 months.

Filling the milk storage bag

- 1 Open the milk storage bag by tearing off the tamper-evident seal at the top of the milk storage bag. Keep the torn-off seal away from children and dispose of it safely. (Fig. 2)
- 2 To open the bag, place your thumbs inside the plastic above the zip and gently pull both sides apart (Fig. 3).
- 3 Pour the expressed milk into the bag. Do not fill the bag beyond the 180ml/6oz level, as breast milk expands when frozen (Fig. 4).
- 4 Carefully remove excess air by flattening the part of the bag above the milk.
- 5 Seal the bag by closing the zip (Fig. 5).

- 6 Use a pen to write down the date of expression and other information on the labelling area. Do not write on the filling part to prevent puncturing the bag (Fig. 6).

Defrosting breast milk

To defrost breast milk overnight, place the milk storage bag in the refrigerator. To defrost breast milk prior to feeding, place the milk storage bag in warm water.

Pouring milk from the milk storage bag

- 1 Tear off the labelling area to eliminate potential contamination of the breast milk during pouring (Fig. 7).

- 2 To open the bag, place your thumbs inside the plastic above the zip and pull both sides apart (Fig. 8).

- 3 Hold the bag by the zip and pour the milk into a container (Fig. 9).

Tip: Lightly squeeze the zip to make pouring easier.

- 4 Throw away the milk storage bag after use. Do not reuse the milk storage bag.

- 5 To heat the milk, place the container with defrosted milk in a bottle warmer or in a bowl of hot water.

Note: Always discard any breast milk that is left over at the end of a feed.

Guarantee and support

If you need information or support, please visit www.philips.com/support.

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