ENGLISH

Introduction

Congratulations on your purchase and welcome to Philips! To fully benefit from the support that Philips offers, register your product at www.philips.com/welcome.

Digital thermometer (SCH420)

Intended use

The digital thermometer is for domestic/home use. The digital thermometer measures your or your child's body temperature in the rectum, under the tongue or in the armpit. For children younger than three years we advise you to use the digital thermometer only in the rectum.

Important safety information

Read this important information carefully before you use the device and save it for future reference.

Warnings

- Keep the device out of reach of infants, children or pets. These products contain items with sharp edges and small parts. Inhalation or swallowing of small parts can be a choking hazard.

- Avoid leaving the thermometer. The digital thermometer contains a battery button; if the button battery is swallowed, it can cause severe internal burns in just 2 hours and can lead to death. Battery compartment cover of the digital thermometer must be replaced properly if removed.

- Clean the digital thermometer after each use (see "Cleaning"). After using the digital thermometer rectally or in the armpit, do not use it for oral measurement for hygiene reasons, not even after cleaning.

- Batteries must not be exposed to extreme temperatures or heat. Do not store or leave any product in direct sunlight or near any source of heat.

- Never attempt to recharge the battery.

- The digital thermometer is not suitable for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the product by a person responsible for their safety.

- Always check the device for damage before use. Do not use the device when damaged; this could result in injuries.

- Do not use the digital thermometer as part of the device at any time, this could result in injuries.

- Do not use the digital thermometer near metal objects, such as coins, keys, radio transmitters or microwave ovens. This could cause measurement errors and reduce the essential performance of the digital thermometer.

- Do not use the digital thermometer within 30 cm (12 inch) of an RF device (e.g. mobile phone, baby monitor, DECT telephone etc.). This could cause measurement errors and reduce the essential performance of the digital thermometer.

Cautions

The digital thermometer is suitable for use at room temperatures between 10 °C (50 °F) and 40 °C (104 °F). When used outside this temperature range, accuracy of the measurement is not guaranteed.

Do not drop the digital thermometer or expose it to shocks; this might damage the device and lead to inaccurate measurements.

- Do not bend or disassemble the tip of the digital thermometer.

- Do not use the digital thermometer if the display shows a dot on the bottom right of the screen. This means that the battery is low and therefore measurements will be inaccurate.

- Do not use the digital thermometer tip while you measure temperature orally. Prevent your mouth from biting the tip while you measure temperature orally. Simply place the tip under the tongue.

Electromagnetic fields (EMF)

This Philips device complies with all applicable standards and regulations regarding exposure to electromagnetic fields.

Measuring temperature of children under the age of three years

Rectal measurement

(*) Note: Measure the temperature in the rectum as the thermometer most accurate result when measuring the temperature of children under the age of three years.

1 Remove the protection cap from the digital thermometer. Make sure that the metal sensor of the flexible tip is firmly attached to the thermometer.

2 Hold the digital thermometer by pressing the button "On/Off". All display items will turn up briefly and then disappear ("Off" / "Last temperature").

3 Put water-based lubricant on the tip of the digital thermometer.

4 Lay the child on its stomach or side.

5 With the digital thermometer in the rectum, gently slide the tip of the digital thermometer no more than 12 mm (0.5 inch) into the rectum.

Make sure that the probe of the thermometer is fully inserted. If you detect any resistance, gently withdraw the probe and try again.

6 Firmly hold the child to prevent it from pulling the digital thermometer out of the rectum.

7 After removing the digital thermometer in place until the "C/F" indication stops flashing and the digital thermometer beeps.

Number of beeps What it means

1 beep per second No fever detected (<37.8 °C/100 °F)

3 beeps per second Possible fever detected (>37.8 °C/100 °F)

8 Remove the digital thermometer and reset the display. If the display shows "Up" (bottom right of the screen), the digital thermometer cannot measure the temperature. Switch off the thermometer and switch it on again. Repeat step 5 above to repeat the measurement.

(*) Note: The thermometer only indicates fever. It also beeps once per second when your child's temperature is below normal. Results may vary based on the individual.

9 Switch off and clean the digital thermometer after each use (see "Cleaning"). It switches off automatically after 10 minutes.

Measuring temperature of children above the age of three years

Armpit measurement

(*) Note: Measure the temperature in the rectum for the most accurate result when measuring the temperature of children above the age of three years. See the instructions for rectal measurement above. You can also select one of the other two options to measure your or your child's temperature in the rectum or under the tongue. The rectum is the best place to measure the temperature. If the rectal measurement is <37.8 °C (99.8 °F), then the rectal measurement is >37.8 °C (99.8 °F).

1 Remove the protection cap from the digital thermometer. Make sure that the probe of the flexible tip is firmly attached to the thermometer.

2 Hold the digital thermometer by pressing the button "On/Off". All display items will turn up briefly and then disappear ("Off" / "Last temperature").

3 Put water-based lubricant on the tip of the digital thermometer.

4 Lay the child on its stomach or side.

5 Gently hold the child to prevent it from pulling the digital thermometer out of the rectum.

6 After removing the digital thermometer in place until the "C/F" indication stops flashing and the digital thermometer beeps.

Number of beeps What it means

1 beep per second No fever detected (<37.8 °C/100 °F)

3 beeps per second Possible fever detected (>37.8 °C/100 °F)

8 Remove the digital thermometer and reset the display. If the display shows "Up" (bottom right of the screen), the digital thermometer cannot measure the temperature. Switch off the thermometer and switch it on again. Repeat step 5 above to repeat the measurement.

(*) Note: The thermometer only indicates fever. It also beeps once per second when your child's temperature is below normal. Results may vary based on the individual.

9 Switch off and clean the digital thermometer after each use (see "Cleaning"). It switches off automatically after 10 minutes.

Measuring temperature of children above the age of three years

Tongue measurement

(*) Note: Measure the temperature in the rectum for the most accurate result when measuring the temperature of children above the age of three years. See the instructions for rectal measurement above. You can also select one of the other two options to measure your or your child's temperature in the rectum or under the tongue. The rectum is the best place to measure the temperature. If the rectal measurement is <37.8 °C (99.8 °F), then the rectal measurement is >37.8 °C (99.8 °F).

1 Remove the protection cap from the digital thermometer. Make sure that the probe of the flexible tip is firmly attached to the thermometer.

2 Hold the digital thermometer by pressing the button "On/Off". All display items will turn up briefly and then disappear ("Off" / "Last temperature").

3 Put water-based lubricant on the tip of the digital thermometer.

4 Lay the child on its stomach or side.

5 Gently hold the child to prevent it from pulling the digital thermometer out of the rectum.

6 After removing the digital thermometer in place until the "C/F" indication stops flashing and the digital thermometer beeps.

Number of beeps What it means

1 beep per second No fever detected (<37.8 °C/100 °F)

3 beeps per second Possible fever detected (>37.8 °C/100 °F)

8 Remove the digital thermometer and reset the display. If the display shows "Up" (bottom right of the screen), the digital thermometer cannot measure the temperature. Switch off the thermometer and switch it on again. Repeat step 5 above to repeat the measurement.

(*) Note: The thermometer only indicates fever. It also beeps once per second when your child's temperature is below normal. Results may vary based on the individual.

9 Switch off and clean the digital thermometer after each use (see "Cleaning"). It switches off automatically after 10 minutes.

Features

Celsius (°C)/Fahrenheit (°F)

When the digital thermometer is off, press and hold the on/off button for 2 seconds. This changes the measurement unit.

Last temperature recall

The digital thermometer has a memory function. When you switch on the digital thermometer, the display shows the last measured temperature for 2 seconds. When the display shows "Up", you can perform the measurement.

Battery-low signal

The battery is low when you see a dot in the bottom right corner of the display (Fig. 16). You can no longer measure the temperature correctly. Replace the battery (see "Remove and insert batteries").

Children's thermometer

A child's thermometer normally is between 36.5 °C (97.6 °F) and 37.5 °C (99.5 °F). A child's thermometer depends on age, activity, time of day, etc. Some illnesses do not have high temperature as a symptom.

Important: Thermometers are not suitable for use by very young children. See the measurement method when you report your child's temperature to a healthcare professional. The thermometer is intended as an aid. Never leave your child alone. Always make sure there is someone present to look after the child and take care of its needs.

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