© 2019 Koninklijke Philips N.V.
All rights reserved
Philips Consumer Lifestyle BV
Tussendiepen 4, 9206AD Drachten,
Netherlands


www.philips.com 3000.033.1691.2 (07/2019)


PHILIPS
AVENT


Philips Avent anti-colic baby bottle
 Philips Avent anti-colic Bòtòl bayi
 Philips Avent anti-colic Bình cho trẻ em
 Philips Avent anti-colic ขวดนมสำหรับทารก
 Philips Avent anti-colic 嬰兒奶瓶
 Philips Avent anti-colic 젖병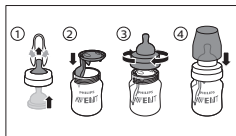


Philips Avent trainer cup
 Philips Avent Cawan latihan
 Philips Avent Cốc tập uống
 Philips Avent ถ้วยหัดดื่ม
 Philips Avent 學習水杯
 Philips Avent 트레이닝 컵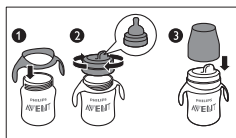

Philips Avent Milk Storage container
 Philips Avent Bekas Penyimpanan Susu
 Philips Avent Hộp bảo quản sữa
 Philips Avent ภาชนะเก็บนม
 Philips Avent 乳汁儲存容器
 Philips Avent 모유/분유 보관 용기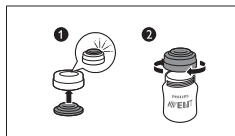

Philips Avent breast pump
 Philips Avent Pam susu badan
 Philips Avent Dụng cụ hút sữa
 Philips Avent เครื่องปัมนานม
 Philips Avent 吸乳器
 Philips Avent 유축기

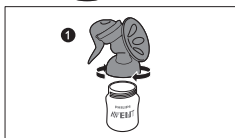For your child's safety and health

WARNING!

- Keep the cap away from children to avoid suffocation.
- Always use this product with adult supervision.
- Never use feeding teats as a soother, to prevent choking hazard.
- Continuous and prolonged sucking of fluids will cause tooth decay.
- Always check food temperature before feeding.
- It is not recommended to use a microwave to warm up baby's food or drinks. Microwaves might alter the quality of food/drinks and destroy some valuable nutrients and may produce localized high temperatures. Therefore, take extra care if and when you heat up food in a microwave. When heating up liquid or food in the microwave, only place the container without screw ring, AirFree™ vent insert, teat and cap. Always stir heated food to ensure even heat distribution and check the temperature before serving.
- Keep all components not in use out of the reach of children.
- Do not allow child to play with small parts or walk/run while using bottles or cups.
- Do not place in a heated oven.
- Drinks other than milk and water, such as fruit juices and flavored sugary drinks are not recommended. If used, they should be well diluted and only used for limited periods, not constantly sipped.

- Expressed breast milk can be stored in sterilized Philips Avent bottles/containers in the refrigerator for up to 48 hours (not in the door) or in the freezer for up to 3 months. Never refreeze breast milk or add fresh breast milk to already frozen milk.
- Always discard any breast milk that is left over at the end of a feed.
- Do not use abrasive cleaning agents or anti-bacterial cleaners. Do not place components directly on surfaces that have been cleaned with anti-bacterial cleaners.
- Plastic material properties may be affected by sterilizing and high temperatures. This can affect the fit of the cap.
- Inspect the feeding teat before use and pull the feeding teat in all directions.
- Throw away any part at the first signs of damage or weakness.
- Do not alter the product or parts of it in any way. This may result in unsafe functioning of the product.

AirFree™ vent insert

Benefit: The AirFree™ vent insert keeps the teat full of milk during feeding, even while the bottle is held horizontally. This supports an upright feeding position for your baby in order to reduce air ingestion. This helps reduce common feeding issues including wind and reflux (GER).

Assembly: Place the AirFree™ vent insert in the bottle before you assemble the screw ring with teat.

Warning: When you prepare formula, make sure that all the formula is fully dissolved to avoid blocking the AirFree™ vent insert.

Note: Always make sure the teat is full of milk by turning the bottle upside down before feeding your baby. While feeding, make sure that the AirFree™ vent insert points upwards, in line with your baby's nose.

Compatibility: The AirFree™ vent insert only works with 4 oz/125 ml, 9 oz/260 ml and 11 oz/330 ml Philips Avent Anti-colic bottle and Classic+ bottles, teat and screw ring. The AirFree™ vent insert is not compatible with the (I/II/III) variable flow teat and the Natural bottle range.

Note: When using an 11 oz/330 ml bottle, the AirFree™ vent insert will keep the teat full of milk but it could be that you see fewer air bubbles.

Philips Avent Anti-colic bottles and Classic+ bottles can also be used without the AirFree™ vent insert.

Before first and every use

Disassemble all parts and clean them in warm soapy water and rinse them thoroughly. Sterilize the parts in boiling water for 5 minutes or by using a Philips Avent sterilizer. This is to ensure hygiene. During sterilizing in boiling water, make sure that the pan contains enough water and prevent the bottle or other parts from touching the side of the pan. This can cause irreversible product deformation, defect or damage that Philips cannot be held liable for. Make sure that you wash your hands thoroughly and that the surfaces are clean before contact with sterilized parts. Place all bottle parts on a clean paper towel or on a clean drying rack and allow them to air dry. Excessive concentration of detergents may eventually cause plastic components to crack. Should this occur, replace immediately. This product is dishwasher safe – food colourings may discolour parts. Clean and sterilize all parts before each use. For hygiene reasons, we recommend replacing teats after 3 months.

Assembly

When you assemble the bottle, make sure you place the cap vertically onto the bottle so that the teat sits upright (see image for more details). The teat is easier to assemble if you wriggle it upwards instead of pulling it up in a straight line.

Maintenance teat and AirFree™ vent insert

Do not leave a feeding teat and/or AirFree™ vent insert in direct sunlight or heat, or leave in disinfectant ('sterilizing solution') for longer than recommended, as this may weaken the parts. Keep parts in dry, covered container.

Choosing the right teat for your baby

The Philips Avent teats are separately available with different flow rates to help your baby with drinking. Over time you can change the teat accommodating the individual needs of your baby. Philips Avent teats are clearly numbered on the side to indicate flow rate. Make sure you use a teat with the correct flow rate when you feed your baby. Use a lower flow rate if your baby chokes, is leaking milk or has trouble adjusting to the drinking speed. Use a higher flow rate if your baby falls asleep during feeding, gets frustrated or when feeding takes a very long time. These Philips Avent teats can be used on the Philips Avent feeding bottle and Philips Avent storage cups.

(1) The teat for 0m+ is ideal for newborn and breastfed babies of all ages. The number 1 is shown on the teat.

(2) The teat for 1m+ is ideal for breastfed babies of all ages. The number 2 is shown on the teat.

(3) The teat for 3m+ is ideal for bottle fed babies at 3 months of age and up. The number 3 is shown on the teat.

(4) The teat for 6m+ ideal for bottle fed babies at 6 months of age and up. The number 4 is shown on the teat.

(I/II/III) Variable flow: has a bite resistant silicone teat. The sturdier teat, with an adjustable flow rate to babies convenience is ideal for bottle fed babies at 3 months of age and up. The teat has one slot cut on the top for fluid and the markings I, II, III on the edges of the teat.

(Y) Thick feed: The thick feed teat designed for thicker feeds is ideal for bottle fed babies at 6 months of age and up. The teat has one Y cut for fluid and the symbol Y on the teat.

Note: Every baby is unique and your baby's individual needs may vary from the description of flow rates.

Ordering accessories

To locate a retailer or purchase replacement bottle, AirFree™ vent insert, teat or any Philips Avent baby product, visit our website at www.philips.com/avent. The AirFree™ vent insert can be purchased separately as a replacement or for use with any compatible Philips Avent Anti-colic and Classic+ bottle.

Compatibility

Make sure to choose the correct teat corresponding to the correct screw ring of your bottle: teats from the Philips Anti-colic range only fit in screw rings of the Philips Anti-colic range, and Philips Natural teats only fit in Philips Natural screw rings. The high quality Philips Avent bottle is compatible with most Philips Avent breast pumps, spouts, sealing discs and cup tops.

Support

If you need information or support, please visit www.philips.com/support.

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