Pumping, storing and feeding in one easy stop.

Pumping

Storing

Feeding

## 1. Important safety notice

**For your child's safety and health WARNING!**

- Clean the product before first use.
- For the safety and health of the child, always use this product with adult supervision.
- During inspection of the teat, pull the teat in all directions. Throw away at the first signs of damage or weakness.
- Before each use, inspect the feeding bottle and ensure the components are correctly attached. Do not use if the bottle is damaged or split.
- Do not use the teat as a soother.

## 2. Before each use

- Before each use, detach all components and thoroughly sterilise the product.
- To ensure cleanliness, place all components in boiling water for five minutes or sterilise using a Philips Avent electric or microwave steam steriliser.

## 3. Assembly

- Wash both hands thoroughly before touching sterilised components.
- Make sure you use a feeding teat with the correct flow rate when you feed your baby.

- Ensure the anti-colic valve (the valve on the side of the teat) is open. If the anti-colic valve is closed, pull the teat to the side to open it.
- Always assemble teats and screw ring when wet.
- To ensure the bottle functions correctly and to prevent leaks, always ensure you:
  - remove any debris or residue that may collect around the rim of the bottle assembly;
  - avoid over-tightening the screw when assembling onto the bottle base.

Anti-colic valve

## 4. Feeding suggestions

- The teat that comes with this bottle should only be used for milk or water. For fruit juice or soup, use the Philips Avent standard cross cut teat, sold separately.
- To avoid scalding, do not pour hot liquids into the feeding bottle.
- Avoid warming contents in a microwave oven as this may cause uneven heating and could scald your baby.
- After sterilising, do not rinse out the bottle and teat to cool down.
- Hold the bottle rather than the bottle top.
- To avoid scalding, do not prepare milk powder feeds close to the baby.
- When preparing formula milk, attach the protective cap when shaking to prevent leaks.
- After preparing the milk powder feed, keep the bottle upright (placing the bottle on an uneven surface may result in leakage).
- Check the temperature of the food before feeding. Test the temperature of the food on the palm of the hand, and begin feeding once the temperature is deemed appropriate.
- Feeding should commence within 2 hours of preparing the milk powder. If feeding does not occur within this timeframe, discard the feed.
- To avoid damage to the teat, do not allow babies that are teething to feed for extended periods.
- Extended feeding of liquids may lead to tooth decay.
- Keep sterilised components on clean surfaces.
- Expressed breast milk may be kept in a sterilised Philips Avent feeding bottle and stored in a refrigerator (not in the door) for up to 48 hours, or in the freezer for up to 3 months.
- Do not refreeze breast milk, or add fresh breast milk to already frozen milk.

- For hygiene reasons, after three months we recommend replacing teats.

## 5. Cleaning and Sterilising

- Do not use corrosive or anti-bacterial cleaners, chemical solutions, or concentrated cleaners.
- Do not place components directly on surfaces that have been cleaned with anti-bacterial cleaners.
- Food colorings may discolor components.
- Remove components and clean the product thoroughly after each use.
- If necessary, clean with the Philips AVENT bottle and teat brush, sold separately.
- Place in boiling water for five minutes or sterilise using a Philips Avent electric or microwave steam steriliser. Consult the steriliser user manual for details.
- If steriliser solution is used, do not leave teats submerged for longer than the recommended time as this can weaken the teat.
- Do not place feeding bottles in a heated drier or sterilising cabinet.

## 6. Storage

- Keep small parts out of the reach of children.
- Keep bottles and components away from heat sources and direct sunlight as this can weaken the bottle and components.

- For hygiene reasons, remove all components and store in a dry container with a lid.

## 7. Accessories

- Spare teats can be purchased separately.
- Note: Philips Avent Essential bottles can only be used with Philips Avent Essential teats.

www.philips.com/AVENT

MY: 1800-880-180

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TH: 02-652-8652

PH: +632 7139435/+639228452144

ID: 0-800-133-3000

\* Based on December 2011 TNS online satisfaction survey conducted among more than 36,000 female users of childcare brands and products in the USA, the UK, France, Germany, Italy, the Netherlands, Poland, Spain, Russia and China.

† BPA free – following EU Directive 2011/8/EU

Philips AVENT, a business of Philips Electronics UK Ltd. Philips Centre, Guildford Business Park, Guildford, Surrey, GU2 8XH

Trademarks owned by the Philips Group.

© Koninklijke Philips N.V. All Rights Reserved.

Made in China in adherence to global quality of Philips AVENT

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