Always here to help you

Register your product and get support at
www.philips.com/welcome


HP3631

Question?
Contact
Philips

PHILIPS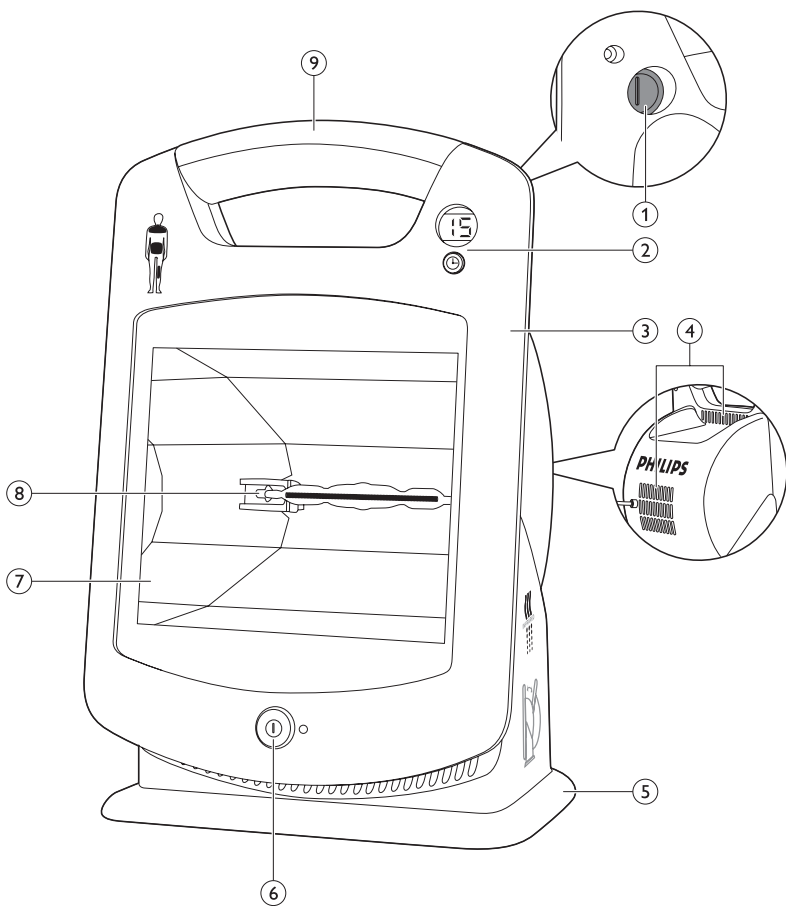

ENGLISH 6
DEUTSCH 14
ESPAÑOL 22
FRANÇAIS 30
ITALIANO 38
NEDERLANDS 46
PORTUGUÊS 54

Introduction

Congratulations on your purchase and welcome to Philips! To fully benefit from the support that Philips offers, register your product at www.philips.com/welcome.

With the InfraCare you can comfortably treat muscular problems and relieve muscular pain at home. Just like the sun, the appliance emits infrared light. The special filter only passes the type of light required to achieve the intended therapeutic effect.

General description (Fig. 1)

- 1 Battery compartment
- 2 Timer
- 3 Lamp housing with adjustable angle
- 4 Fan
- 5 Base
- 6 On/off button with power-on light
- 7 Filter
- 8 Infrared halogen lamp
- 9 Handgrip

Important

General

- The symbols on the appliance have the following meaning:
 - 1 This symbol warns you against hot surfaces (Fig. 2).
 - 2 This symbol indicates that you have to read the user manual carefully before you use the appliance. Save the user manual for future reference (Fig. 3).
 - 3 This symbol indicates that this appliance is double insulated (Class II) (Fig. 4).
- This appliance meets the requirements of Medical Device Directive MDD93/42/EEC.

Danger

- Water and electricity are a dangerous combination! Do not use this appliance in wet surroundings (e.g. in the bathroom or near a shower or swimming pool).
- Do not let water run into the appliance or spill water onto the appliance.

Warning

- Check if the voltage indicated on the appliance corresponds to the local mains voltage before you connect the appliance.
- If the mains cord is damaged, you must have it replaced by Philips, a service centre authorised by Philips or similarly qualified persons in order to avoid a hazard.
- Do not leave the appliance unattended when it is switched on.

- This appliance is not intended for use by persons with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- This appliance is not intended for use by children. Children should be supervised to ensure that they do not play with the appliance.
- This appliance is intended for the treatment of muscle and joint pain and muscle and joint stiffness, not for the treatment of abdominal, facial and other sorts of pain.
- To prevent overheating of the skin, avoid very short treatment distances. The warmth should be comfortable throughout the treatment.
- To prevent eye damage, do not look into the light of the lamp when it is on.
- Do not fall asleep during treatment.
- Prevent the treatment area from cooling down too fast immediately after treatment.
- If there is no improvement after 6-8 treatments, stop using the appliance and consult your doctor.
- ***Contraindications for treatment:***
- Do not use the appliance without consulting your doctor first if you suffer from severe diseases such as heart disease, acute inflammatory diseases, thrombosis, blood coagulation disorders, adrenal suppression, systemic lupus erythematosus or malignant diseases.
- Do not use the appliance if you are oversensitive to infrared light or if you have oedema. If in doubt, consult your doctor.
- Do not use the appliance if you are insensitive to infrared radiation and heat due to a disease (e.g. advanced diabetes) or the use of painkillers. If in doubt, consult your doctor.
- Do not use the appliance if you suffer from conditions associated with nerve demyelination such as carpal tunnel syndrome, multiple sclerosis and ulnar neuropathy. If in doubt, consult your doctor.
- Do not use the appliance if you have an implanted device, particularly not if it is located in the vicinity of the treatment area. If in doubt, consult your doctor.
- If you are pregnant, do not use the appliance to treat areas close to your abdomen or your lower back area, to avoid overheating of these areas. If in doubt, consult your doctor.
- Do not use the appliance if you have a swelling or an inflammation, as heat could aggravate the complaints. If in doubt, consult your doctor.
- Do not use the appliance on damaged or irritated skin or on wounds. If in doubt, consult your doctor.

- Do not use the appliance to treat areas on which you have recently applied topical counterirritants (creams or ointments containing e.g. menthol), as this can cause a mild inflammatory reaction.

Caution

- If the wall socket used to power the appliance has poor connections, the plug of the appliance becomes hot. Make sure you plug the appliance into a properly connected wall socket.
- Do not subject the appliance to heavy shocks.
- Always unplug the appliance after use and in case of a power failure.
- Place the appliance on a stable, level surface. Make sure there is at least 15cm free space around it for HP3621, 25cm free space for HP3631 and 35 cm free space for HP3643 to prevent overheating.
- Do not use the appliance at room temperatures lower than 10°C or higher than 35°C.
- Make sure the vents in the back of the appliance remain open during use.
- Do not cover the appliance with, for example, a cloth or a piece of clothing when it is switched on.
- The appliance is equipped with an automatic protection against overheating. If the appliance is insufficiently cooled (e.g. because the cooling vents are covered), the appliance switches off automatically. Unplug the appliance and remove the cause of overheating. Once the appliance has cooled down, it can be switched on again. To do so, plug the appliance in again and press the on/off switch.
- Let the appliance cool down for approx. 15 minutes before you clean or store it.
- If you have just been swimming or have just taken a shower, make sure you dry your skin properly before you use this appliance.
- Never use the appliance when the filter is damaged, broken or missing.
- Fully unwind the mains cord.
- Make sure the distance with your body and the appliance is large enough to prevent accidental contact with the hot filter.
- Do not let the mains cord come in contact with the filter when the filter is hot.
- Do not attempt to replace the lamp if the lamps stops working. Contact the Philips Consumer Care Centre for information.
- HP3643: To make sure all parts are properly assembled, check if the spring-loaded lock buttons are visible in the holes.
- HP3643: To avoid getting your fingers stuck when you adjust the height, hold the lamp housing by one of the handgrips or by the hinge as long as the height adjustment rings are unlocked.

- HP3643: To prevent unintentional unlocking of the height adjustment rings, do not put your hand on a height adjustment ring while you carry or lift the appliance.

Electromagnetic fields (EMF)

This Philips appliance complies with all standards regarding electromagnetic fields (EMF). If handled properly and according to the instructions in this user manual, the appliance is safe to use based on scientific evidence available today.

Using the appliance

The effect of the InfraCare

The InfraCare produces infrared light, which penetrates deeply into the skin and warms the tissues. Warmth modulates the neuronal signals to the brain and in this way reduces pain. Warming stimulates the blood circulation and dilates the blood vessels, which accelerates the transport of substances necessary to rebuild and nourish the body's tissues. Infrared light also speeds up the metabolic process and the removal of waste substances from the body. Because it makes tissues more pliable, warmth also reduces stiffness and makes joints more flexible. Because of these effects, treatment with the InfraCare can relieve pain due to muscular and joint problems.

Therapeutic applications

The InfraCare is a local warmth therapy appliance.

Possible applications:

- Treatment for occasional or chronic muscle pains, stiff muscles and stiff joints
- Treatment of lower-back ache (lumbago)
- Treatment of chilblains
- Preparation for another treatment such as a massage
- Treatment of sprains and bruises
- To relieve pain, e.g. due to rheumatic disorders

► For more information about the health benefits, see www.philips.com, search word InfraCare (Fig. 5).

Preparing for use

- 1 Fully unwind the mains cord.**
- 2 Place the appliance on a stable, level surface and make sure there is at least 25 cm free space around it to prevent overheating (Fig. 6).**
The appliance can be used in many different ways. Below you find some examples.
 - To treat a sore shoulder, a sore neck or part of your upper back area, place the appliance on a table, tilt the lamp housing backwards. Sit down on a chair or stool with your back towards the lamp (Fig. 7).
 - To treat a lower-back ache, place the appliance on a table beside a chair. Sit down on a chair or stool with your back towards the lamp (Fig. 8).
 - To treat a sore elbow, place the appliance on a table and sit down beside it (Fig. 9).
 - Place the appliance on a bedside table and lie down on the bed with one leg pulled up towards the lamp to treat a sore knee (Fig. 10).
- 3 Adjust the direction of the light beam by tilting the lamp housing forwards or backwards (Fig. 11).**

4 Sit or lie down at the right distance from the appliance.

- At a distance of 30-40 cm to the body, the InfraCare treats an area of about 40 cm by 30 cm (w x h). At a larger distance, the area treated becomes larger and the infrared light and therefore the warmth less intense. At a smaller distance, the area treated is smaller and the infrared light and therefore the warmth more intense.
- Make sure the distance between your body and the appliance is large enough to prevent accidental contact with the hot filter.
- Choose a distance at which the heat is comfortable.

Note: The warmth of the appliance does not reach its full intensity until several minutes after switching the appliance on. This does not mean, however, that the appliance is not effective during these first minutes.

Duration of treatment

The duration of the treatment depends on the person to be treated and the type of use. We recommend a few short treatments a day, e.g. 2 treatments of 15 minutes, on a number of consecutive days to get a result.

Note: Practice has shown that a few short treatments a day produce a better result than one longer treatment.

Always consult your doctor in case of doubt!

Using the InfraCare

- 1** Insert the plug into a wall socket.
- 2** Switch the appliance on by pressing the on/off button. The green power-on light will go on (Fig. 12).
- 3** Set the desired treatment time (0-99 minutes) by pressing the timer button. Keep the button pressed to adjust the time more quickly. The set time is shown on the timer display.
 - Two seconds after releasing the timer button, the timer starts to count down in minutes. The last minute is displayed in seconds. During countdown the remaining time blinks on the display.
 - When the set time has elapsed, you hear an intermittent beep and 00 blinks on the display.
 - Press the timer button to stop the beeping.
 - If you don't press the timer button, the beeping will automatically stop after 45 seconds.
 - To change the set time during treatment, keep the timer button pressed for two seconds until 00 appears on the display. Then use the timer button to set a new treatment time.

Note: The timer does not switch the appliance off. It only signals that the set time has elapsed.

Cleaning

Unplug the appliance and let it cool down for about 15 minutes before cleaning it.

Never use abrasive liquids such as scouring agents, petrol or acetone to clean the appliance.

- 1** Clean the lamp housing and the base with a moist cloth. Do not let any water run into the appliance.
- 2** You can clean the outside of the filter with a soft cloth sprinkled with some methylated spirits.

Storage

Do not let the cord come into contact with the filter when it is hot.

- 1 Unplug the appliance and let it cool down for about 15 minutes before you store it.
 - Always lift or carry the appliance by its handgrip (Fig. 13).
- 2 Store the InfraCare in a dry place.

Replacement

Filter

The filter protects against exposure to undesired light.

If the filter is damaged or broken, do not use the appliance anymore. Contact the Philips Consumer Care Centre for information.

Note: If the filter breaks or becomes damaged during the guarantee period, contact the Consumer Care Centre to have the appliance replaced.

Lamp

The effectiveness of the lamp does not diminish in the course of time.

Do not attempt to replace the lamp if the lamp stops working. Contact the Philips Consumer Care Centre for information.

Note: If the lamp stops working during the guarantee period, do not attempt to replace it yourself, but contact the Consumer Care Centre to have the appliance replaced.

Battery

The battery compartment of the timer is located in a recess in the back of the appliance.

- 1 Remove the cover of the battery compartment by turning it to the left with a screwdriver.
- 2 Insert a new battery. Make sure the - and the + poles of the new battery are pointing in the right direction.

Note: We advise you to use a 1.5V alkaline button battery, type LR54.

- 3 To close the battery compartment, put back the cover and turn it to the right with a screwdriver.

Environment

- Do not throw the appliance away with the normal household waste at the end of its life, but hand it in at an official collection point for recycling. By doing this you will help to preserve the environment (Fig. 14).
- Do not throw the battery away with the normal household waste but hand it in at an official collection point (Fig. 15).

Guarantee and support

If you need information or support, please visit www.philips.com/support or read the separate worldwide guarantee leaflet.

Troubleshooting

If the appliance does not work or does not work properly, first check the list below. If the problem is not mentioned in this list, the appliance probably has a defect. In that case we advise you to contact the Philips Consumer Care Centre.

Problem	Cause	Solution
The appliance does not go on.	The plug has not been inserted properly into the wall socket.	Insert the plug properly into the wall socket.
	There may be a power failure.	Check if the power supply works by connecting another appliance.
	The lamp may be defective.	Contact the Philips Consumer Care Centre.
	The mains cord of the appliance may be damaged.	Check the mains cord. If the mains cord is damaged, it must be replaced by Philips or similarly qualified persons in order to avoid a hazard.

Specifications

Model	HP3631
Rated voltage (V) Hongkong/Singapore/Europe	220-230
Rated voltage (V) China/Korea	220
Rated voltage (V) Malaysia	240
Rated voltage (V) Taiwan	110
Rated frequency (Hz) Hongkong/Singapore/ Europe	50-60
Rated frequency (Hz) China/Malaysia	50
Rated frequency (Hz) Korea/Taiwan	60
Rated input (W)	300
Class	II
Classification	not classified
Ingress of water	IPX0
IR-type	IR-A/B/C
Irradiance (W/m ²)	approx. 1000
Field dimensions (in cm)	40x30
Battery type	LR54

Operation conditions

Temperature	from +10°C to +35°C
Relative humidity	from 30% to 90%

Storage conditions

Temperature	from -20°C to +50°C
Relative humidity	from 30% to 90% (no condensation)

Circuit diagrams, component part lists and other technical descriptions are available on request. As user of this appliance, you should take measures to prevent the occurrence of electromagnetic interference or other types of interference between this appliance and other appliances.

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