
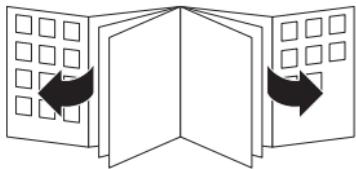
# InfraCare focused treatment

HP3621
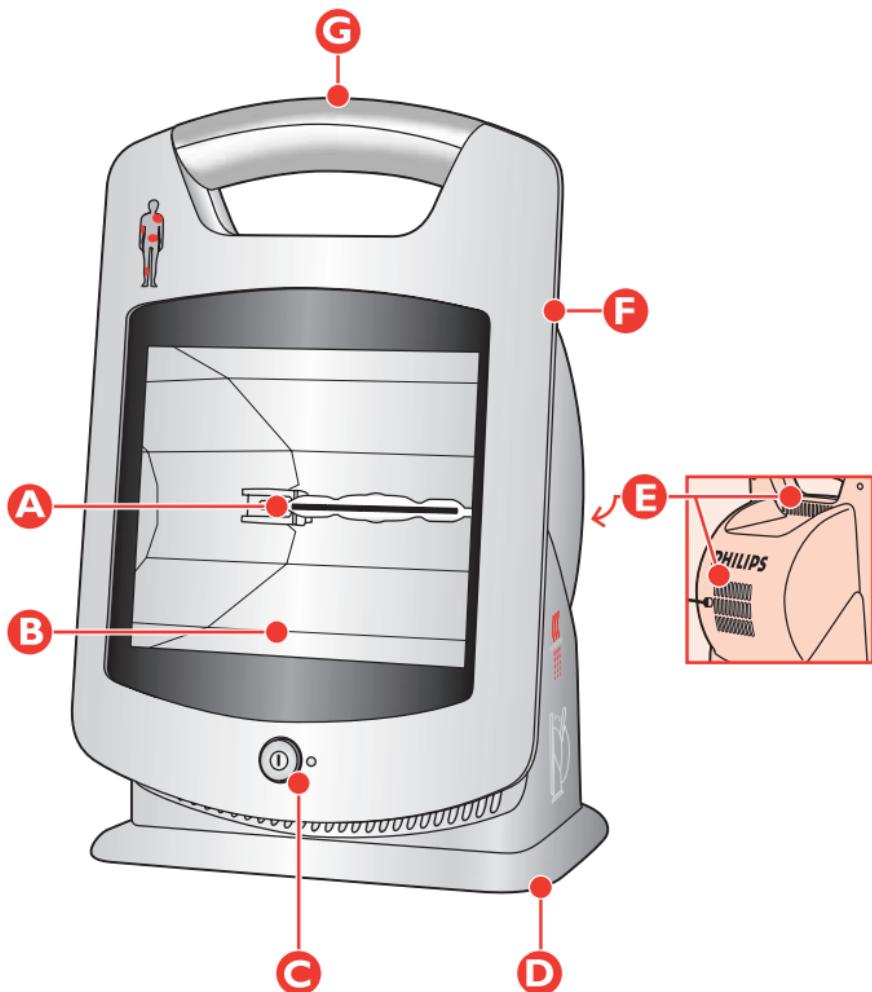
# PHILIPS

<b>ENGLISH</b>	6
<b>DEUTSCH</b>	14
<b>FRANÇAIS</b>	22
<b>NEDERLANDS</b>	30
<b>ESPAÑOL</b>	38
<b>ITALIANO</b>	46
<b>PORTUGUÊS</b>	54
<b>NORSK</b>	62
<b>SVENSKA</b>	70
<b>SUOMI</b>	77
<b>DANSK</b>	84
<b>ΕΛΛΗΝΙΚΑ</b>	92
<b>TÜRKÇE</b>	101
简体中文	108
繁體中文	114

## Introduction

With the InfraCare you can comfortably treat muscular problems and relieve pain at home. Just like the sun, the appliance emits infrared light. The special filter only passes the type of light required to achieve the intended therapeutic effect.

## General description (fig. I)

- A** Infrared halogen lamp
- B** Filter
- C** On/off button
- D** Base
- E** Lamp housing with adjustable angle
- F** Handgrip

## Important

The symbols on the InfraCare have the following meaning:

- Read these instructions for use carefully before using the appliance and save them for future reference (fig. 2).
- This appliance is double insulated according to Class II. You can therefore safely plug it into an unearthing wall socket (fig. 3).
- The surface marked with this symbol becomes hot (fig. 4).
- This appliance meets the requirements of Medical Device Directive MDD93/42/EEC.
- Check if the voltage indicated on the appliance corresponds to the mains voltage in your home before you connect the appliance.
- If the wall socket used to power the appliance has poor connections, the plug of the appliance will become hot. Make sure you plug the appliance into a properly connected wall socket.
- If the mains cord is damaged, it must be replaced by Philips, a service centre authorised by Philips or similarly qualified persons in order to avoid a hazard.
- Do not subject the appliance to heavy shocks.
- Water and electricity are a dangerous combination! Do not use

this appliance in wet surroundings (e.g. in the bathroom or near a shower or swimming pool).

- ▶ Do not let water run into the appliance or spill water onto the appliance.
- ▶ Do not leave the appliance unattended when it is switched on.
- ▶ Always unplug the appliance after use and in case of a power failure.
- ▶ Keep the appliance out of the reach of children.
- ▶ Place the appliance on a stable, level surface and make sure there is at least 15 cm free space around it to prevent overheating.
- ▶ Do not suspend the appliance from a ceiling or wall.
- ▶ Do not use the appliance at room temperatures lower than 10°C or higher than 35°C.
- ▶ Make sure the vents of the appliance remain open during use.
- ▶ The appliance has been fitted with an automatic protection against overheating. If the appliance is insufficiently cooled (e.g. because the vents are covered), the appliance will switch off automatically. Once the cause of overheating has been removed and the appliance has cooled down, the appliance will switch on again automatically.
- ▶ Let the appliance cool down for approx. 15 minutes before cleaning or storing it.
- ▶ If you have just been swimming or have just taken a shower, make sure you dry your skin properly before using this appliance.
- ▶ Do not look into the lamp's light when it is on. Keep your eyes closed.
- ▶ The filter becomes very hot when the appliance is switched on. Do not touch it!
- ▶ Never use the appliance when the filter is damaged, broken or missing.
- ▶ Avoid too intense cooling of the body part immediately after treatment.
- ▶ Do not fall asleep during treatment.
- ▶ Painkillers reduce the sensitivity to heat. Do not use this appliance when you are on painkillers without having consulted your doctor first.

- If you are oversensitive to infrared light, you should be extra careful when using this appliance. Consult your doctor in case of doubt.
- Do not use the appliance if you have a swelling or an inflammation, as heat could aggravate the complaints. If in doubt, consult your doctor.
- If there is no improvement after 6-8 treatments, stop using the appliance and consult your doctor.

## **Using the appliance**

### **The effect of the InfraCare**

---

The InfraCare produces infrared light, which penetrates deeply into the skin. This warms the skin, dilating the blood vessels and thus stimulating blood circulation.

The blood transports the substances necessary for rebuilding and nourishing the body's tissues more rapidly, while at the same time waste substances are removed more quickly.

This means treatment with the InfraCare can relieve muscular problems and pain.

### **Therapeutic applications**

---

The InfraCare is a local warmth therapy appliance.

Possible applications:

- Treatment for occasional or chronic muscle pains, stiff muscles and stiff joints
- Treatment of lower-back ache (lumbago)
- Treatment of chilblains
- Preparation for another treatment such as a massage
- Treatment of sprains and bruises
- Treatment of common colds
- To relieve pain, e.g. due to rheumatic disorders

For more information about the health benefits, see [www.philips.com](http://www.philips.com) searchword InfraCare.

## Preparing for use

---

- 1** Fully unwind the mains cord.
- 2** Place the appliance on a stable, level surface and make sure there is at least 15 cm free space around it to prevent overheating (fig. 5).

The appliance can be used in many different ways. Below you find some examples.

- ▶ Place the appliance on a table, tilt the lamp housing backwards and sit down on a chair or stool with your back towards the lamp to treat a sore shoulder or a sore neck or part of your upper back area (fig. 6).
  - ▶ Place the appliance on a table beside a chair and sit down on a chair or stool with your back towards the lamp to treat a lower-back ache (fig. 7).
  - ▶ Place the appliance on a table and sit down beside it to treat a sore elbow (fig. 8).
  - ▶ Place the appliance on the floor, tilt the lamp housing backwards and sit down on a chair or stool to treat a sore calf (fig. 9).
  - ▶ To treat a common cold, place the appliance on a table and sit down on a chair in front of the appliance. Tilt the lamp housing backwards so that the beam of light is directed towards your face (fig. 10).
- 
- 3** Adjust the direction of the light beam by tilting the lamp housing forwards or backwards (fig. 11).

### 4 Sit or lie down at the right distance from the appliance.

- At a distance of 25-30 cm to the body, the InfraCare treats an area of about 30 cm by 20 cm (w x h). At a larger distance, the area treated becomes larger and the infrared light and therefore the warmth less intense. At a smaller distance, the area treated is smaller and the infrared light and therefore the warmth more intense.
  - Make sure the distance between your body and the appliance is large enough to prevent accidental contact with the hot filter.
  - Choose a distance at which the heat is comfortable.
- Please note that the warmth of the appliance does not reach its full intensity until several minutes after switching the appliance on. This does not mean, however, that the appliance is not effective during these first minutes.

### Duration of treatment

---

The duration of the treatment depends on the person to be treated and the type of use. We recommend a few short treatments a day, e.g. 2 treatments of 15 minutes, on a number of consecutive days to get a result.

- Practice has shown that a few short treatments a day produce a better result than one longer treatment.

**Always consult your doctor in case of doubt!**

### Using the InfraCare

---

**1** Insert the plug into a wall socket.

**2** Switch the appliance on by pressing the on/off button.

### Cleaning

**Unplug the appliance and let it cool down for about 15 minutes before cleaning it.**

Never use abrasive liquids such as scouring agents, petrol or acetone to clean the appliance.

- 1** Clean the lamp housing and the base with a moist cloth. Do not let any water run into the appliance.
- 2** You can clean the outside of the filter with a soft cloth sprinkled with some methylated spirits.

## Storage

- 1** Unplug the appliance and let it cool down for about 15 minutes before storing it.
- Do not allow the cord to come into contact with the filter when it is hot.
- Always lift or carry the appliance by its handgrip (fig. 12).
- Store the InfraCare in a dry place.

## Replacement

### Filter

The infrared filter protects against exposure to undesired light. It is therefore important to have a damaged or broken filter replaced right away.

### Lamp

The effectiveness of the lamp does not diminish in the course of time.

- Have the lamp replaced when it stops functioning.

Have the 200W Philips infrared halogen lamp replaced by a service centre authorised by Philips. They have the knowledge and skills required for this job and the original spare parts for your appliance.

- The lamp has been developed to last at least 250 operating hours.

**Environment**

- Do not throw the appliance away with the normal household waste at the end of its life, but hand it in at an official collection point for recycling. By doing this you will help to preserve the environment (fig. 13).

The lamp of this appliance contains substances that can be harmful to the environment. When you discard the lamp, do not throw it away with the normal household waste but hand it in at an official collection point.

**Guarantee & service**

If you need service or information or if you have a problem, please visit the Philips website at [www.philips.com](http://www.philips.com) or contact the Philips Customer Care Centre in your country (you will find its phone number in the worldwide guarantee leaflet). If there is no Customer Care Centre in your country, turn to your local Philips dealer or contact the Service Department of Philips Domestic Appliances and Personal Care BV.

**Guarantee restrictions**

---

The infrared lamp is not covered by the terms of the international guarantee.

## Troubleshooting

If the appliance does not work or does not work properly, first check the list below. If the problem is not mentioned in this list, the appliance probably has a defect. In that case we advise you to take the appliance to your dealer or to an authorised Philips service centre.

Problem	Possible cause	Solution
The appliance does not go on.	The plug has not been inserted properly into the wall socket.	Insert the plug properly into the wall socket.
	There may be a power failure.	Check if the power supply works by connecting another appliance.
	The lamp may be defective.	Have the lamp replaced (see chapter 'Replacement').
	The mains cord of the appliance may be damaged.	Check the mains cord. If the mains cord is damaged, it must be replaced by Philips or similarly qualified persons in order to avoid a hazard.

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**PHILIPS DAP B.V.**