**EN:** User manual  
**ES:** Manual del usuario  
**FR:** Manuel d'utilisation  
**PT:** Manual do usuário

Philips NL9206AD-4 Drachten

4213.354.3789.1

**EN: For your child's safety and health WARNING!**

Always use this product with adult supervision. Never use feeding nipples as a pacifier. Continuous and prolonged sipping of fluids will cause tooth decay. Always check food temperature before feeding. Keep all components not in use out of the reach of children. Inspect before each use and pull the feeding nipple in all directions. Throw away at the first signs of damage or weakness. Do not store a feeding nipple in direct sunlight or heat, or leave in disinfectant ('sterilizing solution') for longer than recommended, as this may weaken the nipple. Do not place in a heated oven. Before first use place in boiling water for 5 minutes. This is to ensure hygiene. Clean before each use. Wash in warm soapy water and rinse thoroughly, then sterilize using a Philips Avent sterilizer suitable for Natural bottles or boil for 5 minutes. Do not warm contents in a microwave oven, as this may cause uneven heating and could scald your baby. Dishwasher safe - food colorings may discolor components. Do not allow child to play with small parts or walk/run while using bottles or cups. For hygiene reasons, we recommend replacing nipples after 3 months. For optimum performance, pinch and massage the number on the nipple between index finger and thumb prior to feed. Keep in a dry, covered container. Only use Philips Avent Natural nipples with Philips Avent Natural feeding bottles.

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