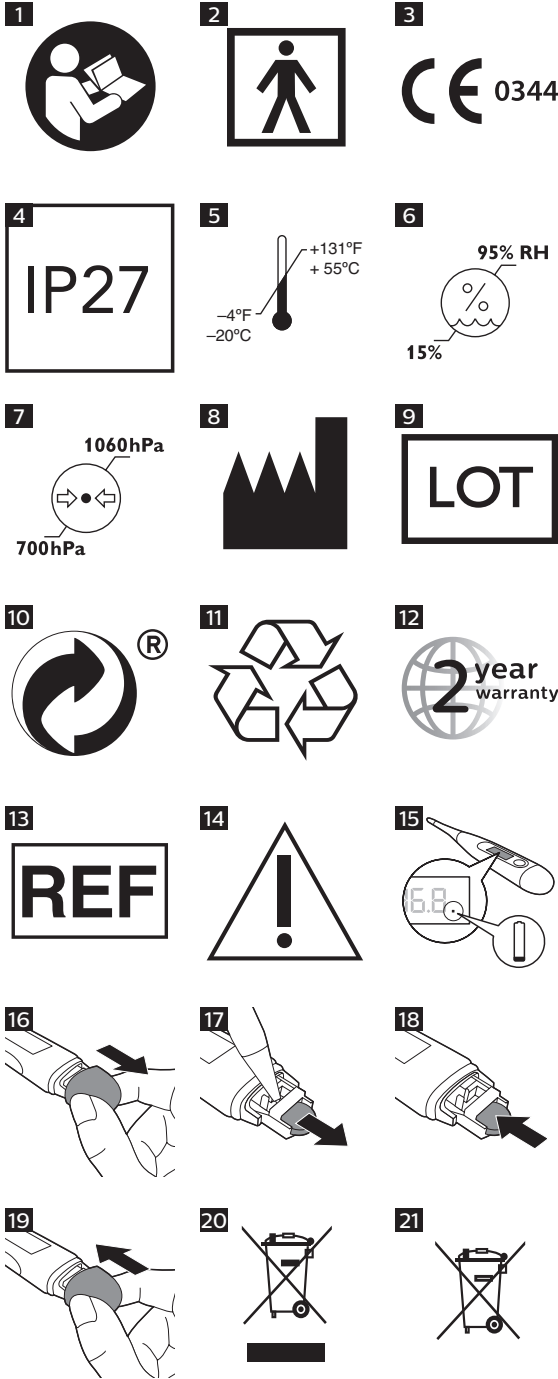
ENGLISH

Introduction

Congratulations on your purchase and welcome to Philips! To fully benefit from the support that Philips offers, register your product at www.philips.com/welcome.

Digital thermometer (SCH420)

Intended use

The digital thermometer is for domestic/home use. The digital thermometer measures your or your child's body temperature in the rectum, under the tongue or in the armpit. For children younger than three years we advise you to use the digital thermometer only in the rectum.

Important safety information

Read this important information carefully before you use the device and save it for future reference.

Warnings

- Keep the device out of reach of infants, children or pets. These products contain items with sharp edges and small parts. Inhalation or swallowing of small parts can be a choking hazard.
- Risk of swallowing batteries! The digital thermometer contains a button battery. If the button battery is swallowed, it can cause severe internal burns in just 2 hours and can lead to death. Battery compartment cover of the digital thermometer must be replaced properly if removed.
- Clean and dry all products before and after each use (see „Cleaning“). After using the digital thermometer rectally or in the armpit, do not use it for oral measurement for hygienic reasons, not even after cleaning.
- Batteries may explode if exposed to high temperatures or fire. Do not store or leave any product in direct sunlight or near any source of heat.
- Never attempt to recharge the battery.
- The thermometer is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the thermometer by a person responsible for their safety.
- Always check the device for damage before use. Do not use the device when damaged, this could result in injuries.
- Do not take apart, repair or change any part of the device at any time, this could result in injuries.
- Do not use the digital thermometer near strong electric or magnetic fields, for example magnets, radio transmitters or microwave ovens. This could cause measurement errors and reduce the essential performance of the digital thermometer.
- Do not use the digital thermometer within 30 cm (12 inch) of an RF device (e.g. mobile phone, baby monitor, DECT telephone etc.). This could cause measurement errors and reduce the essential performance of the digital thermometer.

Cautions

- The digital thermometer is suitable for use at room temperatures between 10 °C (50 °F) and 40 °C (104 °F). When used outside this temperature range, accuracy of the measurement is not guaranteed.
- Do not drop the digital thermometer or expose it to shocks, this might damage the device and lead to inaccurate measurements.
- Do not bend or disassemble the tip of the digital thermometer.
- Do not use the digital thermometer if the display shows a dot on the bottom right of the screen. This means that the battery is low and therefore measurements will be inaccurate.
- Do not bite the metal tip while you measure temperature orally. Prevent your child from biting the metal tip while you measure temperature orally. Simply place the tip under the tongue.

Electromagnetic fields (EMF)

This Philips device complies with all applicable standards and regulations regarding exposure to electromagnetic fields.

Measuring temperature of children under the age of three years

Rectal measurement

Note: Measure the temperature in the rectum for the most accurate result when measuring the temperature of children under the age of three years.

- 1 Remove the protection cap from the digital thermometer. Make sure that the metal sensor of the flexible tip is firmly attached to the digital thermometer.
- 2 Switch on the digital thermometer by pressing the on/off button. All display items light up briefly. Wait until the display shows 'Lo' (see „Last temperature recall“).
- 3 Put water-based lubricant on the tip of the digital thermometer.
- 4 Lay the child on its stomach or its side.
- 5 With one hand, separate the child's buttocks. With the other hand, gently slide the tip of the digital thermometer no more than 12 mm (0.5 inch) into the rectum. Make sure that the metal probe of the thermometer is fully inserted. If you detect any resistance, stop right away. Do not push the flexible tip too far into the rectum.
- 6 Firmly hold the child to prevent it from turning while the digital thermometer is in the rectum.
- 7 Keep the digital thermometer in place until the °C/°F indication stops flashing and the digital thermometer beeps.

Number of beeps	What it means
1 beep per second	No fever detected (<37.8 °C/100 °F)
3 beeps per second	Possible fever detected (>37.8 °C/100 °F)

- 8 Remove the digital thermometer and read the display. If the display shows 'Lo' (temperature is <32.0 °C/90.0 °F) or 'Hi' (temperature is >42.9 °C/109.9 °F) continuously after the measurement, the digital thermometer cannot measure the temperature. Switch off the thermometer and switch it on again. Refer to step 5 above to repeat the measurement.

Note: The thermometer only indicates fever. It also beeps once per second when your child's temperature is below normal. Results may vary based on the individual.

- 9 Switch off and clean the digital thermometer after each use (see „Cleaning“). It switches off automatically after 10 minutes.

>75% recycled paper

Philips Consumer Lifestyle BV
Tussendiepen 4, 9206 AD Drachten, The Netherlands / les Pays-Bas
www.philips.com/avent
Trademarks are the property of Koninklijke Philips N.V.
© 2020 Koninklijke Philips N.V.
All rights reserved. / Tous droits réservés.
4213.354.3941.4 (2020-06-05)

Measuring temperature of children above the age of three years and adults

Measure the temperature in the rectum for the most accurate result when measuring the temperature of children under the age of three years. See the instructions for rectal measurement above. You can also select one of the other two options to measure your or your child's temperature. When you measure in the armpit or under the tongue, the result is usually 0.5 °C (0.9 °F) to 1 °C (1.8 °F) lower than actual body temperature.

Note: The beep sound is only for the rectal measurement. Do not stop at the beep when using the thermometer in the armpit or under the tongue. Ignore any beep and wait for 5 minutes, with the thermometer at the measurement location, before reading the measurement from the display.

Measurement in the armpit

For armpit measurements:

- 1 Follow steps 1 to 2 under ‚Rectal measurement‘.
- 2 Make sure your or your child's armpit is dry. Place the tip of the digital thermometer in the armpit.
- 3 Press the arm against the body to hold the digital thermometer in place for 5 minutes. Ignore any beeps during this time. After 5 minutes, the display shows the temperature measurement.
- 4 Follow steps 8 to 9 under ‚Rectal measurement‘.

Measurement under the tongue

For measurements under the tongue:

- 1 Follow steps 1 to 2 under ‚Rectal measurement‘.
 - 2 Place the tip of the digital thermometer under your or your child's tongue and keep it in place for 5 minutes. Make sure the lips are sealed.
 - 3 Ignore any beeps during this time. After 5 minutes, the display shows the temperature measurement.
 - 4 Follow steps 8 to 9 under ‚Rectal measurement‘.
- Results may vary based on the individual and the method you used.

Temperature	What it means
<37.8 °C/100 °F	No fever detected
>37.8 °C/100 °F	Possible fever detected

Children's temperature

A child's temperature normally is between 36.5 °C (97.7 °F) and 37.5 °C (99.5 °F). A child's temperature varies depending on age, activity, time of day, etc. Some illnesses do not have high temperature as a symptom.

Important

Temperature measurement is not a substitute for going to your doctor. State the measurement method when you report your child's temperature to a healthcare professional. This thermometer is intended as an aid. Never leave your child alone. Always make sure there is someone present to look after the child and take care of its needs.

Features

Celsius (°C)/Fahrenheit (°F)

You can select Celsius (°C) or Fahrenheit (°F) to measure the temperature. When the digital thermometer is off, press and hold the on/off button for 2 seconds. This changes the measurement unit.

Last temperature recall

The digital thermometer has a memory function. When you switch on the digital thermometer, the display shows the last measured temperature for 2 seconds. When the display shows ‚Lo‘, you can perform the measurement.

Battery-low signal

The battery is low when you see a dot in the bottom right corner of the display (Fig. 16). You can no longer measure the temperature correctly. Replace the battery (see ‚Remove and insert batteries‘).

Cleaning

Clean with warm water and some washing-up liquid.

- Clean the digital thermometer and its parts gently to avoid damage.
- Do not use abrasive cleaning agents. Do not use anti-bacterial cleaning agents. Do not use chemical solvents. A very high concentration of cleaning agents may cause plastic to crack.
- Do not immerse the digital thermometer in alcohol or any other liquid.
- Never clean the digital thermometer in a dishwasher, microwave sterilizer or by boiling in water.

Storage

Put the protection cap on the digital thermometer after use. Store it in a clean and dry place.

Technical specifications

Digital thermometer	
Battery	1.5 V (LR41 or SR41)
Battery life	>1000 measurements
Measurement range	32 °C to 42.9 °C (89.6 °F to 109.2 °F)
Accuracy	±0.1 °C (±0.2 °F) for 35.5 °C to 42.0 °C (95.9 °F to 107.6 °F) ±0.2 °C (±0.4 °F) outside these ranges
Compliance	Conforms to MDD 93/42/EEC (class IIa medical device)
Operating and storage temperature	-20 °C to 55 °C (-4 °F to 131 °F)
Operating and storage humidity level	15–95 %
Operating and storage atmospheric pressure	700–1060 hPa
Expected service life	2 years. After the expected service life, accuracy cannot be guaranteed. If you question the accuracy of the measurement at any time, contact the Philips Consumer Care Center in your country.

Remove and insert batteries

- 1 Press the top and the bottom of the battery cover. Pull gently to remove it (Fig. 17).
- 2 Use a pen to remove the battery (Fig. 18). Do not pull the battery tray more than 1 cm out of the digital thermometer. The sensor can break off.
- 3 Dispose of the battery in a proper way.
- 4 Insert a new battery (Fig. 19).
- 5 Press the battery cover back onto the digital thermometer until it clicks into place (Fig. 20).

Error signal

If the digital thermometer does not work properly, the display shows 'Err'. Remove the batteries and wait 1 minute. Insert the batteries again and switch the digital thermometer back on.

Additional baby care products

Nasal aspirator

**Warnings:**

- **Never use the nasal aspirator in the child's mouth or ears.**
- **Never point the tip directly in the center of the nostril.**

- 1 Remove the cap of the nasal aspirator.
- 2 Squeeze the aspirator bulb and place the tip gently into the child's nostril and tilt the tip to the side. Be careful not to insert the aspirator into the child's nose too far.
- 3 Release the bulb slowly to let the suction draw mucus out of the child's nose.
- 4 Repeat the process with the other nostril.

Cleaning

- 1 Disassemble all parts of the nasal aspirator and wash them in warm water with washing-up liquid.
- 2 Clean the nasal aspirator in boiling water for 10 minutes and dry it afterwards.

Finger toothbrush

Place the finger toothbrush on your fingertip to massage the baby's gums gently.

Cleaning

Clean with a sterilizer or boiling water. Let it cool down and dry.

Brush and comb

Brush/comb the baby's hair from root to tip.
Tip: Carefully keep a hand on the roots while brushing or combing to avoid pulling the baby's scalp.

Cleaning

Clean with warm water and some washing up liquid. Rinse them thoroughly and let them dry.

Nail cutter and scissors

Be careful not to cut the nails too close to the baby's skin.
Tip: We advise you to cut the baby's nails after a bath when the nails are soft, or when the baby sleeps.

Cleaning

Wipe the nail cutter and the scissors with a damp cloth and dry them.

**Note:** You can sterilize the nail cutter and the scissors by wiping their cutting edges with alcohol.

Emery boards

Gently glide the emery board across the baby's nails.

Pouch

Save all items in the pouch and store them in a dry place.

Explanation of symbols

The warning signs and symbols are essential to ensure that you use this product safely and correctly and to protect you and others from injury. Below you find the meaning of the warning signs and symbols on the label and in the user manual.

- This symbol means: Read this manual (Fig. 1).
- This symbol is to identify a type BF applied part complying with IEC 60601-1 (Fig. 2).
- This symbol means: conforms to EU Directives. CE stands for „Conformité Européenne“. 0344 is the number of the notified body (Fig. 3).
- This symbol means: protected from touch by fingers and objects greater than 12 mm (0.47 inch). Protected from immersion between 15 cm (5.9 inch) and 1 m (39.3 inch) in depth (Fig. 4).
- This symbol means: storage temperature (Fig. 5).
- This symbol means: relative humidity (Fig. 6).
- This symbol means: atmospheric pressure (Fig. 7).
- This symbol means: the manufacturer, as defined in EU Directive 93/42/EEC (Fig. 8).
- This symbol means: numbers next to this logo indicate the manufacturer's Lot number (Fig. 9).
- This symbol means: the Green Dot („Der Grüne Punkt“ in German) is the license symbol of a European network of industry-funded systems for recycling the packaging materials of consumer goods (Fig. 10).
- This symbol means that an object is capable of being recycled - not that the object has been recycled or will be accepted in all recycling collection systems (Fig. 11).
- This symbol means: 2 year Philips warranty (Fig. 12).
- This symbol means: the manufacturer's catalog number of the device (Fig. 13).
- This symbol indicates caution. The user should consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself (Fig. 14).
- This symbol indicates usage tips, additional information or a note (Fig. 15).

Recycling

- This symbol means that this product shall not be disposed of with normal household waste (2012/19/EU) (Fig. 21).
- This symbol means that this product contains disposable batteries which shall not be disposed of with normal household waste (2006/66/EC) (Fig. 22).
- Follow your country's rules for the separate collection of electrical and electronic products and batteries. Correct disposal helps prevent negative consequences for the environment and human health.

Removing disposable batteries

To remove disposable batteries, see the instructions for placing and/or removing batteries in the user manual.

Always remove empty disposable batteries from the product. Take any necessary safety precautions when you dispose of batteries.

**Note:** Clean the digital thermometer thoroughly before disposal.

Warranty and support

If you need information or support, please visit www.philips.com/support or read the international warranty leaflet.

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