Nipple Shield Štitnik za bradavice

Philips Consumer Lifestyle B.V.
Tussendijken 4, 9206 AD Drachten,
The Netherlands
www.philips.com/avent

Trademarks are the property of Koninklijke Philips N.V.
© 2022 Koninklijke Philips N.V.
All rights reserved.
3000.039.9751-3 (2022-06-28)

Intended use. The Philips Avent Nipple Shield is intended to use with a breastfeeding baby who does not support breastfeeding. The device is intended for a single user.

Indications for use. The Philips Avent Nipple Shield is indicated for use when the mother has sore, sensitive, cracked nipples or suffers from nipple pain. It is also indicated to help the baby develop better latch-on behavior in case of oral anomalies or when the mother has flat/inverted nipples. Moreover, it can be used to alleviate milk supply difficulties when the baby is not transferring the milk well, has an uncommon sucking technique (i.e. weak sucking, small/premature babies) or to help the baby in copious milk transfer.

Contraindications. There are no contraindications. **Side effects.** When using the device, undesirable side effects may occur and protect the nipples of lactating women to support breastfeeding. The shield is intended for a single user.

Important safety information. Read this important information carefully before you use the product and save it for future reference.

Warnings to avoid choking and swallowing. Never leave the nipple shield and the hygiene case unattended and keep out of reach of children. Inspect before each use. Throw away if the first signs of damage or weakness or after 2 months of usage.

Warning to avoid contamination and to ensure hygiene. Follow the instructions described in the Cleaning and Disinfection sections.

Recommendations. Use the product only in case of breastfeeding problems and if you are not confident in your own ability to resolve the problem. Recommended to be used with advice from healthcare professional. If problems or pain occur, consult your healthcare professional for advice on the correct use of the product.

An electronic copy of these instructions can be found at www.philips.com/support

In any case incidents or damage unrelated in relation to this device should be reported to Philips via www.philips.com/support and to the competent authority of the Member state in which the user and/or patient is established.

Before usage

Warning
Do not use abrasive cleaning agents or aggressive chemicals or substances when you clean the nipple shield as this might cause damage. Clean the nipple shield before first use and after each use.

Manual cleaning

1 Rinse with cold, drinking-quality water (approx. 20°C) for 10-15 seconds.

2 Wash in warm, drinking-quality water (approx. 35°C) with washing-up liquid, preferably without artificial fragrances or coloring.

3 Rinse with cold, drinking-quality water (approx. 20°C) for 10-15 seconds.

4 Dry with a clean cloth or leave to dry on a clean surface.

Dishwasher cleaning

1 Place on top rack of the dishwasher.

2 Run standard program with common household dishwasher detergent.

3 Do not fully dry after dishwasher program, dry with a clean cloth or leave to dry on a clean surface.

Step 2 - Disinfection

Warning
Disinfect before first use and at least once a day. Be careful, after disinfection, the nipple shields may be very hot. The hygiene case remains in the case may still be hot. To prevent burns, only use after cooling down for 5 minutes.

Warnings to avoid contamination and to ensure hygiene

If you use the hygiene case for disinfection:

1 Clean the hygiene case before first use.

2 Use the correct water level, microwave power and time. Do not do so can lead to reduced disinfection and can damage the hygiene case or nipple shields.

3 Only use the hygiene case provided.

Boiling

1 Clean as described under "Cleaning".

2 Soak the nipple shield in dry, only the hygiene case provided.

3 Leave to dry on a clean surface.

Microwave Alternatively, use only the hygiene case provided according to the instructions below.

1 Clean the hygiene case as described under "Cleaning".

2 Add 25 ml (0.9 fl. oz.) of drinking-quality water to the hygiene case, up to the indicated line (Fig. 1).

3 Place the nipple shield(s) in the hygiene case and securely close the lid (Fig. 2).

4 Place the filled hygiene case in the microwave for 4 minutes at 750-1000 W (Fig. 3).

5 Let the hygiene case cool down for 5 minutes (Fig. 4).

6 Drain water from the hygiene case (Fig. 5).

7 Dry the nipple shield(s) with a clean cloth or leave to dry on a clean surface.

Usage

Warning
Wash your hands and breasts thoroughly with soap and water before you touch the nipple shields to prevent contamination.

Applying the product - Follow the instructions below when placing the device on your breast:

1 Remove the cleaned and disinfected nipple shield from the hygiene case.

2 Place drinking-quality water or breast milk on the skin-facing side of the rim to help the shield stick to your skin (Fig. 6).

3 Hold the shield at the base of the nipple area and press the nipple cavity partially inside out, so that it becomes half way inserted (Fig. 7).

4 Rotate the shield so that the baby can have nose and chin contact with your breast skin (Fig. 8).

5 Center the shield over your nipple and slightly stretch the wings of the shield as you stick the product onto your breast (Fig. 9).

6 With the shield applied, you can now breastfeed as normal.

Storing - Store the clean nipple shield(s) in the hygiene case or in a dry, clean container. Storing the nipple shield(s) in an unclear or wet container could cause contamination.

Disposal - According to local regulations.

Explanation of symbols
The warning signs and symbols are essential to ensure that you use this product safely and correctly and to protect you and others from injury. Below you find the meaning of the warning signs and symbols on the label and in the user manual.

CE This symbol indicates that this medical device is compliant with the general safety and performance requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices.

ES This symbol indicates the hygiene case is microwave compatible.

Water This symbol indicates to fill the hygiene case with water.

Wait This symbol indicates to wait 5 minutes before taking the hygiene case out of the microwave.

Recycle The ♻️ inside this symbol is the recycling code which the hygiene case is made, which is polypropylene (PP), to facilitate recycling or other reprocessing.

Green Dot The Green Dot ("Der Grüne Punkt" in German) is the license symbol of a European network of industry-funded systems for recycling the packaging materials of consumer goods.

REF This symbol indicates the manufacturer's catalog number of the nipple shield.

I This symbol indicates to follow the instructions for use.

FSC This symbol indicates the Forest Stewardship Council.

MD This symbol indicates that the device is a medical device.

UDI This symbol indicates there is a UDI Data Matrix.

Technical specifications

- Two sizes of the nipple shield: small 15 mm/0.6 in.; medium: 21 mm/0.8 in.

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