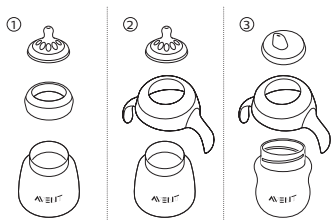
FSC

Fig. 1

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ENGLISH

Important

To fully benefit from the support that Philips offers, register your product at www.philips.com/welcome. Read this user manual carefully before you use the spout cup and save it for future reference.

For your child's safety and health**WARNING!**

- Check the spout cup before each use. If any damage or crack is detected, stop using the spout cup immediately.
- Always use this product with adult supervision.
- Never use the spout as a soother.
- Continuous and prolonged sucking of fluids will cause tooth decay.
- Always check food temperature before feeding.
- Prevent your child from running or walking while drinking.
- Keep all components not in use out of the reach of children.
- Do not heat up any liquid or food in the microwave.

Caution

- Before first use, disassemble all parts (Fig. A) and clean and sterilize them thoroughly. Place the spout cup in boiling water for 5 minutes. This is to ensure hygiene.
- Make sure all parts are properly assembled (Fig. B) before you give the spout cup to your child.
- This spout cup is intended for use with diluted drinks.
- Thick, fizzy or pulpy drinks may block the valve and cause the spout to break or leak.
- This spout cup is not designed for preparing infant formula. Do not use the spout cup to mix and shake infant formula as this can clog the holes and cause the spout cup to leak.

- Do not use the drinking top with a glass bottle.
- Do not put the spout cup in the freezer compartment.
- To prevent scalding, let hot liquid cool down before you fill the spout cup.
- Spouts are available separately. Use only Philips Avent spouts.

Cleaning and sterilization

Clean the spout cup before first use and after each use to ensure optimal hygiene. The spout cup is dishwasher-safe. Food colorings may discolor parts.

- Disassemble all parts (Fig. A).
- Clean all parts in warm water with some washing-up liquid. You can also clean all parts on the top rack of the dishwasher.

Caution: Do not use abrasive or anti-bacterial cleaning agents or chemical solvents. Excessive concentration of cleaning agents may eventually cause the spout to crack. If this occurs, replace the spout immediately.

Caution: Do not place parts directly on surfaces that have been cleaned with anti-bacterial cleaners.

- Sterilize the parts in a sterilizer or boiling water for max. 5 minutes. The spout cup is also suitable for all standard sterilization methods.

Note: Wash your hands thoroughly and make sure surfaces are clean before contact with sterilized items.

Do not sterilize any part of the spout cup in the microwave without using a microwave sterilizer.

- After sterilization, let the parts cool down for 5 minutes.

Assembling and disassembling

See Fig. B on how to assemble the spout cup and Fig. A on how to disassemble the spout cup. Ensure the spout is properly assembled by pulling the spout fully through the screw ring.

Storage

- For hygiene and maintenance, disassemble the spout cup (Fig. A) and store it in a dry and covered container.
- Keep the spout cup away from heat sources and direct sunlight.

Tip: The parts of the spout cup are compatible with all Philips Avent Natural bottles (Fig. 1).

Ordering accessories

To buy accessories or spare parts, visit www.philips.com/parts-and-accessories or go to your Philips dealer. You can also contact the Philips Consumer Care Center in your country.

Support

If you need information or support, please visit www.philips.com/support.

Compatibility (Fig. 1)

Mix and match Philips Avent bottle and cup parts and create the product that works for you, when you need it. All Philips Avent Natural bottles and cups are compatible, excluding glass bottles and My First Big Kid Cup/Grown Up Cup (SCF78X).

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