
PHILIPS

AVENT

Philips NL9206AD-4 Drachten

3000.050.6207.1 (17/1/2020)

FSC

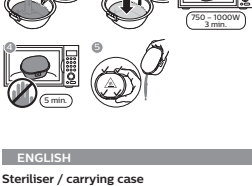ENGLISH

Steriliser / carrying case

This steriliser / carrying case allows you to sterilise your baby's soother in the microwave, and carry it with you.

Before use

- Remove tamper-proof closure and any stickers from the case.
- Wash all items and make sure that they are clean.
- Remove any dirt, food residue, fat or oils, etc. from all items as these can impair sterilisation and may damage the soother or case.

Note: Only use this steriliser / carrying case with enclosed soother.

To sterilise soother:

Caution: The steriliser / carrying case, water and soother may still be very hot after a cooling period of 5 minutes.

Warnings for steriliser/carrying case

Always check the temperature of the soother before use. Incorrect water level, microwave power or microwave time can lead to insufficient sterilisation and can damage the steriliser / carrying case or soother. Only use enclosed soother and keep the case out of reach of children.

Soother

For your child's safety

WARNING!

Inspect carefully before each use. Pull the soother in all directions. Throw away at the first signs of damage or weakness. Do not soak in water and then put in the freezer. This may weaken the soother. Only use dedicated soother holders tested to EN 12586. Never attach other ribbons or cords to a soother, your child may be strangled by them. Do not tie the soother around child's neck as it presents a strangulation danger.

Always make sure your baby uses the correct soother size. Philips Avent soothers come in different sizes, based on age range.

Before first use: Place the soother in boiling water for 5 minutes. Let it cool down for 5 minutes before you give it to your baby. Let water trapped inside the teat cool down and then squeeze it out. This is to ensure hygiene.

Cleaning and sterilisation: Clean the soother before every use with warm water and mild soap. DO NOT use abrasive cleaning agents or aggressive chemicals/substances on soothers.

Excessive combinations of detergents may eventually cause plastic components to crack. If this occurs, replace your soother immediately. Soothers are suitable for all commonly used forms of sterilisation. Clean surfaces and your hands thoroughly before contact with sterilised soothers.

Using: Always use this product under adult supervision. Replace the soother after 4 weeks of use, for safety and hygiene reasons. If the soother becomes lodged in the mouth, DO NOT PANIC; it cannot be swallowed and is designed to cope with such an event. Remove from the mouth with care, as gently as possible. Never dip the teat in sweet substances or medication, your child may get tooth decay.

Storage: Store the soother in the steriliser / carrying case or in a dry covered container. Do not leave a soother in direct sunlight or near a source of heat, or leave in sterilising solution for longer than recommended, as this may weaken the teat.

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