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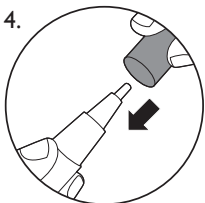
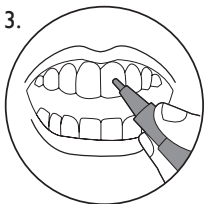
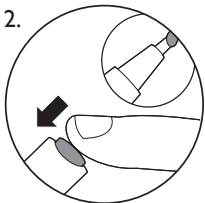
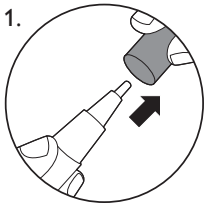
© 2014 Discus Dental, LLC. All rights reserved. Philips and the Philips shield are registered trademarks of Koninklijke Philips N.V. Zoom is a registered trademark of Discus Dental, LLC.

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ENGLISH

Icons Used on Product

- Read user manual before use
- Irritant
- Keep away from children
- Temperature limits
- Open shelf life
- Avoid sunlight
- Expiration date

Product description

Each Philips Zoom Whitening Pen contains whitening gel with 5.25% Hydrogen Peroxide.

Intended Use

The Philips Zoom Whitening Pen is for whitening of natural teeth only.

Whitening Pen

Instructions for Use

WARNING:

- Contains Hydrogen Peroxide (5.25%).
- Harmful when swallowed - Avoid ingesting during use of this product.
- Avoid contact with eyes as irritation or injury may occur. Rinse with water immediately if this occurs.
- Keep out of reach of children.
- Not for use by persons under 13 years of age.
- Avoid direct contact with gums or salivary flow. If irritation occurs (such as redness, swelling, or soreness), discontinue use and consult a dentist.
- Use of this product may result in tooth sensitivity. If this occurs, consult your dental professional.
- Store at room temperature (15°C-25°C / 59°F-77°F).
- Do not freeze.

Instructions for Use

1. Remove cap from Philips Zoom Whitening Pen (Fig. 1).
2. Refresh the brush tip of Philips Zoom Whitening Pen by clicking the pen button at the opposite end, until a small droplet is visible. You may have to click the pen up to 20 times the first time you use it (Fig. 2).
3. Brush a thin layer of whitening gel directly onto the front surfaces of the teeth (Fig. 3).
4. Replace cap when you are finished (Fig. 4).

For the best result, it is recommended that you brush your teeth before application. Philips Zoom Whitening Pen can be used day or night, and requires no waiting after application. Apply to teeth twice daily for two weeks. Philips Zoom Whitening Pen is an easy way to maintain your bright smile at work, at home or on the go. Philips Zoom Whitening Pen is not designed to be a substitute for professional teeth whitening or regular dental care. Please visit your dentist regularly. Philips Zoom Whitening Pen will not whiten crowns, veneers, fillings or dentures. It is recommended that you don't eat or drink for thirty minutes after application to ensure the Philips Zoom Whitening Pen gel has been absorbed. Store this product in a cool location out of direct sunlight. This product's effectiveness will decrease as it nears its expiration date or 6 months after it has been initially opened.

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